

***Refer to the device directions for use for complete instructions on device use.***

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

---

***Reuse Warning***

For single use only. Do not reuse or reprocess. Reuse, reprocessing or sterilizing may compromise the structural integrity of the device and/or lead to device failure. Reuse or reprocessing may also create a risk of contamination of the device or may lead to improper scope cleaning.

***Intended Use/ Indications For Use***

The Koala Cleaning Sponge plus Pure Enzymatic is intended to be soaked in water to wipe the surface of the endoscope after the endoscopic procedure as a pre-cleaning step prior to disinfection or sterilization of the endoscope.

***Contraindications***

None known.

***Warnings***

**DANGER:** May cause allergy or asthma symptoms or breathing difficulties if inhaled.

***Precautions***

- Ensure proper personal protective equipment (PPE) is worn during use of this device.
- Avoid contact with eyes and skin. In case of contact, flush with water. If irritation persists, contact a physician.
- Avoid breathing enzymatic dust. If inhaled: If breathing is difficult, move person to fresh air and keep comfortable for breathing. If experiencing respiratory symptoms: contact a physician.

***Adverse Events***

- Allergic Reaction
- Asthma Exacerbation

***Clinical Benefit Statement***

Remove debris and reduce bioburden from the exterior surface of endoscopes during pre-cleaning steps of endoscope reprocessing.