



## ***Refer to the device directions for use for complete instructions on device use.***

**Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.**

---

### ***Reuse Warning***

For single use only. Do not reuse, reprocess or sterilize. Reuse, reprocessing or sterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or sterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

### ***Indications For Use/ Intended Use***

Hedgehog cleaning brushes are intended for use to clean accessory and suction channel, working channel and auxiliary water channel of the endoscope, or to dean valves, valve openings and dials of the endoscope control head.

### ***Contraindications***

None known.

### ***Warnings***

- Do not use as a cytology brush.
- Endoscope channels may be damaged if brushes are forced through when channels are occluded or if resistance is met during brush passage. If sharp bends in the plastic sheath or wire cable are observed during insertion, dispose of brush to avoid damaging the endoscope.
- Channel Brushes: Do not twist the channel brush while the brush is in the scope.
- Valve Brushes: Avoid twisting or torquing the brush handle.

### ***Precaution***

- Use of this device with any other endoscope channel sizes or endoscope valves could potentially impact brush performance.
- Review endoscope manufacturer's care and cleaning guidelines prior to use of brushes.

### ***Adverse Events***

- None known.

All trademarks are the property of their respective owners.

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

Indications, contraindications, warnings and instructions for use can be found in the product labeling supplied with each device.

2023 Copyright © Boston Scientific Corporation or its affiliates. All rights reserved.