

Refer to the device directions for use for complete instructions on device use.

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

Reuse Warning

For single use only. Do not reuse, reprocess or sterilize. Reuse, reprocessing or sterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or sterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

Note: The Porpoise Air/Water Cleaning Adapter is to be used with Olympus 160, 180 and 190 gastrointestinal endoscopes.

User Information

The Porpoise Air/Water Cleaning Adapter is intended to be used by technicians, nurses, and other Health Care Professionals who are trained in endoscope reprocessing.

Intended Use/ Indications For Use

The intended use of the Porpoise Air/Water Cleaning Adapter is to be fitted to an endoscope air/water channel to flush the endoscope during the “precleaning” step of endoscope reprocessing after therapeutic and diagnostic endoscopic procedures. Not for use during a patient procedure.

Clinical Benefit Statement

The clinical benefit of the Porpoise Air/Water Cleaning Adapter is to facilitate a step of the precleaning process to flush the endoscope to help aid in the prevention of buildup of potentially infectious bioburden and biofilm within endoscopes.

Contraindications

None known.

Warnings

Do not use the Porpoise Air/Water Cleaning Adapter for patient procedures. It will cause continuous insufflation and could result in patient injury.

Precaution/ Adverse Events

None known.

How Supplied

The Porpoise Air/Water Cleaning Adapter is supplied non-sterile.

Device Details

- Do not use if package is damaged or unintentionally opened before use.
- Do not use if labeling is incomplete or illegible.

Handling and Storage: This product has no special handling or storage requirements.