


1 Remote Control Buttons



Therapy refers to patterns of stimulation used to reduce the awareness of pain.

Home Screen Display

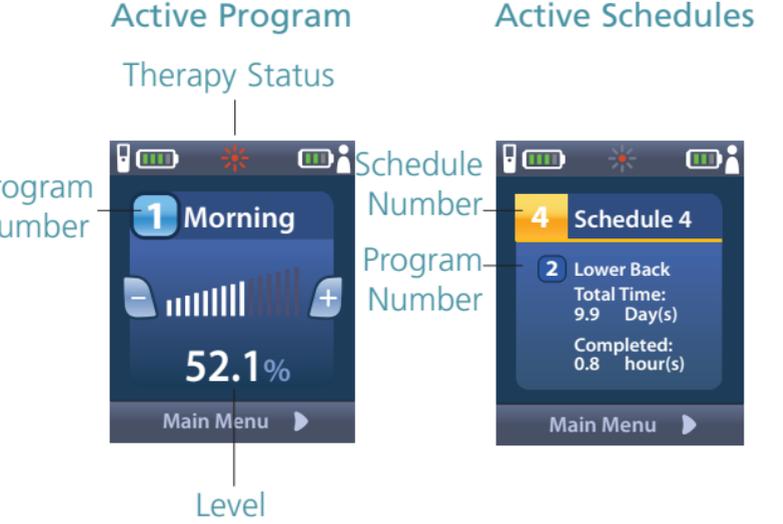

Note: You will see the icon for the Stimulator battery only if you have a rechargeable battery.

Note: The Home Screen Displayed is dependent on the therapy prescribed by your physician.

2 Getting Started



1 Unlock/Lock

2 Start/Stop Therapy

3 Decrease/Increase Stimulation Strength

4 Main menu

The settings displayed are dependent on your Stimulator and the therapy configured by your healthcare provider.

3 Accessing Programs



If your healthcare provider has configured Programs that can be changed, there are two ways to view the Programs menu:

- Press the Programs button on the left side of the Remote
- From the Main Menu , select the Programs menu and press

4 Changing Programs



1 From the Programs Menu press to enter the Change Program menu

2 Press to select the desired Program

3 Press to activate the selected Program

My Healthcare Provider

Name

Phone

Hospital