



the **FAST TRACK** to
your therapy delivery in a single solution

VersaCross™ Workflow

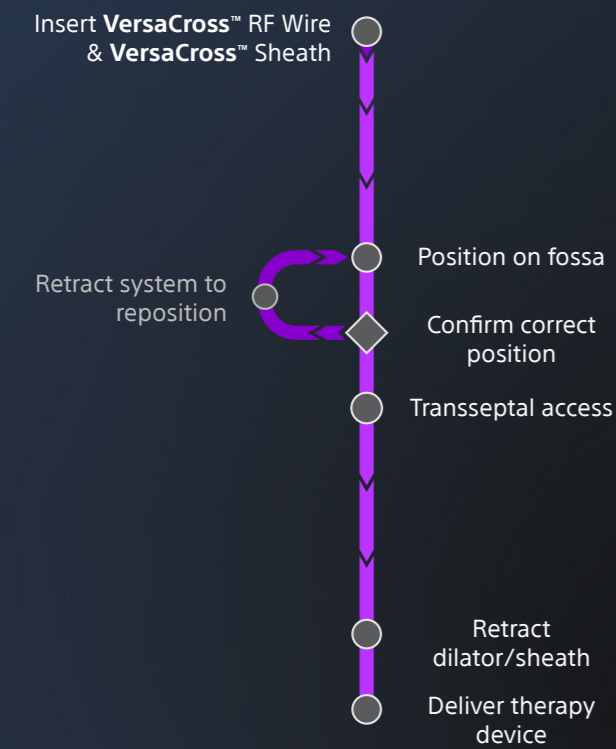

Mechanical Needle Workflow

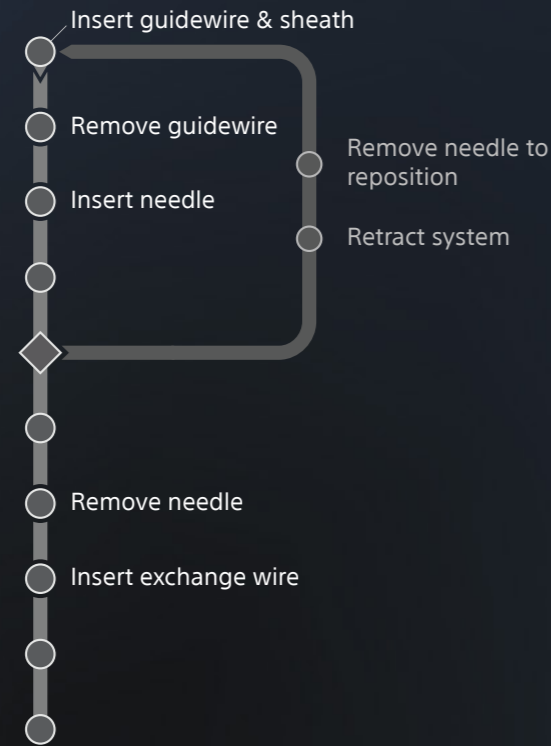

VersaCross™ RF Transseptal Solution

Full platform of tools to personalize your solution

Personalize your solution

1 Choose your **VersaCross™** RF Wire



VersaCross™ RF Wire

RF Wire length: 180 cm, 230 cm
Wire diameter: 0.035"
Curve diameter: 9 mm (J-tip), 24 mm (Pigtail)

2 Choose your **VersaCross™** Sheath to complete your solution available in Dilator curves D1 or D0 with **TRUform™** Shapeable Technology



A VersaCross™ Transseptal Sheath

French size compatibility: 8.5F (2.84 mm)
Sheath usable length: 63 cm, 81 cm
Dilator usable length: 67 cm, 85 cm




B VersaCross™ Steerable Sheath

Sheath curves: S (17 mm), M (22 mm), L (50 mm)
Bidirectional angles: 90 CCW, 180 CW



VersaCross™
RF Transseptal Solution

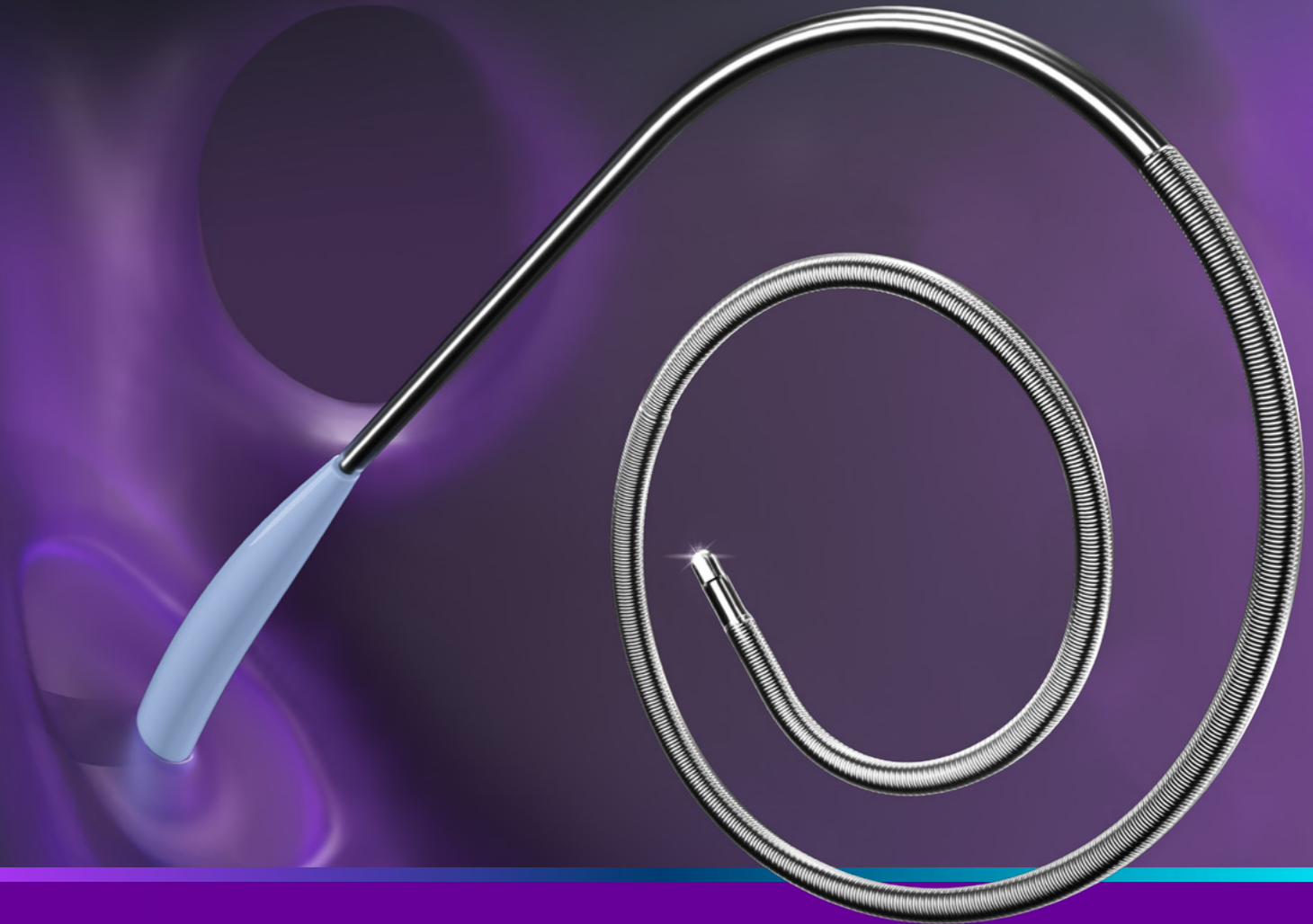

Brief Summary | VersaCross™ RF Wire

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician. Rx only. Prior to use, please see the complete "Instructions for Use" for more information on Indications, Contraindications, Warnings, Precautions, Adverse Events, and Operator's Instructions.

INDICATIONS FOR USE: The VersaCross™ RF Wire is indicated for creation of an atrial septal defect in the heart.

CONTRAINDICATIONS: The VersaCross™ RF Wire is not recommended for use with any conditions that do not require the creation of an atrial septal defect. The Connector Cable is not recommended for use with any other Baylis RF Generator or any other device.

WARNINGS: Laboratory staff and patients can undergo significant x-ray exposure during RF puncture procedures due to the continuous usage of fluoroscopic imaging. This exposure can result in acute radiation injury as well as increased risk for somatic and genetic effects. Therefore, adequate measures must be taken to minimize this exposure. • The VersaCross™ RF Wire and Connector Cable are intended for single patient use only. Do not attempt to sterilize and reuse either devices. Reuse can cause patient injury and/or the communication of infectious disease(s) from one patient to another. Reuse may result in patient complications. • The VersaCross™ RF Wire must be used with the Connector Cable provided. Attempts to use it with other connector cables can result in electrocution of the patient and/or operator. • Do not use the VersaCross™ RF Wire with electrocautery or electrosurgery generators, connector cables or accessories as attempted use can result in patient and/or operator injury. • The Connector Cable must only be used with the RFP-100A Baylis RF Generator and the included VersaCross™ RF Wire. Attempts to use it with other RF Generators and devices can result in electrocution of the patient and/or operator. • The VersaCross™ RF Wire must be used with 0.035" compatible transseptal sheath and/or dilator devices. Use of incompatible accessory devices may damage the integrity of the VersaCross™ RF Wire or accessory devices and may cause patient injury. • The VersaCross™ RF Wire has only been validated for transseptal puncture use through VersaCross™ dilators which have been demonstrated to provide the required support for optimal function. • The VersaCross™ RF Wire is not intended for use with neonatal patients (i.e. less than one month of age). Do not attempt to treat neonatal patients with the VersaCross™ RF Wire.

PRECAUTIONS: • In order to prevent the risk of ignition, ensure that flammable materials are not present in the room during RF power application. • Careful manipulation of the VersaCross™ RF Wire must be performed to avoid vessel trauma. If resistance is encountered, DO NOT use excessive force to advance or withdraw the VersaCross™ RF Wire or ancillary sheath and/or dilator assembly. Excessive force may lead to bending or linking of the device limiting advancement and retraction of sheath and/or dilator device. • The Baylis RF Generator is capable of delivering significant electrical power. Patient or operator injury can result from improper handling of the VersaCross™ RF Wire and/or DIP electrode, particularly when operating the device. • During power delivery, the patient should not be allowed to come in contact with ground metal surfaces. • If using electroanatomical mapping guidance, it is recommended to use it along with alternative imaging modality in the event there is loss of visibility of the device.

ADVERSE EVENTS: Adverse events that may occur while creating an atrial septal defect include: • Tamponade • Sepsis/Infection • Thromboembolic episodes • Vessel perforation • Atrial Fibrillation • Myocardial Infarction • Vessel spasm • Sustained arrhythmias • Atrial Flutter • Hemorrhage • Vascular thrombosis • Perforation of the myocardium • Hematoma • Allergic reaction to contrast medium • Ventricular Tachycardia • Pain and Tenderness • Arteriovenous fistula • Pericardial effusion • Tachycardia • Vascular Trauma • Additional Surgical Procedure • Wire entrapment/ entanglement • Foreign body/wire fracture
EP-1504711-AA

Brief Summary | VersaCross™ Transseptal Dilator

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician. Rx only. Prior to use, please see the complete "Instructions for Use" for more information on Indications, Contraindications, Warnings, Precautions, Adverse Events, and Operator's Instructions.

INDICATIONS FOR USE: The VersaCross™ Transseptal Dilator is indicated for introducing various cardiovascular catheters to the heart, including the left side of the heart through the interatrial septum.

CONTRAINDICATIONS: There are no known contraindications for this device.

WARNINGS: Laboratory staff and patients can undergo significant x-ray exposure during interventional procedures due to the continuous usage of fluoroscopic imaging. This exposure can result in acute radiation injury as well as increased risk for somatic and genetic effects. Therefore, adequate measures must be taken to minimize this exposure. • The VersaCross™ Steerable Sheath kit is intended for single patient use only. Do not attempt to sterilize and reuse the VersaCross™ Steerable Sheath kit. Reuse can cause the patient injury and/or the communication of infectious disease(s) from one patient to another. Failure to follow this instruction may result in patient complications. • Maintain continuous hemodynamic monitoring throughout procedure. • Provide continuous heparinized saline infusion while the introducer remains in vessel.

PRECAUTIONS: • Careful manipulation must be performed to avoid cardiac damage, or tamponade. Sheath, dilator and guidewire advancement should be done under fluoroscopic guidance. If resistance is encountered, DO NOT use excessive force to advance or withdraw the device.

ADVERSE EVENTS: Adverse events that may occur while using the VersaCross™ Sheath include: • Infection • Air embolus • Local nerve damage • Vasovagal reaction • Dissection • Vessel spasm • AV fistula formation • Atrial septal defect • Pseudoaneurysm • Aortic puncture • Arrhythmias • Perforation and/or tamponade • Hematoma • Hemorrhage • Catheter entrapment • Thromboembolic events • Stroke • Valve damage • Myocardial infarction • Pacemaker/defibrillator lead displacement • Pulmonary edema • Coronary artery spasm and/or damage • Vessel trauma • Pericardial/pleural effusion
EP-1506213-AA

Brief Summary | VersaCross™ Transseptal Sheath

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a physician. Rx only. Prior to use, please see the complete "Instructions for Use" for more information on Indications, Contraindications, Warnings, Precautions, Adverse Events, and Operator's Instructions.

INDICATIONS FOR USE: The VersaCross™ Transseptal Sheath kit is used for the percutaneous introduction of various types of cardiovascular catheters and guidewires to all heart chambers, including the left atrium via transseptal perforation / puncture.

CONTRAINDICATIONS: There are no known contraindications for this device.


WARNINGS: Laboratory staff and patients can undergo significant x-ray exposure during interventional procedures due to the continuous usage of fluoroscopic imaging. This exposure can result in acute radiation injury as well as increased risk for somatic and genetic effects. Therefore, adequate measures must be taken to minimize this exposure. The use of echocardiography is recommended. • The VersaCross™ Transseptal Sheath kit is intended for single patient use only. Do not attempt to sterilize and reuse the VersaCross™ Transseptal Sheath kit. Reuse can cause patient injury and/or the communication of infectious disease(s) from one patient to another. • Do not attempt direct percutaneous insertion of the sheath without the dilator as this may cause vessel injury. • Careful manipulation must be performed to avoid cardiac damage or tamponade. Sheath advancement should be done under fluoroscopic guidance. Echocardiographic guidance is also recommended.

PRECAUTIONS: • Careful manipulation must be performed to avoid cardiac damage or tamponade. Sheath, dilator and guidewire advancement should be done under fluoroscopic guidance. If resistance is encountered, DO NOT use excessive force to advance or withdraw the device. • The VersaCross™ Transseptal Sheath is compatible with introducer sheaths 11Fr or larger. • The VersaCross™ Transseptal Sheath and Dilator are compatible with transseptal devices and guidewires .035" or smaller. • The VersaCross™ Transseptal Sheath kit is NOT compatible with transseptal needles such as the "NRG™ Transseptal Needle".

ADVERSE EVENTS: Adverse events that may occur while using the VersaCross™ Transseptal Sheath kit include: • Infection • Air embolus • Local nerve damage • Hemorrhage • Embolic events • Vessel spasm • AV fistula formation • Atrial septal defect • Pseudoaneurysm • Perforation and/or tamponade • Arrhythmias • Pericardial/pleural effusion • Hematoma • Vessel trauma • Valve damage • Catheter entrapment
EP-1506605-AA

Boston Scientific is a Global Company. Please note that model numbers, indications, contraindications, warnings and specifications may differ depending on geographic region. Not all information displayed in this brochure may be licensed in accordance with Canadian law. Please contact your Boston Scientific representative for local labeling, product specifications and licensed model numbers. **CAUTION:** The law restricts this device to sale by or on the order of a physician. Rx only. Indications, Contraindications, Warnings, and Instructions For Use can be found in the product labeling supplied with each device or at www.baylismedical.com. Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries. This material not intended for use in France. All trademarks are property of their respective owners. Patents Pending and/or issued.


Eliminate exchanges

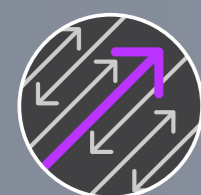

Access with precision



Secure effortless delivery

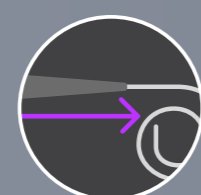

TRANSSEPTAL REIMAGINED


Fast track access-to-delivery with a single solution

the only **EXCHANGELESS*** solution
for access-to-delivery of left heart therapy devices

V E R S A T I L I T Y

ELIMINATE EXCHANGES

Left heart access using a **SINGLE SOLUTION**
from start to finish

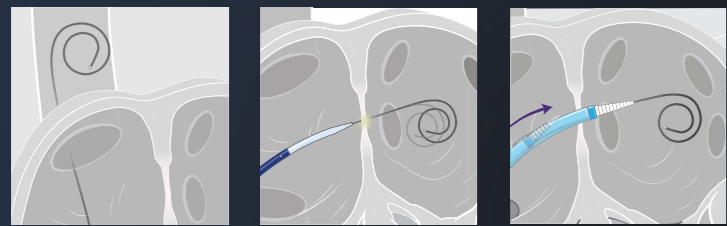

Rewire and drop down to
optimize transeptal location
WITHOUT THE HASSLE
of exchanging a needle

ACCESS WITH PRECISION

PRECISION RF PUNCTURE TECHNOLOGY
to optimize transeptal location
for any anatomy



TRUform™ SHAPEABLE TECHNOLOGY

Shapeable dilator
with true-to-form
curve retention



Snap-fit to
maintain hands-
free sheath-dilator
alignment

SECURE EFFORTLESS DELIVERY

Instantly gain and maintain
access **WITHOUT EXCHANGES**




Sturdy 0.035" rail
to deliver therapy devices
with confidence

Flexible tip cushions against
left atrial wall and reduces
risk of perforation

*VersaCross™ RF Wire can be used, without exchanges, as a guidewire, as a transeptal puncture device or as an exchange rail for delivering therapy sheaths.

Know where you are
at all times with

OMNIVIZ™ Technology

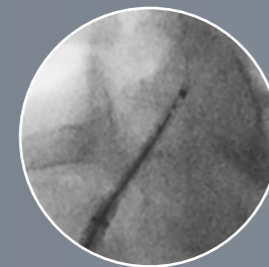


RADIOPAQUE

Visualize your entire solution
on fluoroscopy




ECHOGENIC

Reliably locate your devices on ultrasound
to reduce reliance on fluoroscopy

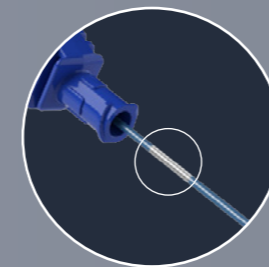

POSITIONAL MARKERS

Visibly confirm position of
RF tip within dilator

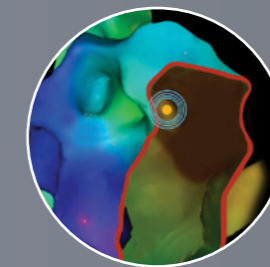

MAPPING

Track and mark RF tip position
on your mapping system

VersaCross™
RF Transeptal Solution