Boston
Scientific

LithoVue™ System Workstation

User's Manual

LithoVue™ System Workstation

User's Manual

LithoVue System Workstation User's Manual.....	3
Manual del usuario de la estación de trabajo del sistema LithoVue	46
Manuel d'utilisation du poste de travail du système LithoVue.....	96
LithoVue System-Arbeitsstation – Bedienungsanleitung.....	145
Manuale dell'utente della postazione di lavoro del sistema LithoVue.....	197
Gebruiksaanwijzing LithoVue-systeemwerkstation	248
Manual do utilizador da estação de trabalho do sistema LithoVue	299

TABLE OF CONTENTS

1 INTRODUCTION	5
1.1 SAFE USE REQUIRES READING THE USER'S MANUAL	7
2 CLINICAL INFORMATION.....	7
2.1 INDICATIONS FOR USE AND INTENDED USE.....	7
2.2 CONTRAINDICATIONS	7
2.3 USER TRAINING, KNOWLEDGE, AND SKILLS.....	7
2.4 WARNINGS	8
2.5 PRECAUTIONS	9
2.6 ADVERSE EVENTS.....	10
3 HOW SUPPLIED	10
3.1 SYSTEM WORKSTATION COMPONENTS.....	11
3.2 SYSTEM WORKSTATION ASSEMBLY.....	12
Assembling the Cart.....	12
Assembling the Cart Base and Cart Post	12
Attaching the Touch PC Mounting Plate to the Cart Post.....	13
Attaching the Cart Handle	14
Attaching the Power Transformer Bracket.....	15
Installing the Touch PC onto the Cart.....	15
Cart Adjustment.....	18
Adjusting Touch PC Height	19
Adjusting Touch PC Tilt.....	19
Confirming Installation	19
Connecting an External Monitor or DVI Switcher with the DVI Cable	20
3.3 TRANSPORTATION, USE AND STORAGE.....	21
3.4 SPECIFICATIONS AND DEVICE COMPATIBILITY.....	22
Electrical	22
Physical (nominal).....	22
Illumination Output (nominal)	23
Medical Electrical Classifications	23
LithoVue™ Flexscope Fluid Compatibility.....	23
System Workstation Device - Accessory Compatibility	23
Software Information.....	23
4 LITHOVUE SYSTEM.....	24
4.1 SYSTEM WORKSTATION FEATURES AND USER INTERFACE	24
System Workstation Front Panel Features and User Interface	24
System Workstation Rear Panel Features	25
4.2 SYSTEM WORKSTATION CART FEATURES	25
4.3 MAIN OPERATING SCREEN	27

Light On/Off Button.....	27
Image Brightness Control Buttons	28
Image Brightness Indicator.....	28
Time Remaining Indicator	28
Controls Swap Button.....	29
Live Video Image Display Area	30
Message Screens	30
5 SYSTEM OPERATION.....	31
5.1 STARTING THE SYSTEM WORKSTATION	31
5.2 CHECKING THE QUALITY OF THE VIDEO IMAGE.....	33
5.3 ADJUSTING THE BRIGHTNESS OF THE VIDEO IMAGE.....	33
5.4 COMPLETING A PROCEDURE	33
Using the LithoVue™ System in an Ureteroscopy Procedure	33
5.5 SAFELY SHUTTING DOWN THE SYSTEM WORKSTATION	35
5.6 DISPOSAL OF THE PRODUCT, ACCESSORIES, AND PACKING MATERIALS.....	35
6 CLEANING AND MAINTENANCE	35
6.1 CLEANING THE SYSTEM WORKSTATION	35
6.2 MAINTENANCE, SERVICE AND REPLACEMENT PARTS.....	36
7 TROUBLESHOOTING.....	36
7.1 ISOLATING THE SYSTEM WORKSTATION FROM THE MAINS POWER SUPPLY.....	36
7.2 LOSS OF IMAGE.....	36
7.3 LOSS OF ARTICULATION CONTROL ON THE LITHOVUE FLEXSCOPE	37
7.4 SYSTEM WORKSTATION POWER LOSS	39
7.5 SYSTEM WORKSTATION BECOMES NON-OPERATIONAL.....	39
8 SERVICE AND WARRANTY	42
8.1 Limited Warranty.....	42
8.2 Obtaining Warranty Service from Boston Scientific Corporation	42
9 ELECTROMAGNETIC COMPATIBILITY (EMC) REQUIREMENTS.....	42
9.1 GUIDANCE AND MANUFACTURER'S DECLARATION - ELECTROMAGNETIC EMISSIONS	42
9.2 GUIDANCE AND MANUFACTURER'S DECLARATION - IMMUNITY ALL EQUIPMENT AND SYSTEMS	43
9.3 GUIDANCE AND MANUFACTURER'S DECLARATION - EMISSIONS AND EQUIPMENT AND SYSTEMS THAT ARE NOT LIFE-SUPPORTING	44
9.4 RECOMMENDED SEPARATION DISTANCE OF OTHER EQUIPMENT	45

Rx ONLY

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

1 Introduction

This user's manual describes how to use, maintain, and troubleshoot the LithoVue™ System (see **Figure 1.1**).

The LithoVue System is a software-controlled digital flexible ureteroscope system that consists of the LithoVue System Workstation (Touch PC and Cart) and the LithoVue Single-Use Digital Flexible Ureteroscope (sterile, single-use disposable). The LithoVue System Workstation is referred to as the *System Workstation*, and the LithoVue Single-Use Digital Flexible Ureteroscope is referred to as the *LithoVue Flexscope* in these instructions.

The LithoVue Flexscope connects to the System Workstation via the Flexscope Connector Cable Plug Receptacle (hereafter referred to as *Workstation Receptacle*) on the front of the System Workstation (see **Figure 1.1**).


The LithoVue System is designed to allow physicians to access, visualize, and perform procedures in the urinary tract, using appropriate accessory devices (e.g., baskets, laser fibers, and forceps).

The LithoVue System is available in the configurations shown in **Table 1.1**:

Table 1.1 LithoVue System Available Configurations

Model Number (UPN)	Description
M0067911000	LithoVue System Workstation Global
M0067913500	LithoVue Single-Use Digital Flexible Ureteroscope (Standard Deflection) - Global
M0067913600	LithoVue Single-Use Digital Flexible Ureteroscope (Reverse Deflection) - Global

Figure 1.1 – LithoVue™ System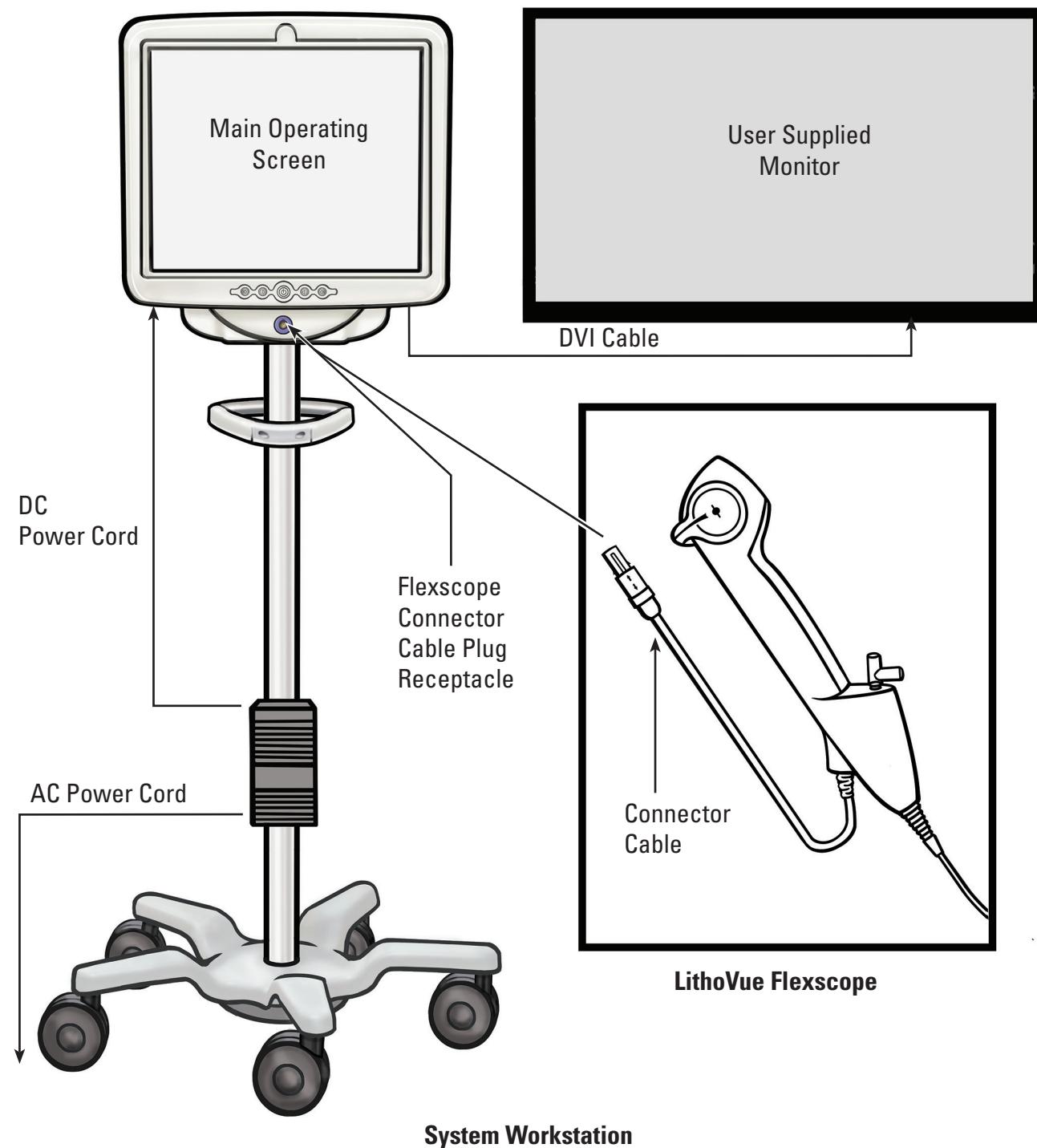

The System Workstation does the following:

- Provides power for the illuminating LED and video image hardware in the LithoVue Flexscope.
- Receives video signals from the LithoVue Flexscope and processes the images for display on the System Workstation screen or on a user-supplied monitor, if connected.
- Provides illumination control. The illumination provided by the LithoVue Flexscope is adjusted automatically by the system to ensure adequate light levels at the treatment site. Additionally, the user may also turn the illumination on/off and adjust the illumination levels using the controls provided on the System Workstation.

- (d) Displays the video images. The system workstation can also send the video signals via a DVI Cable for display on a user-supplied video monitor. When using a user-supplied video monitor, the system will duplicate the video image, but will not duplicate the system controls.
- (e) Provides the user interface.

1.1 SAFE USE REQUIRES READING THE USER'S MANUAL

Prior to using the LithoVue™ System, you must read this User's Manual and the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use to become familiar with the LithoVue System's features and controls. This manual and the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use provide the information necessary for inspecting, preparing and operating the system.

Failure to thoroughly understand and follow all instructions, cautions and warnings provided in this User's Manual and the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use may result in injury to the patient and/or user, and/or may result in damage to, or malfunction of, this equipment. Additionally, damage to other equipment or property may result.

Follow all instructions, cautions, and warnings provided with all products and equipment to be used in conjunction with the LithoVue System to avoid any possible hazards due to device incompatibility.

If the instructions are unclear to you, contact Boston Scientific for assistance, using the information found in **Section 8 "Service and Warranty"**.

Keep this User's Manual in an accessible location.

2 Clinical Information

2.1 INDICATIONS FOR USE AND INTENDED USE

The LithoVue System is intended to be used to visualize organs, cavities and canals in the urinary tract (urethra, bladder, ureter, calyces and renal papillae) via transurethral or percutaneous access routes. It can also be used in conjunction with endoscopic accessories to perform various diagnostic and therapeutic procedures in the urinary tract.

2.2 CONTRAINDICATIONS

Contraindications for this device are those specific to urinary tract endoscopy.

Diagnostic or therapeutic ureteroscopy is contraindicated in people with an untreated urinary tract infection.

Other contraindications to therapeutic ureteroscopy (e.g., lithotripsy, endopyelotomy, tumor therapy) are more numerous and can mirror those associated with the corresponding open surgical interventions. Patients on anticoagulants or with coagulopathies should be managed appropriately.

2.3 USER TRAINING, KNOWLEDGE, AND SKILLS

Read the entire User's Manual and LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use before using the LithoVue System. Study labeling thoroughly for safe handling and storage. Use the LithoVue System as intended.

The LithoVue System Workstation should only be assembled and installed by a trained biomedical technician.

Only physicians who have ureteroscopic, diagnostic, and therapeutic training should use the LithoVue System. A thorough understanding of the techniques, principles, clinical applications, and risks associated with endoscopic urinary tract procedures is necessary before using this device. This includes, but is not limited to devices used in laser lithotripsy, administration of irrigation/contrast, and use of procedural accessories such as baskets/forceps, access sheaths, guidewires, and antiretropulsion devices.

Nursing support staff should have reasonable experience with common medical equipment and urology procedures.

2.4 WARNINGS

- Do not use the LithoVue™ System in the presence of flammable fluids and gases such as alcohol or oxygen. Doing so can result in fire and burns to the operator and patient.
- Do not insert or advance the LithoVue Flexscope unless there is a clear live endoscopic view of the lumen through which the scope is being advanced (or confirm with visualization by other imaging modalities). Doing so can cause patient injury such as perforation, avulsion, hemorrhage, or urothelial damage.
- In the event that the live endoscopic image is lost, do not advance or insert the LithoVue Flexscope and do not insert, advance or actuate accessories. Doing so can cause patient injury such as perforation, avulsion, hemorrhage, or urothelial damage.
- Do not use excessive force when advancing or withdrawing the scope. Doing so can cause patient injury such as perforation, hemorrhage or urothelial damage or damage to the LithoVue Flexscope. If resistance is felt during advancement or withdrawal of the scope, investigate the source of resistance and take remedial action (e.g., fluoroscopy, contrast injection).
- Do not force the distal tip of the LithoVue Flexscope against the sidewall of the ureter or renal pelvis. Doing so can cause patient injury such as perforation, avulsion, hemorrhage or urothelial damage.
- Do not use excessive force when advancing or withdrawing an accessory within the LithoVue Flexscope. Doing so can cause patient injury such as perforation, avulsion, hemorrhage, urothelial damage or damage to the LithoVue Flexscope.
- When inserting or using accessories, maintain continuous visualization of the distal tip. Ensure the distance between the distal tip of the LithoVue Flexscope and the object in view is greater than the LithoVue Flexscope's minimum visible distance. Failure to do so may result in the accessories causing patient injury such as perforation, hemorrhage or urothelial damage.
- Do not withdraw a laser fiber back into the LithoVue Flexscope while the laser is firing. Doing so may cause patient injury and/or scope damage.
- Do not look directly into the light emitted from the LithoVue Flexscope. Doing so can result in eye injury.
- Verify ground isolation when setting up and using accessories from different manufacturers. Failure to do so can result in shocks and accessory malfunction leading to patient injury.
- Do not open the handle of the LithoVue Flexscope. Doing so can damage the waterproof seals and result in risk of electric shock.
- The LithoVue Flexscope is a single-use device and there are no serviceable parts. Do not repair damaged or non-operating LithoVue Flexscopes. Do not use the LithoVue Flexscope if damage is discovered or suspected.
- Do not excessively bend the flexible shaft or the articulating section of the LithoVue Flexscope as this may break or kink the shaft.
- If damage to the LithoVue Flexscope occurs or it malfunctions during a procedure, stop using the LithoVue Flexscope immediately. See **Section 7 "Troubleshooting"** for more information. Continue the procedure with a new LithoVue Flexscope, as appropriate.
- No modification of this equipment is allowed. Do not attempt to repair or alter any components/parts of the LithoVue System Workstation. The LithoVue System Workstation contains no operator-serviceable components. All repairs and servicing are to be performed only by authorized Boston Scientific service personnel. See **Section 8 "Service and Warranty"** for additional information.
- To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earth. Prior to installation, ensure that the selected hospital's electrical outlet has a proper ground connection and complies with the information listed on the label located on the rear of the Touch PC.

- Prior to use of a cardiac defibrillator, remove the LithoVue™ Flexscope from the patient. Failure to remove the LithoVue Flexscope from a patient during use of a cardiac defibrillator could result in damage to the system due to the discharge of the cardiac defibrillator.
- The use of accessories, a power supply and/or cables other than those specified or supplied as spare parts from Boston Scientific may increase electromagnetic emissions or decrease immunity of the LithoVue System.
- When the LithoVue System is used with other electrical medical equipment, the applied parts must be Type BF or Type CF applied parts.
- Components added to the system by the user must be certified to the respective IEC standards (IEC 60601-1 for medical equipment, IEC 60950 for data processing equipment, and IEC 60065 for A/V equipment) or other country equivalent. In addition, the user must ensure the new configuration complies with local regulations and hospital requirements for electrical safety.
- If the System Workstation stops functioning during a procedure, follow the procedure described in **Section 7 “Troubleshooting”**.
- Using a System Workstation without disinfecting it can expose the operator to bio-hazardous materials. To prevent exposure to bio-hazardous materials, disinfect the System Workstation between uses using the cleaning procedure describe in **Section 6.1 “Cleaning the System Workstation”**.

2.5 PRECAUTIONS

- Only use the LithoVue Flexscope in conjunction with the LithoVue System Workstation. Connection to other devices may cause device or property damage or operator injury.
- Only physicians with adequate ureteroscopic training should perform procedures with the LithoVue Flexscope. Consult the medical literature regarding techniques, complications, and hazards prior to any procedure.
- Use the LithoVue Flexscope with caution in patients who have undergone previous urinary tract reconstructive surgery or with known strictures. These conditions may prevent passage of the flexible scope shaft.
- Failure to thoroughly understand and follow all instructions, cautions and warnings provided in this User’s Manual and the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use may result in injury to the patient and/or user; and/or may result in damage to, or malfunction of, this equipment. Additionally, damage to other equipment or property may result. Follow all instructions, cautions and warnings provided with all products and equipment to be used in conjunction with the LithoVue System to avoid any possible hazards due to device incompatibility.
- The LithoVue System has been tested and shown compatible with laser lithotripsy devices. The use of other energized procedural devices may cause loss of image, device damage, or patient injury.
- When using the LithoVue Flexscope with a laser lithotripsy device, all personnel within the treatment room shall wear protective laser eyewear in accordance with the laser manufacturer’s Instructions for Use.
- Do not use accessories that fail to meet the compatibility requirements of the LithoVue Flexscope as stated in the Lithovue Single-Use Digital Flexible Ureteroscope Directions for Use. Doing so may cause damage to the LithoVue Flexscope, System Workstation and/or accessory.
- To ensure satisfactory LithoVue System performance, perform the prescribed inspections and operational checks on the LithoVue Flexscope described in the Lithovue Single-Use Digital Flexible Ureteroscope Directions for Use before use. Additionally, verify the System Workstation is properly installed. See **Section 3.2. “Confirming Installation”** for more information.

- The distal tip of the LithoVue™ Flexscope should be straight when inserting and withdrawing accessories. Follow the accessory directions for use regarding inserting the accessory into a flexible ureteroscope. Failure to follow the accessory directions for use may result in patient injury, or accessory and LithoVue Flexscope damage or malfunction.
- The LithoVue Flexscope features a strain relief at the transition from the handle to the shaft. The strain relief protects the device during use. To prevent damage to the shaft and/or light fiber, do not bend the shaft sharply.
- Use only those fluids/lubricants recommended in the compatibility section of the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use or see **Section 3.4 “LithoVue Flexscope Fluid Compatibility”**.
- Do not use accessories that fail to meet the compatibility requirements of the LithoVue Flexscope as stated in the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use. Doing so may cause damage to the LithoVue Flexscope and/or accessory.
- The LithoVue Flexscope Connector Cable (hereafter referred to as *Flexscope Cable*) should connect to the Workstation Receptacle easily. If the Flexscope Cable does not connect, verify that the arrows on the Flexscope Cable and the Workstation Receptacle line up and that the Workstation Receptacle is not damaged. Forcing the Flexscope Cable into the Workstation Receptacle may damage the scope and/or System Workstation.
- Do not insert a wet, contaminated, or damaged Connector Cable plug into the System Workstation as poor video performance or damage to the system may result.
- Do not remove the Connector Cable from the System Workstation by pulling on the cable as poor video performance or damage to the system may result. Slide the locking collar on the Connector Cable's plug toward the cable and pull the plug out to remove the cable.
- Placing the System Workstation near other medical electrical equipment may result in electromagnetic interference (EMI) which may degrade the video image. Additionally, EMI from the System Workstation may interfere with other equipment in the operating room. Verify that all electrical equipment is working properly before starting the procedure. Failure to do so may result in the equipment not working properly resulting in either a delay of the procedure or an adverse event.
- Spilling liquids on the System Workstation can damage it or cause it to shut down. Do not place liquids above or near the System Workstation.
- During assembly and before each use, inspect all components for damage. Do not use a component if it appears damaged. Contact Boston Scientific for service using the information found in **Section 8 “Service and Warranty”**.

2.6 ADVERSE EVENTS

Possible complications include, but may not be limited to:

- | | | | |
|----------------------|--------------|---|---------------------|
| • Bleeding | • Pain | • Sepsis | • Ureteral reflux |
| • Avulsion | • Discomfort | • Renal Failure | • Hematoma |
| • Stenosis/Stricture | • Urinoma | • Perforation (ureter, renal pelvis or bladder) | • Urothelial damage |
| • Inflammation | • Infection | | |
| • Laceration | • Fever | • Hematuria | |

3 How Supplied

CAUTION: Examine the System Workstation packaging. Do not use if packaging has been opened or the components appear damaged. Do not use if labeling is incomplete or illegible.


**CAUTION: Inspect the components for damage. Do not use a component if it appears damaged.
Contact Boston Scientific for service.**

3.1 SYSTEM WORKSTATION COMPONENTS

The LithoVue™ System Workstation is shipped in a cardboard shipping crate. Inside the shipping crate, the components of the System Workstation are packaged separately, as noted in **Figure 3.1**. The lists below detail the contents included in each component box.

NOTE: Depending on the model, the power cord may be packaged either in the Touch PC box or in the cardboard shipping crate.

Figure 3.1 – LithoVue System Workstation Packaging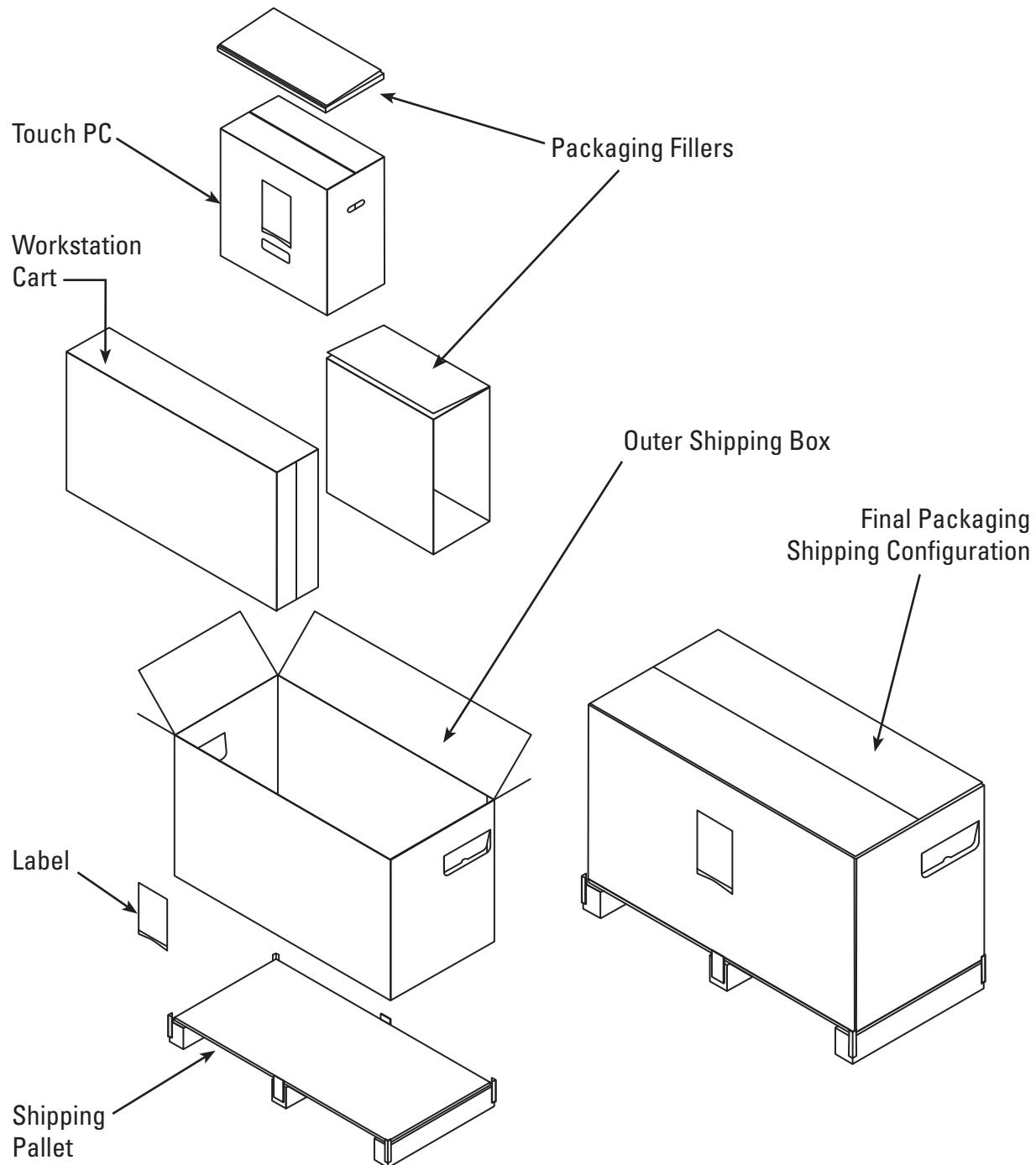

Touch PC Packaging:

- Touch PC
- Power Transformer
- (8) No. M4 x 10 mm pan-head Phillips Mounting Screws {(4) required for mounting the Touch PC to the Touch PC Mounting Plate} Note: (4) extra Mounting Screws are provided
- DVI Cable
- LithoVue™ System Workstation User's Manual

Cart Packaging:

- Cart Base with attached wheels
- Cart Post
- Touch PC Mounting Plate
- Cart Handle with (2) No. 10-32 x 9/16" Phillips-head screws pre-installed
- Power Transformer Bracket Kit containing:
 - Power Supply Holder with Velcro straps and Clamping Block
 - Cord Wrap Hook with Clamping Block
 - (2) No. 10-32 x 9/16" Phillips-head screws
 - Power cord strain relief kit containing: (1) nylon cable clamp, (1) #8 flat washer and (1) 8-32 x 1/4" Phillips-head screw
- Hardware bag containing:
 - (3) No. 10-32 x 9/16" flat-head Phillips screws (for Touch PC Mounting Plate)
 - #2 Phillips-head Screwdriver
 - 9/64" Hex Wrench (used to adjust Tilt Tension Adjustment Screws)
 - 1/2" (13 mm) Socket Wrench
 - (1) Manual Tilt Lexan Cover (optional - cover the screws on Touch PC Mounting Plate)
 - (1) 5/16" flat washer, (1) 5/16" split lock washer and (1) 5/16"-18 x 1" Hex Head Cap Screw

3.2 SYSTEM WORKSTATION ASSEMBLY

CAUTION: Do not attempt to repair or alter any components/parts of the LithoVue System Workstation. The LithoVue Touch PC contains no operator-serviceable components. All repairs, upgrades and servicing are to be performed only by authorized Boston Scientific service personnel. See *Section 8 "Service and Warranty"* for additional information.

Assembling the Cart


Follow these steps to assemble the Cart:

Assembling the Cart Base and Cart Post

1. Remove the Cart components from the box. The Cart Post, Cart Base, Touch PC Mounting Plate, Cart Handle, Power Transformer Bracket and hardware are all packaged separately within the outer Cart Box.
2. Confirm that all components listed in **Section 3.1, "System Workstation Components"**, are included in the packaging and that the components are not damaged.
3. Insert the Cart Post into the Cart Base and lay the assembly on its side for access to the bottom of the Cart Base.

- Assemble the (1) 5/16" hex head cap screw, (1) 5/16" flat washer and (1) 5/16" split lock washer as shown in **Figure 3.2**. Start the screw by hand before using the 1/2" (13 mm) wrench to fully tighten the screw. Tighten the screw until fully seated and the post no longer rotates.

Figure 3.2 – Attaching the Cart Post to the Cart Base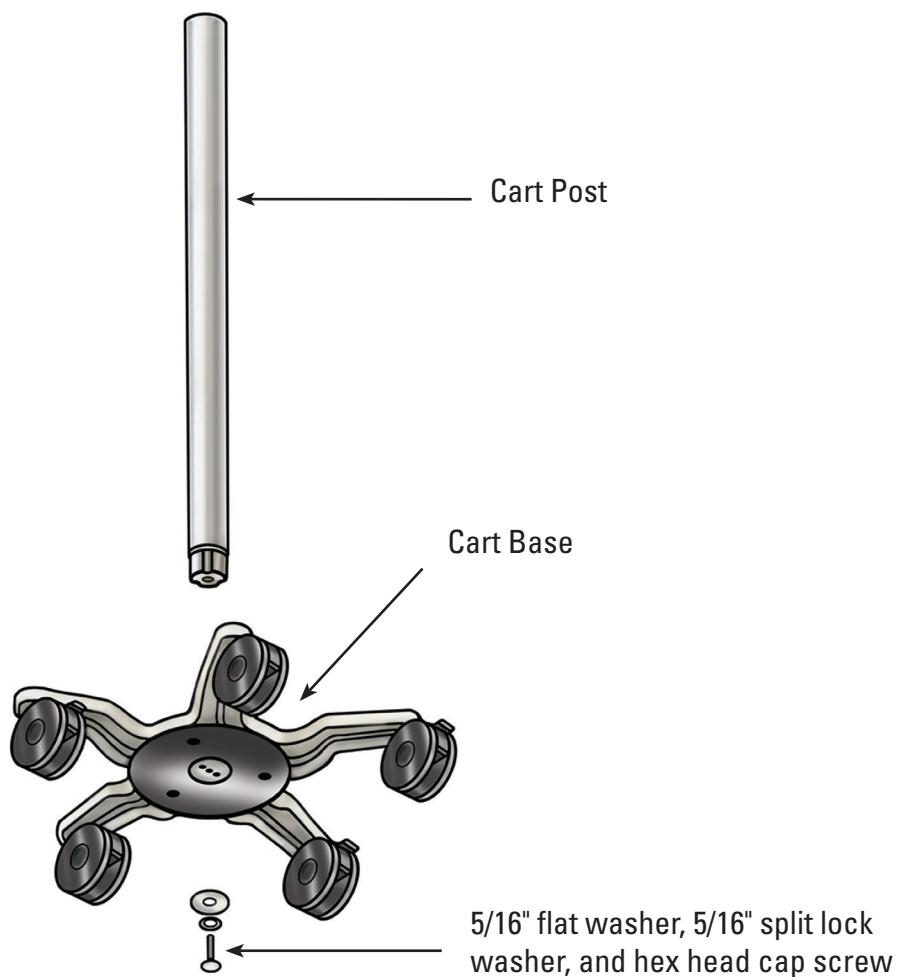


- Return the Cart to the upright position and confirm that the three caster locks are pressed down to lock the Cart into position.

Attaching the Touch PC Mounting Plate to the Cart Post

The three-hole pattern on the top of the Cart Post is designed to orient the Touch PC Mounting Plate in one direction only. The Touch PC Mounting Plate is designed to always position the Touch PC opposite the Cart's Height Adjustment Knob. Secure the Touch PC Mounting Plate to the Cart Post as follows:

- Place the Touch PC Mounting Plate on the Cart Post, aligning the three-hole pattern.
- Insert the (3) flat-head Phillips Mounting Screws into the Touch PC Mounting Plate and Cart Post. Start the screws by hand before tightening with the Phillips-head screwdriver. Tighten the screws until they are snug and fully seated. **See Figure 3.3.**
- Optional – if desired, cover the top of the mount with the included adhesive Lexan cover.

Figure 3.3 – Installing the Touch PC Mounting Plate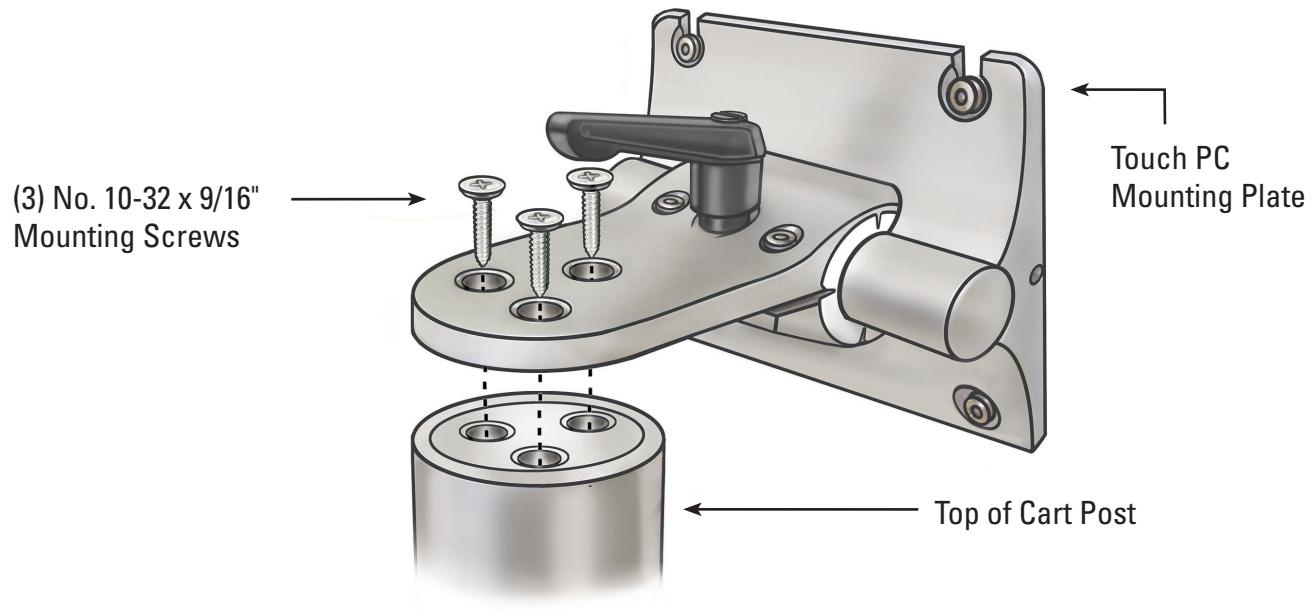

Attaching the Cart Handle

1. Separate the two pieces of the Cart Handle by using a Phillips-head screwdriver to remove the (2) pre-installed screws.
2. Attach the two pieces of the Cart Handle together around the Cart Post below the Cart label. The dark part of the handle should face in the same direction as the Cart's Height Adjustment Knob. See **Figures 3.4 and 3.5**.
3. Re-insert the (2) screws and use a Phillips-head screwdriver to secure the two handle pieces firmly around the Cart Post. Be careful not to overtighten the screws.

Figure 3.4 – Attaching the Cart Handle

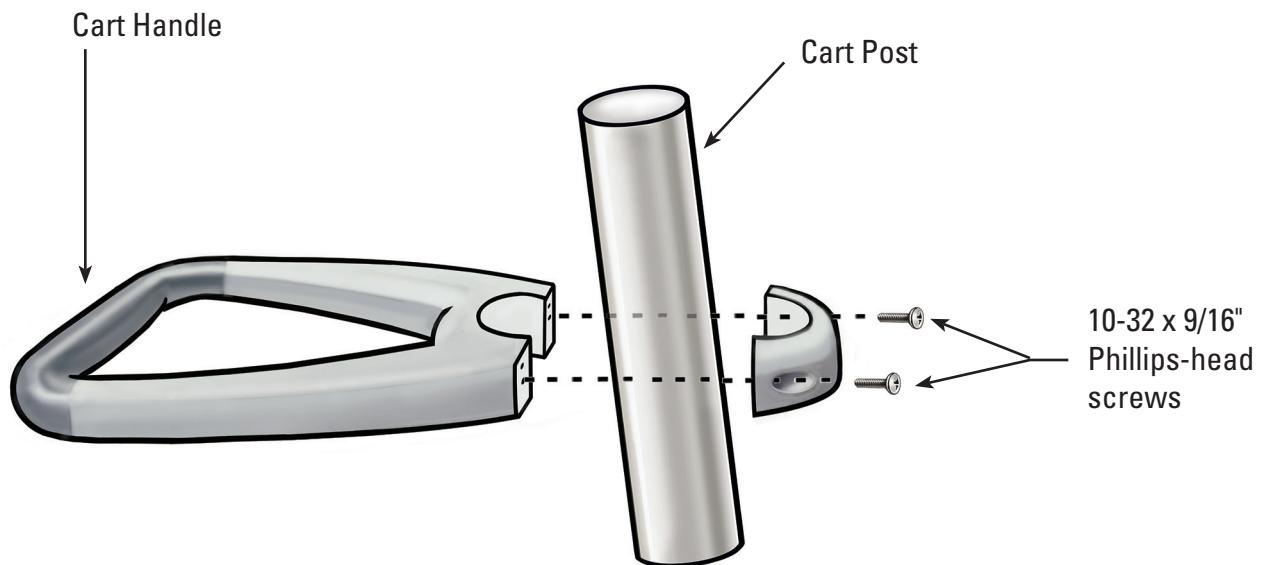
Figure 3.5 – Proper Placement of the Cart Handle

Attaching the Power Transformer Bracket

1. Attach the Cord Wrap Hook and Power Transformer Bracket's Clamping Blocks together around the Cart Post approximately 40 cm (16") from the Cart Base. Ensure the Velcro straps for the Power Transformer are facing forward (same direction as the Touch PC Mount). See **Figure 3.9** for correct installation with the Power Transformer.
2. Insert the (2) screws using a Phillips-head screwdriver to secure the Cord Wrap Hook and Power Transformer Bracket firmly to the Cart Post. Be careful not to overtighten the screws.


Installing the Touch PC onto the Cart

Follow these steps to install the Touch PC onto the Cart:

1. Remove the Touch PC and ancillaries from the box.
2. Confirm that all components listed in **Section 3.1 “System Workstation Components”** are included in the packaging and that the components are not damaged.
3. Loosen the Tilt Adjustment Lever and adjust the mounting plate so that it is approximately vertical. See **Figure 3.6**.

NOTE: if the Touch PC Mounting Plate is difficult to adjust, the tilt tension adjustment screws will need to be loosened. See **Section 3.2 “Adjusting Touch PC Tilt”** for instructions on how to adjust the tilt tension adjustment screws.

Figure 3.6 – Touch PC Tilt Adjustment Lever

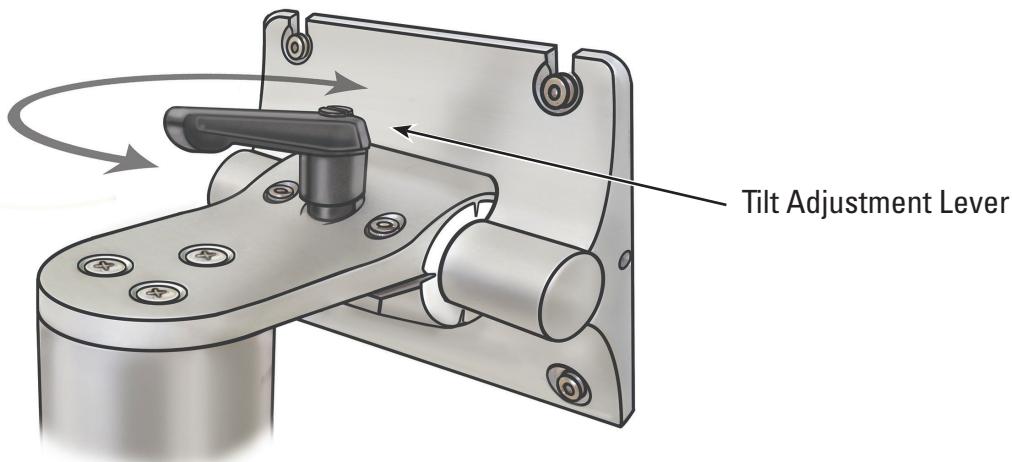
4. Tighten the Tilt Adjustment Lever to prevent movement of the mounting plate during assembly.
5. Using a Phillips-head screwdriver, install two of the four M4 x 10 mm Mounting Screws in the two upper holes of the inner set of the Touch PC's VESA-mounting holes (See **Figure 3.7**). Leave about 1/4" of space between the head of the screw and the Touch PC.

Figure 3.7 – Touch PC Mounting Holes

6. Have an assistant lift the Touch PC using the handholds molded into the sides of the Touch PC, and slide the two M4 x 10 mm Mounting Screws on the Touch PC into the two top slots on the mounting plate. The assistant should hold the Touch PC in this position.
7. Using a Phillips-head screwdriver, insert and tighten the lower two M4 x 10 mm Mounting Screws in the two lower holes of the Touch PC Mounting Plate. See **Figure 3.8**.


Figure 3.8 – Installing the Touch PC Mounting Plate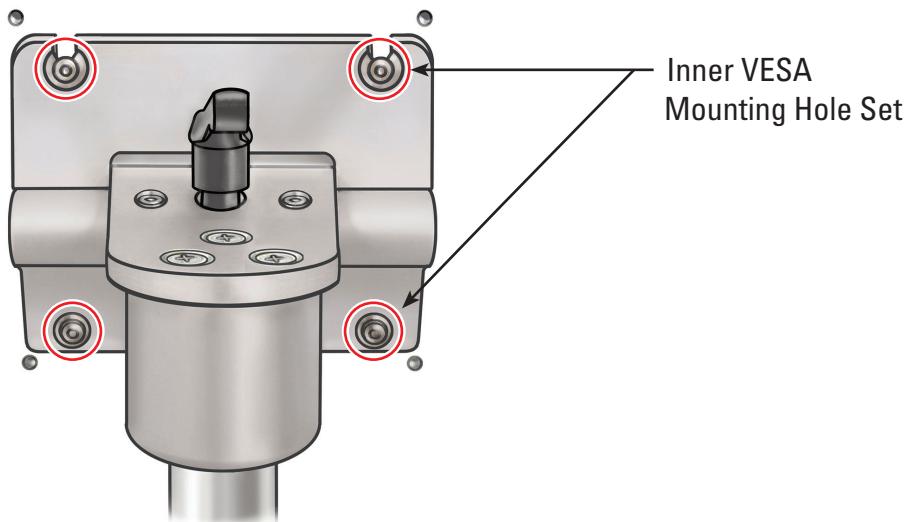

8. Tighten the top two M4 x 10 mm Mounting Screws holding the Touch PC to the mounting plate.
9. Install the Power Transformer into the Power Transformer Bracket with the power switch facing **down** and hold it in place using the Velcro straps. **See Figure 3.10.**
10. Attach the AC Power Cord to the Power Transformer. Place the Nylon Cable Clamp (from the Power Cord Strain Relief Kit) over the AC Power Cord.
11. Mount the Nylon Cable Clamp to the bottom of the Power Transformer Bracket using the #8 flat washer and 8-32 x 1/4" Phillips-head screw (from the Power Cord Strain Relief Kit).


NOTE: The loop of the Nylon Cable Clamp should point out the side of the Power Transformer Bracket. See **Figure 3.9.**

Figure 3.9 – Installing the Power Cord Strain Relief Kit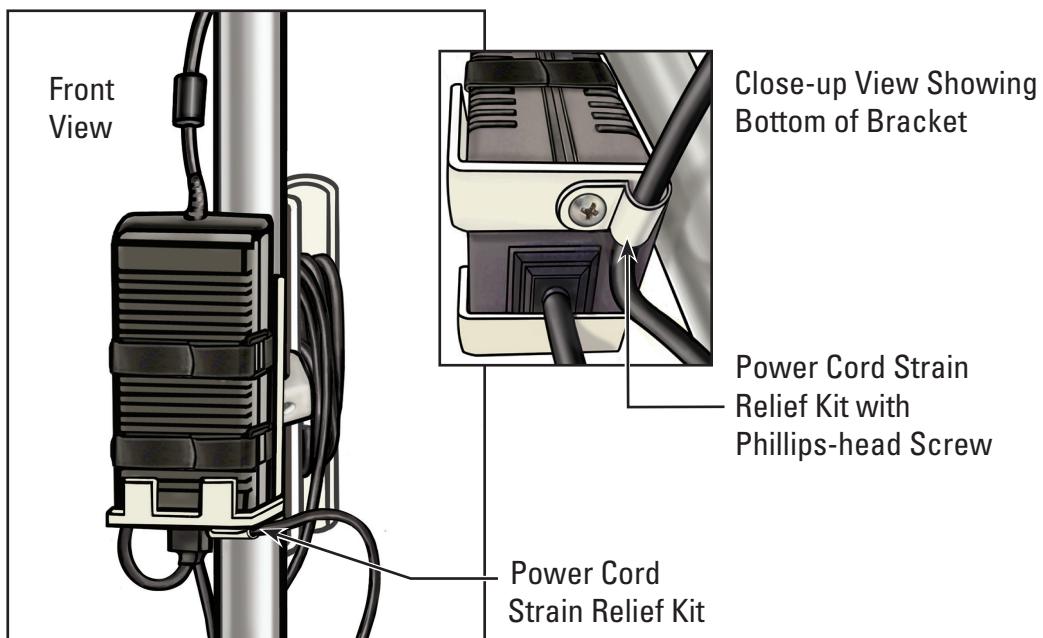

12. Organize the Power Cord in the Cord Wrap Hook.
13. Run the DC plug up the outside of the cart post and plug it into the DC power receptacle on the bottom of the Touch PC.


Figure 3.10 – Installing the Power Transformer Bracket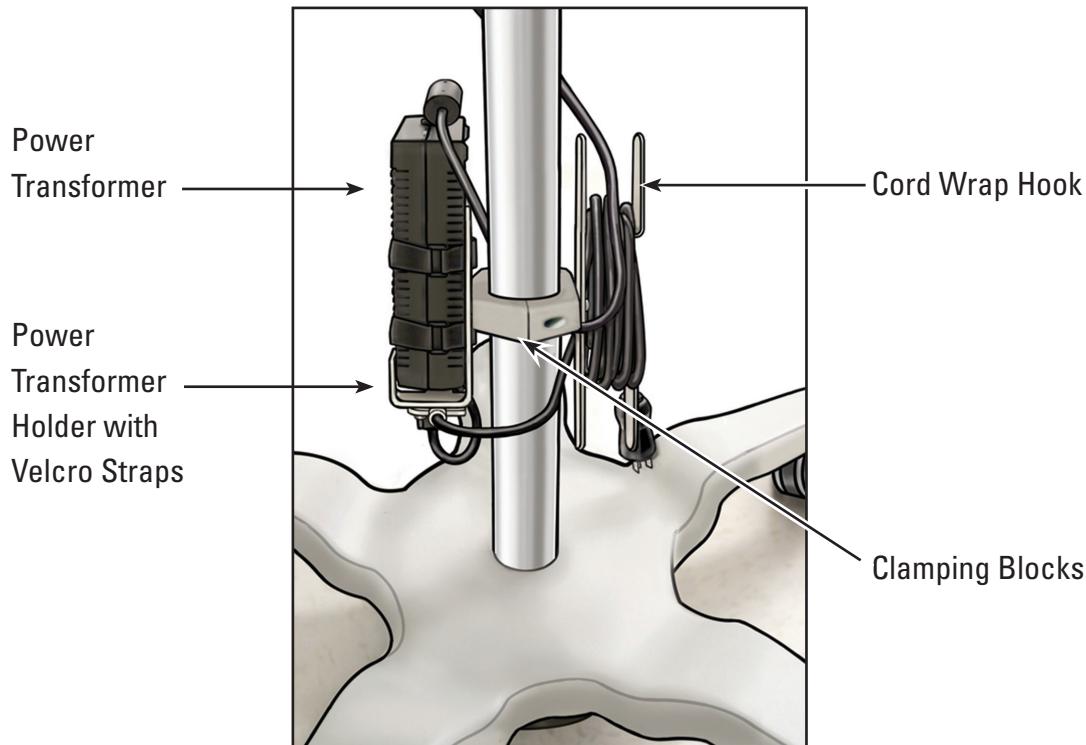

Cart Adjustment

The Cart is equipped with (2) adjustment mechanisms, one for adjusting the Touch PC's height and another for adjusting the Touch PC's tilt. See **Figure 3.11**.

Figure 3.11 – Cart Adjustments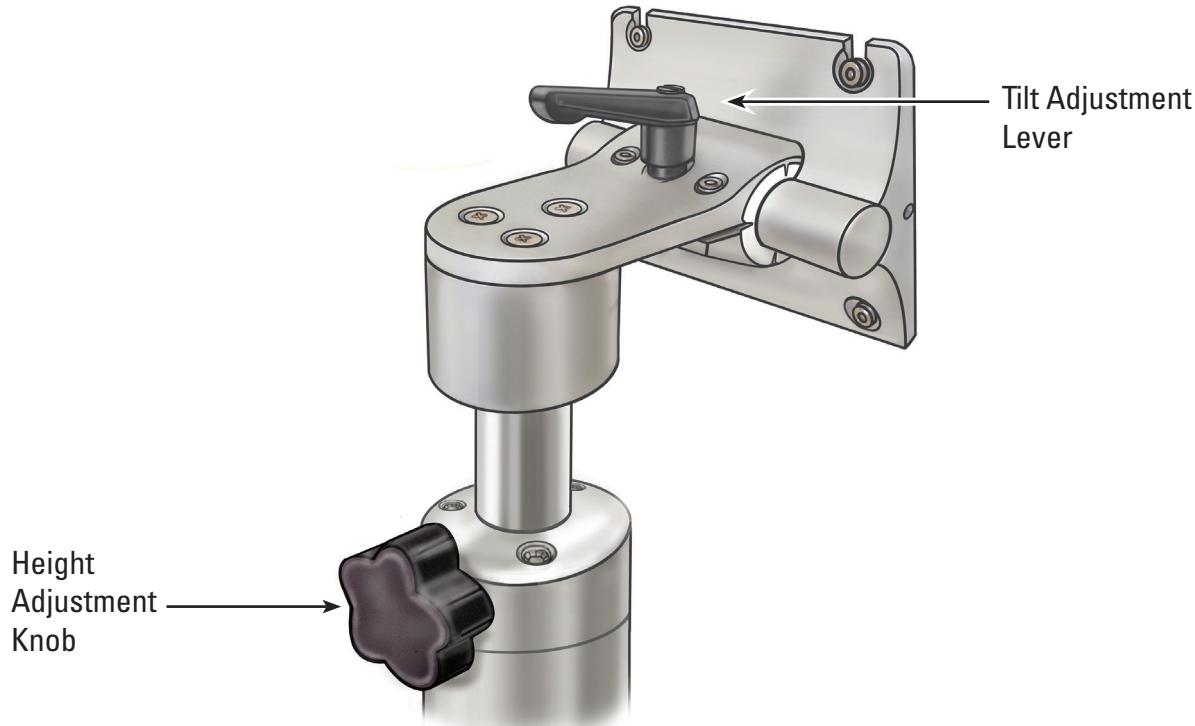

Adjusting Touch PC Height


CAUTION: The Touch PC Cart Post operates by pneumatic pressure and will travel upward when disengaged. Ensure that nothing obstructs upward travel before turning the Height Adjustment Knob. Grasp the Touch PC using the side handholds to control the upward travel of the Touch PC when the Height Adjustment Knob is loosened.

1. To adjust the Touch PC's height, loosen the Cart's Height Adjustment Knob while supporting the Touch PC using the handholds molded into the sides of the Touch PC. Move the Touch PC to its desired location.
2. Tighten the Cart's Height Adjustment Knob.

Adjusting Touch PC Tilt

1. To adjust the Touch PC's tilt, loosen the Cart's Tilt Adjustment Lever.
2. Grasp the handholds molded into the sides of the Touch PC and tilt to the desired angle.
3. Retighten the Tilt Adjustment Lever.
4. If the Touch PC does not hold its tilt position, use the 9/64" (3.5 mm) hex wrench provided with the Cart to tighten the tilt tension adjustment screws equally. See **Figure 3.12**.
5. If the Touch PC is too difficult to tilt, use the 9/64" (3.5 mm) hex wrench provided with the Cart to loosen the tilt tension adjustment screws equally. See **Figure 3.12**.

Figure 3.12 – Adjusting Cart Tilt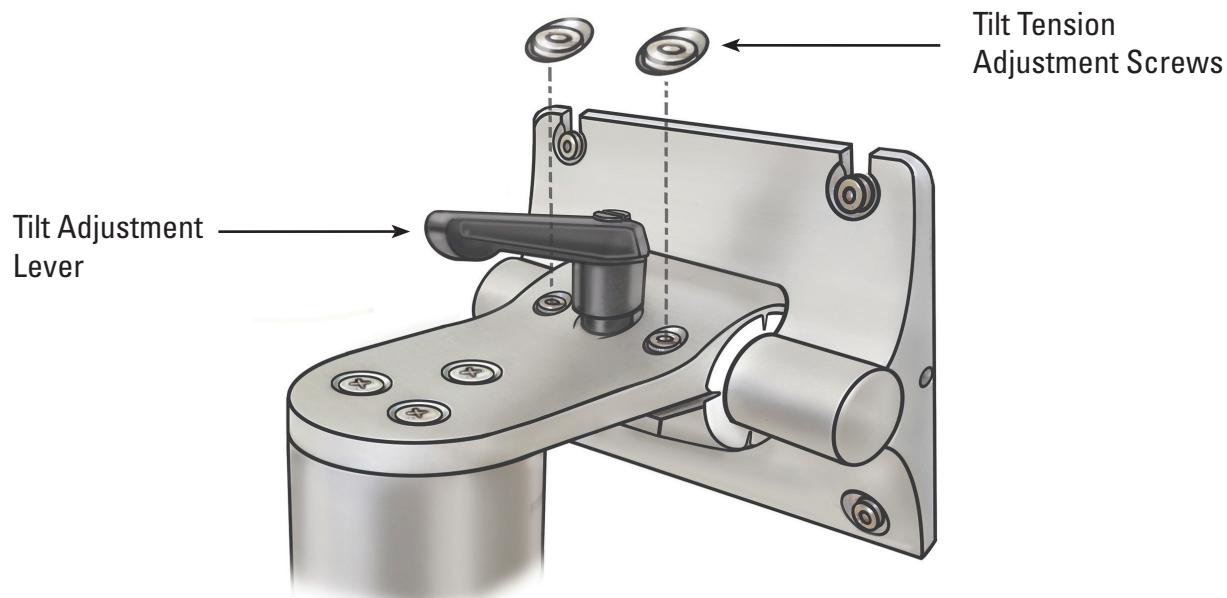

Confirming Installation


Confirm the correct installation of the System Workstation after assembly by following these steps:

1. Verify that all fasteners on the Cart and Touch PC mounting system are secure.
2. Plug the System Workstation into the hospital's electrical outlet and ensure the switch on the Power Transformer is turned on. The green LED on the Power Transformer will light when the Power Transformer is turned on.

CAUTION: To avoid risk of electric shock, ensure the Power Cord is plugged into the Touch PC before plugging the System Workstation into the hospital's electrical outlet.

3. Press the power button on the front panel to power on the System Workstation. (see **Figure 3.13**).

Figure 3.13 – Front Panel Power Button

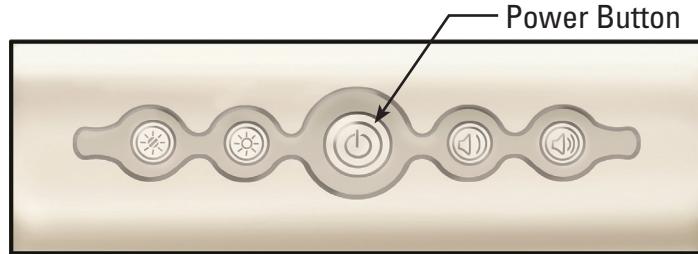
The System Workstation displays the boot screen (see **Figure 3.14**).

Figure 3.14 – Boot Screen

4. The System Workstation will display the LithoVue™ Flexscope disconnected message screen (see **Figure 3.15**) in the image display area. Once this image has been displayed, the System Workstation is functioning correctly and is now ready for use.

Figure 3.15 – Flexscope disconnected message screen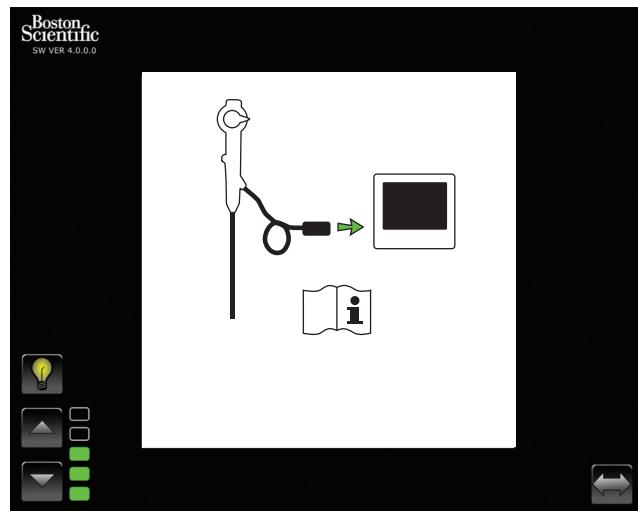

Connecting an External Monitor or DVI Switcher with the DVI Cable

If desired, you can display the live video image on an external DVI-compatible monitor or split the image signal using a DVI switcher by following these steps:

1. Confirm that the external monitor and/or DVI switcher meets the specifications described in **Section 3.4 "System Workstation Device - Accessory Compatibility"**.
2. Connect the supplied DVI Cable from the DVI output jack on the bottom of the System Workstation to the DVI In jack on the external monitor or DVI switcher.
3. Turn on the external monitor. The video image (but no touch screen buttons) will be displayed on the external monitor.

CAUTION: Users are responsible for using appropriate audio-visual equipment and for verifying proper installation prior to use.

3.3 TRANSPORTATION, USE AND STORAGE

The following instructions provide information for proper movement and storage of the System Workstation.

1. Ensure the Power Cord is securely looped around the Cart's Cord Wrap Hook to prevent cord damage during transportation.
2. Lift the pedal on the locking casters to unlock the System Workstation and move to the desired location. To ensure safe movement of the system, check that the wheels are free of obstacles.
3. Store the System Workstation in an area that is safe from impact or other accidental damage.
4. Confirm that storage area environment is within the limits provided in **Table 3.1**.
5. Confirm that the storage area is free from risk of water leakage or splashes.
6. Push pedal on the locking casters down to secure the system's position. This will prevent accidental rolling and potential impact damage.

Table 3.1 Transportation, use, and storage environmental limits

Environmental Limits during Use	
Ambient temperature (°C)	10 to 35
Relative humidity (%)	30 to 75 RH (non-condensing)
Atmospheric pressure (hPa)	700 to 1060
Environmental Limits during Transport and Storage	
Ambient temperature (°C)	-20 to 60
Relative humidity (%)	10 to 90 RH (non-condensing)
Atmospheric pressure (hPa)	500 to 1060

3.4 SPECIFICATIONS AND DEVICE COMPATIBILITY

Electrical

Table 3.2 Power Transformer and Power Cord Specifications

CAUTION: The Power Cords and Power Transformer must be compatible with the LithoVue™ System. A compatible Power Cord and Power Transformer are provided with each System Workstation for this purpose.

Power Transformer and Power Cord Specifications	
Power Transformer, Mfg: Sinpro Electronics Co., Ltd. Model: MPU101-105	External, 100 watts, 100-240 V ~ 47-63 Hz AC in, 13-16 V DC out, medical grade
North America Power Cord	Length—3.0 meters Voltage Rating—125 VAC Current Rating—10 amps
International Power Cords (EU, UK, Italy, Denmark, Switzerland, Australia/New Zealand, China, South Africa)	Length—2.5 meters Voltage Rating—250 VAC Current Rating—10 amps
Japan Power Cord	Length—2.5 meters Voltage Rating—125 VAC Current Rating—12 amps

Physical (nominal)

Table 3.3 Touch PC Specifications

Touch PC Physical Specifications	
Height:	40.9 cm (16.1 in.)
Width	41.9 cm (16.5 in.)
Depth	8.2 cm (3.2 in.)
Weight	6.4 kg (14.1 lbs.)
LCD Panel Size	17" 5:4
Resolution	1280 x 1024

Table 3.4 Cart Specifications

Cart Physical Specifications	
Height:	Approximately 124 cm to 157 cm (49 to 62 inches) to the top of the Cart Post
Base Diameter	63.5 cm (25 inches)
Weight (unpackaged)	Approximately 18 kg (40 lbs)

Illumination Output (nominal)

Table 3.5 Illumination Output

Illumination Output	
Type	LED
Wavelength	375-800 nm

Tested per IEC 62471 - Standard for Photobiological Safety of Lamps and Lamp Systems

Medical Electrical Classifications

Table 3.6 Medical Electrical Classifications

The LithoVue™ System meets the requirements of the following standards: IEC 60601-1 and CAN/CSA-C22.2 No. 60601-1.

Medical Electrical Classifications	
Type of Protection Against Electric Shock	Class 1 equipment
Degree of Protection Against Electric Shock	Type BF Applied Part (LithoVue Flexscope)
Degree of Protection Against Ingress of Liquids, System Workstation	IPX0
Degree of Protection Against Ingress of Liquids, LithoVue Flexscope	IPX2
Mode of Operation	Continuous Operation
Suitability for use in oxygen rich environments	Equipment not suitable for use in the presence of flammable mixtures
Installation and Use	Mode

LithoVue Flexscope Fluid Compatibility

The LithoVue Flexscope is known to withstand exposure to the following procedural fluids: blood, urine, saline, contrast media, petroleum-based and water-based lubricants.

System Workstation Device - Accessory Compatibility

NOTE: All attached equipment (DVI switcher, external monitor, etc.) must comply with applicable electrical safety standards. Proper system functioning must be verified before use.

- The System Workstation is ONLY compatible with the LithoVue Flexscope.
- DVI cable - DVI dual-link cables up to 15 feet may be used with this system.
- DVI switcher - DVI dual-link powered switchers may be used with this system. The LithoVue System has been tested and shown to be compatible with the Extron DVI Switcher (P/N 60-964-01).
- Monitor compatibility requirements - Must have a DVI input and a minimum resolution of 1280 x 1024 at any of the following aspect ratios: 5:4 and 16:9. The LithoVue System has been tested and shown to be compatible with the NDS Radiance SC-SX19-A1511.

Software Information

The software revision level is controlled by Boston Scientific. The current revision level is provided below the BSC logo on the System Workstation display.

4 LithoVue™ System

4.1 SYSTEM WORKSTATION FEATURES AND USER INTERFACE

System Workstation Front Panel Features and User Interface

The System Workstation Front Panel (see **Figure 4.2**) has the following features:


- **Main Operating Screen** – A color liquid crystal touch-screen display (LCD) provides the live video image and graphical user interface (GUI). The user interacts with the system via the touch-sensitive controls on the screen. Buttons on the main operating screen are activated with a finger press.

NOTE: The System Workstation has been demonstrated to function with gloves.

- **Power Button** – When the System Workstation is connected to a power source, pushing the front panel Power Button powers up the System Workstation. When the System Workstation is powered on, another press of the Power Button will power the system down. The Power Button illuminates blue when the System Workstation is powered on.


NOTE: There are also two volume control buttons on the front of the Touch PC. These volume control buttons (see **Figure 4.1**) are not functional. There are also two screen brightness buttons on the front of the System Workstation. Adjusting these will change the screen brightness.

Figure 4.1 – Front Panel Power Button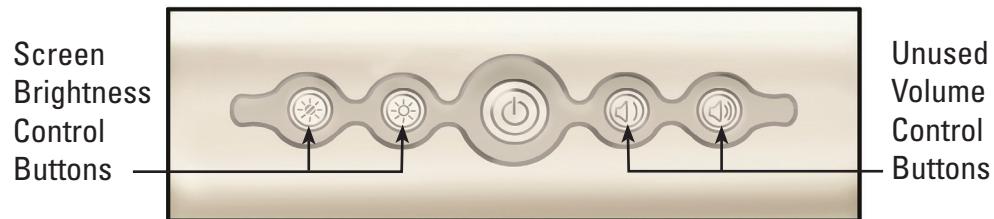

- **Flexscope Connector Cable Plug Receptacle (Workstation Receptacle)** – The LithoVue Flexscope Connector Cable's Plug plugs into the receptacle on the front of the LithoVue System Workstation.

Figure 4.2 – System Workstation Front Panel Features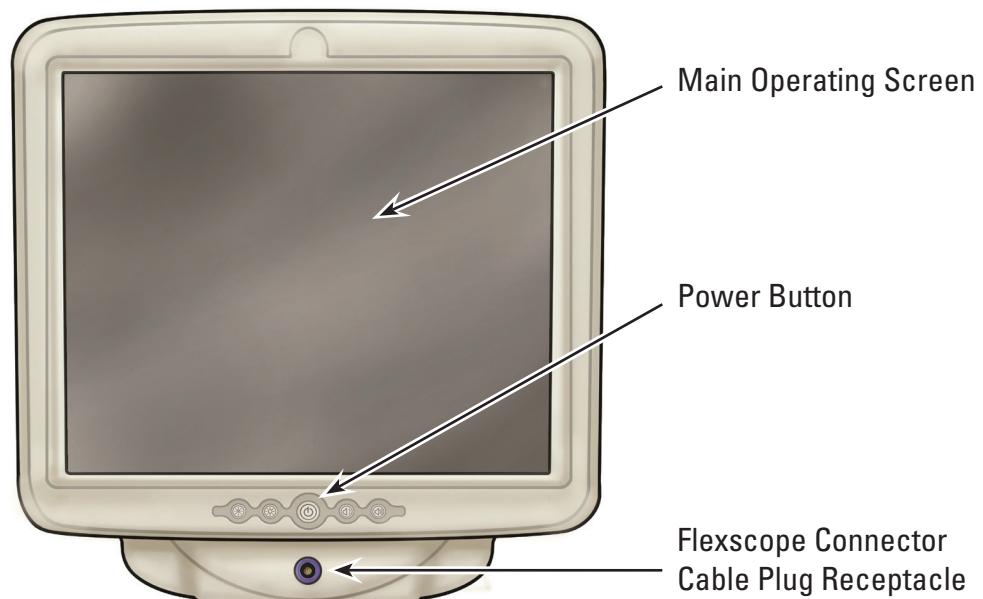


System Workstation Rear Panel Features

The rear panel includes these features:

- **DVI Output Connector**—(See **Figure 4.3**) Video output connector for compatible DVI-capable monitors or a DVI Switcher. For more information regarding compatible monitors and DVI Switchers, see **Section 3.4 “System Workstation Device - Accessory Compatibility”**.
- **DC Power Cord Connector**—(See **Figure 4.3**) Connection point for the Touch PC’s power cord.
- **Labels**—Provide regulatory and manufacturing information.

Figure 4.3 – System Workstation Rear Panel Features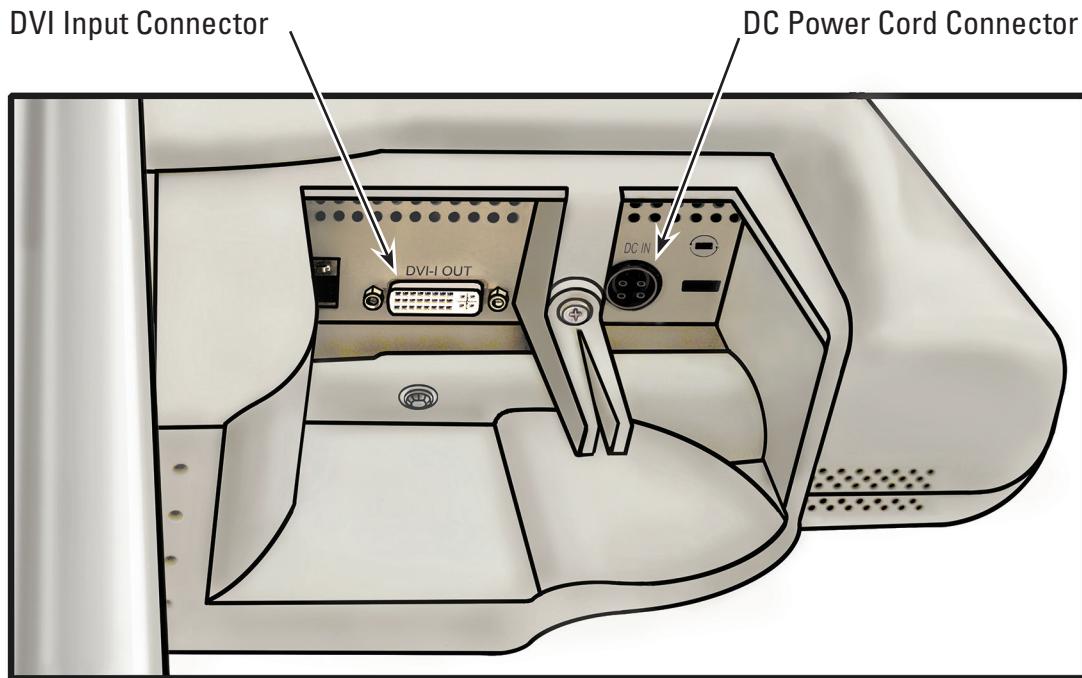


4.2 SYSTEM WORKSTATION CART FEATURES

The Cart includes the following features (see **Figure 4.4**):

- **Locking Casters** – Three of the Cart casters lock, providing stability during use and storage.
- **Handle** – Facilitates repositioning of the System Workstation.
- **Cord Wrap Hook** – Provides storage for the power cord during use and storage.
- **Power Transformer Bracket** – Secures the Power Transformer to the Cart.
- **Height Adjustment** – The height of the Touch PC can be adjusted from 124 cm to 157 cm (49 to 62 inches). For more information on how to adjust the height of the Touch PC, see **Section 3.2 “Cart Adjustment”**.
- **Tilt Adjustment** – The Touch PC viewing angle can be adjusted using the Cart Tilt Adjustment lever. For more information on how to adjust the tilt angle of the Touch PC, see **Section 3.2. “Cart Adjustment”**.

Figure 4.4 – Cart Features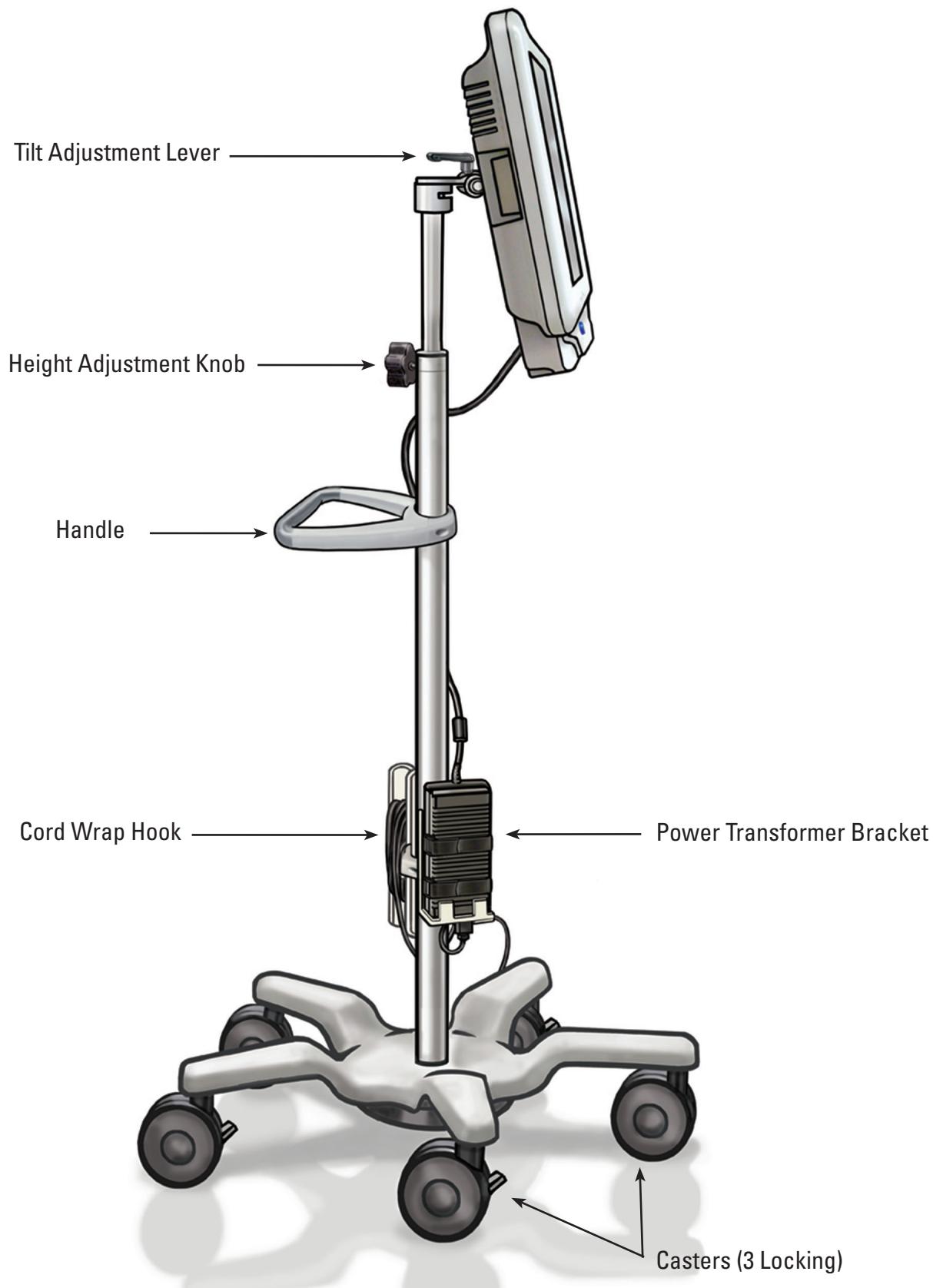


4.3 MAIN OPERATING SCREEN

Main Operating Screen Features

The main operating screen (see **Figure 4.5**) includes the features described in the sections below.

NOTE: On-screen buttons are activated by a finger-press.

Figure 4.5 – Main Operating Screen

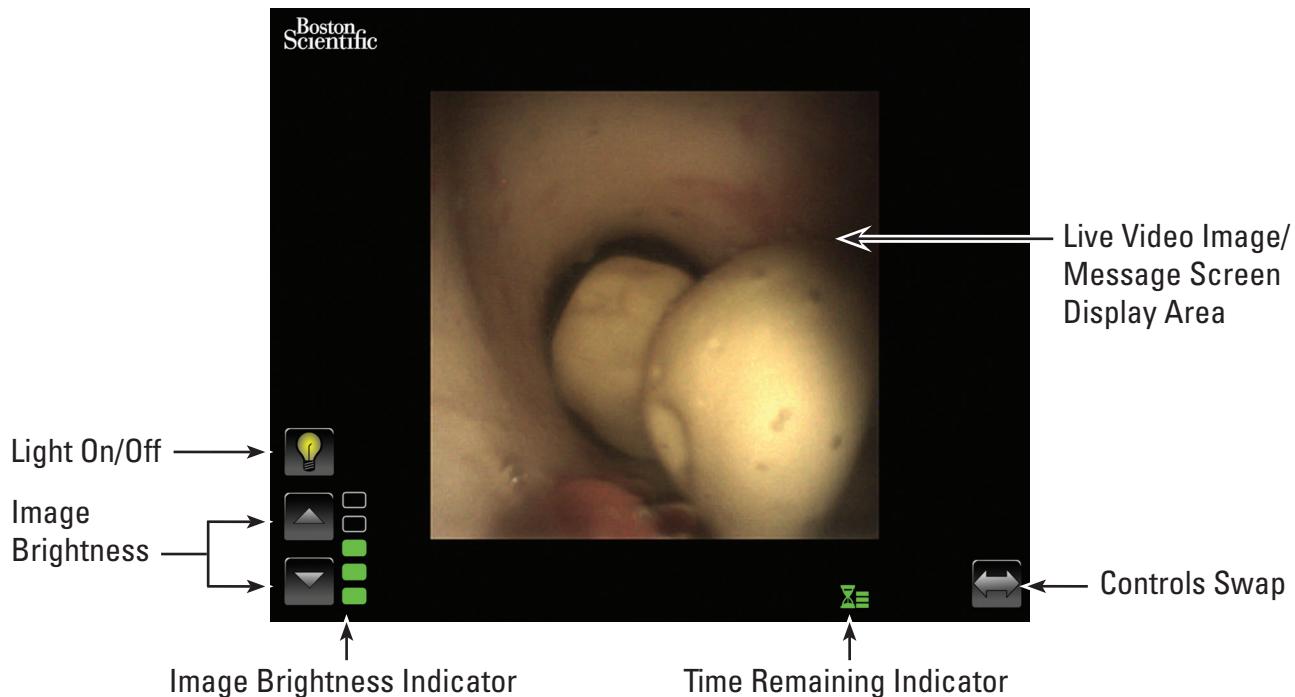Light On/Off Button

When the System Workstation is turned on and a new LithoVue™ Flexscope is connected, the system will default to **Light On**. Any subsequent scope connections while the System Workstation is turned on will default to on or off depending on the status of the **Light On/Off** button.

Each press of the **Light On/Off** button will toggle the scope illumination between on and off.

Default Light On indicator

When the **Light On** button is pressed, the icon will change and briefly illuminate to acknowledge the button press.

Light Off after a button press

After a short period of time, the **Light On/Off** button will then change to display the default **Light Off** indicator.

Default Light Off indicator

When the **Light Off** button is pressed, the icon will change and briefly illuminate to acknowledge the button press.

Light On after a button press

After a short period of time, the **Light On/Off** button will then change to display the default **Light On** indicator noted above.

Image Brightness Control Buttons

When the light is on, pressing the \triangle **button** increases the image brightness. Pressing the ∇ **button** decreases the image brightness. After each button press, the buttons will briefly illuminate to acknowledge the button press before returning to the default state.

Default Image Brightness Control Buttons

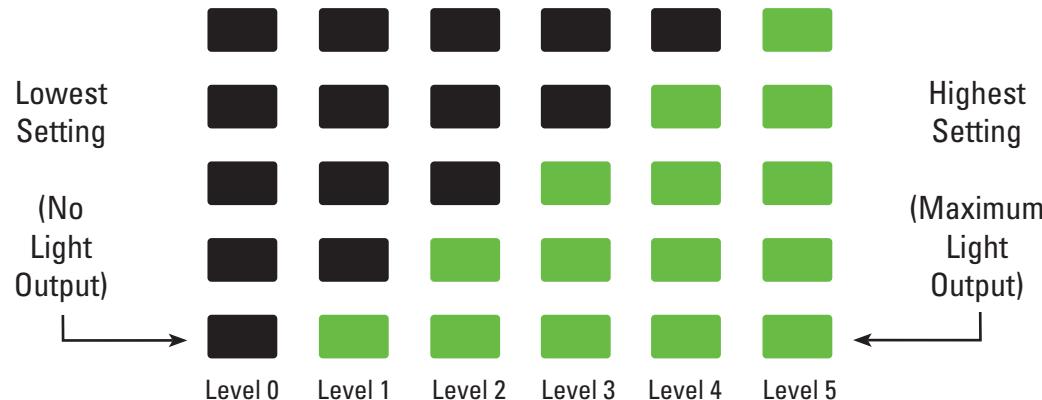
Image Brightness Control Buttons after a button press

Image Brightness Indicator

The bars of this indicator illuminate to indicate the brightness of the image. There are five brightness levels available for user adjustment using the **Image Brightness Control Buttons**.

Time Remaining Indicator

The LithoVue™ Flexscope is a single-use device. To help mitigate the potential for re-sterilization and re-use of the LithoVue Flexscope, the System Workstation monitors how long the LithoVue Flexscope has been connected and provides graphical feedback to the user.

The bars and color of this indicator change to indicate LithoVue Flexscope status, as noted in **Table 4.1**. Once the Orange Blinking **Time Remaining Indicator** appears, the user has 30 minutes to complete the procedure before the LithoVue Flexscope ceases to function.

When there are zero minutes remaining, the **Time Remaining Indicator** will disappear and the “**Use Time Exceeded**” message screen will appear in the image display. See **Section 4.3 “Message Screens”** for more information on the use time exceeded message screen.

If additional time is needed for the procedure, after the “Use Time Exceeded” message displays, a new LithoVue™ Flexscope will need to be used.

Table 4.1 Symbols indicating operating time remaining on the LithoVue Flexscope

Color/State of Icon	Green Icon	Yellow Icon	Orange Icon	Orange Icon Blinking
Icon				
User Response	Proceed with case, no action required	Proceed with case, no action required	Proceed with case, being cautious of time remaining	30 minutes of scope use remaining. Obtain another scope if additional time is needed.

Controls Swap Button

Pressing the **Controls Swap** button moves the control buttons and indicators to the opposite side of the screen for user convenience (see **Figures 4.6a** and **4.6b**). At the next system startup, the system will default to displaying control buttons and indicators on whichever side was last selected by the user.

Default Controls Swap Button

After pressing the **Controls Swap** button, the button will briefly illuminate to acknowledge the button press before returning to the default state.

Controls Swap Button after a button press

Figure 4.6a – Main Operating Screen – Controls on Left

Figure 4.6b – Main Operating Screen – Controls on Right

Live Video Image Display Area

This is the area where the live video image is displayed. The live video image area is also used to display system message screens in lieu of the video image. For more information on message screens, see **Section 4.3 "Message Screens"**.

Message Screens

The system can display five message screens in the main image area to indicate system status or problems (see **Figure 4.7**). See **Section 7 "Troubleshooting"** for more information on troubleshooting the System Workstation when an error message is displayed.

Figure 4.7 – System Workstation Message Screens

Icon	Meaning
	Problem with LithoVue™ Flexscope
	LithoVue Flexscope disconnected
	LithoVue Flexscope initializing
	System Workstation hardware malfunction
	LithoVue Flexscope use time exceeded

5 System Operation

5.1 STARTING THE SYSTEM WORKSTATION

Follow the steps below to start the System Workstation:

1. Plug the System Workstation into the hospital's electrical outlet and ensure the switch on the Power Transformer is turned on. The green LED on the Power Transformer will light when the Power Transformer is turned on.

CAUTION: To avoid risk of electric shock, ensure the Power Cable is plugged into the Touch PC before plugging the System Workstation into the hospital's electrical outlet. Additionally, before starting the System Workstation, ensure the Power Cord is fully inserted into the hospital's electrical outlet.

CAUTION: Ensure that the System Workstation Power Cable is located such that it will not pose a trip risk to operators or cause an accidental disconnection of the Power Cord.


2. Press the power button on the front panel to power on the System Workstation.
3. The System Workstation displays the boot screen (see **Figure 5.1**).

Figure 5.1 – Boot Screen

4. If the LithoVue™ Flexscope has not yet been plugged in, the System Workstation displays the Flexscope disconnected message screen (see **Figure 5.2**) in the image display area.

Figure 5.2 – LithoVue Flexscope Disconnected Message Screen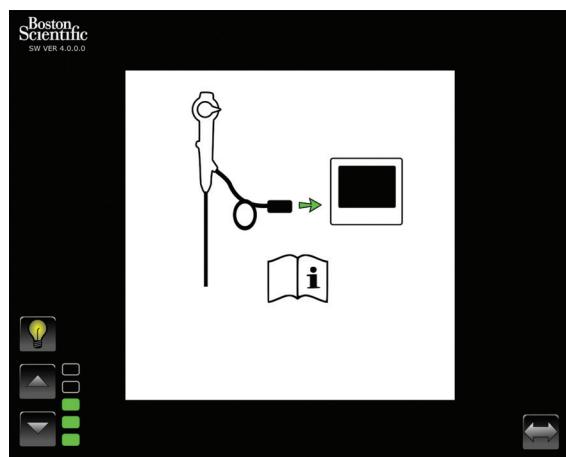

5. Plug the LithoVue™ Flexscope Cable into the Workstation Receptacle on the front panel of the System Workstation by lining up the triangle on the Connector Cable's Plug with the triangle on the front panel of the System Workstation.

NOTE: The LithoVue Flexscope may be plugged in before the System Workstation has been powered on. In this case, the system will proceed directly to scope initialization and then display of the preliminary live image.

CAUTION: The Flexscope Cable should connect to the Workstation Receptacle easily. If the Connector Cable's Plug does not connect, verify that the arrows on the Plug and Workstation Receptacle line up. Forcing the Connector Cable's Plug into the cable receptacle may damage the System Workstation.

6. Once the LithoVue Flexscope is connected, the System Workstation will display the Flexscope Initialization screen (See **Figure 5.3**). After the LithoVue Flexscope has completed the initialization process, the System Workstation displays a preliminary image. A white progress bar, located at the bottom of the display screen, indicates the System Workstation is downloading calibration information from the LithoVue Flexscope (see **Figure 5.4**). The image will not be final until the LithoVue Flexscope has finished calibration.

Figure 5.3 – LithoVue Flexscope Initialization Screen


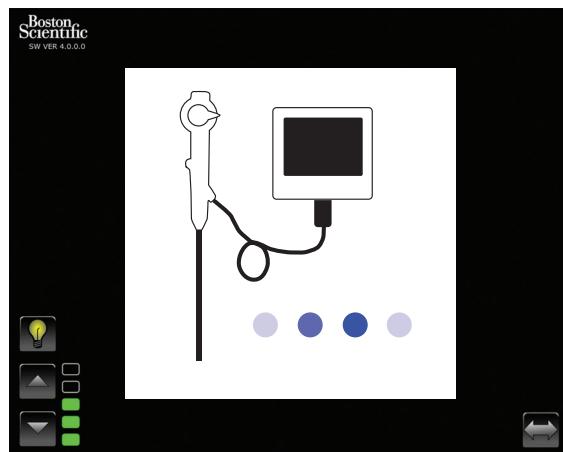
Figure 5.4 – Preliminary image display and calibration loading progress bar

7. After downloading the LithoVue Flexscope calibration information, the System Workstation displays the high quality live video image in the image display area (see **Figure 5.5**).

Figure 5.5 – LithoVue™ System displaying final high quality live video image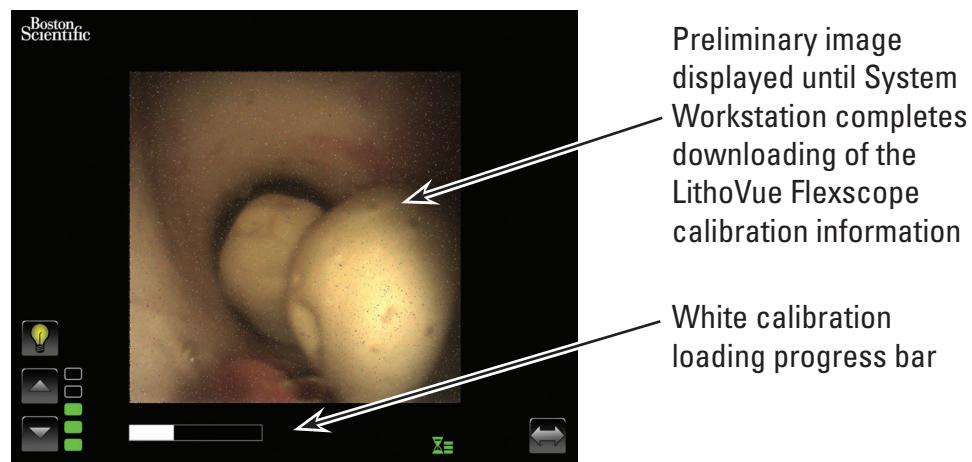

5.2 CHECKING THE QUALITY OF THE VIDEO IMAGE

Before performing a procedure, hold the distal tip of the LithoVue Flexscope close to an object (about the same distance as the width of the shaft) and ensure the video monitor displays a clear, sharply focused, high-quality image. Confirm the “real time” appearance of the video image on the video monitor. Confirm the image is oriented correctly.

If the image is not displaying in real time, is not oriented correctly, or is not clear and sharply focused, see **Section 7 “Troubleshooting”** to resolve the problem before beginning the procedure.

If a DVI switcher and/or external monitor are being used, ensure the image displays properly before beginning the procedure.

5.3 ADJUSTING THE BRIGHTNESS OF THE VIDEO IMAGE

The LithoVue System will automatically adjust the illumination to ensure adequate visibility at the treatment site. However, the brightness of the video image may be further modified by the user.

To increase the brightness of the video image as displayed on the live video image screen, press the \triangle button. To decrease the brightness of the video image, press the ∇ button. The image brightness scale provides visual feedback of the relative brightness of the light source.

5.4 COMPLETING A PROCEDURE

The instructions below describe the procedural use of the System Workstation and assumes you have received, inspected, assembled and confirmed installation of the System Workstation following the instructions in **Section 3 “How Supplied”**.

Using the LithoVue System in an Ureteroscopy Procedure

Use of the LithoVue System in a ureteroscopy procedure involves the following steps:

1. Plug the System Workstation into the hospital electrical supply outlet and ensure the switch on the power transformer is turned on.
2. Place the System Workstation in a position such that it is convenient to the user during the procedure (including adjusting the Touch PC’s height and tilt as described in **“Section 3.2 Cart Adjustment”**).
3. Once the System Workstation is in position, lock the Cart casters to prevent unintentional movement during use. Unlock the casters prior to attempting additional movement, then relock prior to system use. Ensure the ventilation areas on the System Workstation are kept clear.

CAUTION: The Touch PC Cart Post operates by pneumatic pressure and will travel upward when disengaged. Ensure that nothing obstructs upward travel before turning the Height Adjustment Knob. Grasp the Touch PC using the side handholds to control the upward travel of the Touch PC when the Height Adjustment Knob is loosened.

4. Press the power button on the front panel to power on the System Workstation. Verify the System Workstation powers up appropriately prior to proceeding with the procedure.
5. If desired, press the “Controls Swap” button to change the orientation of the on-screen controls, such that they are more accessible to the user.
6. Remove the LithoVue™ Flexscope from its packaging. Prepare and inspect the LithoVue Flexscope as described in the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use.
7. Insert the Connector Cable plug into the Workstation Receptacle on the front panel of the System Workstation by lining up the triangle on the Connector Cable’s Plug with that on the front panel of the System Workstation.

NOTE: the LithoVue Flexscope may be plugged in before the System Workstation has been powered on. In this case, the LithoVue Flexscope disconnected message screen will not display.

CAUTION: The Flexscope Cable should connect to the Workstation Receptacle easily. If the Connector Cable’s Plug does not connect, verify that the arrows on the Plug and Workstation Receptacle line up. Forcing the Connector Cable’s Plug into the cable receptacle may damage the System Workstation.

8. Confirm that the LithoVue Flexscope is functioning properly by following the “Inspection and Operational Checks” instructions in the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use.
9. Confirm the quality and real-time appearance of the video image. If desired, press the “Light On/Off” button to turn the LithoVue Flexscope light on and off, as needed. If the LithoVue Flexscope is not to be used immediately for a procedure, it is recommended that the light be turned off until needed.
10. If desired, connect an irrigation/contrast source to the LithoVue Flexscope, as described in the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use.
11. Introduce the LithoVue Flexscope to target site, as described in the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use, using imaging, as needed, to confirm position.
12. Press the “Light On/Off” button to turn the Lithovue Flexscope light on or off and use the “Image Brightness” control buttons to adjust brightness, as needed, throughout the procedure.
13. Use the LithoVue Flexscope’s Articulation Lever to deflect the distal tip as needed to facilitate diagnosis and therapy. See the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use for more information on articulating the distal tip.

WARNING: In the event that the live video image becomes lost or unusable, **STOP – DO NOT ADVANCE THE URETEROSCOPE, DO NOT INSERT, ADVANCE, or ACTUATE PROCEDURAL DEVICES IN THE WORKING CHANNEL.** See the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use or **Section 7 “Troubleshooting”** for troubleshooting information.

14. Use the LithoVue Flexscope’s working channel to deliver accessories to complete the diagnosis and therapy. See the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use for more information on using accessories.
15. Remove the LithoVue Flexscope from the patient, as described in the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use..
16. Unplug the LithoVue Flexscope from the System Workstation by sliding the locking collar on the connector cable plug toward the cable and pull the plug out to remove the cable.

CAUTION: Do not remove the Flexscope Cable from the System Workstation by pulling on the cable. Improper removal of the Flexscope Cable may damage the System Workstation.

Dispose of the LithoVue Flexscope in accordance with hospital, administrative and/or local government policy.

5.5 SAFELY SHUTTING DOWN THE SYSTEM WORKSTATION

Follow these steps to shut down the System Workstation at the end of a procedure or during a procedure:

1. If a restart is required during a procedure, press the “Power” button on the front of the System Workstation to power it down. Another press of “Power” button will restart the system.
2. To shut down the system at the end of a procedure:
 - a) Remove the LithoVue™ Flexscope from the patient and prepare for disposal, as described in the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use.

NOTE: the LithoVue Flexscope may be unplugged from the System Workstation before or after the System Workstation has been powered down.

- b) Press the “Power” button on the front of the System Workstation to power it down.
- c) Clean the System Workstation after each use as described in **Section 6 “Cleaning and Maintenance”**.

5.6 DISPOSAL OF THE PRODUCT, ACCESSORIES, AND PACKING MATERIALS

Dispose of all products, accessories and packaging materials in accordance with hospital, administrative and/or local government policy.

6 Cleaning and Maintenance

6.1 CLEANING THE SYSTEM WORKSTATION

CAUTION: To avoid the potential for electric shock, disconnect the Power Cord from the hospital's electrical outlet before cleaning the System Workstation.

The System Workstation may be cleaned with mild, non-abrasive cleaning solutions commonly used in the hospital environment. The System Workstation has been tested and demonstrated to be compatible with the following cleaning solutions:

- 70% Isopropyl alcohol
- Ammonium Chloride 10%
- Sodium Hypochlorite 6%

Do not spray the Touch PC with cleaning solution. Apply the cleaning solution to a cleaning cloth and wipe all external surfaces.

CAUTION: To avoid System Workstation damage, do not allow fluids to enter the Touch PC enclosure, Power Cord connections, connector cable receptacle, or other component/accessory connections.

CAUTION: Using cleaners and disinfectant solutions that contain long-life surfactants can leave conductive residues on the contacts in the Workstation Receptacle. The conductive residues can lead to malfunctions of the System Workstation. To prevent electrical malfunctions due to conductive residue buildup on the contacts, do not use cleaners and disinfectant solutions that contain long-life surfactants.

6.2 MAINTENANCE, SERVICE AND REPLACEMENT PARTS

- **Touch PC**

Do not open the Touch PC housing. The Touch PC has no serviceable components and does not require routine maintenance or calibration. Contact Boston Scientific for repair or service using the information found in **Section 8 "Service and Warranty"**.

- **Cart**

All fasteners associated with the Touch PC mounting system should be inspected periodically and tightened as necessary.

Before each use, check the function of the tilt and height adjustment features of the Cart. For directions on how to adjust the cart tilt and height, see **Section 3.2 "Cart Adjustment"**.

- **Power Transformer and Power Cord**

In the event of a failure of the Power Transformer or Power Cord, replacement components can be ordered from Boston Scientific.

WARNING: Use of a non-approved power transformer or power cord may damage the System Workstation.

The new Power Transformer and Power Cord may be replaced by a trained biomedical technician, following the installation instructions provided in **Section 3.2 "System Workstation Assembly"**.

7 Troubleshooting

Most operating problems are easily resolved. If the LithoVue™ System does not operate as expected, try to resolve the problem using the instructions contained in this section before contacting Boston Scientific for technical support.

This section contains directions on how to isolate the System Workstation from the mains power supply, as well as troubleshooting instructions for the following situations:

1. Loss of image (including image problems such as lost image, frozen image, delayed image or other image ill-effects).
2. Loss of articulation control on the LithoVue Flexscope.
3. System Workstation power loss.
4. System Workstation becomes non-operational due to a hardware or software error.

7.1 ISOLATING THE SYSTEM WORKSTATION FROM THE MAINS POWER SUPPLY

To isolate the System Workstation from the hospital electrical supply outlet, disconnect the power cord from the System Workstation.

CAUTION: Do not position the System Workstation such that it is difficult to disconnect from the Mains Power Supply.

7.2 LOSS OF IMAGE

CAUTION: In the event that the live video image becomes lost or unusable, **STOP – DO NOT ADVANCE THE URETEROSCOPE, DO NOT INSERT, ADVANCE, or ACTUATE PROCEDURAL DEVICES IN THE WORKING CHANNEL.** Blind manipulation of the ureteroscope can result in patient injury and/or scope damage.

If a user message is present on the System Workstation, refer to **Table 7.1** for the potential cause and troubleshooting steps. If no user message is displayed, conduct the following steps to correct the issue:

1. Disconnect the LithoVue Flexscope from the System Workstation.
2. Ensure that the Connector Cable and plug and Workstation Receptacle are undamaged.

3. Ensure that the Connector Cable's Plug and the Workstation Receptacle are clean and dry.
4. Reconnect the LithoVue™ Flexscope to the System Workstation, firmly pushing in the connector until it is fully inserted.
5. Check for live video image. If video image is unusable, continue to Step 6.
6. Power off the System Workstation.
7. Power on the System Workstation.
8. If live image cannot be restored, discontinue use of the LithoVue Flexscope. Conduct the following steps to remove the LithoVue Flexscope:
 - a. Return the Articulation Lever to Neutral position – this will cause the distal tip to return to an unarticulated position. Reference the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use (Section "**Ureteroscope Features and Controls**"). The articulation state of the LithoVue Flexscope can be confirmed with imaging.
 - b. Slowly withdraw the LithoVue Flexscope from the patient. If you feel resistance, investigate the source of resistance with imaging before continuing to withdraw the LithoVue Flexscope.
 - c. Dispose of the LithoVue Flexscope in accordance with hospital, administrative and/or local government policy and use another scope to complete the procedure.

If the image issues still do not resolve after connection of a new LithoVue Flexscope, contact Boston Scientific using the information found in **Section 8 "Service and Warranty"**.


7.3 LOSS OF ARTICULATION CONTROL ON THE LITHOVUE FLEXSCOPE

CAUTION: In the event of a loss of articulation control, **STOP – DO NOT ADVANCE THE URETEROSCOPE, DO NOT INSERT, ADVANCE, or ACTUATE PROCEDURAL DEVICES IN THE WORKING CHANNEL.**

In the event that the articulation control becomes unresponsive, discontinue use of the LithoVue Flexscope by conducting the following steps:

1. Return the Articulation Lever to the Neutral position (non-articulated state). Reference the LithoVue Single-Use Digital Flexible Ureteroscope Directions for Use (Section "**Ureteroscope Features and Controls**").
2. Use imaging to confirm that the distal tip is in a nonarticulated state.
3. If the distal tip remains articulated, utilize the Articulation Lever and imaging to straighten the tip.
4. If the Articulation Lever cannot be used to straighten the distal tip, an 0.035" (.89 mm) or larger super stiff guidewire may be passed, floppy end first, through the working channel to aid in straightening the distal tip using the following steps:
 - a. Stabilize the handle of the LithoVue Flexscope and insert the guidewire into the LithoVue Flexscope using the accessory access port (see **Figure 7.1**).

Figure 7.1 – LithoVue™ Flexscope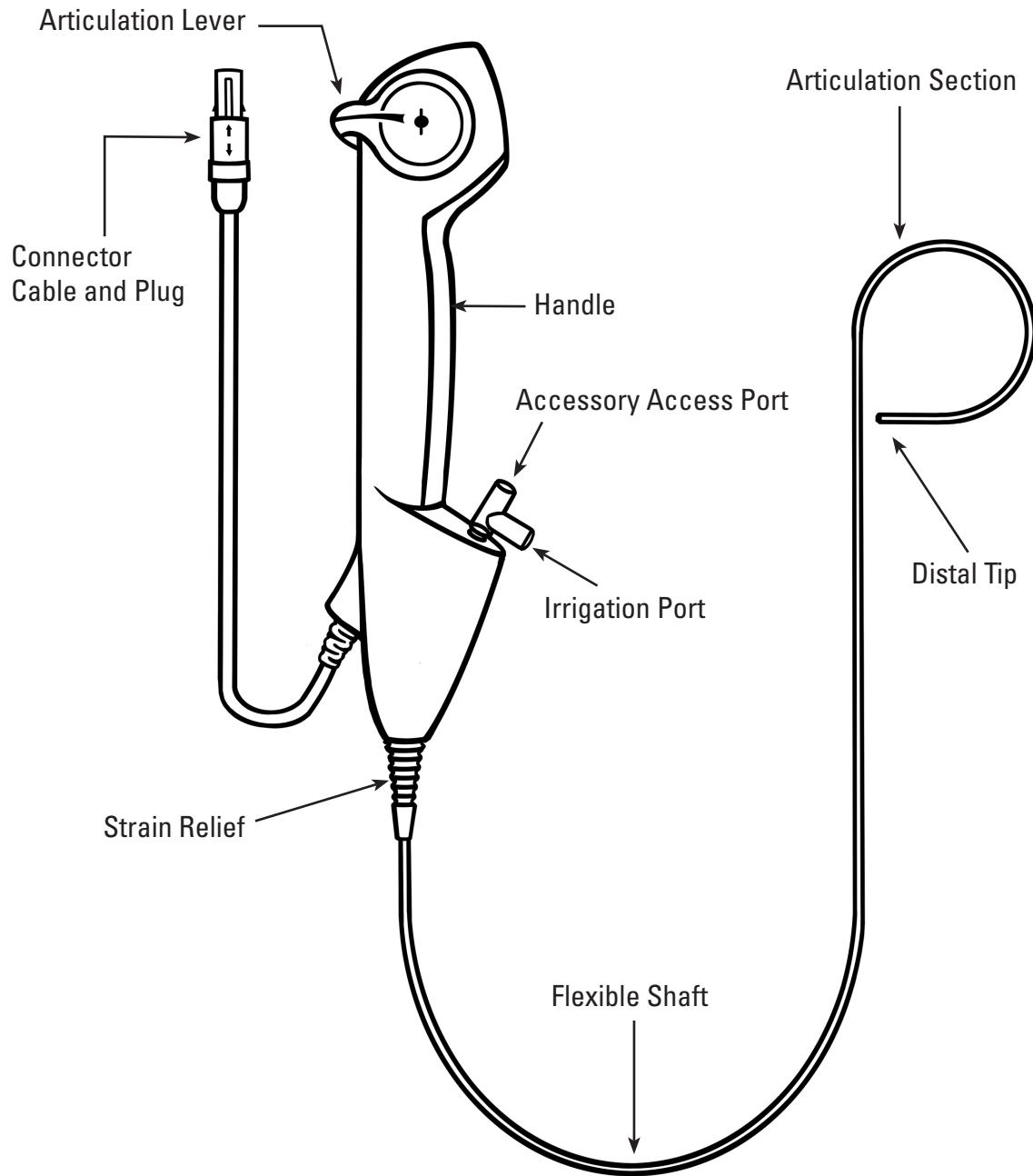

- b. Advance the guidewire slowly, observing the live video image for initial entry of the guidewire into the field of view.
- c. Stop advancing the guidewire when the stiff portion of the guidewire emerges from the distal tip and is visible on the live video image.
- d. Use fluoroscopy or other imaging modality to confirm that the LithoVue Flexscope's shaft has straightened.
- e. Slowly withdraw the LithoVue Flexscope from the patient. Monitor the withdrawal using spot fluoroscopic imaging or other imaging modality. If you feel resistance, investigate the source of resistance and take remedial action before continuing to withdraw the LithoVue Flexscope.
- f. Dispose of the LithoVue Flexscope in accordance with hospital, administrative and/or local government policy.

7.4 SYSTEM WORKSTATION POWER LOSS

CAUTION: In the event of a loss of System Workstation power, **DO NOT ADVANCE THE URETEROSCOPE OR ANY PROCEDURAL DEVICES IN THE WORKING CHANNEL WITHOUT VIEWING THE LIVE IMAGE.**

Follow these steps to recover from an electrical power loss:

1. Press the Power Button to restart the System Workstation.
2. If the System Workstation fails to power on after the Power Button is pressed, verify that the unit is plugged into the hospital's electrical outlet, that the Power Cord is plugged into the System Workstation, AND that the Power Transformer switch is turned ON (the green LED on the Power Transformer will be lit).
3. Press the Power button to restart the System Workstation.
4. If the System Workstation fails to start, have biomedical personnel verify correct functioning of the hospital's electrical outlet and restart the System Workstation.
5. If the System Workstation fails to start after the hospital's electrical outlet has been verified, contact Boston Scientific using the information found in **Section 8 "Service and Warranty"** for System Workstation service.


7.5 SYSTEM WORKSTATION BECOMES NON-OPERATIONAL


CAUTION: In the event of a system lockup, **DO NOT ADVANCE THE URETEROSCOPE OR ANY PROCEDURAL DEVICES IN THE WORKING CHANNEL WITHOUT VIEWING THE LIVE IMAGE.**

Follow these steps to recover if the System Workstation becomes non-operational:

1. Press the Power button until the System Workstation shuts down.
2. Press the Power button to restart the System Workstation.
3. If the System Workstation fails to start, contact Boston Scientific using the information found in **Section 8 "Service and Warranty"** for System Workstation service.

Table 7.1 LithoVue™ System Workstation Troubleshooting Chart

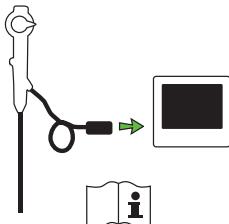User Message	Possible Cause	Corrective Action
LithoVue Flexscope Disconnected icon is displayed in video field. 	Flexscope Cable is not properly or fully inserted into the cable receptacle on the System Workstation.	Remove the LithoVue Flexscope. Ensure the connection terminals are dry and clean. Ensure the plug on the Flexscope Cable is properly aligned with the key on the Workstation Receptacle. Ensure the plug is fully inserted into the receptacle. If User Message remains after a repeated connection, replace the LithoVue Flexscope.
	LithoVue Flexscope is broken or defective.	Remove the LithoVue Flexscope. Ensure the connection terminals are dry and clean. Ensure the plug on the Flexscope Cable is properly aligned with the key on the Workstation Receptacle. Ensure the plug is fully inserted into the receptacle. If User Message remains after a repeated connection, replace the LithoVue Flexscope.
	Connection of unapproved device.	Replace unapproved device with a LithoVue Flexscope.

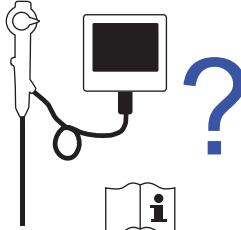User Message	Possible Cause	Corrective Action
Problem with LithoVue™ Flexscope icon is displayed in video field. 	LithoVue Flexscope malfunction, missing or corrupt calibration data.	Remove the LithoVue Flexscope. Ensure the connection terminals are dry and clean. Ensure the plug on the Flexscope Cable is properly aligned with the key on the Workstation Receptacle. Ensure the plug is fully inserted into the receptacle. If User Message remains after a repeated connection, replace the LithoVue Flexscope.
Hardware Malfunction icon is displayed in the video field.  	LithoVue Flexscope malfunction.	Remove the LithoVue Flexscope. Ensure the connection terminals are dry and clean. Ensure the plug on the Flexscope Cable is properly aligned with the key on the Workstation Receptacle. Ensure the plug is fully inserted into the receptacle. If User Message remains after a repeated connection, proceed to step below.
	System Workstation internal hardware communication failure.	Reboot the System Workstation. If the problem still exists, contact Boston Scientific for support.
LithoVue Flexscope Use Time Exceeded icon is displayed in the video field.  	LithoVue Flexscope use time exceeded.	Dispose of the LithoVue Flexscope and replace with a new one.
The System Workstation does not start when you press the power button.	Power Transformer's power switch in the Off position.	Ensure the Power Transformer's switch is turned on and the green LED is lit.
	The Power Transformer's AC power cord not properly connected to power transformer, or the AC Power Cord is not connected tightly to the hospital's electrical outlet.	Ensure the ends of the AC Power Cord are firmly connected to the Power Transformer and to a power source.
	The Power Transformer's DC Power Cord is not connected tightly to the back of the Touch PC.	Ensure the end of the DC Power Cord is firmly connected to receptacle on the Touch PC.
	No electrical power at the AC power receptacle.	Check the hospital's electrical outlet.
	Defective power transformer.	Contact Boston Scientific using the information found in Section 8 "Service and Warranty" .

User Message	Possible Cause	Corrective Action
System Workstation front panel power button is illuminated, but the boot screen does not display.	Boot problem.	Shut down the System Workstation by pressing and holding the Power Button. Press the Power Button to restart the System Workstation. If problem reoccurs, contact Boston Scientific using the information found in Section 8 "Service and Warranty" .
Video image is too dark.	Illumination light off.	Press the Light On/Off button to turn illumination on.
	Illumination is too low.	Adjust brightness using brightness control buttons. If brightness setting changes do not correct the problem, replace the LithoVue™ Flexscope.
	Debris covering the distal tip of the LithoVue Flexscope.	Clean the distal tip by wiping it with 15 to 70 percent alcohol in purified water solution using a cotton swab.
Video image is too bright.	Illumination setting is too high.	Adjust brightness using brightness control buttons.
Poor video quality.	System Workstation too close to other electrical medical equipment.	Ensure the System Workstation's location meets the Electromagnetic Compatibility Requirements discussed in Section 9 "Electromagnetic Compatibility (EMC) Requirements" . Shutdown other medical electrical equipment to determine which equipment is causing the symptom. Properly place other electrical medical devices according to their directions for use.
	Image processing issue.	Shut down the System Workstation by pressing and holding the Power Button. Press the Power Button to restart the System Workstation. If problem reoccurs, contact Boston Scientific found in Section 8 "Service and Warranty" .
	Debris covering the distal tip of the LithoVue Flexscope.	Clean the distal tip by wiping it with 15 to 70 percent alcohol in purified water solution using a cotton swab. Check irrigation and increase the irrigation flow rate or pressure. A pressure controller such as the Boston Scientific Single Action Pumping System can be used to help regulate irrigation pressures.
Problems with secondary display (poor video quality, no display)	The external video monitor or DVI switcher is not compatible or not setup properly with the LithoVue DVI output signal	Check all cabling to ensure connections are secure. Check compatibility requirements in Section 3.4 "System Workstation Device - Accessory Compatibility" . Check external monitor's user guide to setup the monitor to the DVI video standard.

If you experience any problems that cannot be resolved using the troubleshooting instructions above, contact BSC using the information found in **Section 8 "Service and Warranty"** for service.

8 Service and Warranty

8.1 LIMITED WARRANTY

Boston Scientific Corporation (BSC) warrants for one year from the date of purchase that reasonable care has been used in the design and manufacture of this product. This warranty is in lieu of and excludes all other warranties not expressly set forth herein, whether expressed or implied by operation of law or otherwise, including, but not limited to, any implied warranties of merchantability or fitness for a particular purpose. Handling, cleaning and storage of the product as well as other factors relating to the patient, diagnosis, treatment, surgical procedures, and other matters beyond Boston Scientific Corporation's control may directly affect the product and results obtained from it. Boston Scientific Corporation shall repair or replace, at its option, any part of the product that Boston Scientific Corporation determines was defective at time of shipment if notice thereof is received within one year of shipment. Boston Scientific Corporation shall not be liable for any incidental or consequential loss, damage, or expense directly or indirectly arising from the use of the product. Boston Scientific Corporation neither assumes, nor authorizes any other person to assume for it, any other or additional liability or responsibility in connection with the product. BSC assumes no liability with respect to product use by a non-qualified physician; use contrary to documentation; use with a non-LithoVue catheter. Buyer shall be responsible for the ongoing support and maintenance of the product not covered by this one-year warranty and after the one year warranty period has expired. Buyer may, at its sole cost and expense, purchase an extended warranty from Boston Scientific Corporation (BSC) to extend the term of this warranty.

8.2 OBTAINING WARRANTY SERVICE FROM BOSTON SCIENTIFIC CORPORATION

Contact the Customer Service Department at Boston Scientific Corporation at 800-949-6708 to report any problem with the System Workstation and obtain a return authorization number, if required.

Return the System Workstation to Boston Scientific Corporation. All shipments to Boston Scientific Corporation must be insured and safely and securely packaged, preferably in the original shipping carton, and should include a letter explaining the problem. Reference the return authorization number.

All transportation and insurance charges and risk of loss are the responsibility of the customer and must be prepaid. A purchase order must be issued to Boston Scientific Corporation to cover all transportation and insurance charges for return shipment after service.

You will receive a return goods authorization (RGA) tracking number for the returning product. Write the RGA number on the outside of the return packaging.

9 Electromagnetic Compatibility (EMC) Requirements

9.1 GUIDANCE AND MANUFACTURER'S DECLARATION-ELECTROMAGNETIC EMISSIONS

The LithoVue™ System is intended for use in the electromagnetic environment specified below (**Table 9.1**). You should ensure the operating environment meets these requirements before using the System Workstation.

Table 9.1 Acceptable environment for operation of the System Workstation

Emissions Test	Compliance	Electromagnetic Environment—Guidance
RF Emissions CISPR 11	Group 1	The LithoVue™ System uses radio frequency (RF) energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF Emissions CISPR 11	Class A	
Harmonic Emissions IEC 61000-3-2	Class A	
Voltage Fluctuations/Flicker Emissions IEC 61000-3-3	Complies	The LithoVue System is suitable for use in all establishments other than domestic and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.

9.2 GUIDANCE AND MANUFACTURER'S DECLARATION – IMMUNITY ALL EQUIPMENT AND SYSTEMS

The LithoVue System is intended for use in the electromagnetic environment specified below (**Table 9.2**). You should ensure the operating environment meets these requirements before using the System Workstation.

Table 9.2 Acceptable environment for operation of the System Workstation

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment (Guidance)
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV for power supply lines ± 1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	< 5 % U_T (> 95% dip in U_T) for 0.5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles < 5% U_T (> 95% dip in U_T) for 5 sec	< 5 % U_T (> 95% dip in U_T) for 0.5 cycle 40% U_T (60% dip in U_T) for 5 cycles 70% U_T (30% dip in U_T) for 25 cycles < 5% U_T (> 95% dip in U_T) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the LithoVue System requires continued operation during power mains interruptions, it is recommended that the LithoVue System Workstation be powered from an uninterruptible power supply or battery.

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment (Guidance)
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Note – UT is the AC mains voltage prior to application of the test level.			

9.3 GUIDANCE AND MANUFACTURER'S DECLARATION – EMISSIONS AND EQUIPMENT AND SYSTEMS THAT ARE NOT LIFE-SUPPORTING

The LithoVue™ System is intended for use in the electromagnetic environment specified below (**Table 9.3**). You should ensure the operating environment meets these requirements before using the System Workstation.

Table 9.3 Guidance and manufacturer's declaration – emissions and equipment and systems that are not life-supporting

Immunity Test	IEC 60601 Test Level	Compliance Level	Electromagnetic Environment (Guidance)
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the LithoVue System, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	<p>Recommended separation distance $d=1.2\sqrt{P}$</p> <p>$d=1.2\sqrt{P}$ 80 MHz to 800 MHz $d=2.3\sqrt{P}$ 800 MHz to 2.5 GHz</p> <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range^b.</p> <p>Interference may occur in the vicinity of equipment marked with the following Symbol:</p> 
Note 1 – At 80 MHz and 800 MHz, the higher frequency range applies.			
Note 2 – These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

- a)** Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the LithoVue System is used exceeds the applicable RF compliance level above, the LithoVue System should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the LithoVue System.

- b)** Over the frequency range 150 kHz to 80 MHz, the field strengths should be less than 3 V/m.

9.4 RECOMMENDED SEPARATION DISTANCE OF OTHER EQUIPMENT

The LithoVue™ System is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. You should help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the LithoVue System as recommended below, according to the maximum output power of the communications equipment (**Table 9.4**).

Table 9.4 Recommended separation distances between an operating LithoVue System and portable and mobile RF communications equipment

Rated Max Output Power of Transmitter (Watts)	Separation (m) 150 kHz to 80 MHz	Separation (m) 80 MHz to 800 MHz	Separation (m) 800 MHz to 2.5 GHz
	$D=(3.5/V1) \sqrt{P}$	$D=(3.5/E1) \sqrt{P}$	$D=(7/E1) \sqrt{P}$
0.01	.12	.12	.23
0.1	.38	.38	.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance (d) in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power of the transmitter in watts (W) according to the manufacturer.

Note 1 – At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2 – These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Description of Symbols

U	LithoVue Flexscope Tip Movement Up	— — —	Direct current IEC 60878-5031
D	LithoVue Flexscope Tip Movement Down	ETL CLASSIFIED  Intertek 3161809	Medical Electrical Equipment and Devices
	System power On/Off IEC 60878-5009		

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© 2015 Boston Scientific Corporation or its affiliates. All rights reserved.
Boston Scientific (Australia) Pty Ltd
Boston Scientific Tip Gereçleri Ltd. Şti.