AMS™ Quick Connect

Sutureless Window Connectors

Instructions for Use

English **AMS™ Quick Connect**
Sutureless Window Connectors
Instructions for Use..... 3

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

Sutureless Window Connectors

Instructions for Use


AMS Quick Connect Sutureless Window Connector Assembly Instructions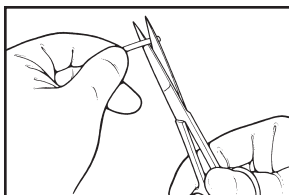


1. Trim tubing squarely.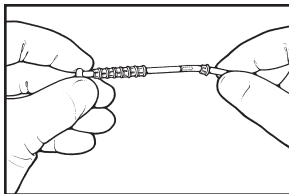


2. Attach collet rings.
Place one ring on end of each tubing.

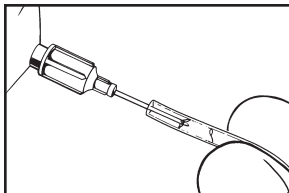
3. Flush.
Flush ends of both tubing using appropriate solution.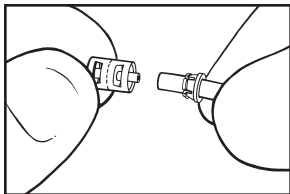


- 4.** Insert first tubing.
Firmly push tubing to middle wall of connector as seen through the connector window.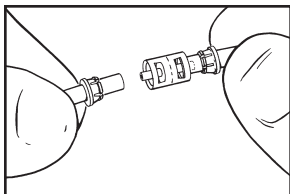


- 5.** Connect.
Firmly push second tubing to middle wall. Check windows to make certain both tubing ends are still touching the middle wall.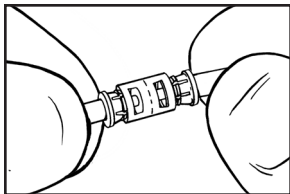


- 6.** Approximate the collets against the connector prior to applying the assembly tool.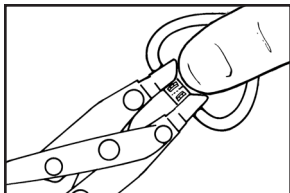

- 7.** Do not hold the connection in the assembly tool. This may contribute to the collet misaligning with the connector and result in collet prong(s) missing the connection.

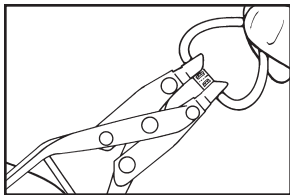
8a. Assure that the connection is inserted completely into the assembly tool. Slowly squeeze the handle, closing the tool. If a prong of the collet does not enter into the connector prior to completing the connection, release the pressure on the connection and allow the collet to realign with the connector. Complete the connection. The closure stop must touch the opposite handle.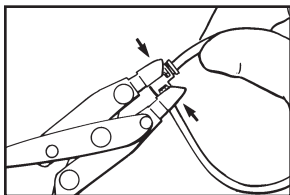


8b. When using a right angle connector, the assembly tool must be used twice—once on each end of the connector. Again, make certain the tubing is touching the middle wall on both sides of the connector. Closure stop of assembly tool must touch opposite handle each time.

CAUTION: Before you close the assembly tool, check the tubing. The tubing must not be trapped between the assembly tool jaw and the connector. The tubes must exit straight from the ends of the connector, through the slots in the assembly tool.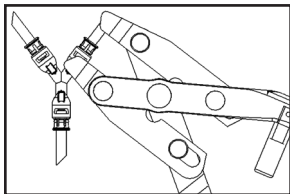


8c. When using a 3-way connector, the assembly tool must be used 3 times—once on each end of the connector. The assembly tool must be engaged *from the side* of each connector end. Advance the tubing into each connector end until the tubing touches the inner connector wall as seen through the connector window.


CAUTION: The long branch of the 3-way connector must be connected to the control pump.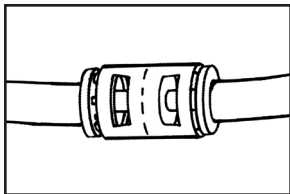

9. Check.
After completing the connection, examine both ends of the connection thoroughly. If prong(s) of the collet do not enter the connector and are protruding, do not close. Remake the connection by cutting out the connection and making a completely new connection.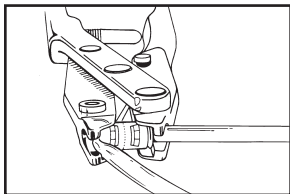

Incorrect
Perforates tubing.

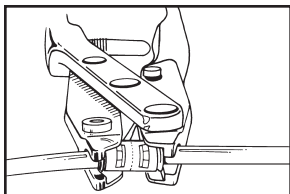Correct
Tubes exit straight from the ends of the connector. After using the AMS Quick Connect™ Assembly Tool, the tubing should partially bulge through the window of the connector. This indicates that the tubing is still firmly against the middle wall of the connector.

The AMS Quick Connect System cannot be resterilized. Conventional hospital sterilization will damage the connector components. (The assembly tool can be resterilized.)

NOTE: AMS Quick Connect Sutureless Window Connectors should not be used on revision tubing.

How Supplied and Storage

WARNING: Contents supplied STERILE. Do not use if sterile barrier is damaged. If damage is found, call your AMS representative.

For single patient use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient. After use, dispose of product and packaging in accordance with hospital, administrative and/or local government policy.

Store product in a clean, dry, dark area at room temperature.

This page intentionally left blank.
Diese Seite wurde absichtlich freigelassen.
Esta página se deja en blanco intencionadamente.
Page laissée vierge intentionnellement.
Questa pagina è stata lasciata intenzionalmente vuota.
Esta página foi intencionalmente deixada em branco.

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All rights reserved. All trademarks are property of
Boston Scientific (Australia) Pty Ltd
American Medical Systems, Inc.
Minnetonka, MN 55343 U.S.A.
American Medical Systems Europe B.V.
©2016 Boston Scientific Corporation or its affiliates.