



50736129-01

2018-09  
< en >

## Sterile Lubricating Jelly

### WARNING

Contents supplied STERILE. Do not use if sterile barrier is damaged. If damage is found, call your Boston Scientific representative.

For single use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

After use, dispose of product and packaging in accordance with hospital, administrative and/or local government policy.

### DEVICE DESCRIPTION

Lubricating Jelly is water soluble, greaseless, and unscented.

#### Ingredients

Water, propylene glycol, hydroxypropyl methylcellulose, carbomer 934P, methylparaben, propylparaben, sodium hydroxide.

### INTENDED USE/ INDICATIONS FOR USE

Lubricating Jelly is a sterile, water-soluble, latex-free gel intended for use on intact skin, on mucous membranes and in natural body orifices.

#### User Information

For use by a professional in an endoscopy suite.

#### Contents

Provided in a 56.7g (2 oz.) tube.



### WARNING

May cause eye irritation. Avoid contact with eyes. After eye contact, flush eyes with plenty of 1% physiological saline solution. If no saline is available, flush with copious amounts of water.

### CONTRAINDICATIONS

Sensitivity to ingredients listed in Device Description.

### ADVERSE EVENTS

Irritation or allergic reaction to ingredients.

### HOW SUPPLIED

Do not use if package is opened or damaged.

Do not use if labeling is incomplete or illegible.


#### Handling and Storage

Store in a cool, dry, dark place.


### OPERATIONAL INSTRUCTIONS

Sanitize tube tip and cap prior to opening. Remove cap, remove foil seal and replace cap prior to use.

**REF** Catalog Number

 Consult instructions for use.

 Contents

 Legal Manufacturer

**LOT** Lot


 Recyclable Package

 Use By


**AUS** Australian Sponsor Address

**ARG** Argentina Local Contact

**BRA** Brazil Local Contact

 For single use only. Do not reuse.

 Do Not Restерilize

 Do not use if package is damaged.

**STERILE** Sterile

**ARG** **Argentina  
Local Contact**

Para obtener información de contacto de Boston Scientific Argentina SA, por favor, acceda al link [www.bostonscientific.com/arg](http://www.bostonscientific.com/arg)

**AUS** **Australian  
Sponsor Address**

Boston Scientific (Australia) Pty Ltd  
PO Box 332  
BOTANY  
NSW 1455  
Australia  
Free Phone 1800 676 133  
Free Fax 1800 836 666

**BRA** **Brazil  
Local Contact**

Para informações de contato da Boston Scientific do Brasil Ltda, por favor, acesse o link [www.bostonscientific.com/bra](http://www.bostonscientific.com/bra)

 **Legal  
Manufacturer**

HR Pharmaceuticals, Inc.  
2600 Eastern Blvd., Suite 201  
York, PA 17402

**DISTRIBUTED BY:**  
Boston Scientific Corporation  
300 Boston Scientific Way  
Marlborough MA 01752 USA  
USA Customer Service 888-272-1001

 **Do not use if package  
is damaged.**

 **Recyclable  
Package**

© 2018 HR Pharmaceuticals Inc. All rights reserved.