

**Implantable Materials Kit**

*Model SC-4600*

**Description**

The Boston Scientific Implantable Materials Kit contains representative sample pieces of the implantable components that have direct patient contact and are included in the Precision<sup>®</sup> System.

Material	Letter Code	Principal Component Where Used
Titanium, Ti-6-4	T6	IPG Case
Stainless Steel, 316L	SS	IPG Connector Block
MP35N Alloy	MP	IPG Port Plug
Platinum-Iridium	PI	Distal Contacts on the Percutaneous Lead
Tecothane <sup>®</sup>	TC	IPG Port Plug
Hysol <sup>®</sup> Epoxy	--	IPG Header
Pellathane <sup>®</sup>	PL	Percutaneous Lead Body
MED-4860	60	Silicone Suture Sleeves
Medical Adhesive	--	Inside Silicone Suture Sleeve ( <i>must be ordered separately</i> )

**NOTE:**

- *Pure Titanium and pure Platinum are not included in the Implantable Materials Kit because alloys containing these pure elements are included in the kit.*
- *Only one silicone grade is included in the Implantable Materials Kit and is representative of the following silicone grades NOT included in this kit: MED-4224, MED-4780, MED-4850, and MED-4870.*
- *If you are planning on using Medical Adhesive in the implant, please order Medical Adhesive, SC-4320, from your Boston Scientific representative since it is not included in this kit.*

**Indications**

The Implantable Materials Kit is intended for use by medical professionals only.

**Contraindications**

The materials of the Implantable Materials Kit are **NOT STERILE** and are **NOT** intended for implantation.

**WARNINGS:**

- Carefully examine all product packaging before use. If the product packaging is damaged, ensure that the materials are clean and intact before use.
- The materials in this package are **NOT STERILE**.
- Do **NOT** ingest the materials in this package. They are **NOT** for oral consumption.
- Boston Scientific assumes no responsibility for results of any testing obtained from the use of this Implantable Materials Kit.

**Directions for Use**

For any testing associated with the materials included in the Implantable Materials Kit, please seek the counsel of a qualified medical professional.

**Customer Service - USA**

In USA, to order replacement parts, or for answers to any questions you might have, please contact the Customer Service Department:

- US Toll Free Phone: (866) 360-4747
- Phone: (661) 949-4747, Fax: (661) 949-4022

Boston Scientific Neuromodulation  
25155 Rye Canyon Loop  
Valencia, CA, 91355, USA

