

# LATITUDE<sup>®</sup>

Patient Management System



*Freedom to do more*

**Boston  
Scientific**  
*Delivering what's next.™*

## FOCUS

LATITUDE® gives you control and focus for patients who most need your attention.



## TRUST

The LATITUDE® system incorporates the most reliable patient monitoring technology.



## FREEDOM TO DO MORE

The best possible outcomes for you and your patient.

# LATITUDE® PATIENT MANAGEMENT SYSTEM GIVES YOU ...

## ... FREEDOM TO DO MORE

LATITUDE® Patient Management System offers **safe and convenient** remote patient management for patients with Boston Scientific's COGNIS® CRT-D and TELIGEN® ICD implants.

It gives you **greater control to more effectively manage your patient's condition, optimizing care** and reducing hospital appointments.

Patients may have to attend **fewer hospital visits** and can enjoy life reassured that their cardiac condition is being regularly monitored by their physician.

*You can trust the LATITUDE® Patient Management System to help you focus on the essentials.*



# FOCUS: ON TECHNOLOGICAL INNOVATION WITH ...




## ... A USER-FRIENDLY INTERFACE

LATITUDE® is a complete service with a user-friendly interface which will be instantly familiar to physicians used to COGNIS® CRT-D and TELIGEN® ICD devices.


**Confidential patient data can be accessed anywhere and at any time on our access protected website**, which is supported by a totally secured and backed-up European server.

*The user-friendly interface gives you freedom to do more...*

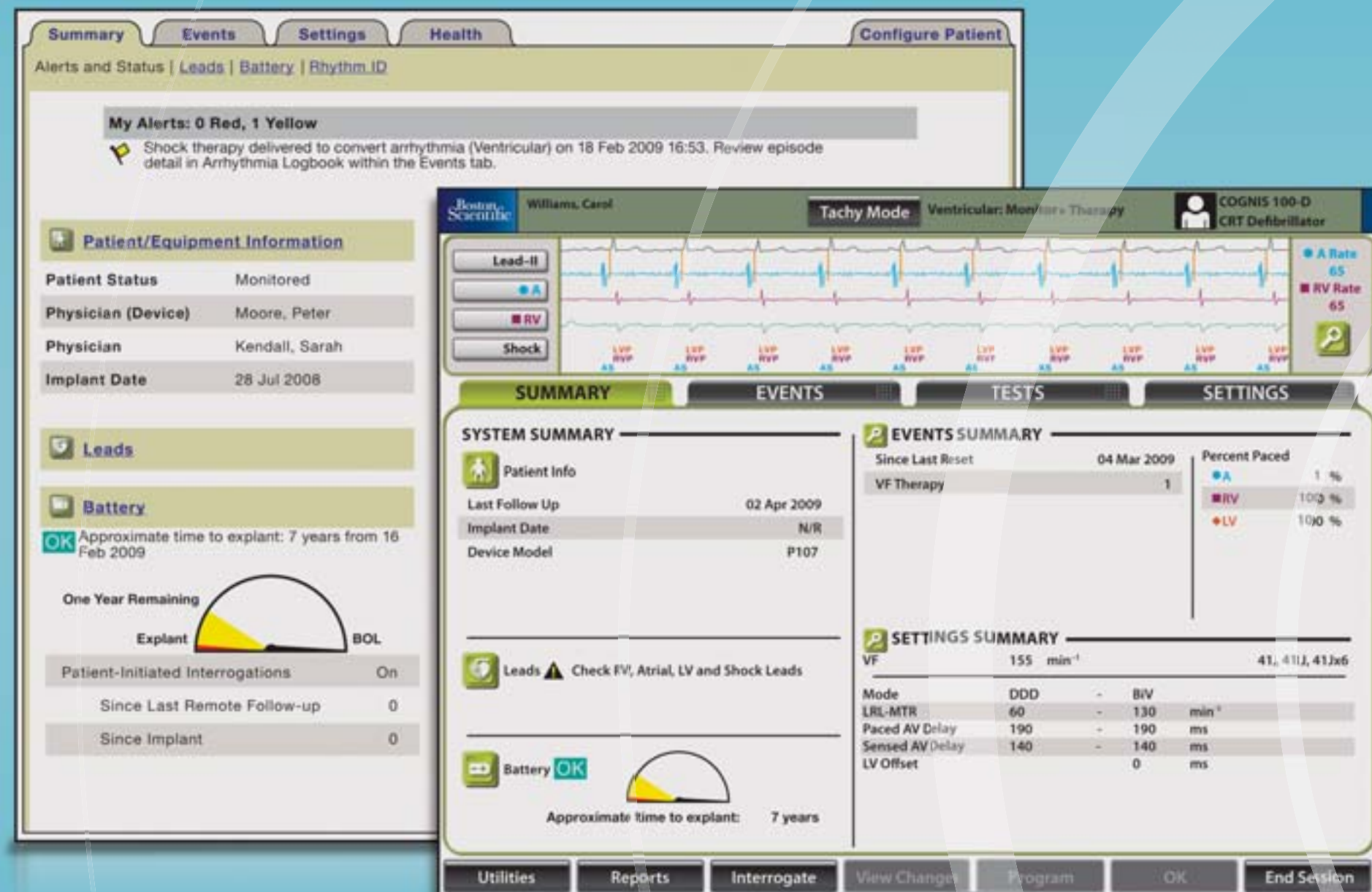



# FOCUS: ON YOUR PATIENT'S NEEDS WITH ...

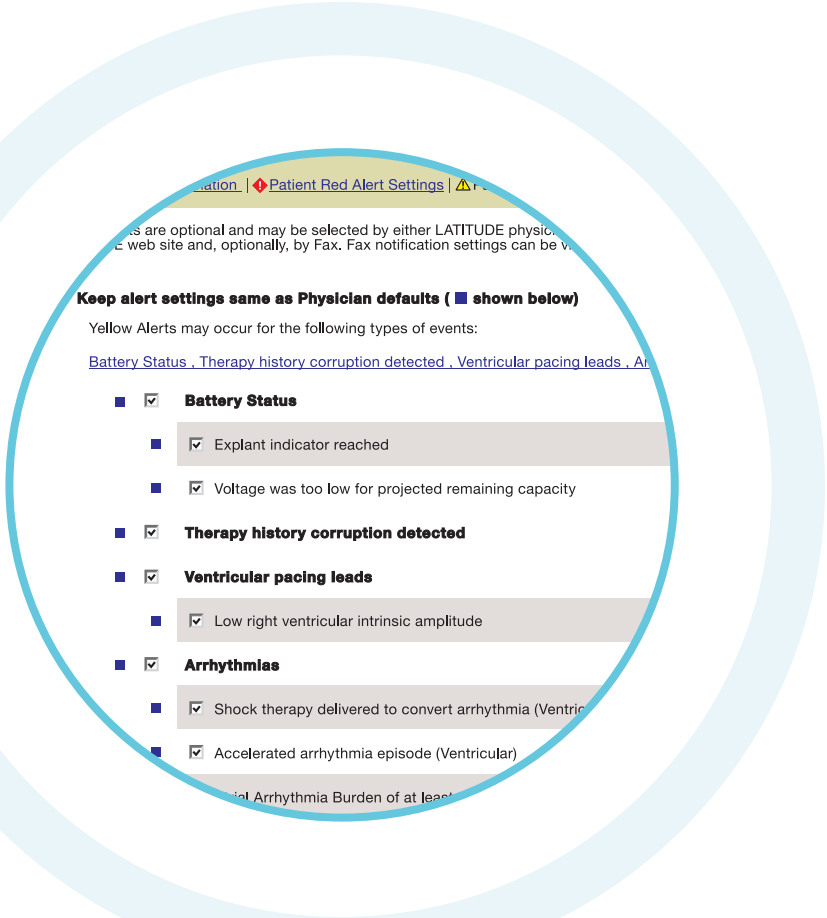

## ... INFORMATION AT A GLANCE

The LATITUDE® communicator transmits a clear and comprehensive set of data, providing two levels of alerts.

A **red alert** indicates **urgent** conditions, examples of which are end of or low battery life, high or low shock lead impedance and possible device malfunction.

**Yellow alerts** are **optional** and you can select the ones you would like to see. They include various indicators, such as explant indicator reached, arrhythmias or weight change for each individual patient.

In addition you can **re-configure yellow alerts** according to each patient's clinical indications **without bringing your patient into the hospital.**

Therefore you can focus your attention on patients who need it most.

*Information that gives you freedom to do more...*



# FOCUS: ON EFFECTIVE TIME MANAGEMENT WITH ...



## ... REMOTE PATIENT FOLLOW-UP

You can establish a **recurring follow-up** schedule according to the **patient's needs** and **hospital protocol**, while maintaining freedom to accommodate changes in the patient's condition, **potentially reducing in-hospital follow-ups**.

LATITUDE® is compatible with the industry Health Level 7 data standard.

## ... REMOTE PATIENT MONITORING

LATITUDE® offers all the advantages of a remote patient management system, in particular the ability to **monitor patients between scheduled follow-ups** to detect changes earlier and to take action to positively impact clinical outcomes.

*Remote patient follow-up and remote monitoring gives you freedom to do more...*

LATITUDE® Patient Management

PATIENT SEARCH HELP/CONTACT US LOG OUT

**Boston Scientific**

Patient Detail  
Mitchell, Maria (Moore, Peter)

Logged In: Hill, George  
02 Mar 2009  
Language: English (UK) Edit

PATIENTS FOR REVIEW ALL PATIENTS PATIENT UTILITIES CONFIGURE

Reports Menu Dismiss Patient

Patient: Mitchell, Maria  
Patient ID: 120919970  
Device: TELIGEN 100 F110/219007 | Monitor + Therapy  
Last Office Interrogation: 01 Dec 2008 12:23  
Latest Send: 23 Feb 2009 04:26

Date of Birth: 21 Mar 1976

Summary Events Settings **Configure Patient**

Patient/Equipment Information | **Patient Red Alert Settings** | Patient Yellow Alert Settings | Patient Schedule

Save Reset

**Remote follow-up schedule**  
(How often a patient will appear on the Patients for Review List)


☒ Keep settings same as Physician defaults: Remote follow-up once every three months on Monday

Schedule remote follow-ups: **Once Every Three Months** on: **Monday**

The last remote follow-up was: 08 Sep 2008  
The next remote follow-up is: 18 May 2009

Note: Changes to schedule can take up to 8 days to take effect in patient's home.

# FOCUS: ON INFORMATION TO OPTIMIZE ...

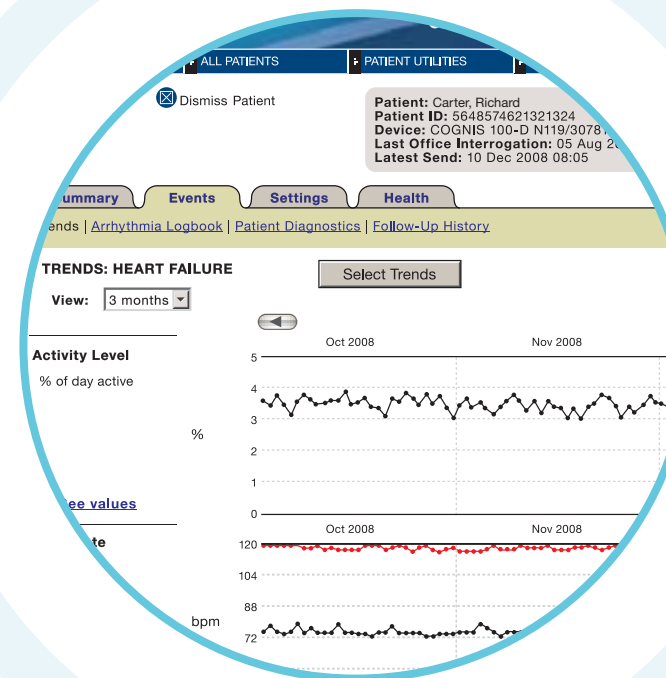

## ... CLINICAL OUTCOMES

LATITUDE® offers two unique external sensors to optimize the monitoring of heart failure patients.

The **wireless weight scale** and the **blood pressure monitor** enable patients to send you daily information, providing an early indication of changes in their condition.

LATITUDE® is the only patient monitoring system that meets **ESC Class 1 recommendations** for heart failure patients to monitor their weight and blood pressure daily. A set of 'quality of life' questions could give further indication of any changes in their condition.

Constant focus optimizes patient outcomes.

*Enhanced clinical outcomes give you freedom to do more...*





# TRUST: IN SUPPORT AND TECHNOLOGY...



## SUPPORT AND SERVICE

Your hospital staff will benefit from **onsite product and service training** by qualified Boston Scientific experts.

Be reassured that, should you need help, the **LATITUDE® customer support's team of experts** is **only a telephone call away**. They can offer immediate assistance on a range of subjects, including system set-up, alert configuration and device interrogation options.

## ALERTS


**You can be confident that LATITUDE® will alert you to any urgent clinical event. Red alerts** will always appear on the secure website.

## PATIENT INITIATED INTERROGATION

**With your permission**, the communication can also be **initiated by the patient**, who simply has to push a button on the communicator to begin the interrogation process. This gives your patients the peace of mind of being in contact with you when needed.

*Boston Scientific's support and service give you freedom to do more...*

# TRUST: IN A DESIGN THAT INSPIRES CONFIDENCE...

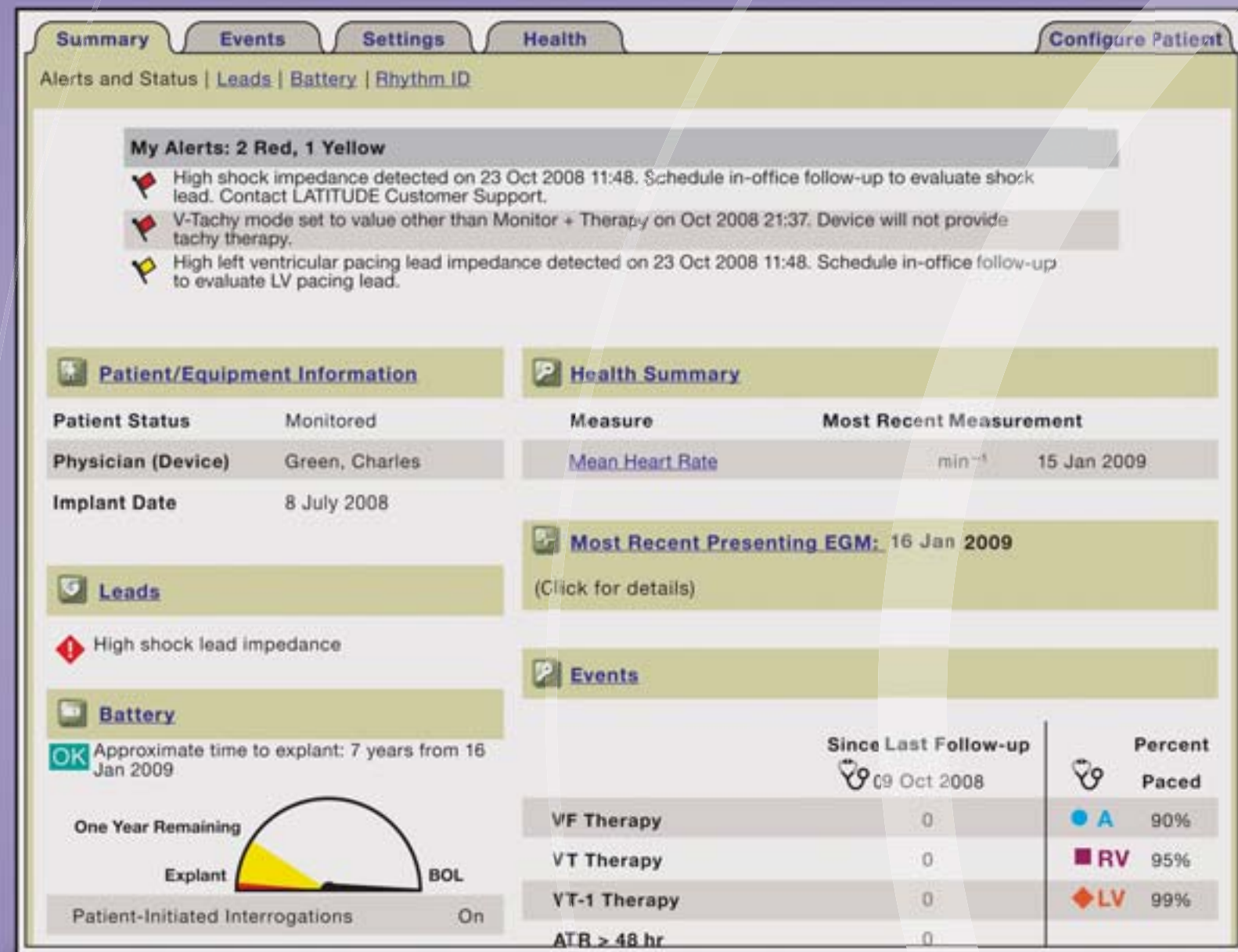

## LONGEVITY

Feel secure with a system which, as it works in conjunction with COGNIS® CRT-D and TELIGEN® ICD implanted devices, benefits from a best-in-class warranty and ensures high energy, stable charge time and longer life. In addition LATITUDE® offers the potential to **preserve longevity by communicator-initiated interrogation.**

## ZIP™ RF TELEMETRY

Thanks to reliable ZIP™ RF TELEMETRY Boston Scientific can offer the **smallest and thinnest high energy implantable ICD and CRT-D devices.**

It also transmits data faster than comparable systems which gives the potential for lower battery consumption and, therefore, longer life.

*Preserved longevity gives you freedom to do more...*



## WE ARE HERE TO HELP

Our multilingual support team is only a toll-free telephone call away. For help please email [latitude.international@bsci.com](mailto:latitude.international@bsci.com) or call your local number:

### AUSTRIA

☎ 0800 202289 📠 01206091097

### BELGIUM

☎ 0800 80697 📠 022006115

### DENMARK

☎ 70100182 📠 43682205

### FINLAND

☎ 010804819 📠 0972519245

### FRANCE

☎ 0811640071 📠 0157323452

### GERMANY

☎ 06951709481 📠 06951709485

### ITALY

☎ 848 781164 📠 0291483117

### NETHERLANDS

☎ 0800 0292077 📠 0202061368

### NORWAY

☎ 81000047 📠 24159841

### PORTUGAL

☎ 800844729 📠 0217616252

### SPAIN

☎ 901010840 📠 912754448

### SWEDEN

☎ 0201605707 📠 0851761653

### SWITZERLAND

☎ 0844 000110 📠 0445118205

### UNITED KINGDOM

☎ 0845 602 9283 📠 02030273478

MODEL NUMBERS	DESCRIPTION
<b>6468</b>	<b>LATITUDE® Communicator</b>
6417	LATITUDE® Weight Scale*
6483	LATITUDE® Blood Pressure Monitor*
6484	LATITUDE® Blood Pressure Cuff*† Small Medium Large
6485	
6486	

\*Optional †The LATITUDE® Blood Pressure Cuff has to be ordered separately.

All cited trademarks are the property of their respective owners. CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labelling supplied with each device. Information for the use only in countries with applicable health authority product registrations. All names are fictional and are used for demonstration purposes only.

Printed in the UK by Gosling.



**Boston  
Scientific**

*Delivering what's next.™*

[www.bostonscientific-international.com](http://www.bostonscientific-international.com)

© 2009 Boston Scientific Corporation  
or its affiliates. All rights reserved.  
DINCRM0362EA