



EndoVive™ Y-Port Feeding Adapter

Initial Gastrostomy Tube Y-Port Adapter

**Boston
Scientific**
Advancing science for life™

Refer to the device directions for use for complete instructions on device use.

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

Device Description

The EndoVive Y-Port Feeding Adapter is a disposable device used to attach a pump, syringe or gravity enteral feeding device to a gastrostomy tube for direct feeding.

The catheter is Magnetic Resonance (MR) Safe.

Precautions

Do not use this product without reading and understanding the complete instructions enclosed herein.

Caution: Carefully examine the unit to verify that the contents have not been damaged in shipment. DO NOT USE if damaged. Immediately return damaged product to Boston Scientific Corporation.

Directions

- Firmly push the feeding adapter into feeding tube. Verify that the feeding adapter is fully inserted.
- Attach pump, syringe or gravity enteral feeding device.
- Prior to discharge, instruct the patient on device cleaning, monitoring and medication administration (when applicable). Ensure that the patient is provided with a Patient Care guide for additional device monitoring instructions.
- Re-attach cap after use.

How Supplied

The EndoVive Y-Port Feeding Adapter is separately packaged as one (1) unit. The device is non-sterile. Do not use if package is opened or damaged. Do not use if labeling is incomplete or illegible.

Handling and Storage: Store in a cool, dry, dark place. Do not expose to organic solvents, ionizing radiation or ultraviolet light. Rotate inventory so that products are used prior to the sterilization expiration date on package label.

Warranty

Boston Scientific Corporation (BSC) warrants that reasonable care has been used in the design and manufacture of this instrument. This warranty is in lieu of and excludes all other warranties not expressly set forth herein, whether express or implied by operation of law or otherwise, including, but not limited to, any implied warranties of merchantability or fitness for a particular purpose. Handling, storage, cleaning and sterilization of this instrument as well as other factors relating to the patient, diagnosis, treatment, surgical procedures and other matters beyond BSC's control directly affect the instrument and the results obtained from its use. BSC's obligation under this warranty is limited to the repair or replacement of this instrument and BSC shall not be liable for any incidental or consequential loss, damage or expense directly or indirectly arising from the use of this instrument. BSC neither assumes, nor authorizes any other person to assume for it, any other or additional liability or responsibility in connection with this instrument. BSC assumes no liability with respect to instruments reused, reprocessed or sterilized and makes no warranties, express or implied, including but not limited to merchantability or fitness for a particular purpose, with respect to such instruments.

All trademarks are the property of their respective owners.

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

Indications, contraindications, warnings and instructions for use can be found in the product labeling supplied with each device.

2023 Copyright © Boston Scientific Corporation or its affiliates. All rights reserved.

ENDO-1554705-AA