

# AdVance™ Male Sling System

## Patient selection criteria<sup>1</sup>

### Six (6) patient selection criteria for AdVance Sling (post-prostatectomy incontinence):

- ▶ **1** <4 pads/day or <300g daily pad weight
- ▶ **2** Intact appearing external urinary sphincter without segmental defects on cystoscopy; ability to volitionally contract their external urinary sphincter
- ▶ **3** No history of pelvic radiation or cryotherapy
- ▶ **4** No history of previous surgical procedures to treat incontinence
- ▶ **5** Volitional detrusor contraction when voiding
- ▶ **6** Post-void residual (PVR) urine volume <100 mL

	Patient meets all 6 criteria	Patient does not meet all 6 criteria	P value
<b>N=95</b>	72	23	
Mean daily preoperative <b>pad use</b> (range)	2.6 (1-4)	4.4 (1-9)	<.05
Mean daily preoperative <b>pad weight, g</b> (range)	131 (10-280)	520 (80-1200)	<.05
Mean daily postoperative <b>pad use</b> (range)	<b>0.6</b> (0-6)	2.4 (0-7)	<.05
Mean daily postoperative <b>pad weight, g</b> (range)	<b>16</b> (0-310)	201 (0-800)	<.05
Pad reduction percentage	<b>77%</b>	46%	
Pad weight reduction percentage	<b>88%</b>	61%	
Patient satisfaction	<b>92%</b>	30%	

Mean follow-up: 28 months

1. Sturm RM, Guralnick ML, Stone AR, et al. Comparison of clinical outcomes between "ideal" and "nonideal" transobturator male sling patients for treatment of postprostatectomy incontinence. *Urology*. 2014 May;83(5):1186-8.

Prior to use, please review the Instructions for Use for a complete listing of indications, contraindications, warnings, precautions and potential adverse events.

CAUTION: Federal Law restricts this device to sale by or on the order of a physician. This product is only intended for physicians trained in its use.

INDICATIONS: The AdVance™ male sling system is intended for the placement of a suburethral sling for the treatment of male stress urinary incontinence (SUI).

CONTRAINDICATIONS: Patients with urinary tract infections, blood coagulation disorders, a compromised immune system or any other condition that would compromise healing, with renal insufficiency, and upper urinary tract relative obstruction.

POTENTIAL ADVERSE EVENTS: Extrusion, erosion through the urethra or other surrounding tissue, migration of the device from the desired location, fistula formation and inflammation. Rx Only



Boston Scientific Corporation  
300 Boston Scientific Way, Marlborough, MA 01752  
www.amsmenshealth.com  
All trademarks are the property of their respective owners.  
©2016 Boston Scientific Corporation or its affiliates. All rights reserved.  
MH-393613-AA MAY 2016