FAQ

Living With Your Implanted Device

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
Medical Professionals:
1.800.CARDIAC (227.3422)
Patients and Families:
1.866.484.3268

© 2011 by Boston Scientific Corporation
or its affiliates. All rights reserved.

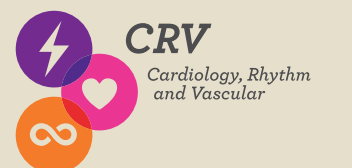CRM9-3675-0411

Frequently Asked Questions: Living With Your Device

Recovery and Follow-Up

Q.How do most people feel emotionally after receiving an implanted device?

A. Having an implanted device can be a major turning point in your life. Afterward, you may feel sad or worried. It's an event that may change your thoughts and feelings about your health. Some people may feel vulnerable at first because they depend on an implanted device.

As you return to daily life, your level of confidence and comfort with the implanted device will likely grow. Talk to your doctor if you continue to feel anxious, scared, or depressed. You might want to talk to a therapist or find a support group.

Q.Now that I have an implantable device, do I need to have regular appointments to have it checked?

A. Yes. Your doctor will want to check your device and your overall health on a regular basis. With certain Boston Scientific devices, your doctor may be able to monitor your device remotely while you are at home.

Remote monitoring can be scheduled to replace a clinic visit. Remote follow-up can be done daily or weekly, depending on your doctor's settings, which allows information to come to the clinic between device follow-up times.

- Some pacemakers may be checked with transtelephonic monitoring (TTM). Your doctor must give authorization for you to participate in TTM. Patients with pacemakers from certain manufacturers may use remote monitoring.
- Patients with ICD or CRT defibrillators may use remote monitoring.

Ask your doctor for more information. Your doctor's office will have records of all your follow-up appointments, whether they are done in-person or remotely. These records include your device settings and pertinent information reported from your device's memory.

Your Lifestyle

Q.Are there items I need to keep a certain distance from, or not use?

A. Yes. Some appliances and pieces of equipment give off electromagnetic signals at close range. The influence of these signals is called electromagnetic interference (EMI). Pacemakers, implantable cardioverter defibrillators (ICDs), and cardiac resynchronization therapy (CRT) devices can sometimes detect and respond to these signals. Check with your doctor if you have questions about specific pieces of equipment.

Q.What does EMI do to my device? Is there any type of shield I can use to keep my device safe?

A. An ICD or CRT-D may respond to EMI from some appliances and pieces of equipment by delivering an unnecessary shock or withholding a necessary shock. A pacemaker may withhold necessary pacing. This response to EMI is usually temporary. It will not usually harm your device. There is no way to shield your device from all of the signals. For more information about EMI, refer to your patient handbook, talk to your doctor, or call Boston Scientific Patient Services at 1.866.484.3268 and select the option for technical questions.

Q.When can I start driving again?

A. Your doctor will talk with you about driving restrictions. Decisions about driving are made on an individual basis and in compliance with the laws in your state.

Your Device's Operation

Q.What should I do if I receive a shock from my ICD or CRT-D?

A. If you receive a shock and do not feel OK afterward, or if you get several shocks in quick succession, seek immediate medical attention. Make a plan in advance with your doctor about what to do, or whom to call, if you receive a shock from your device.

Q.Do devices make beeping tones? What should I do if I hear beeping?

A. *Pacemakers and CRT-P devices:* These devices do not make beeping tones.

ICDs and CRT-D devices: These devices can make beeping tones when there is a condition that needs to be checked by a doctor. The device will beep sixteen times in a row and repeat this sequence every 6 hours.

Some strong magnetic fields may also cause an ICD or CRT-D device to make beeping tones. If you hear beeping tones, immediately move away from the object that may be causing the beeping.

Always contact your doctor if you hear beeping.

Q.Does my ICD or CRT-D need to be turned off for certain medical procedures? How do I know which ones and how do I arrange this?

A. Certain procedures may require deactivation of your ICD or CRT-D. Tell the doctor who will be ordering or performing the test that you have an ICD or CRT-D. The doctor can then contact Boston Scientific Technical Services for specific recommendations.

Q.Does an ICD or CRT-D device also have a pacemaker?

A. Yes. All ICDs and CRT-D devices also have a pacemaker inside. They have the full capability of a pacemaker in addition to the functions of the defibrillator in the device.

Batteries and Device Replacement

Q.What conditions cause my device's battery to lose energy? How will I know when the battery gets low, and what happens then?

A. Like any battery, the energy in your device's battery will naturally decrease over time. The battery's life is affected by how much energy is programmed for therapy, and how often your device delivers that therapy. Whenever your doctor or nurse checks your device with the programmer, the battery status is reported.

When the energy in your device's battery decreases to a certain level, your doctor will recommend replacement of your device. The battery in your device cannot be removed and replaced separately. A new device is connected to the existing leads.

Contacting Your Doctor or Boston Scientific

Q.When should I call my doctor?

A. Your doctor will provide guidelines for when you should contact him or her between visits. Your doctor is the best resource to:

- Answer your questions about symptoms and your overall health
- Address your concerns about how your incision is healing
- Answer questions about how your device is programmed, functioning, or performing
- Answer questions about follow-up appointments or scheduling concerns

Q.What services and information does Boston Scientific Patient Services provide?

A. A Boston Scientific Patient Services representative can:

- Answer technical questions about your device
- Provide you with a new Medical Device ID card
- Provide educational materials and handbooks about your device or other Boston Scientific products
- Help you locate a facility where your device can be checked when traveling

Types of implantable heart rhythm devices

- Pacemaker
- ICD: implantable cardioverter defibrillator (also has a pacemaker)
- CRT-D: cardiac resynchronization therapy defibrillator (also has a pacemaker)
- CRT-P: cardiac resynchronization therapy pacemaker

Questions to ask your doctor

We encourage you to talk with your healthcare provider about your condition and treatment options. To help with that conversation, we've listed some questions you may wish to discuss in addition to your own.

Medications

- What medications do I need to take?
- How long do I need to take them?
- Are there any side effects?
- Do I need to take them if I have an implanted device?

Treatment

- How will my heart disease be monitored?
- Will I need other procedures?

Lifestyle and Recovery

- Will I need cardiac rehabilitation?
- When can I return to my regular activities?
- What is the best way to manage my heart disease?

New or Returned Symptoms

- What symptoms or problems should I watch for?
- Who should I contact if I experience symptoms?

For more information

Contact Boston Scientific Patient Services at 1.866.484.3268. Boston Scientific wants to help you lead a full, active life. We know that you may have questions as you adjust to living with your device. While we always recommend that you talk with your doctor first about your specific therapy, we also encourage you to:

Read the patient handbook that came with your device

For additional copies of the handbook, call Boston Scientific Patient Services toll free at 1.866.484.3268 and select the option for technical questions.

Visit www.lifebeatonline.com

Visit www.lifebeatonline.com for stories of other heart patients, explanations of common heart conditions, resources, EMI, travel, and much more.

Our website offers:

- Animations that show how the heart works
- Clear explanations of common heart and blood vessel conditions
- Stories about other heart patients
- Pictures and descriptions of implantable devices
- Concise, printable summaries specific to your Boston Scientific device
- Information on how to be safe around appliances and equipment
- Worldwide clinic locator that identifies where devices can be checked

To contact Boston Scientific Patient Services call

1.866.484.3268

You or your family can call to speak personally with a Patient Services representative.

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Important Safety Information

Cardiac Resynchronization Therapy Devices

Cardiac resynchronization therapy pacemakers (CRT-P) and defibrillators (CRT-D) are designed to treat heart failure patients who may or may not have symptoms (CRT-D), or who may have symptoms despite the best available drug therapy (CRT-P and CRT-D). They are also designed to help your heart pump more effectively and meet your body's need for blood flow. CRT-P and CRT-D systems are not for everyone, including people with separate implantable cardioverter-defibrillators (CRT-P only) or certain steroid allergies. Procedure risks include infection, tissue damage, and kidney failure. Patients who should not receive this device include: patients who have additional medical conditions that may not allow the pacemaker to function appropriately (CRT-P only) and patients whose ventricular rhythm disturbances or heart failure have a reversible or temporary cause. In some cases, the device may be unable to detect or appropriately respond to your heart rhythm (CRT-P and CRT-D) or may deliver inappropriate shocks (CRT-D only). In rare cases severe complications or device failures can occur. Electrical or magnetic fields can affect the device. Only your doctor knows what is right for you. These devices are available by prescription only. Individual results may vary. Your physician should discuss all potential benefits and risks with you. For further safety information, refer to the Physicians Instructions for Use or call 1/866-484-3268.

Implantable Cardioverter Defibrillators

An implantable cardioverter defibrillator is designed to monitor and treat heart rhythm problems, greatly reducing the risks associated with them. But it is not for everyone, including people with certain steroid allergies. Procedure risks include infection, tissue damage, and kidney failure. Patients who should not receive this device include: patients whose ventricular rhythm disturbances have a reversible or temporary cause and patients with certain types of atrial rhythm disturbances. Procedure risks include infection and tissue damage. In some cases, the device may not respond to irregular heartbeats or may deliver inappropriate shocks. In rare cases severe complications or device failures can occur. Electrical or magnetic fields can affect the device. Only your doctor knows what is right for you. This device is available by prescription only. Individual results may vary. Your physician should discuss all potential benefits and risks with you. For further safety information, refer to the Physicians Instructions for Use or call 1/866-484-3268.

Pacemakers

A pacemaker system is designed to monitor and treat your heart rhythm problems, greatly reducing the risks associated with them. But it is not for everyone, including patients with certain steroid allergies. Procedure risks include infection, tissue damage, and kidney failure. Patients who have additional medical conditions that may not allow the pacemaker to function appropriately should not receive a device. In rare cases severe complications or device failures can occur. Electrical or magnetic fields can affect the device. Only your doctor knows what is right for you. This device is available by prescription only. Individual results may vary. Your physician should discuss all potential benefits and risks with you. For further safety information, refer to the Physicians Instructions for Use or call 1/866-484-3268.

Device Quality and Reliability

It is Boston Scientific's intent to provide implantable devices of high quality and reliability. However, these devices may exhibit malfunctions that may result in lost or compromised ability to deliver therapy. Refer to Boston Scientific's CRM product performance report on www.bostonscientific.com for more information about device performance, including the types and rates of malfunctions that these devices have experienced historically. While historical data may not be predictive of future device performance, such data can provide important context for understanding the overall reliability of these types of products. Also, it is important that you talk with your doctor about the risks and benefits associated with the implantation of a device.

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