

CONTAK RENEWAL[®] 3 and 4 Tighten All Eight (8) Setscrews

SUMMARY

Like CONTAK RENEWAL[®] and CONTAK RENEWAL 2 cardiac resynchronization therapy defibrillators (CRT-Ds), CONTAK RENEWAL 3 and 4 CRT-Ds each have a total of eight (8) setscrews to tighten while connecting the leads to the pulse generator. However, the setscrew locations have changed for CONTAK RENEWAL 3 and 4 CRT-Ds.

CONTAK RENEWAL 3 and 4 CRT-Ds each have a total of eight (8) setscrews to tighten while connecting the leads to the pulse generator. As the diagram below illustrates, four (4) setscrews are on top of the header (Figure 1); three (3) setscrews are on the back of the header (Figure 2); and one (1) setscrew is on the front of the header (Figure 3).

Remember: Tighten all eight setscrews for CONTAK RENEWAL 3 and 4 CRT-Ds.

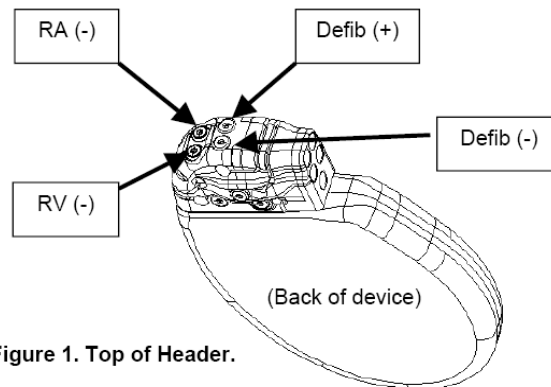


Figure 1. Top of Header.

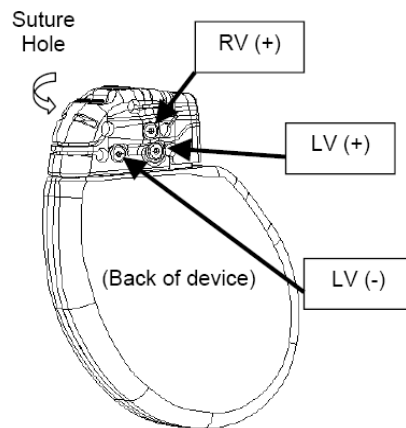


Figure 2. Back of header.

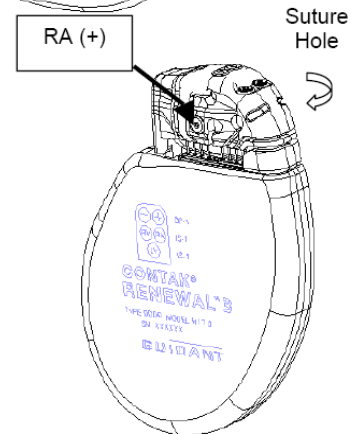


Figure 3. Front of header.

CRM PRODUCTS REFERENCED

The following are trademarks of Cardiac Pacemakers Inc., a Boston Scientific company: CONTAK RENEWAL. Products referenced herein may not be approved in all geographies. For comprehensive information on device operation, reference the appropriate product labeling.

CRM CONTACT INFORMATION

United States & Canada

www.bostonscientific.com

Technical Services – U.S.
1.800.CARDIAC (227.3422)
Tech.Services@bsci.com

LATITUDE Clinician Support
1.800.CARDIAC (227.3422)
latitude@bsci.com

Patient Services
1.866.484.3268

International

www.bostonscientific-international.com

Technical Services – Europe
+32 2 416 7222
eurtechservice@bsci.com

Patient Services
001.651.582.4000 – International