

CircuCool® Fluid Circulation Pump Set-up Guide



1) Connect the Model 2071 TTT Box to the Model 822T APM™ Device.



2) Turn the knob on the front of the CircuCool Pump counterclockwise until loose and open the front of the occlusion bed.



3) Lift the latch handle and open the occlusion bed.




4) Using sterile technique, hold the catheter with lumens on the bottom and connect and tighten the luers to the catheter luers.



5) Using sterile technique, connect the sterile cable to the catheter.




6) Hand off the tubing set to someone in a non-sterile field

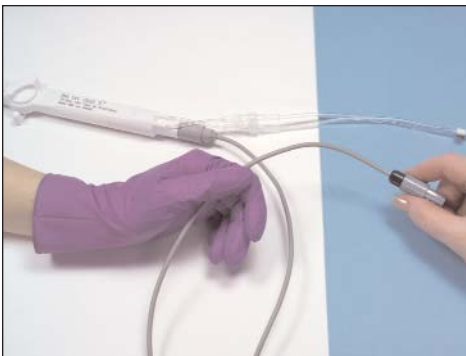


7) Hand off the catheter cable to a someone in a non-sterile field.




8) Connect catheter cable to the TTT Box. Turn on the TTT Box; verify "on" light is illuminated.

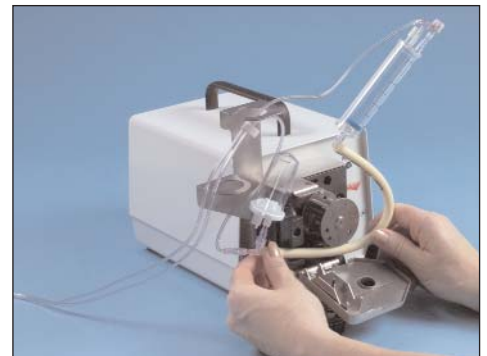

9) Insert the hard Peristaltic tubing into the lower portion of the occlusion bed.

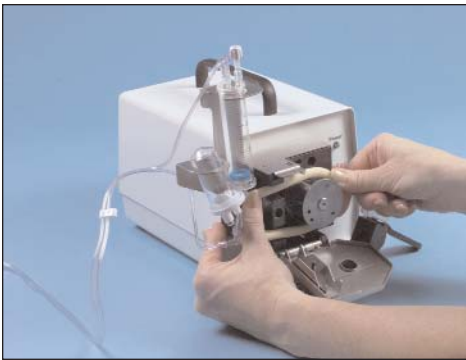

10) Slide the hard Peristaltic tubing into the upper slot of the occlusion bed. Make sure the Peristaltic tubing is inside the occlusion bed.

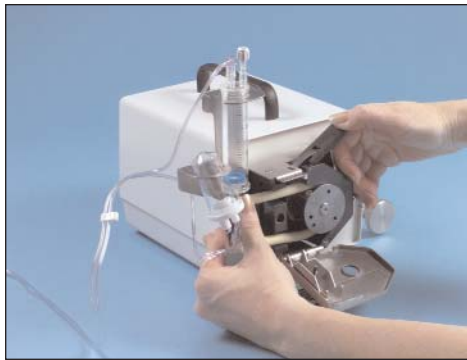

11) Close the occlusion bed latch. Make sure that the burette chamber is not distorted by the holder.



12) Close the door. Push the knob in and turn it clockwise until snug. Ensure the burette chamber and pulse dampener are in their slots.



13) Using a syringe, place 30cc of sterile D5W into the burette chamber.




14) Check that the blue float seal is floating, then turn on the pump.



15) As the system circulates, the fluid level will drop. When it drops by 10cc, add an additional 10cc of sterile D5W. The unit is ready to use.

16) Caution: Perform system integrity check. Always prime the tubing prior to inserting the catheter into the patient.

Chilli® Cooled Ablation System Connection Overview

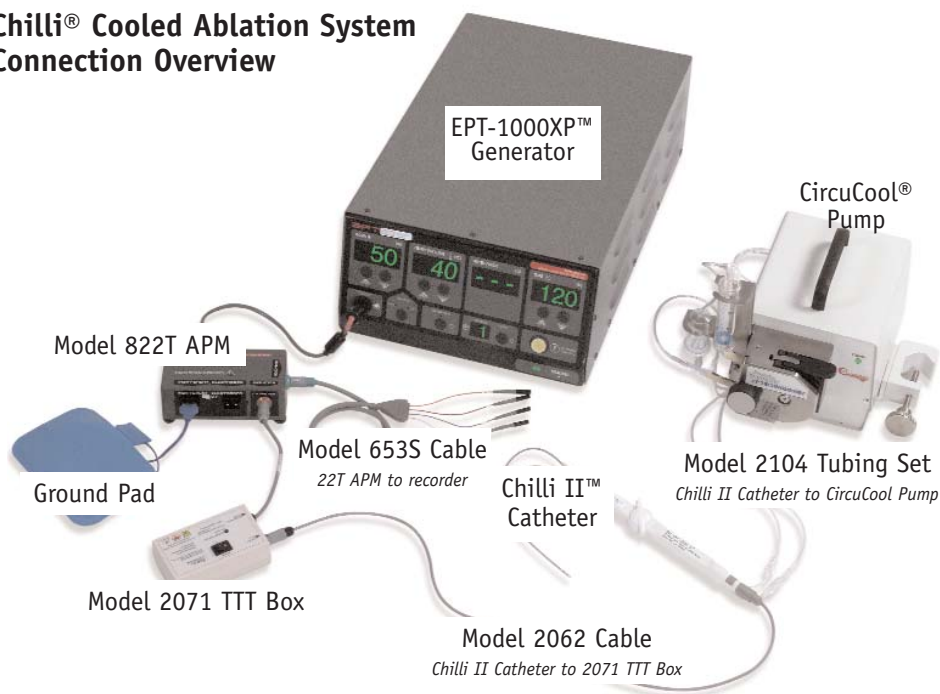

Boston Scientific

Boston Scientific Corporation
2710 Orchard Parkway
San Jose, CA 95134

www.bostonscientific.com

Customer Service
1-888-272-1001

Caution: Federal law (USA) restricts these devices to sale by or on the order of a physician. See the appropriate technical manuals for detailed information regarding instructions for use, indications, contraindications, warnings and precautions, and potential adverse events.

© 2005 Boston Scientific Corporation
or its affiliates. All rights reserved.

EPT-10159_09/05