

Captivator® II Snares

Single-Use Polypectomy Snares

CAPTIVATOR II SNARES

Order Number	Description	Loop Width (mm)	Sheath O.D. (mm)	Working Length (cm)
M00561291	Rounded Shape	33	2.4	240

PRESCRIPTIVE INFORMATION

Refer to the operator's manual for complete instructions for use.

INDICATIONS

The Single-Use Polypectomy Snares are indicated for use in the removal and cauterization of:

- Diminutive Polyps
- Sessile Polyps
- Pedunculated Polyps

CONTRAINDICATIONS

Contraindications for these devices are those specific to endoscopic Polypectomy.

WARNINGS

Contents supplied STERILE using an ethylene oxide (EO) process. Do not use if sterile barrier is damaged. If damage is found call your Boston Scientific representative.

For single use only. Do not reuse, reprocess, or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

After use, dispose of product and packaging in accordance with hospital, administrative and/or local government policy. Precautions may be found in the complete instructions for use.

POTENTIAL COMPLICATIONS

Potential adverse effects include, but are not limited to:

- perforation
- fulguration
- immediate or delayed hemorrhages
- transmural burns, characterized by abdominal pain, fever, and transient ileus

Check for proper position of the snare loop using direct vision. Positioning the snare loop in an improper location may lead to patient injury.

Please be aware that potential complications may arise even with the proper use of medical devices. Accordingly, this device should only be used by persons qualified in the procedures for which it is indicated.

CAUTIONS

Cautions can be found in the product labeling supplied with each device. CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.



Boston Scientific
Tel 508.650.8000
www.bostonscientific.com

Ordering Information
1.800.225.3226

©2007 Boston Scientific Corporation
or its affiliates. All Rights Reserved.