


Lynx™

Suprapubic Mid-Urethral Sling System

Boston Scientific

PROCEDURAL STEPS


1

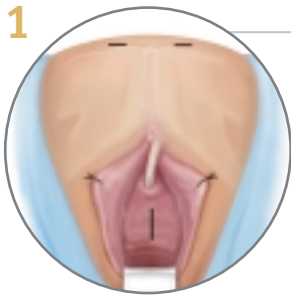

Patient Preparation

- Prepare the lower abdominal and vaginal operative sites.
- Create two small transverse abdominal incisions approximately 0.5 cm to 1 cm on each side of the midline just above the symphysis.
- Incise the anterior vaginal wall and dissect bilaterally in standard fashion.


2

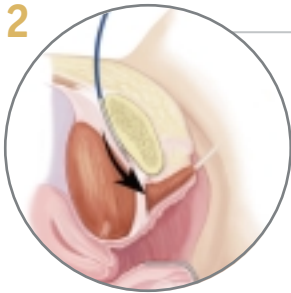

Needle Insertion

- Insert one needle through the abdominal incision, moving the needle downward in a vertical motion.
- Pierce through the rectus fascia into the Space of Retzius.
- Guide the distal end of the needle down along the posterior surface of the pubic bone through the vaginal incision.
- Repeat on the contralateral side with the second needle.


3

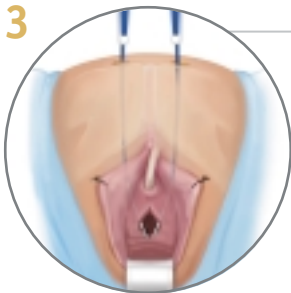

Cystoscopy

- With both needles in place, cystoscopy should be performed to confirm bladder integrity.
- If a needle is seen in the bladder, remove the needle. Visually inspect the needle for integrity. Empty the bladder and repeat Step 2 on the side that the bladder perforation occurred.


4

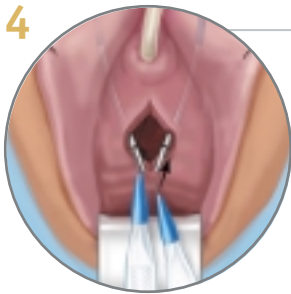

Loop Engagement

- Once it is determined that the bladder is intact, engage one association loop to the distal end of each needle protruding through the vagina.


5

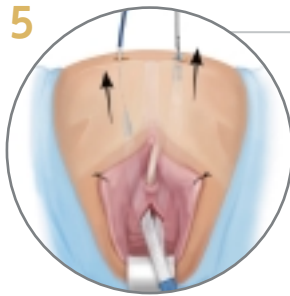

Needle Removal

- Pull the needles up through the abdominal incisions making sure the Mesh Assembly is not twisted and lies flat under the urethra.
- Remove the association loops from the needles.


6

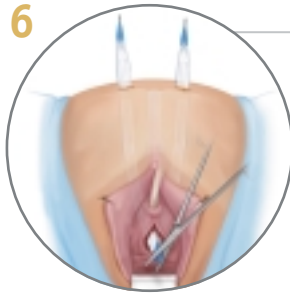

Mesh Adjustment

- Adjust the Mesh Assembly by pulling upwards on the dilators so that the blue centering tab is centered below the urethra.
- Appropriately tension the Mesh Sleeve Assembly according to physician preference.
- Grasp the blue centering tab and cut the tab through the center of the punch hole ensuring that both halves of the blue tab are removed from the vaginal canal.

7

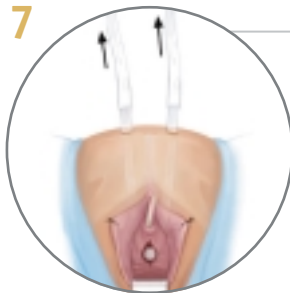

Sleeve Removal

- Pull upwards on the dilators to remove the sleeve leaving the mesh in place.
- Verify the tension of the mesh and adjust as necessary.

8




Closing

- Gently push downward on the abdomen, cut the distal ends of the mesh and confirm that the ends retract into the abdominal incisions.
- Close all incisions according to usual methods.



Suprapubic Mid-Urethral Sling System

REMOVABLE ENGAGEMENT MECHANISM

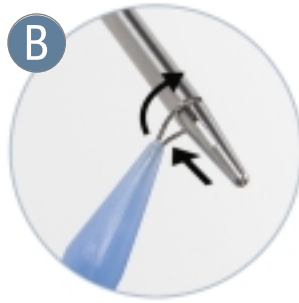

ORDERING INFORMATION

Product Code	Description
M0068503000	Lynx™ Mid-Urethral Sling System (1 Delivery Device System and 1 Mesh Assembly)
*M0068503001	*Lynx Sling System 5 Pack (5 Delivery Device Systems and 5 Mesh Assemblies)

*Check with sales representative for availability.

**Boston
Scientific**

Delivering what's next.™

Boston Scientific Corporation
One Boston Scientific Place
Natick, MA 01760-1537
www.bostonscientific.com

Ordering Information
1.888.272.1001

Copyright © 2004 by
Boston Scientific Corporation
or its affiliates. All rights reserved.

BVU2040A 9M 04/04 - 04/06

CAUTION: Federal Law (USA) restricts this device to sale by or on the order of a physician. Refer to package insert provided with the product for complete Instructions for Use, Contraindications, Potential Adverse Effects, Warnings and Precautions prior to using this product.