

AccuStick™ II Introducer System with Radiopaque Marker

- Radiopaque(RO) Marker
 - RO marker positioned at tip of sheath enhances visualization for more precise placement
- Large Sheath End Hole Inner Diameter
 - Facilitates placement of .038” working guidewire alongside .018” guidewire
- Convenient, Integrated System
 - 21-Gauge diagnostic needle reduces tissue trauma
 - Option of .018” kink-resistant nitinol guidewire is designed to provide strength during tough procedures
 - Coaxial sheath/dilator assembly with locking stiffening cannula is designed for over-the-wire placement
 - .038” heavy duty guidewire in a choice of “J” tip or straight tip provides excellent performance as a working wire
- Sheath/Dilator Assembly Proximal Taper
 - Eases introduction and placement
- Dilator Reference Mark
 - Facilitates alignment of sheath and dilator tips for more accurate wire placement

Product Information

UPN	Order Number	Description	Quantity
M001207020*	20-702	AccuStick II Introducer System with - 1 Extra Stiff, Stainless Steel, 60cm, .018” Guidewire with 7.5cm Floppy Platinum Tip - 1 Heavy Duty, Stainless Steel, 145cm, .038” Guidewire with 3mm “J” Tip	1
M001207030*	20-703	AccuStick II Introducer System with - 1 Extra Stiff, Stainless Steel, 60cm, .018” Guidewire with 7.5cm Floppy Platinum Tip - 1 Heavy Duty, Stainless Steel, 145cm, .038” Guidewire with Straight Tip	1
M001207100*	20-710	AccuStick II Introducer System with - 1 Nitinol, 60cm, .018” Guidewire with 6cm Floppy Platinum Tip - 1 Heavy Duty, Stainless Steel, 145cm, .038” Guidewire with 3mm “J” Tip	1
M001207040*	20-704	AccuStick II Introducer System with - 1 Extra Stiff, Stainless Steel, 60cm, .018” Guidewire with 7.5cm Floppy Platinum Tip	1
M001207050*	20-705	AccuStick II Introducer System without Guidewires	1
M001201081	20-108	Stainless Steel, 60cm, .018” Guidewire with 7.5cm Floppy Platinum Tip	5
M001207071	20-707	Stainless Steel, 60cm, .018” Guidewire with 11.5cm Floppy Platinum Tip	5
M001207111	20-711	Nitinol, 60cm, .018” Guidewire with 6cm Floppy Platinum	5
M001206041	20-604	21 Gauge Diagnostic Needle with Stylet	10

* Each system includes: One AccuStick II Introducer (Coaxial 6F Sheath with RO Marker/4F Dilator, 20cm) with Locking Stiffening Cannula; One 21 Gauge, 15cm Diagnostic Needle with Stylet

AccuStick™ II Introducer System with Radiopaque Marker

Caution: Federal (USA) law restricts this device to sale, distribution and use by or on the order of a physician.

Indications:

The AccuStick II Introducer System with Radiopaque Marker facilitates introduction and placement of a guidewire.

Contraindications:

None known

Warnings:

None known

Potential Adverse Effects:

Potential risks exist for serious complications to include:

Perforation of a vessel or viscus

Laceration of a vessel or viscus

Bleeding

Wire or catheter embolism

Extravasation

Hematoma

Hemothorax

Hydrothorax

Inflammation, necrosis or scarring

Risks normally associated with percutaneous interventional procedures

Pain in region

Skin infection

Edema

These and other complications are well documented in medical literature. Use of the AccuStick II Introducer System with Radiopaque Marker should be reserved by persons knowledgeable of the risks involved and qualified in the procedures.

Cautions:

Sterile. For single use only. Do not reuse or resterilize. These products are supplied sterile. If the catheter package is opened or damaged, do not use products.

Device should be used for non-vascular procedures only.

Advancement and/or withdrawal of 0.46/0.97mm (0.018"/0.038") wires should be smooth and without resistance. **DO NOT USE EXCESSIVE FORCE.** If resistance is felt, carefully remove wire(s) and system as a unit.

**Boston
Scientific**

Delivering what's next.™

Boston Scientific Corporation
Oncology Products
100 Boston Scientific Way
Marlborough, MA 01752
www.bostonscientific.com

Ordering Information
1.800.225.3238

© 2007 Boston Scientific Corporation or its
affiliates. All rights reserved.
ONCPS170WEB