

Precision Spectra™ ImageReady™ MRI Head Only Patient Eligibility

This form provides information about the patient's implanted Precision Spectra Spinal Cord Stimulator System and MRI head scan eligibility. It may be provided to the radiologist to support the confirmation of the patient's scan.

- Prior to performing and MRI Head Scan, confirm that the patient's stimulation is OFF.
- Refer to www.controlyourpain.com/dfu for labeling and safety conditions.
- For additional assistance contact Boston Scientific Customer Service at 866.360.4747

Patient Name: _____ Date: _____
 Physician Name: _____ Physician Signature: _____
 Office Address: _____ (optional)
 Phone: _____

A.	MR CONDITIONAL PRECISION SPECTRA SYSTEM INFORMATION	MODEL #	MRI ELIGIBLE	NOT MRI ELIGIBLE
1.	Implantable Pulse Generator (IPG)			
	• Precision Spectra IPG, 32-Contact IPG	SC-1132	<input type="checkbox"/>	
	• Precision IPG, 16-Contact IPG	SC-1110-xx		<input type="checkbox"/>
	• Other IPG _____			<input type="checkbox"/>
2.	Percutaneous and/or surgical paddle leads (check all that apply)			
	• Linear Lead, 8-contact lead, 30 cm	SC-2158-30	<input type="checkbox"/>	
	• Linear Lead, 8-Contact lead, 50 cm	SC-2158-50	<input type="checkbox"/>	
	• Linear ST Lead, 8-Contact Lead, 30 cm	SC-2218-30	<input type="checkbox"/>	
	• Linear ST Lead, 8-Contact Lead, 50 cm	SC-2218-50	<input type="checkbox"/>	
	• Linear 3-4 Lead, 8-Contact Lead, 50 cm	SC-2352-50	<input type="checkbox"/>	
	• Linear 3-6 Lead, 8-Contact Lead, 50 cm	SC-2366-50	<input type="checkbox"/>	
	• Artisan Paddle Lead, 16-Contact Paddle, 50 cm	SC-8216-50	<input type="checkbox"/>	
	• CoverEdge 32 Paddle Lead, 32 Contact Paddle, 50 cm	SC-8336-50	<input type="checkbox"/>	
	• CoverEdge X Paddle Lead, 32 Contact Paddle, 50 cm	SC-8352-50	<input type="checkbox"/>	
	• Infinion Lead, 16-Contact Lead, 50 cm or 70 cm	SC-2316-xx		<input type="checkbox"/>
	• Leads longer than 50 cm, other leads, Adapters, Extensions, or Splitters:			<input type="checkbox"/>
3.	Surgical Accessories (check all that apply)			
	• Clik Anchor	SC-4316	<input type="checkbox"/>	
	• Clik X Anchor	SC-4318	<input type="checkbox"/>	
	• Med-A	SC-4320	<input type="checkbox"/>	
	• Silicone Suture Sleeves		<input type="checkbox"/>	
	• Other: _____			

Note: Leads should be connected directly into the IPG, Patient should not be implanted with lead extensions, splitters, or adapters.

B.	PATIENT IMPLANT CONFIGURATION INFORMATION (ALL QUESTIONS MUST BE ANSWERED)	MRI ELIGIBLE	NOT MRI ELIGIBLE
1.	The lead distal tip placement is at or below the T5 thoracic spinal level. Only distal lead tip placement between T12 and T5 is allowed	Yes	No
2.	The IPG is implanted in the upper buttock or lower flank	Yes	No
3.	Patient has no abandoned leads or IPGs (i.e. leads or IPGs that are not connected to the functioning Precision Spectra System)	Yes	No
4.	No evidence can be found of fractured leads or compromised IPG-lead system integrity	Yes	No

C.	INSTRUCTIONS FOR THE PATIENT PRIOR TO THE MRI EXAM	MRI ELIGIBLE
1.	Instruct the patient to fully charge their IPG (IPG charge shown as 3 bars on the Remote Control) and bring the Charger to the MRI Center (in case charging is necessary)	<input type="checkbox"/>
2.	Instruct the patient to bring their Remote Control to the MRI exam and turn stimulation off before the MRI Head Scan	<input type="checkbox"/>

Note: The Charger and Remote Control must not be brought into the MRI Scanner Room

Indications for Use. The Precision Spectra™ Spinal Cord Stimulator System (Precision Spectra System) is indicated as an aid in the management of chronic intractable pain of the trunk and/or limbs, including unilateral or bilateral pain associated with the following: failed back surgery syndrome, intractable low back pain, and leg pain.
Contraindications, warnings, precautions, side effects. The Precision Spectra System is contraindicated for patients who: are unable to operate the Precision Spectra System, have failed trial stimulation by failing to receive effective pain relief, are poor surgical risks, or are pregnant. Refer to the Instructions for Use provided with the Precision Spectra System or ControlYourPain.com for potential adverse effects, warnings, and precautions prior to using this product. **Caution:** Federal (U.S.) law restricts this device to sale by or on the order of a physician. The Precision Spectra SCS System with ImageReady MRI Technology is "MR-Conditional" only when exposed to the MRI environment under the specific conditions defined in the ImageReady™ MRI Guidelines for Precision Spectra™ Spinal Cord Stimulator System.