

## IMAGEREADY™ MRI HEAD ONLY PATIENT ELIGIBILITY

This form provides information about the patient's implanted Precision Spectra™ or Spectra WaveWriter™ Spinal Cord Stimulator Systems and MRI head scan eligibility. It may be provided to the radiologist to support the confirmation of the patient's scan.

- Prior to performing an MRI Head Scan, confirm that the patient's stimulation is OFF.
- Refer to [www.bostonscientific.com/imageready](http://www.bostonscientific.com/imageready) for labeling and safety conditions

Name:

Date:

Physician Name:

Physician Signature:

Office Address:

(optional)

Phone:

	MODEL #	MRI ELIGIBLE	NOT MRI ELIGIBLE
<b>1. Implantable Pulse Generator (IPG)</b>			
• Precision Spectra IPG, 32-Contact IPG	SC-1132	<input type="checkbox"/>	
• Spectra WaveWriter IPG, 32-Contact IPG	SC-1160	<input type="checkbox"/>	
• Precision IPG, 16-Contact IPG	SC-1110-xx		<input type="checkbox"/>
• Other:			<input type="checkbox"/>
<b>2. Percutaneous and/or surgical paddle leads (check all that apply)</b>			
• Linear Lead, 8-contact lead, 30 cm	SC-2158-30	<input type="checkbox"/>	
• Linear Lead, 8-Contact lead, 50 cm	SC-2158-50	<input type="checkbox"/>	
• Linear ST Lead, 8-Contact Lead, 30 cm	SC-2218-30	<input type="checkbox"/>	
• Linear ST Lead, 8-Contact Lead, 50 cm	SC-2218-50	<input type="checkbox"/>	
• Linear 3-4 Lead, 8-Contact Lead, 50 cm	SC-2352-50	<input type="checkbox"/>	
• Linear 3-6 Lead, 8-Contact Lead, 50 cm	SC-2366-50	<input type="checkbox"/>	
• Infinion CX Lead, 16-Contact Lead, 50 cm	SC-2317-50	<input type="checkbox"/>	
• Artisan Paddle Lead, 16-Contact Paddle, 50 cm	SC-8216-50	<input type="checkbox"/>	
• Artisan MRI Paddle Lead, 32-Contact Paddle, 50 cm	SC-8416-50	<input type="checkbox"/>	
• CoverEdge 32 Paddle Lead, 32 Contact Paddle, 50 cm	SC-8336-50	<input type="checkbox"/>	
• CoverEdge X Paddle Lead, 32 Contact Paddle, 50 cm	SC-8352-50	<input type="checkbox"/>	
• Infinion Lead, 16-Contact Lead, 50 cm or 70 cm	SC-2316-xx		<input type="checkbox"/>
• Leads longer than 50 cm, other leads, Adapters, Extensions, or Splitters:			<input type="checkbox"/>
<b>3. Surgical Accessories (check all that apply)</b>			
• Clik Anchor	SC-4316	<input type="checkbox"/>	
• Clik X Anchor	SC-4318	<input type="checkbox"/>	
• Med-A	SC-4320	<input type="checkbox"/>	
• Silicone Suture Sleeves		<input type="checkbox"/>	
• Other:			

**Note:** Leads should be connected directly into the IPG. Patient should not be implanted with lead extensions, splitters, or adapters.

B. PATIENT IMPLANT CONFIGURATION INFORMATION (ALL QUESTIONS MUST BE ANSWERED)	MRI ELIGIBLE	NOT MRI ELIGIBLE
1. The lead distal tip placement is at or below the T5 thoracic spinal level. Only distal lead tip placement between T12 and T5 is allowed	Yes	No
2. The IPG is implanted in the upper buttock or lower flank	Yes	No
3. Patient has no abandoned leads or IPGs (i.e. leads or IPGs that are not connected to the functioning Precision Spectra or Spectra WaveWriter System)	Yes	No
4. No evidence can be found of fractured leads or compromised IPG-lead system integrity	Yes	No
C. INSTRUCTIONS FOR THE PATIENT PRIOR TO THE MRI EXAM	MRI ELIGIBLE	
1. Instruct the patient to fully charge their IPG (IPG charge shown as 3 bars on the Remote Control) and bring the Charger to the MRI Center (in case charging is necessary)		<input type="checkbox"/>
2. Instruct the patient to bring their Remote Control to the MRI exam and turn stimulation off before the MRI Head Scan		<input type="checkbox"/>

**Note:** The Charger and Remote Control must not be brought into the MRI Scanner Room

**CAUTION:** The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labelling supplied with each device. Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries. This material not intended for use in France. 2020 Copyright © Boston Scientific Corporation or its affiliates. All rights reserved. Product available in the European Economic Area (EEA) only. Please check availability with your local sales representative or customer service.