Reprocessing Instructions

Non-Sterile Tools

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 Manual Cleaning 6

 Ultrasonic Cleaning 7

 Automated Cleaning Washer/Disinfector 8

 Pre-Sterilization Instructions 9

 Steam Sterilization..... 10

 Post Sterilization Storage Conditions 10

Reprocessing Instructions

Non-Sterile Tools

Rx ONLY

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

These instructions provide information on the use, care, cleaning, maintenance, and sterilization of reusable surgical tools manufactured by Boston Scientific Corporation (BSC) intended for reprocessing in a health care facility. The BSC reusable tools may be safely and effectively reprocessed using the manual or automated cleaning instructions and sterilization parameters provided.

Do not use or reprocess if the product is damaged.

DEVICE DESCRIPTION

These instructions pertain to reusable surgical tools manufactured and/or distributed by BSC, for use with devices manufactured by BSC, including:

Table 1.

Part Number	Product Name	Reusable Tool used with...			
		AMS 700™ Inflatable Penile Prosthesis (with & without InhibiZone™)	AMS 800™ Artificial Urinary Sphincter (with & without InhibiZone™)	AMS Ambicor™ Inflatable Penile Prosthesis	Spectra™ Concealable Penile Prosthesis
720153-01	Sizer	X		X	X
72400155	Closing Tool	X		X	
35400010	Furlow Insertion Tool	X		X	
72400271	Quick Connect Assembly Tool	X	X		
72100005	Insertion Package	X	X		

Note: See applicable product's directions for use for instructions on how to use the tool.

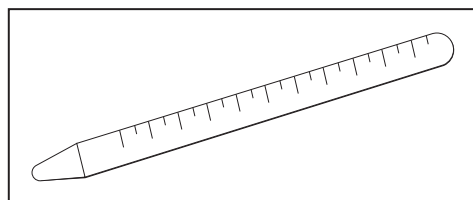
Figure 1. Sizer

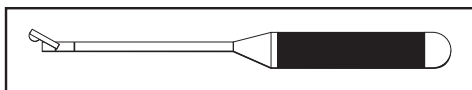
Figure 2. Closing Tool

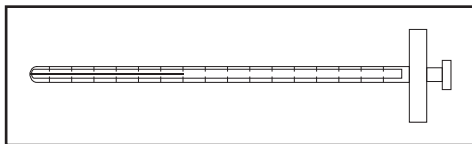
Figure 3. Furlow Insertion Tool

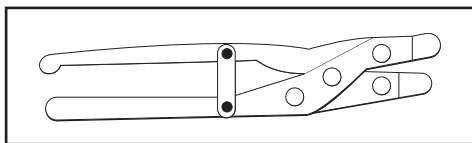
Figure 4. Quick Connect Assembly Tool

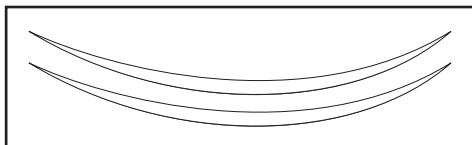
Figure 5. Insertion Package

WARNINGS

- Ethylene Oxide (EO), gas plasma sterilization, and dry heat sterilization methods shall not be used for sterilization of BSC reusable tools.
- Failure to follow instructions could lead to inadequate sterilization. The cleaning procedures contained in this instruction manual must be executed as detailed prior to executing steam sterilization instructions.
- Devices that fail functional checks, have identification markings that are not legible, and/or have visible wear, rust, or pitting should be safely disposed of in accordance with standard biohazard practices. Devices that have visible soil after repeated cleaning should be disposed of in accordance with standard biohazard practices.

PRECAUTIONS

- Tape or other material attached to the tool should be removed before reprocessing.
- Healthcare facility procedures and precautions should be followed by hospital personnel who work with contaminated or potentially contaminated medical devices.
- Caution should be exercised when handling devices with sharp points or cutting edges.
- Do not stack tools or place heavy tools on top of delicate devices.
- Comply with local laws and ordinances.
- The tools may also become contaminated with body fluids containing hepatitis virus, HIV, or other etiological agents and pathogens.
- Fresh cleaning detergents should be prepared when existing detergents become grossly contaminated (turbid).
- Do not allow contaminated devices to dry prior to reprocessing. Do not allow blood, body fluids, bone or tissue debris, saline, or disinfectants to dry on used tools.

- Articulating devices (devices that open/close, such as the Quick Connect Assembly Tool or Furlow Insertion Tool) should be in the open position or disassembled when being cleaned in an automated or manual process. Failure to open or disassemble devices could result in the retention of tissues or fluids that will prevent the device from being used as intended.

PREPARATIONS

BSC does not sterilize the reusable BSC Tools. New and used tools must be thoroughly processed according to these instructions prior to use. Reprocessing includes a manual, ultrasonic, or automated washer/disinfector cleaning, followed by a steam sterilization cycle.

Disassembly

Only the Furlow Insertion Tool requires disassembly prior to cleaning and sterilization. To disassemble, pull the Obturator (round handle) (a) completely out of the Cylinder handle (rectangular handle) (b). Refer to Figure 6.

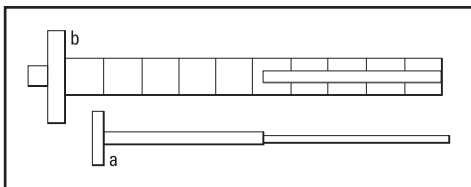
Figure 6. Disassembled Furlow Insertion Tool

Note: Disassemble by hand. Never use aiding instruments to disassemble tools beyond what is recommended.

Reassembly

To reassemble the Furlow Insertion Tool, insert the Obturator into the Cylinder handle.

Materials for Reprocessing

- Personal Protective Equipment (PPE) should be worn when handling or working with contaminated or potentially contaminated materials, devices, and equipment.
- Legally marketed accessories, when applicable, should be used for performing all reprocessing steps.

Note: Cleaning agent should be pH neutral. Saline and cleaning agents containing aldehyde, mercury, active chlorine, bromine, bromide, iodine, or iodide are corrosive and should not be used.

Point of Use Processing

- Cleaning detergent: pH neutral enzymatic detergent (i.e., Prolystica™ Enzymatic Detergent or equivalent)
- Softened tap water
- Absorbent, low-lint cloth

Manual Cleaning

- Cleaning detergent: pH neutral enzymatic detergent (i.e., Prolystica Enzymatic Detergent or equivalent)
- Lumen brush: 0.118 inch (3 mm) diameter nylon brush
- Surface brush: Soft nylon-bristled brush
- Soaking container/tub
- 20 mL or greater volume syringe
- Critical water (RO or ≤ 0.2 micron filtered water)
- Softened tap water
- Absorbent, low-lint cloth

Ultrasonic Cleaning

- Ultrasonic bath: Capable of frequency 45 kHz minimum at 10 minutes
- Cleaning detergent: pH neutral enzymatic detergent (i.e., Prolystica Enzymatic Detergent or equivalent)
- Lumen brush: 0.118 inch (3 mm) diameter nylon brush
- Surface brush: Soft nylon-bristled brush
- Critical water (RO or ≤ 0.2 micron filtered water)
- Softened tap water
- Absorbent, low-lint cloth

Automated Washer/Disinfectant Cleaning

- Cleaning detergent: pH neutral enzymatic detergent (i.e., Prolystica™ Enzymatic Detergent or equivalent)
- Critical water (RO or ≤ 0.2 micron filtered water)
- Absorbent, low-lint cloth
- ISO 15883 washer/disinfectant

Lubrication

- Lubricant that does not interfere with steam sterilization per the manufacturer's label (i.e., Symmetry Surgical Preserve™ Instrument Lube Concentrate or equivalent)

Sterilization

- Sterilization pouch
- Pre-vacuum type OR Gravity type steam sterilizer

REPROCESSING INSTRUCTIONS

Point of Use Processing

1. Disassemble the Furlow Insertion Tool if used.
2. Remove gross soil using absorbent, low-lint cloths and thoroughly rinse the surgical tools with softened tap water.
3. Place devices in a container of distilled water or cleaning detergent, or cover with damp towels.

Note: Soaking in a cleaning detergent prepared according to the manufacturer will facilitate cleaning especially in tools with crevices, hinged joints, rough surfaces, lumens, and areas with moving components or clips.

Containment and Transportation

Start the reprocessing steps within 1 hour of surgery completion. For the containment and transportation of the devices during point of use processing, procedures established by the healthcare facility must be followed.

Manual Cleaning

Notes:

- Metal brushes or scouring pads must not be used during manual cleaning procedures. These materials will damage the surface and finish of the tool.
- Manual scrubbing with brushes should always be performed with the tool below the surface of the cleaning detergent to prevent generation of aerosols and splashing, which may spread contaminants. Cleaning agents must be completely rinsed from device surfaces to prevent accumulation of detergent residue.

-
1. If not already disassembled, disassemble the Furlow Insertion Tool if used.
 2. Prepare a cleaning detergent per detergent manufacturer instructions.

3. Submerge tools completely in the cleaning detergent for a minimum of 5 minutes.

With the tools still completely submerged, for an additional 5 minutes scrub surfaces using a surface brush until all visible soil has been removed.

Notes:

- Minimum submerge and scrub time is 10 minutes.
- All scrubbing should be performed below the surface of the cleaning detergent to minimize the potential of aerosolizing contaminated solution.

-
4. Actuate (open and close) moveable mechanisms and flush lumens with a syringe to ensure contact of the cleaning detergent with all surfaces, and to remove trapped bubbles. Particular attention should be given to crevices, hinged joints, rough surfaces, lumens, and areas with moving components or clips.
 5. Insert the lumen brush into the lumen (if applicable) with a twisting motion while pushing in and out at least 3 times.

Note: The slot on the distal end of the Furlow Insertion Tool stem should be brushed by pushing the head of the brush against the slot starting at the proximal end and moving outwardly toward the distal end of the device. This allows the bristles of the brush to fit into the slot and push gross debris out of the end of the lumen.

-
6. Remove the tools from the cleaning detergent and rinse in critical water 3 times for a total of 1 minute (e.g., rinse 40 seconds, 10 seconds, 10 seconds). Actuate (open and close) all moveable and hinged parts while rinsing.
 7. The Furlow Insertion Tool lumen, if used, should be flushed with critical water 3 times using a 20 mL to 50 mL syringe.
 8. Visually inspect the tools with sufficient magnification and light to verify that all soil and detergent residue has been removed. If it has not, repeat the cleaning process.
 9. Dry tools with an absorbent, low-lint cloth, or clean, filtered compressed air.
 10. Visually inspect the tools with sufficient magnification and light to verify that all soil and detergent residue has been removed. If it has not, repeat the cleaning process.

Ultrasonic Cleaning

Notes:

- Separate stainless steel tools from other metal tools during ultrasonic cleaning to avoid electrolysis.
- Fully open hinged tools and use wire mesh baskets or trays designed for ultrasonic cleaners.

-
1. If not already disassembled, disassemble the Furlow Insertion Tool if used.
 2. Rinse the tools, including lumens (if applicable) with softened tap water to remove any visible soil or detergent residue from all areas (lumen and surface) of the tools using lumen and cleaning brushes.
 3. Prepare an ultrasonic bath with a cleaning detergent per detergent manufacturer instructions.

Note: Ensure devices are fully opened and disassembled. Failure to open devices (e.g., Quick Connect Assembly Tool) could result in the retention of tissues or fluids that will prevent the device from being used as intended.

-
4. Immerse the device in the ultrasonic bath for a minimum of 10 minutes at a frequency of 45 kHz.

5. Remove the tools from the ultrasonic bath and rinse in critical water for a minimum of 1 minute or until there is no sign of soil or detergent residue. Actuate (open and close) all moveable and hinged parts while rinsing.
6. The Furlow Insertion Tool lumen, if used, should be flushed with critical water 3 times using a 20 mL to 50 mL syringe.
7. Visually inspect the tools with sufficient magnification and light to verify that all soil and detergent residue has been removed. If it has not, repeat the cleaning process.
8. Dry tools with an absorbent, low-lint cloth, or clean, filtered compressed air.
9. Visually inspect the tools with sufficient magnification and light, to verify that all soil and detergent residue has been removed. If it has not, repeat the cleaning process.

Automated Cleaning Washer/Disinfector

Precaution: Chemical disinfection programs should not be used due to the potential for chemical residues to remain on the tools. These residues could interfere with sterilization efficacy.

Notes:

- Follow the washer/disinfector manufacturer’s instructions for use.
- Use a washer/disinfector with demonstrated efficacy.
- Increase the dry time as the load size increases. Follow the washer/disinfector manufacturer’s instructions.
- Low-level disinfection must be used as part of a washer/disinfector cycle, but the devices must also be sterilized before use.
- Use critical water for the final rinse.

1. If not already disassembled, disassemble the Furlow Insertion Tool if used.
2. Place tools in a washer/disinfector basket. Observe the manufacturer’s loading requirements. Follow the cycle parameters given in Table 2.
3. Avoid contact between devices as movement during washing could cause damage and washing action could be obstructed.
4. Arrange the Furlow Insertion Tool lumens (if applicable) vertically to assist drainage.
5. The Quick Connect Assembly Tool (if applicable) should be in the open position.

The following minimum wash cycle parameters are recommended:

Table 2. Minimum Automated Washer/Disinfector Cycle for Surgical Tools

Step	Description
1	4 minutes; 50°C - 55°C enzymatic wash
2	2 minutes; 50°C - 55°C wash
3	2 minutes; 50°C - 55°C final rinse
4	10 minutes; 70°C Thermal disinfection
5	15 minutes; 80°C air dry
6 (optional)	If a lubrication cycle is available that applies a water soluble lubricant, it is acceptable to use on the tools.

Note: If the washer/disinfector does not have a lubrication cycle, refer to “Lubrication.”

6. Upon completion, unload the washer/disinfecter.
7. Visually inspect the tools with sufficient magnification and light to verify that all soil and detergent residue has been removed. If it has not, repeat the cleaning process.
8. If needed, dry tools with an absorbent, low-lint cloth, or clean, filtered compressed air.
9. Visually inspect the tools with sufficient magnification and light to verify that all soil and detergent residue has been removed. If it has not, repeat the cleaning process.

Warning: Failure to properly clean could lead to inadequate sterilization. The cleaning procedure outlined in these instructions must be performed as stated prior to steam sterilization.

Pre-Sterilization Instructions

Lubrication

The Quick Connect Assembly Tool must be lubricated. Lubrication of other tools is optional.

Warning: Mineral oil or silicone lubricants should not be used because they prevent proper steam sterilization.

1. Refer to the lubricating agent labeling for preparation and use instructions.
2. Disassemble the Furlow Insertion Tool before lubricating.
3. Dip the tools in the prepared lubricant or spray the prepared lubricant on per manufacturer's instructions, paying special attention to crevices and joints.
4. Remove any excess lubricant with absorbent, low-lint cloth.

Inspection and Functional Check

Visually inspect each device for completeness, damage, and excessive wear. When inspecting devices, look for the following:

- Jaws and teeth should align properly.
- Movable parts should operate smoothly throughout the intended range of motion.
- Long, thin tools should be free of bending or distortion.
- Where tools form part of a larger assembly, all components should be available and should assemble readily.
- The identification marking should be readable.
- There should be no reddish brown rust on the tool.

If damage or wear is observed that might compromise the function of the tool, dispose of product and packaging in accordance with hospital, administrative, and/or local government policy and contact your BSC representative for a replacement.

Packaging

1. Before use, packaging materials should be held at room temperature (20°C - 23°C and RH of 30% - 60%) for a minimum of 2 hours.
2. Tool Preparation
 - a. For the Furlow Insertion Tool, disassemble the tool with both parts together in the same pouch.
 - b. The Quick Connect Assembly Tool shall be in the open position when placing in the pouch.
3. The cleaned and inspected tools should be double pouched.

To double pouch:

- a. Place the tool in a smaller pouch and seal the pouch.
- b. Place the smaller sealed pouch in a larger pouch and seal it.

Steam Sterilization

Warning: Do not use rigid containers for steam sterilization. This configuration could limit steam penetration and prevent effective sterilization of the tools. Do not wrap the tools with tape.

Precaution: Do not use flash sterilization for reusable tools.

Load trays per the sterilization equipment manufacturer's instructions.

Table 3. Steam Sterilizer Parameters

Sterilization Method	Exposure Time (minutes)	Exposure Temperature	Dry time (minutes)
Pre-vacuum*	3	134°C	30
Pre-vacuum*	18	134°C	30
Pre-vacuum	4	132°C	20
Gravity	30	121°C	30

*This (Pre-vacuum: 3 minutes at 134°C and dry time of 30 minutes, 18 minutes at 134°C and dry time of 30 minutes) sterilization cycle is not considered by the United States Food and Drug Administration (US FDA) to be a standard sterilization cycle. Users should only use sterilizers and accessories (such as sterilization wraps, sterilization pouches, chemical indicators, biological indicators, and sterilization containers) that have been cleared by the US FDA for the selected sterilization cycle specifications (time and temperature).

Post Sterilization Storage Conditions

Sterile packaged tools should be stored in a designated, limited access area that is well ventilated, dry, and provides protection from dust, moisture, and temperature/humidity extremes.

Note: Inspect every package before use to ensure that the sterile barrier is not torn, perforated, shows signs of moisture, or appears to be tampered with. If any of those conditions are present then the contents are considered non-sterile and should be completely re-processed.

Polystica is a registered trademark of Steris Corporation.

Preserve is a registered trademark of Codman.

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All rights reserved.
Boston Scientific Limited
Boston Scientific (Australia) Pty Ltd
© 2018 Boston Scientific Corporation or its affiliates.