# Disposable Dilators

Disposable Dilator

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## Disposable Dilator

### Rx ONLY

**Caution:** Federal Law (USA) restricts this device to sale by or on the order of a physician.

#### WARNING

Contents supplied STERILE using a steam process. Do not use if sterile barrier is damaged. If damage is found, call your Boston Scientific representative.

For single use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

After use, dispose of product and packaging in accordance with hospital, administrative and/or local government policy.

#### DEVICE DESCRIPTION

The Boston Scientific Disposable Dilators are used for dilation of the corpora cavernosa prior to penile prosthesis implantation. The length and shape of the dilators are compatible with both penoscrotal and infrapubic surgical techniques.

The Dilators are constructed of Utem™, a smooth, solid, biocompatible thermoplastic material chosen to provide high strength and rigidity.

Each dilator is 22.1 cm (8.7 in) long and the shaft is 7.98 mm (0.314 in) in diameter. The shaft tapers up to the wider dilator area at each end and tapers down to a bullet-shaped tip at each end.

The dilation ends differ by 1 mm in diameter (e.g., one end of the dilator is 9 mm, the opposite end is 10 mm). The Dilator diameters are marked on each shaft near the dilation ends (Refer to Figure 1).

#### User Information

This device is to be used only by physicians who are knowledgeable regarding the use of corporal dilators. This Directions for Use is not intended to be a complete reference.

#### Contents

Set of 2 Dilators (9 mm - 12 mm)

- (1) 9 mm - 10 mm Disposable Dilator
- (1) 11 mm - 12 mm Disposable Dilator

Set of 4 Dilators (9 mm - 16 mm)

- (1) 9 mm - 10 mm Disposable Dilator
- (1) 11 mm - 12 mm Disposable Dilator
- (1) 13 mm - 14 mm Disposable Dilator
- (1) 15 mm - 16 mm Disposable Dilator

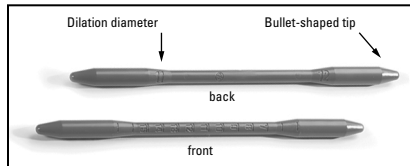
Figure 1.

#### INTENDED USE / INDICATIONS FOR USE

The Disposable Dilators are intended for use as manual, surgical instruments to progressively dilate the corpora cavernosa prior to insertion of a Boston Scientific penile prosthesis.

Variations in use may occur in specific procedures due to individual technique and patient anatomy.

#### CONTRAINDICATIONS

None known.

#### WARNINGS

- Do not use this device in a manner which may cause tissue trauma.
- Pre-existing penile or corporal scarring may make surgical use of the Dilators more complicated or impractical.
- Failure to evaluate and promptly treat tunical trauma and perforations may result in a substantial worsening of the condition and may lead to infection and/or loss of tissue.
- Dilator use will make latent natural or spontaneous erections, as well as other interventional treatment options for erectile restoration, impossible.

#### PRECAUTIONS

- Dilators are provided sterile and intended for single use only. DO NOT RESTERILIZE.
- Do not use beyond Use By date indicated on the package.
- This device is to be used only by physicians who are knowledgeable regarding the use of corporal dilators. This Directions for Use is not intended to be a complete reference.

- This device is not designed to be implanted.
- Inspect the package for damage. If the package integrity is compromised, do not use the Dilators as sterility may have been compromised.
- Use caution when using this device with other surgical instruments that may damage the tool.

#### ADVERSE EVENTS

The following potential adverse events associated with the use of corporal dilators may include (in alphabetical order):

- Allergic reaction
- Altered therapeutic response
- Infection
- Prolonged procedure
- Tissue damage
- Tunical trauma/perforation
- Urethral trauma/perforation

---

**Note:** Refer to the Directions for Use for the selected penile prosthesis for specific prosthesis risks and surgical-related Adverse Events.

---

#### HOW SUPPLIED

The Disposable Dilators are provided sterile.

- Do not use if package is opened or damaged.
- Do not use if labeling is incomplete or illegible.
- Do not use the device if past the "Use By" date.

#### Handling and Storage

Store in a cool, dry, dark place.

#### OPERATIONAL INSTRUCTIONS

1. Make an incision into one of the corpora cavernosa. Using the physician's dilator of choice, introduce the smaller end of the dilator into the distal portion of the corpus cavernosum. Gently advance the dilator, taking care to keep it lateral to avoid urethral trauma and/or corporal perforation.
2. Continue to keep the dilator lateral while observing the position of the tip as it advances against the lateral wall of the tunica albuginea. Take care not to cross over through the intercavernous septum to the contralateral side.
3. Palpate the glans to detect when the dilator reaches the distal portion of the corpus cavernosum.
4. Remove the dilator and insert the same sized end into the proximal corpus cavernosum. Use care to dilate the tissue to the point of corporal insertion at the ischial tuberosity.
5. Repeat the dilation process, increasing the dilator size as needed to progressively expand the corporal tissue.

#### WARRANTY

Boston Scientific Corporation (BSC) warrants that reasonable care has been used in the design and manufacture of this device. **This warranty is in lieu of and excludes all other warranties not expressly set forth herein, whether express or implied by operation of law or otherwise, including, but not limited to, any implied warranties of merchantability or fitness for a particular purpose.** Handling and storage of this device as well as other factors relating to the patient, diagnosis, treatment, surgical procedures and other matters beyond BSC's control may directly affect the device and the results obtained from its use. BSC shall not be liable for any incidental or consequential loss, damage or expense directly or indirectly arising from the use of this device. BSC neither assumes, nor authorizes any other person to assume for it, any other or additional liability or responsibility in connection with this device. **BSC assumes no liability with respect to devices that are reused, reprocessed or resterilized and makes no warranties, express or implied, including but not limited to merchantability or fitness for a particular purpose, with respect to such devices.**

Utem is a trademark of SABIC Global Technologies B.V.

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All rights reserved.
**Boston Scientific Limited**
**Boston Scientific (Australia) Pty Ltd**
© 2019 Boston Scientific Corporation or its affiliates.