

AVOIDING UNNECESSARY THERAPY FOR VENTRICULAR ARRHYTHMIAS \geq 200 BPM: RESULTS FROM MADIT-RIT


BACKGROUND

MADIT-RIT was a prospective, randomized, multi-center trial comparing conventional ICD programming (Arm A) to either high rate therapy \geq 200 bpm (Arm B), or long delay therapy (Arm C) in patients with a primary prevention ICD or CRT-D indication over an average follow-up period of 1.4 years.

METHODS

A new retrospective analysis of the data from MADIT-RIT characterized the rate and type of appropriate therapies for ventricular arrhythmias \geq 200 bpm in all three arms of the MADIT-RIT study.

RESULTS

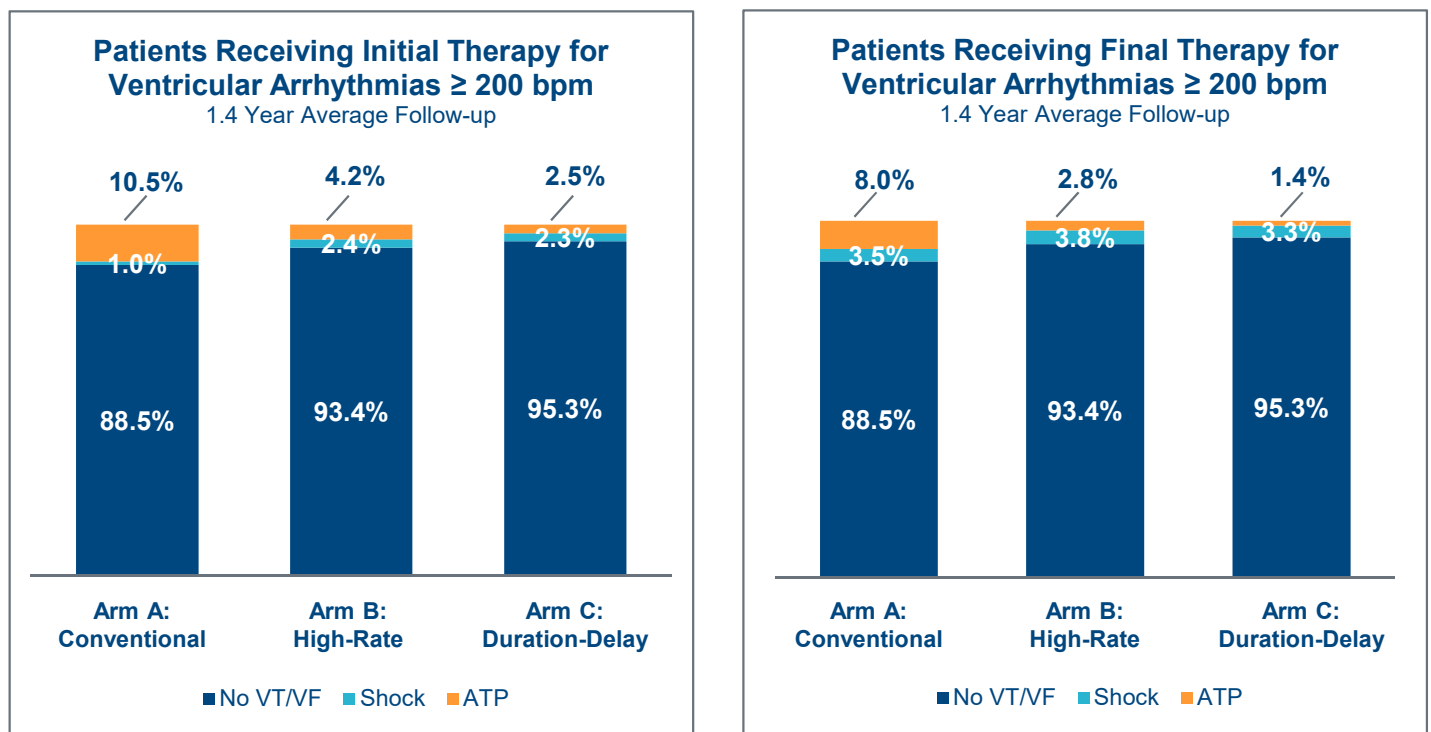

CONCLUSIONS

There was a statistically significant reduction in ATP for ventricular arrhythmias \geq 200 bpm when applying therapy delays, suggesting that many VTs were self-terminating.

Earlier interventions are likely unnecessary and may lead to an overestimation of the value of ATP in terminating these arrhythmias in primary prevention ICD patients.

There was no difference in final shock rate among arms despite the significant difference in use of ATP.

REFERENCES

Schuger, et al. Avoiding Unnecessary Therapy for Ventricular Arrhythmias \geq 200 bpm: Results from MADIT-RIT. S-PO03 at HRS 2019, San Francisco, CA.

ICD Systems – RESONATE™ HF, RESONATE™ EL, PERCIVA™ HF, PERCIVA™, VIGILANT™ EL, MOMENTUM™ EL

INDICATIONS AND USAGE

Boston Scientific implantable cardioverter defibrillators (ICDs) are intended to provide ventricular antitachycardia pacing (ATP) and ventricular defibrillation for automated treatment of life-threatening ventricular arrhythmias.

CONTRAINDICATIONS

Use of these Boston Scientific pulse generators are contraindicated for the following: patients whose ventricular tachyarrhythmias may have reversible cause, such as: digitalis intoxication, electrolyte imbalance, hypoxia, sepsis; or patients whose ventricular tachyarrhythmias have a transient cause, such as: acute myocardial infarction (MI), electrocution, drowning; or patients who have a unipolar pacemaker.

WARNINGS

Read this manual thoroughly before implantation to avoid damage to the pulse generator and/or lead. For single patient use only. Do not reuse, reprocess, or resterilize. Always have external defibrillation equipment available during implant and electrophysiologic testing. Ensure that an external defibrillator and medical personnel skilled in CPR are present during post-implant device testing should the patient require external rescue. Do not use this pulse generator with another pulse generator. Program the pulse generator Tachy Mode(s) to Off during implant, explant, or postmortem procedures to avoid inadvertent high voltage shocks. Do not kink, twist, or braid the lead with other leads as doing so could cause lead insulation abrasion damage or conductor damage. For leads that require the use of a Connector Tool, use caution handling the lead terminal when the Connector Tool is not present on the lead. Do not directly contact the lead terminal with any surgical instruments or electrical connections such as PSA (alligator) clips, ECG connections, forceps, hemostats, and clamps. Do not contact any other portion of the DF4–LLHH or DF4–LLHO lead terminal, other than the terminal pin, even when the lead cap is in place. Do not use atrial tracking modes in patients with chronic refractory atrial tachyarrhythmias. Tracking of atrial arrhythmias could result in ventricular tachyarrhythmias. Advise patients to seek medical guidance before entering environments that could adversely affect the operation of the active implantable medical device, including areas protected by a warning notice that prevents entry by patients who have a pulse generator. RESONATE HF, RESONATE, PERCIVA HF, PERCIVA, and VIGILANT devices with a DF4 right ventricular lead connection are considered MR Conditional. For these devices, unless all of the MRI Conditions of Use are met, MRI scanning of the patient does not meet MR Conditional requirements for the implanted system, and significant harm to or death of the patient and/or damage to the implanted system may result. All other devices covered by this manual are not MR conditional. Do not expose patients with non-MR conditional devices to MRI scanning. For potential adverse events applicable when the Conditions of Use are met or not met, refer to the MRI Technical Guide. Do not subject a patient with an implanted pulse generator and/or lead to diathermy. If desired, ensure that Patient Triggered Monitor is enabled prior to sending the patient home. Once the Patient Triggered Monitor feature has been triggered by the magnet and an EGM has been stored, or after 60 days have elapsed from the day that Store EGM was enabled, the patient should not apply the magnet.

PRECAUTIONS

For specific information on precautions, refer to the following sections of the product labeling: clinical considerations, sterilization and storage, implantation, device programming, environmental and medical therapy hazards, hospital and medical environments, home and occupational environments, follow up testing, explant and disposal, supplemental precautionary information.

POTENTIAL ADVERSE EVENTS

Based on the literature and on pulse generator and/or lead implant experience, the following alphabetical list includes the possible adverse events associated with the included devices: Air embolism; Allergic reaction; Bleeding; Bradycardia; Cardiac tamponade; Chronic nerve damage; Component failure; Conductor coil fracture; Death; Elevated thresholds; Erosion; Excessive fibrotic tissue growth; Extracardiac stimulation (muscle/nerve stimulation); Failure to convert an induced arrhythmia; Fluid accumulation; Foreign body rejection phenomena; Formation of hematomas or seromas; Heart block; Heart failure following chronic RV apical pacing; Inability to defibrillate or pace; Inappropriate therapy (e.g., shocks and antitachycardia pacing (ATP) where applicable, pacing; Incisional pain; Incomplete lead connection with pulse generator; Infection including endocarditis; Insulating myocardium during defibrillation with internal or external paddles; Lead dislodgement; Lead fracture; Lead insulation breakage or abrasion; Lead perforation; Lead tip deformation and/or breakage; Local tissue reaction; Loss of capture; Myocardial infarction (MI); Myocardial necrosis; Myocardial trauma (e.g., tissue damage, valve damage); Myopotential sensing; Oversensing/undersensing; Pacemaker-mediated tachycardia (PMT); Pericardial rub, effusion; Pneumothorax; Pulse generator migration; Shunting current during defibrillation with internal or external paddles; Syncope; Tachyarrhythmias, which include acceleration of arrhythmias and early, recurrent atrial fibrillation; Thrombosis/thromboemboli; Valve damage; Vasovagal response; Venous occlusion; Venous trauma (e.g., perforation, dissection, erosion); Worsening heart failure.

For a list of potential adverse events associated with MRI scanning, refer to the MRI Technical Guide

Patients may develop psychological intolerance to a pulse generator system and may experience the following: Dependency; Depression; Fear of premature battery depletion; Fear of a device malfunction.

Additionally, potential adverse events associated with the implantation of a coronary venous lead system include: Allergic reaction to contrast media; Breakage/failure of implant instruments; Prolonged exposure to fluoroscopic radiation; Renal failure from contrast media used to visualize coronary veins.

Refer to the product labeling for specific indications, contraindications, warnings/precautions and adverse events. Rx only.

(Rev D) 046774 AH

**Boston
Scientific**
Advancing science for life™

Rhythm Management
Boston Scientific Corporation
4100 Hamline Avenue North
St. Paul, MN 55112
Tel: 651.582.4000 Fax: 651.582.4166
Medical Professionals:
1.800.CARDIAC (227.3422)
Patients and Families:
1.866.484.3288
www.bostonscientific.com

©2019 Boston Scientific Corporation
or its affiliates. All rights reserved.

CRM-627902-AA