

Cardiology Order Form

IMAGEREADY™

MR-Conditional Pacing Systems

PATIENT NAME _____

D.O.B. _____

MODEL #s _____

PACEMAKER/CRT-P _____

ATRIAL LEAD _____

RV LEAD _____

LV LEAD _____

According to Boston Scientific's device labeling, the following Conditions of Use **MUST BE MET** for a patient with an ImageReady™ Pacing System to undergo an MR-Conditional scan. Adherence to the Conditions of Use **MUST BE VERIFIED** prior to each scan to ensure that the most up-to-date information has been used to assess the patient's eligibility and readiness for an MR-Conditional scan.

RESOURCES

Confirm that patient has a valid ImageReady™ MR-Conditional Pacing System by referring to the below resources.

- ▶ Boston Scientific MRI Technical Guide, ImageReady™ MR-Conditional Pacing System
- ▶ www.BostonScientific.com/imageready
- ▶ Boston Scientific MRI Hotline 1.844.4.BSC.MRI (1.844.427.2674)

OFF-LABEL MRI SCAN

My patient **DOES NOT HAVE** a valid ImageReady™ MR-Conditional Pacing System and/or **DOES NOT MEET** the Conditions of Use listed below. Because not all Conditions of Use have been met, the scan is off-label. BSC labeling warns of potential risks for off-label MRI scans and does not promote nor encourage this use. Use the Cardiology Order Form *Off-Label MRI Scan* to specify programming parameters during off-label MRI scans.

CONDITIONS OF USE

- Patient is implanted with a valid ImageReady™ MR-Conditional Pacing System.
- Pulse generator implant location is left or right pectoral region.
- At least six (6) weeks have elapsed since implantation and/or any lead revision or surgical modification of the MR-Conditional Pacing System.
- No other active or abandoned implanted devices, components, or accessories present, such as lead adapters, extenders, leads, or pulse generators.
- No evidence of a fractured lead or compromised pulse generator-lead system integrity.
- RA and RV leads programmed to bipolar pacing operation or pacing OFF.
- Pulse generator in MRI Protection Mode during scan.
- Patient **MEETS ALL** MRI Conditions of Use as stated above.

Program MRI Protection Mode during scan with these pacing parameters.

- Pacing OFF
- DOO Pacing Rate _____ PPM or 10 PPM above patient intrinsic
- VOO Pacing Rate _____ PPM or 10 PPM above patient intrinsic
- AOO Pacing Rate _____ PPM or 10 PPM above patient intrinsic
- Atrial Amplitude _____ V@ 1.0 ms PW
- RV Amplitude _____ V@ 1.0 ms PW
- LV Amplitude _____ V @ _____ ms PW
- MRI Protection Timeout 3 H 6 H 9 H 12 H 24 H (nominal) 48 H Time-out OFF

NOTE

Do not leave the pulse generator in MRI Protection Mode any longer than necessary following the scan.

DATE _____

PHYSICIAN SIGNATURE _____

PHYSICIAN NAME _____