

Cardiology Order Form

IMAGEREADY™

MR-Conditional Transvenous Defibrillation Systems

PATIENT NAME _____

D.O.B. _____

MODEL #s

ICD/CRT-D

ATRIAL LEAD

RV LEAD

LV LEAD

According to Boston Scientific's device labeling, the following Conditions of Use **MUST BE MET** for a patient with an ImageReady™ Transvenous Defibrillation System to undergo an MR-Conditional scan. Adherence to the Conditions of Use **MUST BE VERIFIED** prior to each scan to ensure that the most up-to-date information has been used to assess the patient's eligibility and readiness for an MR-Conditional scan.

RESOURCES

Confirm that patient has a valid ImageReady™ MR-Conditional Transvenous Defibrillation System by referring to the below resources:

- ▶ Boston Scientific MRI Technical Guide, ImageReady™ MR-Conditional Defibrillation System
- ▶ www.BostonScientific.com/imageready
- ▶ Boston Scientific MRI Hotline 1.844.4.BSC.MRI **(1.844.427.2674)**

OFF-LABEL MRI SCAN

- ☐ My patient **DOES NOT HAVE** a valid ImageReady™ MR-Conditional Transvenous Defibrillation System and/or **DOES NOT MEET** the Conditions of Use listed below. Because not all Conditions of Use have been met, the scan is off-label. BSC labeling warns of potential risks for off-label MRI scans and does not promote nor encourage this use. Use the Cardiology Order Form *Off-Label MRI Scan* to specify programming parameters during off-label MRI scans.

CONDITIONS OF USE

- ☐ Patient is implanted with ImageReady™ MR-Conditional Transvenous Defibrillation System with all ports occupied by a lead or port plug.
- ☐ No other active or abandoned implanted devices, components, or accessories present, such as lead adaptors, extenders, leads, or pulse generators.
- ☐ Pulse generator implant location restricted to left or right pectoral region.
- ☐ At least six (6) weeks have elapsed since implantation and/or any lead revision or surgical modification of the MR-Conditional Transvenous Defibrillation System.
- ☐ No evidence of a fractured lead or compromised pulse generator-lead system integrity.
- ☐ Patient is judged to be clinically capable of tolerating no Tachycardia protection for the entire duration in which the pulse generator is in MRI Protection Mode.
- ☐ Pulse generator is in MRI Protection Mode during scan.
- ☐ As soon as MRI Protection Mode is programmed, the patient must be continually monitored by pulse oximetry and electrocardiography (ECG). Ensure backup therapy is available (external rescue).
- ☐ Patient **MEETS ALL** MRI Conditions of Use as stated above.

MRI PROTECTION MODE PARAMETERS

Program MRI Protection Mode during scan with these pacing parameters.

☐ Pacing OFF

☐ DOO (Bipolar) Pacing Rate _____ PPM

☐ VOO (Bipolar) Pacing Rate _____ PPM

☐ AOO (Bipolar) Pacing Rate _____ PPM

Ventricular Pacing Chamber ☐ RV ☐ BiV

Atrial Amplitude _____ V@ 1.0 ms PW

RV Amplitude _____ V@ 1.0 ms PW

LV Amplitude _____ V@ _____ ms PW

MRI Protection Timeout ☐ 3 H ☐ 6 H (nominal) ☐ 9 H ☐ 12 H ☐ Time-out OFF

NOTE

Do not leave the pulse generator in MRI Protection Mode any longer than necessary following the scan. Beeper may no longer be usable following MRI scan.

DATE _____

PHYSICIAN SIGNATURE _____

PHYSICIAN NAME _____