


Boston Scientific

Advancing science for life™

LATITUDE™ NXT 4.0

Quick Reference Guide

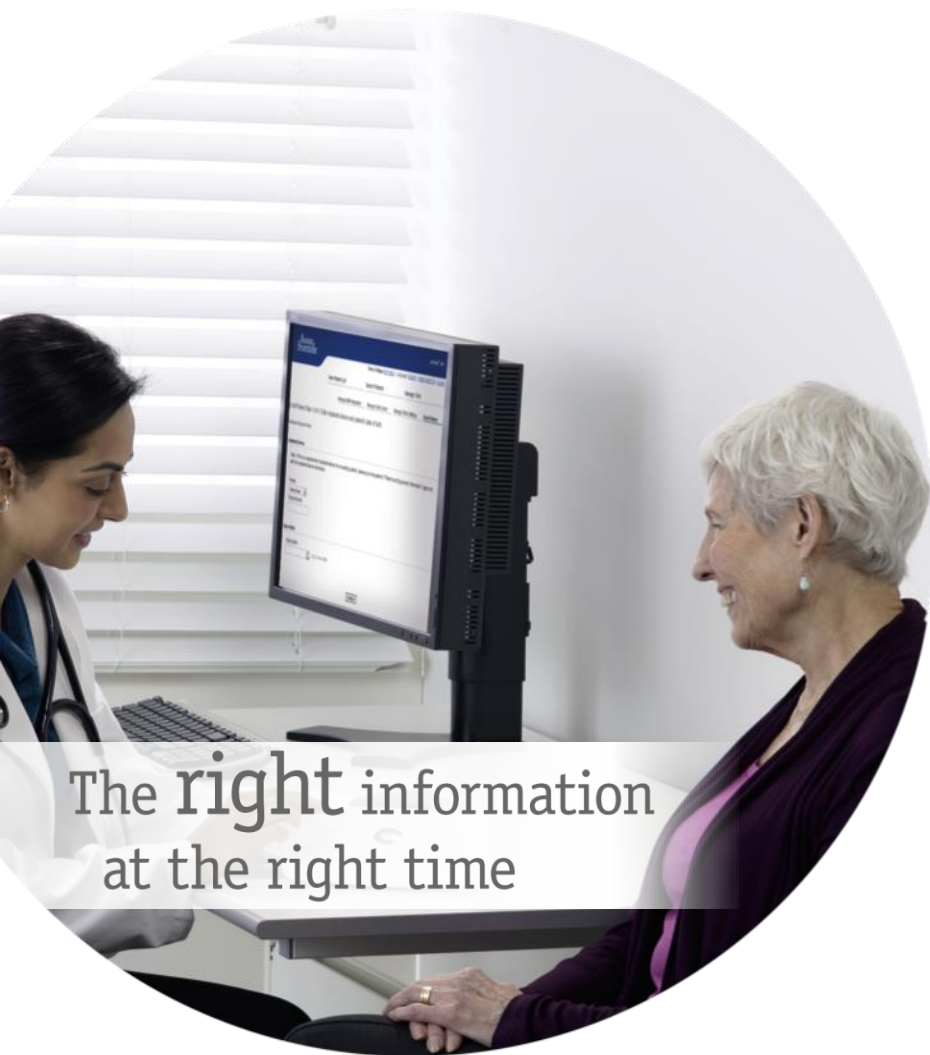

The **right** information
at the right time

LATITUDE™ NXT is compatible with Boston Scientific's most advanced pacemaker and defibrillator families¹, and now also with the EMBLEM™ sub-cutaneous defibrillator.



Definitions

- [Overview](#)

Getting Started

- [Website Login](#)
- [Enrolling a Patient](#)

Basic Navigation

- [Navigating the Site](#)
- [View Patient List](#)
- [Patient Search](#)
- [Managing Patient Group](#)
- [Add / Delete a Patient Group](#)

Alerts

- [Transvenous Devices](#)
- [Sub-cutaneous Devices](#)

Reviewing Data

- [Reviewing Data Online](#)
- [Transvenous Data](#)
- [Sub-cutaneous Data](#)
- [Reports](#)

Configuration

- [Scheduled Follow-Ups](#)
- [Patient-Initiated Interrogation](#)
- [Monitoring Schedule and Alerts](#)
- [Individual Configurations](#)

Additional Alert Notifications

- [Email and SMS](#)

Patient-Initiated Interrogation

- [Settings Up Preferences](#)

LATITUDE™ NXT Integration

- [Data Management Integration \(EMR\)](#)

LATITUDE™ NXT Communicator

- [Buttons](#)
- [Connectivity](#)
- [Troubleshooting](#)

Country Code Switches

- [Dial-Out Number](#)
- [Country Code](#)

LATITUDE™ Heart Failure Management

- [Sensors](#)

Support

- [Local LATITUDE Customer Support Hotline Numbers](#)

Quick Reference Guide:

Definitions

Clinic Account Manager (CAM) – Clinic user who is responsible for managing the clinic and the clinic’s users and patients. The CAM has access to all patients in all Patient Groups and can create User Accounts.

Dismiss Patient – Removes the patient from the **For Review** filtered list on the View Patient List webpage. Patient data are still accessible from the **All Patients** filtered list.

Latest Device Transmission – Column on the Patient List that displays the date of the last full interrogation.

LATITUDE™ Heart Failure Management (LHFМ) – Optional Weight Scale and Blood Pressure Monitor equipment that transmit measurements over a wireless connection to a patient’s Communicator.

Monitoring Status – Indicates the current monitoring status of the patient. It may also include a link to troubleshooting steps.

Next Remote Follow-up – Column on the Patient List that displays the date of the patient’s next scheduled remote follow-up.

Patient Group – A clinic can organize its patients into Patient Groups, which provide default alert and schedule configuration settings for their associated patients. Patient Groups allow the clinic to organize patients by criteria that make sense for their practice. Examples include physician name or satellite clinic.

Patient ID – An alphanumeric identifier assigned by the clinic to facilitate coordination of medical records. Clinics may add this on the Edit/View Patient and Equipment Information page, or upon enrollment for new LATITUDE™ NXT patients. This is required for enrollment.

Patient Initiated Interrogation (PII) – If enabled, patients are able to initiate an unscheduled interrogation of their implanted device by pressing the Heart button on the Communicator.

Quick Reference Guide:

Definitions



Red Alert – Implanted device conditions that could potentially leave the patient without available device therapy result in a red alert.

Review Status – Information on the status of the patient’s review. Statuses include New Data, Viewed, or Dismissed.

Review Reason – Indicates the reason the patient was added to the For Review patient list. Valid review reasons include Scheduled, Patient Initiated, Weight Change, Device Alert, and Multiple Reasons. Click the reason to see more detailed information.

Getting Started

1. Work with your sales representative to complete the Clinic / CAM Enrollment Form.
1. Clinic account managers (CAMs) will receive a temporary user ID and password from a Boston Scientific sales representative. Clinic users will receive a temporary user ID and password from their clinic account manager.
2. Navigate to www.latitudelogin.com.
3. If this is the first time you are accessing the LATITUDE™ NXT website, select your country and your preferred language.



1. Click the LATITUDE™ NXT button
2. Enter your user ID and password and click the login button.

LATITUDE™ NXT

Please enter your User ID and Password to enter the Boston Scientific LATITUDE Clinician website

*Indicates Required Field

*User ID:

*Password:

[Change Country/Language](#): United Kingdom/English
[Forgot Password?](#)

For additional assistance, call [LATITUDE™ Customer Support](#)

7. If you are a first-time user, you will be prompted to change your temporary password and complete a set of personal security questions.

CHANGING YOUR PASSWORD

1. Click the **My Profile** link next to your name at the top of any page on the website.
2. Click the **Change Password** link in the Login Information section.
3. Enter your old and new password.
4. Click the **Save** and **Close** button.

Enrolling a Patient

1. Click the **Manage Clinic** link from the top navigation bar.
2. Click the **Enroll Patient** link.

Boston Scientific LARITUDE™ 1001

Rivas, Philippe (p.rivas) | Language: English | Help/Contact Us | Logout

View Patient List Search Patients Manage Clinic

Manage EMR Integration Manage Clinic Users Manage Clinic Settings **Enroll Patient**

Enroll Patient Step 1 of 4: Enter implanted device and patient's date of birth.

*Indicates Required Field

Implanted Device

Note: If this is a replacement implanted device for an existing patient, please go to the patient's "Patient and Equipment Information" page and edit the implanted device information.

*Model
-Select Model-

*Serial Number:

Date of Birth

*Date of Birth: (e.g.: 24 Jan 1950)

3. Select the patient's device model from the drop-down and type in the patient's device serial number.
4. Enter in the patient's date of birth.
5. Click **Continue**.


6. Enter patient information in the designated fields. Note: Patient phone number is not mandatory but strongly recommended in order to facilitate troubleshooting.
7. Assign a Patient Group from the Patient Group Membership drop-down menus. (See Patient Group Tab for instructions on creating patient groups).
8. Select the Communicator model and fill in the Communicator Serial Number, found on the outside of the box or underside of the communicator.
9. If the patient has been prescribed a weight scale or blood pressure monitor, those model and serial numbers may be entered at enrollment or later through the Edit / View Patient and Equipment Information pages.
10. Check the box to accept confirming Patient Consent and click Continue.
11. Click the Enroll Patient button.
12. You may choose to print the Confirmation Page by clicking on the "Print Confirmation Page" button.

Basic Navigation

The new LATITUDE™ NXT website offers you greater control over how you receive, review, and manage patient and device data.

NAVIGATING THE SITE

Top banner and navigation buttons that appear on all web pages.

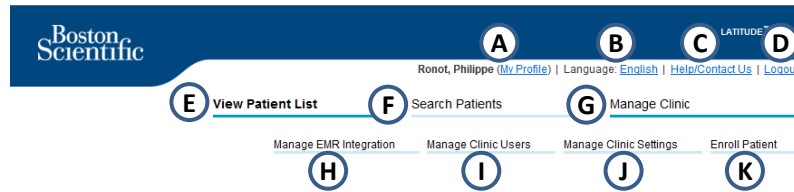

- A. Link to the Update Clinic User page for the current user.
- B. Display selected language. Links to the Configuration profile page where the selected language can be changed.
- C. Link to contact information and other resources.
- D. Ends the user's session.
- E. Link to a page that provides a list of patients to which the current user has access. (The View Patient List Page)
- F. Link to the Patient Search page.

- G. Displays links related to Managing your Clinic (H–K).
- H. Link to a page that enables clinicians to configure integration with their clinic's EMR system.
- I. Provides a list of clinic users and associated configuration information.
- J. Manage clinic and Patient Group demographics and membership and associated configuration information.
- K. Link to enrollment form that enables clinicians to enroll new patients.

View Patient List Page

The View Patient List Page displays all patients for all your assigned Patient Groups.

You can choose to display patients according to their Patient Group using the “Viewing Patient Group” drop-down menu. **(A)**

You can also further filter patients using the following Filters: **(B)**

All Patients – Lists all patients in the selected Patient Group(s).

For Review – Lists patients with reviewable data that have not been dismissed. This includes data associated with alerts, remote scheduled follow-ups or patient-initiated interrogations.

Missed Follow-up – Lists patients who had a scheduled remote follow-up but a remote interrogation has not yet been completed successfully.

Not Monitored – Lists patients who currently are not being monitored.

Not Scheduled – Lists patients who currently do not have a scheduled remote follow-up.

View Patient List

Viewing Patient Group: All Patient Groups

All Patients (5) | For Review (4) | Missed Follow-up (0) | Not Monitored (2) | Not Scheduled (0)

Patient ID/Device	Review Status	Latest Device Transmission	Alerts	Review Reason	Next Remote Follow-up	Monitoring Status	Actions
NG3 PG Genn, Otto AUTOGEN X4 CRT-D G179	Viewed	08 Jul 2013	🔴	Multiple Reasons	02 Sep 2013	Monitored	<input checked="" type="checkbox"/> Dismiss From Review List Reports Menu
F100001 Intuition, Poor INTUA CRT-P W273	New Data	08 Jul 2013	🟡	Multiple Reasons	02 Sep 2013	Monitored	<input checked="" type="checkbox"/> Dismiss From Review List Reports Menu
T-Teligan 0058 Teligen, Trevor TELIGEN 100 F110	Viewed	03 Jul 2013	🟡	Multiple Reasons	17 Jul 2013	Monitored	<input checked="" type="checkbox"/> Dismiss From Review List Reports Menu
SS1 Cogswell, Cognis COGNIS 100-D P107	Viewed	10 Jun 2013	None	Multiple Reasons	02 Sep 2013	Implanted Device Not Found	<input checked="" type="checkbox"/> Dismiss From Review List Reports Menu

NOTE

As soon as patients appear in the "Not Monitored" tab, it is very important to contact patient and/or LATITUDE™ Customer support for troubleshooting.

View Patient List Page

The Patient List itself is organized into columns: **(C)**

- **Patient ID/ Patient / Device** column displays the patient's ID / name. Click the patient ID / name to see detailed information for that patient.
- The **Review Status** column provides information on the status of the patient's review such as New Data, Viewed, or Dismissed. Click the status to see additional detail.
- The **Latest Device Transmission** column displays the date of the last full interrogation.
- The **Alerts** column indicates the patient has one or more alerts that have not been dismissed.
 - Click the flag to see a detailed list of alerts.
- The **Review Reason** column indicates the reason the patient was added to the For Review patient list. Valid review reasons include: Scheduled, Patient Initiated, Weight Change, Device Alert and Multiple Reasons. Click the reason to see more detailed information related to the review.
- The **Next Remote Follow-Up** column displays the date of the patient's next scheduled remote follow-up. Click the date to set a new date for a patient's next scheduled remote follow-up.

- The **Monitoring Status** column indicates the current monitoring status of the patient.
- The **Actions** column provides links to dismiss the patient from the review list or generate reports for the patient.

PATIENT SEARCH

To search for a patient record:

1. From any page, click **Search Patients** link from the top navigation bar.
2. Enter your desired search criteria.
3. Click **Search**.

The search results will be displayed below the search criteria in a table similar to the View Patient List web page. From the table, you can print the search results list, access details of a patient appearing in the list, print reports for a selected patient(s) or dismiss a patient that is for review.

Managing Patient Groups

1. A clinic is organized into one or more Patient Groups.
2. Clinic users can be assigned to any number of Patient Groups.
3. If assigned to a Patient Group, a clinic user has access to all patients enrolled in that Patient Group.
4. Patient Groups provide default alert and schedule configuration settings for their associated patients. A patient can have customized settings that differ from the Patient Group default settings.
5. Clinics that organize patients into Patient Groups have the option of associating a patient with two Patient Groups.
6. One Patient Group is primary and manages the patient's device.
7. The other Patient Group is secondary and also monitors the patient's condition but will not be notified of red alerts and will not see any follow-up or alert history associated with the primary Patient Group.

Clinic Account Managers have access to all patients in all Patient Groups and can add or remove Patient Groups.

Within LATITUDE™ NXT, Clinic Users can change patient demographic and equipment information, change patient groups, transfer patients and unenroll patients. Refer to the LATITUDE™ Clinician Manual for more information.

NOTE

If is recommended to start on LATITUDE™ with a limited set of patient groups, and to eventually add new ones when the number of enrolled patents has significantly increased.

Managing Patient Groups

TO ADD A PATIENT GROUP:

1. Click the **Manage Clinic** link from the top navigation bar.
2. Click the **Manage Clinic Settings** link.
3. Click the Add **Patient Group** button.
4. Fill out required fields and assign clinic members that will have access to the patients in this Patient Group.
5. Click **Save and Close**.

Edit/View Demographics and User Membership: TACHY_VA_White_River_Junction_2505990

*Indicates Required Field

Demographics

*Name:
TACHY_VA_White_River_

Description:

Clinic Member Assignment

Note: The assigned clinic members will have access to the patients followed by this group. The type of access is dependent on the clinic member's privileges

Members:
 Agte, Prathi (Complete Access)
 Baird, Doug (Limited Access)
 Bell, Bill (Complete Access)
 Campbell, Dean (Limited Access)
 Hubbert, Susan (Complete Access)
 Radhikani, Madhav (Complete Access)
 Reister, Craig (Complete Access)
[Select All](#) / [Deselect All](#)

TO DELETE A PATIENT GROUP:

1. Click the **Manage Clinic** link from the top navigation bar.
2. Click the **Manage Clinic Settings** link.
3. Click the **Edit / View Demographics and User**
4. Membership button associated with the Patient Group. Click the **Remove Patient Group** button.

NOTE

Patient Groups can only be removed if there are no patients enrolled in the Patient Group.

Alerts (transvenous devices)



LATITUDE™ NXT features Daily Alert Monitoring with optional SMS text message and e-mail alert notification, and parameters which allow you to configure yellow alerts on / off without bringing the patient in the clinic.

LATITUDE™ Red Alerts

Implanted device conditions that could potentially leave the patient without available device therapy result in a red alert. Red alert notifications occur for the following conditions:

BATTERY

- Remote monitoring disabled due to limited battery capacity

RIGHT VENTRICULAR LATITUDE LEAD CHECK+™

- Shock lead impedance out of range
- Low shock lead impedance detected when attempting to deliver a shock
- High shock lead impedance detected when attempting to deliver a shock
- Right ventricular or single chamber pacing lead impedance out of range
- Right ventricular pacing lead impedance abrupt change*
- Right ventricular non-physiologic signal detected*

* These alerts are nominally off and can only be configured at the patient level

TACHY MODE

- V-Tachy mode set to other than Monitor + Therapy

OTHERS

- Possible device malfunction
- High voltage detected on shock lead during charge
- Device in Safety Mode
- Device in Electrocautery Protection mode



Individually optional & customized Alerts without required Patient presence

BATTERY

- Explant indicator reached
- Voltage too low for projected remaining capacity

VENTRICULAR/SINGLE CHAMBER PACING LEADS

- Right Ventricular or single chamber intrinsic amplitude out of range
- Right ventricular automatic threshold as > programmed amplitude or suspended
- Left ventricular intrinsic amplitude out of range
- Left ventricular pacing lead impedance out of range
- Left ventricular automatic threshold as > programmed amplitude or suspended

ATRIAL PACING LEADS

- Atrial intrinsic amplitude out of range
- Atrial pacing lead impedance out of range
- Atrial automatic threshold detected as > programmed amplitude or suspended

MORE DETAILS

ARRHYTHMIAS

- Shock therapy delivered to convert arrhythmia
- Accelerated arrhythmia episode
- VT Episodes (V > A)
- Atrial Arrhythmia burden > 0, 0.5, 1, 3, 6, 12, 18, 24 hours
- Patient triggered event Stored

CARDIAC RESYNCHRONIZATION % PACING

- <50, 60, 70, 80, 85, 90, 95 %

RIGHT VENTRICULAR % PACING

- >10, 20, 30, 40, 50 %

OTHERS

- Device Brady Mode Off
- Therapy history corruption detected
- Configurable weight gain alert (xxx kg gain within yyy days)

About all Red and Yellow alerts can be found on the following Boston Scientific website: [Product Resources](#)

NOTE

For sub-cutaneous devices, LATITUDE™ NXT features Weekly Alert Monitoring with optional SMS text message and e-mail alert notification, and parameters which allow you to configure yellow alerts on / off without bringing the patient in the clinic

NON CONFIGURABLE RED ALERTS

- 🚩 Device battery has reached End of Life (EOL)
- 🚩 High electrode impedance
- 🚩 Therapy Off
- 🚩 Possible device malfunction

CONFIGURABLE YELLOW ALERTS

- 🚩 Device battery has reached Elective Replacement Indicator
- 🚩 Shock therapy delivered to convert arrhythmia
- 🚩 Untreated episode
- 🚩 Sensing not fully optimized


- **PATIENT SAFETY**
- **FLEXIBILITY IN RECEIVING THE INFORMATION**
- **WEEKLY FOLLOW UP**



Reviewing Data online

TO REVIEW PATIENT DATA ONLINE:

1. Click the [View Patient List](#) link from the top navigation bar.
2. Click the Patient ID hyperlink

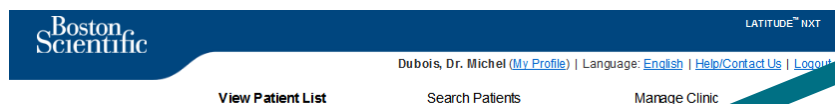

View Patient List

Viewing Patient Group: All Patient Groups

[All Patients \(10\)](#) | [For Review \(5\)](#) | [Missed Follow-up \(2\)](#) | [Not Scheduled \(3\)](#) | [Not Scheduled \(1\)](#)

Patient ID/ Patient/ Device	Review Status	Latest Device Transmission	Alerts	Review Reason	Next Remote Follow-up	Monitoring Status	Actions
55714093 Bertrand, Yves IN/IVE CRT-P W173 Hospital du Val Dieu (Primary)	Viewed	24 Jun 2014		Multiple Reasons	27 Aug 2014	Monitored	Dismiss From Review List Reports Menu
69021930 Martin, Isabelle COGNIS 100-D P107	Viewed	24 Jun 2014		Implanted Device Alert	27 Aug 2014	Monitored	Dismiss From Review List Reports Menu
32101221 Petit, Philippe INGENIO J173	Viewed	24 Jun 2014		Implanted Device Alert	22 Aug 2014	Monitored	Dismiss From Review List Reports Menu
29560849 Laurent, Marie INGENIO J173	Viewed	24 Jun 2014	None	Patient Initiated	22 Sep 2014	Monitored	Dismiss From Review List Reports Menu
27946795 Silva, Max DYNA GEN MIN ICD D022 Hospital du Val Dieu (Primary)	Viewed	24 Jun 2014	None	Scheduled	30 Sep 2014	Monitored	Dismiss From Review List Reports Menu

Patient ID: 55714093

Latest Device Transmission: 24 Jun 2014 06:02 WEST

Patient: Bertrand, Yves
Device: W173 IN/VE CRT-P W173
Date of Birth: 11 Aug 1951

Patient Group: Hospital du Val Dieu (Primary)
Search Tags: LHFH
Monitoring Status: Monitored

Actions: [Edit/View Schedule and Alert Configuration](#), [Edit/View Patient and Equipment Information](#), [Reports Menu](#), [Dismiss From Review List](#)

Navigation: [View Patient List](#) | [Search Patients](#) | [Manage Clinic](#)

Summary | Leads | Events | Health | Follow-up History

My Alerts


- 24 Jun 2014 06:02 WEST Cardiac Resynchronization Therapy pacing of < 95%. Pacing was 92% between 23 Jun 2014 06:02 and 24 Jun 2014 06:02 WEST.
- 24 Jun 2014 06:02 WEST Weight gain of at least 2.27 kg in a week or at least 0.91 kg average over a two or more day period.

For Technical Support contact 1-800-CARDIAC® (Americas) or your local Boston Scientific Representative.

Leads (View detail)	Most Recent In-Office Measurement (28 May 2014)	Most Recent Daily Measurement (23 Jun 2014)	Health (View detail)	Most Recent Daily Measurement (23 Jun 2014)
Atrial			External Sensors	
Intrinsic Amplitude	1.6 mV	3.8 mV	Weight	74 kg (24 Jun 2014)
Pace Impedance	605 Ω	517 Ω	Blood Pressure	152/83 mmHg (23 Jun 2014)
Pace Threshold	1.2 V @ 0.4 ms			
Right Ventricular			Implanted Device Measures	
Intrinsic Amplitude	10.0 mV	10.9 mV	Mean Heart Rate	88 min ⁻¹
Pace Impedance	481 Ω	445 Ω	Median Respiratory Rate	15 min ⁻¹
Pace Threshold	1.1 V @ 0.4 ms		Activity Level	5 % of day
Left Ventricular			HRV Footprint	40 %
Intrinsic Amplitude	8.0 mV	7.4 mV	SDANN	56 ms
Pace Impedance	729 Ω	675 Ω	Autonomic Balance Monitor	2.60
Pace Threshold	1.2 V @ 0.8 ms		Atrial Burden Total Time	0.0 hour(s)
			AP Scan™	55 events/h

Reviewing Data online Events –

Transvenous Devices

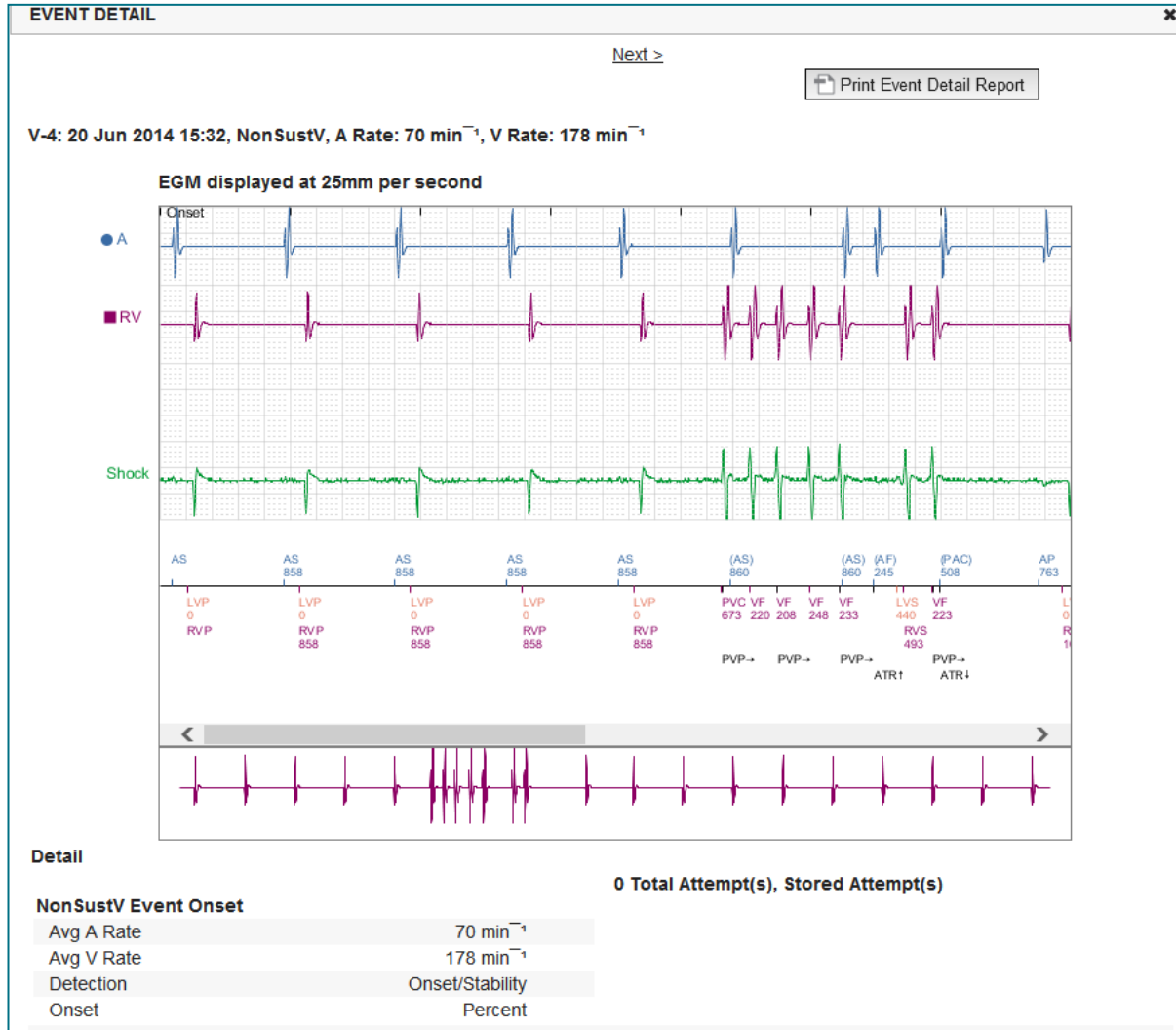

Reviewing Data online Events –

Sub-cutaneous Devices

EVENT DETAIL x

[Next >](#) Print Event Detail Report

007: Jul 13, 2015 13:24 CDT, Treated, Shock Impedance: 75 Ω

S-ECG displayed at 25 mm/sec 2.5 mm/mV

T T T T T T T T T T T S S E C E S S S T S S

< >

Tachy Therapy Settings		
Therapy	ON	S = Sense
Shock Zone	220 bpm	P = Pace
Conditional Shock Zone	200 bpm	N = Noise
Additional Device Settings		T = Tachy Detection
Post Shock Pacing	OFF	C = Charge Start
Gain Setting	1X	E = Charge End
Sensing Configuration	Alternate	• = Discard
Final Shock Polarity	REV	♥ = Event End

Reviewing Data on reports

		Reviewing Data Through Printed Reports / Which Reports to Print					
Typical Follow-up information Requested	Reviewing Data Online	QUICK NOTES™	Most Recent Presenting EGM / S-ECG	Combined Follow-up	HF PERSPECTIV™	Arrhythmia Logbook	Device Settings
Presenting EGM / S-ECG	Summary Tab, click the Presenting EGM / S-ECG link		✓	✓			
Lead Data	Summary Tab, click the Leads link	✓		✓	✓		
Battery Status	Summary Tab, click the Battery link	✓		✓	✓		
Events	Summary Tab, click the Events link or Events Tab	✓		✓		✓	
Counters	Summary Tab, Event Counts / Percent Paced			✓			
Histograms	Review on Report			✓			
Activity Level	Summary Tab or Health Tab			✓			
Heart Rate Variability (HRV) Data	Summary Tab or Health Tab			✓			

Transvenous and Subcutaneous devices

Transvenous devices only

Reviewing Data on reports

		Reviewing Data Through Printed Reports / Which Reports to Print					
Typical Follow-up information Requested	Reviewing Data Online	QUICK NOTES™	Most Recent Presenting EGM / S-ECC	Combined Follow-up	HF PERSPECTIV™	Arrhythmia Logbook	Device Settings
Device Settings	Summary Tab (print report for full settings)	✓		✓	✓		✓
Alert(s)	Summary Tab or Follow-up History Tab	✓		✓	✓		
Weight and Blood Pressure	Summary Tab or Health Tab	✓ <small>S-ICD devices only</small>		✓	✓		
Respiratory Rate Trend	Summary Tab or Health Tab			✓	✓		
AP Scan	Summary Tab or Health Tab			✓	✓		

Transvenous and Subcutaneous devices

Transvenous devices only

CONFIGURING SCHEDULED FOLLOW-UPS FOR PATIENT GROUPS

1. Select **Manage Clinic** from the top navigation bar.
2. Select **Manage Clinic Settings**.
3. Within the selected Patient Group, Click **Edit / View Schedule and Alert Defaults**.
4. Decide if you want **Automatic** (recurring intervals) or **Manual Follow-ups** and select the corresponding radio button.
 - For **Automatic Follow-Ups**, select from a list of **Interval Frequency Options** and **Days of the Week Options**:
 - **Interval Frequency Options**: Once per week, once every two weeks, once per month, and month intervals up to Once every 12 months
 - **Days of the Week Options**: Monday, Tuesday, Wednesday, Thursday, Friday
 - For **Manual Follow-ups**, you may pick a new date each time the previous one has been completed
5. Click **Save** and **Close**.

The screenshot shows the Boston Scientific LATITUDE NXT user interface. At the top, the user is identified as Ronot, Philippe, with options for language and logout. The navigation bar includes 'View Patient List', 'Search Patients', 'Manage Clinic', 'Manage EMR Integration', 'Manage Clinic Users', 'Manage Clinic Settings' (which is active), and 'Enroll Patient'. The main content area is titled 'Edit/View Schedule and Alert Defaults: TACHY_VA_White_River_Junction_2505990'. It features three sections: 'Remote Scheduled Follow-ups' with radio buttons for 'Automatically set the next remote follow-up based on the following interval' (selected) and 'Manually set the next remote follow-up'. The automatic option includes dropdowns for 'Frequency' (set to 'Once Every Three Months') and 'Day of the Week' (set to 'Monday'). The 'Weekly Implanted Device Alert Monitoring' section has a checked checkbox for 'Enable weekly yellow alert monitoring' and a note about its applicability. The 'Patient Initiated Interrogations' section has a checked checkbox for 'Enable patient initiated interrogations'. The 'Alert Configuration' section has a note about alert frequency determination.

CONFIGURING PATIENT-INITIATED INTERROGATIONS FOR PATIENT GROUPS

1. Select **Manage Clinics** from the top navigation bar.
2. Select **Manage Clinic Settings**.
3. Within the selected Patient Group, Click **Edit / View Schedule and Alert Defaults**.
4. Scroll to the Patient Initiated interrogations section.
5. Turn on and off this setting by clicking the check box next to Enable patient initiated interrogations (allows five per week).
6. Click **Save** and **Close**.

The screenshot shows the Boston Scientific LATITUDE NXT web interface. The top navigation bar includes the Boston Scientific logo, the user name 'Ronot, Philippe (My Profile)', and language options 'English', 'Help/Contact Us', and 'Logout'. Below the navigation bar are several menu items: 'View Patient List', 'Search Patients', 'Manage Clinic', 'Manage EMR Integration', 'Manage Clinic Users', 'Manage Clinic Settings' (which is highlighted), and 'Enroll Patient'. The main content area is titled 'Edit/View Schedule and Alert Defaults: TACHY_VA_White_River_Junction_2505990'. It contains three sections: 'Remote Scheduled Follow-ups' with a radio button selected for 'Automatically set the next remote follow-up based on the following interval:' and dropdowns for 'Frequency: Once Every Three Months' and 'Day of the Week: Monday'; 'Weekly Implanted Device Alert Monitoring' with a checked checkbox for 'Enable weekly yellow alert monitoring (for configured implanted device alerts)' and a note about applicability; and 'Patient Initiated Interrogations' with a checked checkbox for 'Enable patient initiated interrogations (allows 5 per week)'. There is also an 'Alert Configuration' section with a note about alert frequency.

CONFIGURING MONITORING SCHEDULE AND ALERTS FOR PATIENT GROUPS

1. Select **Manage Clinics** from the top navigation bar.
2. Select **Manage Clinic Settings**.
3. Within the selected Patient Group, Click **Edit / View Schedule and Alert Defaults**.
4. Scroll to the Alert Configuration section.
5. Use the check boxes to select or deselect the alerts you want or don't want to receive.
6. Click **Save** and **Close**.

Boston Scientific LATITUDE[™] ICD1

Monok, Philippe (My Profile) | Language: English | Help/Contact Us | Logout

View Patient List Search Patients Manage Clinic

Manage EHR Integration Manage Clinic Users **Manage Clinic Settings** Enroll Patient

Edit/View Schedule and Alert Defaults: Dragon 2 Test Clinic Group

Remote Scheduled Follow-up [View Updated By](#)

- Automatically set the next remote follow-up based on the following interval:
Frequency:
Day of the Week:
- Manually set the next remote follow-up.

Weekly Implanted Device Alert Monitoring [View Updated By](#)

- Enable weekly yellow alert monitoring (for configured implanted device alerts)
 Note: This option only applies to Communicator models that support weekly yellow alert monitoring, as opposed to daily yellow alert monitoring.

Patient Initiated Interrogations [View Updated By](#)

- Enable patient initiated interrogations (allow 5 per week)

Alert Configuration [View Updated By](#)

Note: Frequency of alert monitoring (weekly or daily) is determined by the patient's assigned Communicator.
Note: Red alerts only apply if this patient group is selected as a patient's primary group.
Note: Alerts that are enabled will only be received for a particular patient if the patient's implanted device supports the alert and is programmed appropriately to detect it. See patient level configuration for details.

Battery

- Remote monitoring disabled due to limited battery capacity.
- Signal indicator reached.
- Voltage too low for projected remaining capacity.

Right Ventricular LATITUDE[™] Lead Check[™]

- Shock lead impedance out of range.
- Low shock lead impedance detected when attempting to deliver a shock.
- High shock lead impedance detected when attempting to deliver a shock.
- Right ventricular or single chamber pacing lead impedance out of range.
- Right ventricular pacing lead impedance abrupt change.
- Right ventricular non-physiologic signal detected.
- Right ventricular or single chamber intrinsic amplitude out of range.
- Right ventricular automatic threshold detected as programmed amplitude or suspended.

CONFIGURING SCHEDULES AND PATIENT-INITIATED INTERROGATIONS FOR INDIVIDUAL PATIENTS

1. From the View Patient List, select All Patients.
2. Scroll to the patient you want to configure (or use the Search Patients function).
3. Click the patient ID.
4. From the Summary Page, select Edit / View Schedule and Alert Configuration.
5. Deselect the Use Patient Group Defaults checkbox.
6. Scroll to settings and make appropriate changes (similar to changing preferences for patient groups).
7. Click Save and Close.

The screenshot displays the 'Edit/View Schedule and Alert Configuration' page for patient NG3 PG. The interface includes a navigation bar with 'View Patient List', 'Search Patients', and 'Manage Clinic'. The patient information section shows: Patient ID: NG3 PG, Latest Device Transmission: 08 Jul 2013 00:51 CEST, Patient: Genn, Obo, Device: G179 AUTOGEN X4 CRT-D057318, Date of Birth: 03 Apr 2013, Patient Group: Dragon 2 Test Clinic Group (Primary), Search Tags: None, and Monitoring Status: Monitored. The 'Next Scheduled Remote Follow-up' is set for 02 Sep 2013. Under 'Remote Scheduled Follow-ups', the 'Use Patient Group Defaults' checkbox is checked, with 'Automatically set the next remote follow-up based on the following Interval' selected. The frequency is set to 'Once Every Three Months' and the day of the week is 'Monday'. The 'Patient initiated interrogations' section has 'Allow one Patient Initiated Interrogation' checked, and 'Use Patient Group Defaults (Enabled)' is checked, with 'Enable patient initiated interrogations (allows 5 per week)' also checked. The 'Alert Configuration' section has 'Use Patient Group Defaults (shown below as ■)' checked.

Additional Alert Notifications

Primary Alert notification is through the View Patient List on the LATITUDE™ NXT website. However, physicians may opt to receive additional alert notification through SMS text message and e-mail. These options may be configured at the Patient Group level.

1. Select Manage Clinic from the top level navigation bar.
2. Select Manage Clinic Settings.
3. Within the selected Patient Group, Click Edit / View Schedule and Alert Defaults.
4. Scroll to the Additional Alert Notification section.
5. Select which type of Alerts you would like to receive:
 - Red Alerts
 - Yellow Alerts
6. Select Hours of Notification:
 - 8 AM – 5 PM Monday – Friday
 - 24 hours a day, 7 days a week
7. Enter up to three e-mail addresses and three SMS text message numbers.
8. Send a test message to each address / number to verify receipt.
9. Click Save and Close.

Additional Alert Notification

[Last Updated By](#)

Note: The primary means of alert notification is through the Patient List page. Additional notification through SMS text messages and E-mail can be selected below. As this additional notification is dependent on external systems, delivery may be delayed or not occur at all.

Send additional alert notification based on the following settings:

Alert Severity:

- Red Alerts
- Yellow Alerts

Hours of Notification:

- 8 AM - 5 PM, Monday - Friday
- 24 hours a day, 7 days a week

Email Addresses

(e.g. john@doe.com)

Send a Test Message

(e.g. john@doe.com)

Send a Test Message

(e.g. john@doe.com)

Send a Test Message

SMS Text Message Number(s)

(country code + number, e.g. 33 01 23 45 67 89)

Send a Test Message

(country code + number, e.g. 33 01 23 45 67 89)

Send a Test Message

(country code + number, e.g. 33 01 23 45 67 89)

Send a Test Message

Note: Changes to the schedule or alert configuration can take up to seven (7) days to take effect in the patient's communicator.

Patient Initiated Interrogation

The LATITUDE™ Communicator supports unscheduled Patient Initiated Interrogations (PII). Clinicians may enable or disable this feature from the LATITUDE™ NXT website. If enabled, patients can send up to five per week. If disabled or the weekly limit is reached, the clinician can allow one PII by selecting the “Allow one Patient Initiated Interrogation” button on the patient’s Schedule and Alert Configuration page.

- If enabled, patients may press the Heart button on the Communicator to send a full interrogation, including a Presenting EGM.
- If this feature is disabled or the limit has been reached, the Communicator will not perform a PII.
- If disabled, the PII setting may be changed and the patient will be able to perform a PII immediately.

PII preference may be set for all patients in a patient group or for individual patients

TO SET PREFERENCE FOR A PATIENT GROUP:

1. Select [Manage Clinic](#) from the top navigation bar.
2. Select [Manage Clinic Settings](#).
3. Within the selected Patient Group, Click [Edit / View Schedule and Alert Defaults](#).
4. Scroll to the Patient Initiated interrogations section.
5. Turn on and off this setting by clicking the check box next to Enable patient-initiated interrogations (allows five per week).
6. Click [Save](#) and [Close](#).

Patient Initiated Interrogation

TO SET A PREFERENCE FOR AN INDIVIDUAL PATIENT:

1. From the View Patient List, select All Patients.
2. Scroll to the patient you want to configure (or use the Search Patients function).
3. Click the patient ID.
4. From the Summary Page, select Edit / View Schedule and Alert Configuration.
5. Deselect the Use Patient Group defaults checkbox.
6. Scroll to settings and make appropriate changes (similar to changing preferences for patient groups).
7. Click **Save** and **Close**.

The screenshot displays the Boston Scientific LATITUDE NXT user interface. At the top, the user is identified as Ronot, Philippe, with options for language and profile. Navigation tabs include 'View Patient List', 'Search Patients', and 'Manage Clinic'. The main content area is titled 'Edit/View Schedule and Alert Configuration' and shows details for Patient ID: T-Teligen 0058. Key information includes the latest device transmission date (03 Jul 2013 04:01 BST), patient name (Teligen, Trevor), device type (F110 TELIGEN), and monitoring status (Monitored). Below this, there are sections for 'Remote Scheduled Follow-ups' and 'Patient Initiated Interrogations'. The 'Patient Initiated Interrogations' section is currently expanded, showing a checked option to 'Allow one Patient Initiated Interrogation' and another checked option to 'Use Patient Group Defaults (Enabled)', which includes the sub-option 'Enable patient initiated interrogations (allows 5 per week)'.

Patient ID: T-Teligen 0058

Latest Device Transmission: 03 Jul 2013 04:01 BST

Patient: Teligen, Trevor **Patient Group:** Dragon 2 Test Clinic Group (Primary)

Device: F110 TELIGEN **Search Tags:** None

Date of Birth: 31 Dec 1950 **Monitoring Status:** Monitored

Next Scheduled Remote Follow-up: 17 Jul 2013

Remote Scheduled Follow-ups

Use Patient Group Defaults (Once Every Three Months on Monday)

Automatically set the next remote follow-up based on the following interval:

Frequency:

Day of the Week:

Manually set the next remote follow-up.

Patient Initiated Interrogations

Allow one Patient Initiated Interrogation

Use Patient Group Defaults (Enabled)

Enable patient initiated interrogations (allows 5 per week)

EMR Data Integration

The LATITUDE™ NXT EMR system integration feature provides an automated way to export patient device data to a clinic's internal EMR system.

- The Installation of the required software called LATITUDE™ NXT Integration may require the assistance of the IT professional responsible for administering your clinic systems.
- LATITUDE™ NXT exports data using Health Level Seven (HL7) file format standard as well as the IDCO profile.
- The utilized data that will be sent to the EMR is based on the QUICK NOTES™ report (including alerts).
- The HL7 IDCO file format provides further options to embed different PDF based reports:
 - Presenting EGM
 - Combined Follow-up
 - Atrial Arrhythmia
 - Heart Failure Perspectiv
 - For S-ICD: Summary Report, Presenting ECG

View Patient List Search Patients Manage Clinic

Manage EMR Integration Manage Clinic Users Manage Clinic Settings Enroll Patient

Manage EMR Integration: Dragon 2 Test Clinic

Edit/View EMR Configuration

Filter by Device Transmission Date

From: (e.g.: 24 Jan 1950) To: (e.g.: 24 Jan 1950) Filter Clear

Patient ID/ Patient	Review Reason	Device Transmission Date	Status	Status Date/Time	Actions
T-Teigen 0058 Teigen, Trevor	Scheduled	03 Jul 2013	Waiting for Clinic Computer	03 Jul 2013 05:03 CEST	Resend
T-Teigen 0058 Teigen, Trevor	Implanted Device Alert	22 Jun 2013	Waiting for Clinic Computer	27 Jun 2013 23:08 CEST	Resend
NG3 PG Genn, Chtn	Implanted Device Alert	22 Jun 2013	Transferred	28 Jun 2013 23:07 CEST	Resend

View EMR Log 1 - 14 of 14


NOTE

Please refer to the LATITUDE™ NXT Clinician Manual for further instructions on installing the software and configuring the EMR Integration feature.

LATITUDE™ NXT Communicator


OVERVIEW OF THE LATITUDE™ COMMUNICATOR

- The LATITUDE™ Communicator is easy to set up and use.
- Patients may plug the Communicator into a compatible phone line. A GSM solution through our partnership with Vodafone – the largest international telecom company – is available for patients without a compatible telephone line.
 - GSM coverage is provided through Vodafone and its large network of local providers.
 - The GSM data plan is optional and is available for purchase.
 - For communicator model 6290, the GSM capability is available through a USB GSM adapter
- It is recommended that the LATITUDE™ Communicator stay near where the patient sleeps or in another room where he or she spends a significant amount of time.
- The wireless Communicator automatically captures data from the device on a schedule set by the clinic.

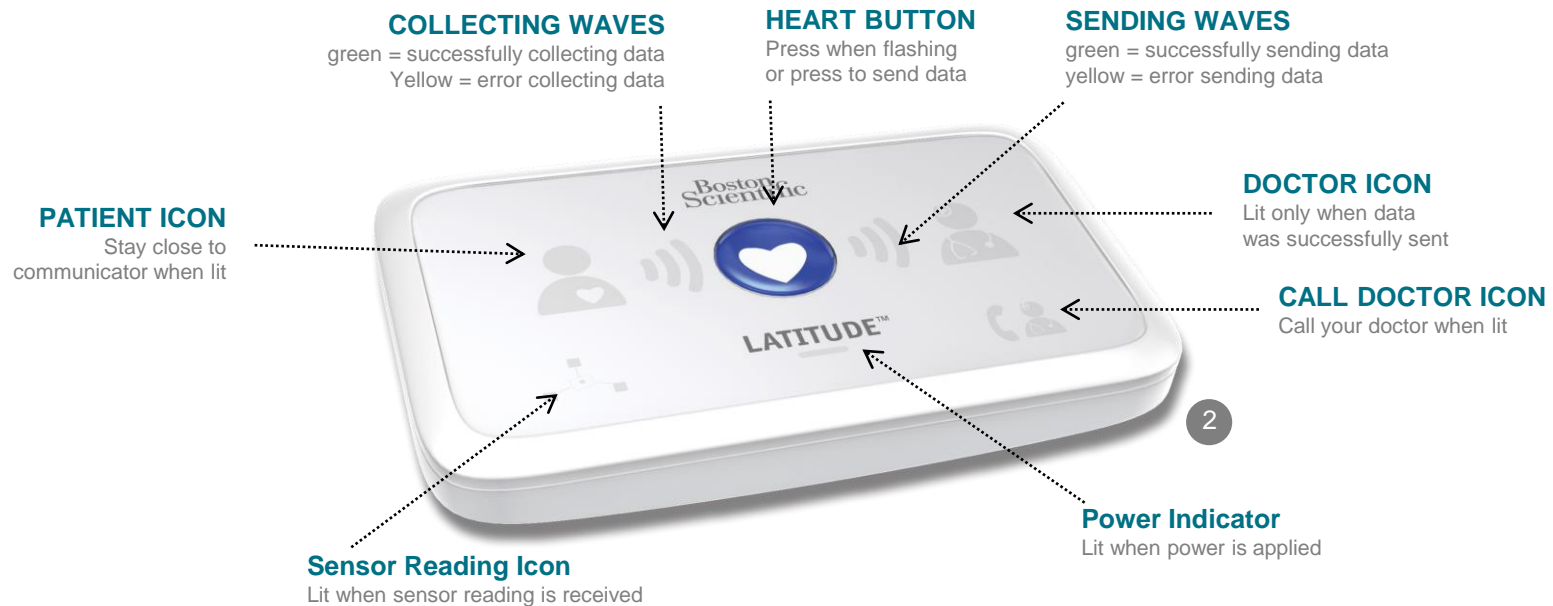


LATITUDE™ NXT Communicator

- If the Status button is pressed for less than 3 seconds, the collecting and sending waves will lit to reflect the status of the last data transmission.
- If the Status button is pressed and held for more than 3 seconds, it will force a communication between the communicator and the LATITUDE server

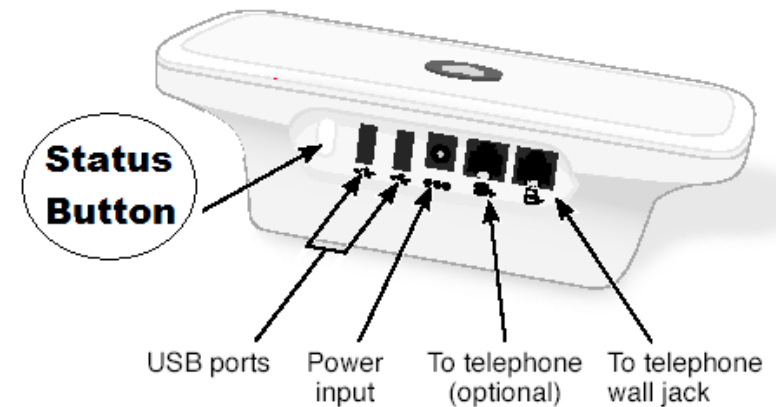



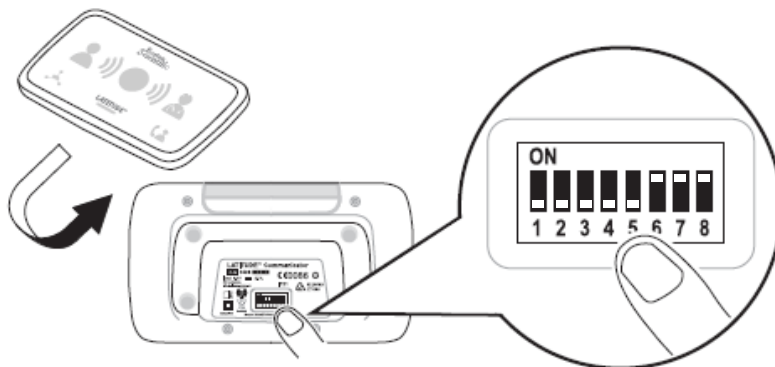



Rear View

Bottom View



- White switches can be slid up or down to match the appropriate setting (see "Dial-Out / Country Code Switches" for more details).
- Once correctly positioned, these switches will rarely have to be modified by patients
- The model and serial numbers of the communicator can also be found on the bottom of the communicator.

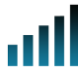Communicators & Connectivity

		LATITUDE™ NXT 2		
Communicator Model		Adapter	6288	6290
Standard Phone Cable Included in the communicator box		✓	✓	✓
DSL Filter Necessary for patients using DSL or VoIP phone lines		✓	✓	✓
Integrated Cell		✓		
Cell Adapter				✓
Ethernet Adapter		✓	✓	✓
Bluetooth Adapter (for Sensors) Necessary for patients provided with weight scale and/or Blood Pressure monitor		✓	✓	✓

TROUBLESHOOTING TIPS FOR THE LATITUDE™ COMMUNICATOR

Below are some quick troubleshooting tips for the LATITUDE™ Communicator.

Please refer to the Clinician Manual for a complete listing of statuses and recommendations.

Patients can press the Status button on the back of the Communicator at any time.

Pushing the button briefly (less than 3 seconds) shows:

- Confirmation of a transmission
- Illuminated collecting and sending waves to assist in troubleshooting

Pushing and holding the button (longer than 3 seconds):







- Forces a call to the LATITUDE™ website

Call Doctor light is red

- Indicates detection of a Red Alert condition that has been unable to be sent to the LATITUDE™ NXT website for more than 24 hours
- Patient should contact clinic

Call Doctor light is yellow


- Indicates a number of conditions, including an unrecoverable Communicator error or other connection problems
- Patient should contact clinic

Troubleshooting status	Brief Description	Suggested Action LATITUDE™ GSM Plan	Suggested Action Telephone Landline
	<p>The Communicator is unable to locate the implanted device.</p>	<ul style="list-style-type: none"> • Ensure Communicator is optimally placed. • Retry interrogation facing the Communicator. • Remain still during interrogation. 	
	<p>The Communicator is able to locate the implanted device, but is not able to complete an interrogation.</p>	<ul style="list-style-type: none"> • Ensure Communicator is optimally placed. • Retry interrogation facing the Communicator. • Remain still during interrogation. 	
	<p>Patient has exceeded PII limit or Communicator unable to connect because patient has cancelled interrogation</p>	<ul style="list-style-type: none"> • Enable additional PII if applicable. • Retry interrogation after 10 min. • Ensure patient doesn't press the heart button while the interrogation is in progress 	
	<p>Communicator not able to find a dial tone (landline) or GSM provider (LATITUDE™ GSM Plan users)</p>	<ul style="list-style-type: none"> • Move the Communicator to another location that may provide a stronger signal. 	<ul style="list-style-type: none"> • Check for a dial tone. • Check that telephone cord is plugged in tightly to wall jack and to Communicator.
	<p>Communicator attempted to connect to the LATITUDE™ NXT website but was unable to complete</p>	<ul style="list-style-type: none"> • Move the Communicator to another location that may provide a stronger signal. 	<ul style="list-style-type: none"> • Check that another device (telephone, computer, answering machine is not using the line). • Check that the Dial-out switches and Country Code switches are set correctly. • Ensure that the DSL filter is in use if the patient has DSL Internet service.
	<p>Communicator connected but was not able to send information to the LATITUDE™ NXT website</p>	<ul style="list-style-type: none"> • Check that the Communicator serial number has been entered correctly. • Check that Dial-out switches and country code switches are set correctly. 	

Dial-Out / Country Code Switches


DIAL-OUT NUMBER

Patients may use the LATITUDE™ Communicator in managed care facilities, hotels or other locations that require a dial-out number or prefix to place a call (e.g., dialing 9 to get an outside line). Switches 1, 2, 3 on the back of the Communicator must be set to match the required dial-out number.

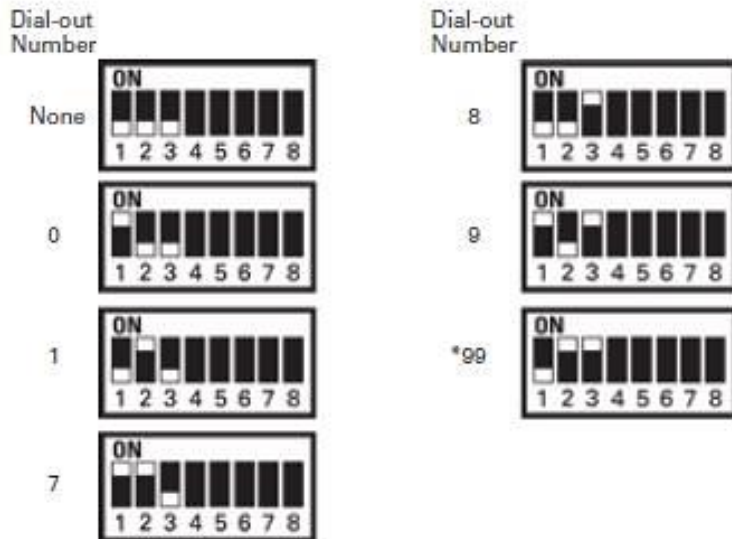

COUNTRY CODE


Country Code switches (switches 4 – 8) need to be set for the country in which the Communicator will be used. Country code switches should be set to the correct country before the patient leaves the clinic.

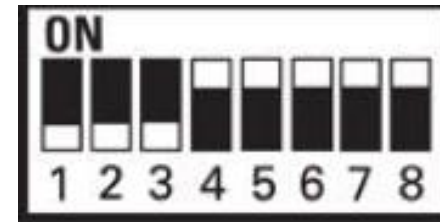

NOTE

If the patient travels to a location that requires a different dial-out prefix or to a different country, appropriate switches must be adjusted. Patients have these switch settings listed in the LATITUDE™ Communicator Patient Manual.

Dial-Out / Country Code Switches

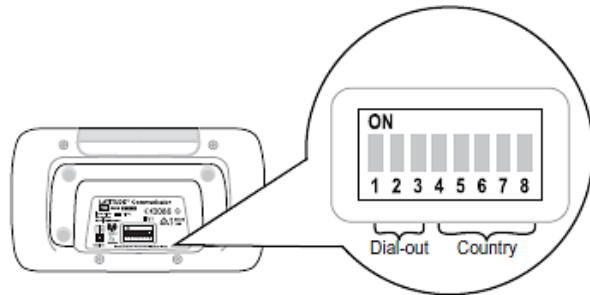

Austria



Belgium



Czech Republic




Denmark



Finland



France

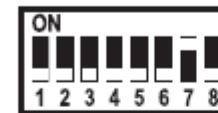

Germany



Greece



Hong Kong



Hungary




Ireland (Republic)



Italy



Netherlands

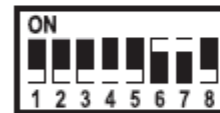

Norway



Poland



Portugal




Slovak Republic



Spain



Sweden

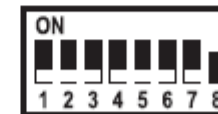

Switzerland



United Kingdom



LATITUDE™ Heart Failure

Management Sensors



The LATITUDE™ NXT Patient Management system includes optional components for managing heart failure patients. This equipment consists of a weight scale and blood pressure monitor. The scale and blood pressure monitor are Bluetooth-enabled and send information to the Communicator that is then transmitted to the secure LATITUDE™ NXT website. Speak to your Boston Scientific representative to find out if the equipment is available for your patients.

Weight is an optional yellow alert that can be configured on or off for all patients in a patient group or individual patients. The alert notifies the clinician of:

Weight gain of at least (0,45, 0,91, 1.36, 1.81, 2.27, 2.72, 3.18, 3.63, 4.08, 4.54) kg(s) in (1, 2, 3, 4, 5, 6, 7) days

Weight and blood pressure can be viewed on the Summary tab and on the Health page or on the Combined Follow-up Report.

Patients for whom the LATITUDE™ Sensors have been prescribed will need to connect the equipment to the Communicator by plugging in an included USB adapter to the back of the Communicator. Only one USB adapter is needed to connect the weight scale and the blood pressure monitor to the Communicator.



LATITUDE™ CUSTOMER SUPPORT

Multilingual, dedicated support to assist clinicians in implementing and using the LATITUDE™ system:

- Guidance regarding LATITUDE™ system set-up and website navigation
- Assistance with alert and scheduling configuration options
- Communicator troubleshooting

BOSTON SCIENTIFIC CRM TECHNICAL SERVICES

Dedicated support for device-related questions, including data on the LATITUDE™ system:

- Comprehensive device, lead and EGM consultation on data displayed in the LATITUDE™ system or on the ZOOM LATITUDE™ Programmer.
- Assistance with Red and Yellow Alert questions.

AUSTRIA 0800 202289	GERMANY 069 51709 481	NETHERLANDS 0800 0292077	SWEDEN 020 160 57 07
BELGIUM 0800 80697	GREECE 442 035 647 788	NORWAY 81 00 00 47	SWITZERLAND 0844 000110
CZECH REPUBLIC 239 016 657	HUNGARY 080 981 579	POLAND 22 306 07 33	UNITED KINGDOM 0845 602 9283
DENMARK 70 10 01 82	IRELAND 1890 812005	PORTUGAL 800844729	
FINLAND 010 80 48 19	ISRAEL 1 809 303 136	SLOVAK REPUBLIC 42 12 686 22 389	
FRANCE 0805 5404 22	ITALY 848 781164	SPAIN 901 010 840	

Disclaimer



LATITUDE™ Patient Management System from Boston Scientific CRM

All cited **trademarks** are the property of their respective owners. CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labelling supplied with each device.

1 Indications, contraindications, warnings, precautions and instructions for use can be found in the product labeling supplied with each device

2 Graphic pictures might not represent the actual size of the products

CRM-335304-AA AUG 2015 Information for the use only in countries with applicable health authority product registrations.

**Boston
Scientific**
Advancing science for life™

www.bostonscientific-international.com

© 2015 Boston Scientific Corporation or its Affiliates. All rights reserved.
DINCRM0756EB