


Step 1

Attach Magnet to Mobile Device Case


- 1A** Remove the blue magnet from the silver box.
- 1B** Use the tab to peel off the adhesive backing from the back of the mobile device case.
- 1C** Place the logo side of the magnet on the adhesive and press firmly.

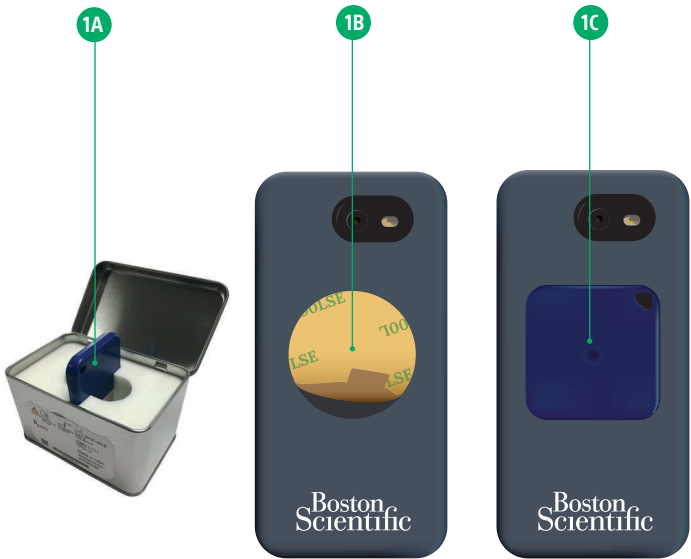

Step 2

Power On your Device


- 2** Press and hold the Power Key.

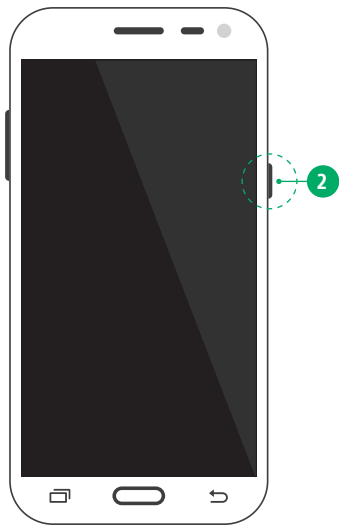

Step 3

Open your App


- 3** Tap the app icon to open your app.
- Touchscreen Tip:** Gently and quickly tap the screen with your fingertip. Do not use your fingernail.

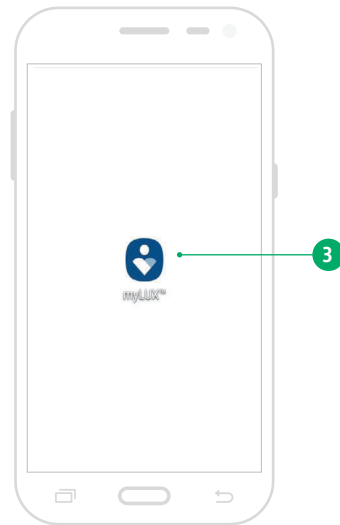

Step 4


If you see this screen...

You're all set up! Please review the section: "Using myLUX" in this Quick Start Guide.

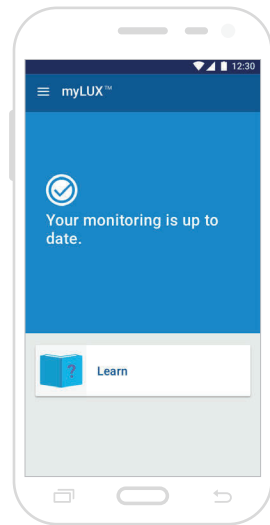


Or, if you see this screen...



- 4A** ... and the app is set to your preferred language, tap **NEXT** and go to step 6.
- 4B** Or, if you prefer, tap **CHANGE LANGUAGE**.

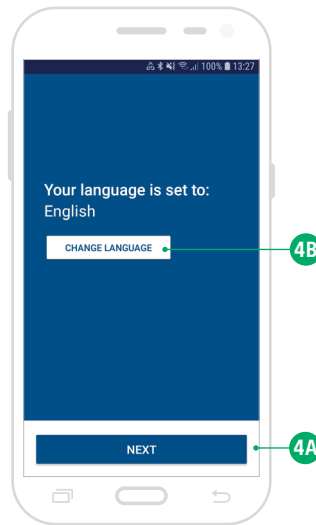

Step 5

Change Language


- 5A** Use the  arrows to drag your preferred language to the top of the list.
 - 5B** Tap **DONE** to confirm preferred language.
- The screen in step 4A will display in the language you selected. Tap **NEXT**.

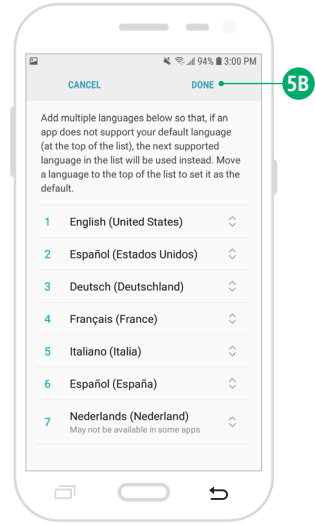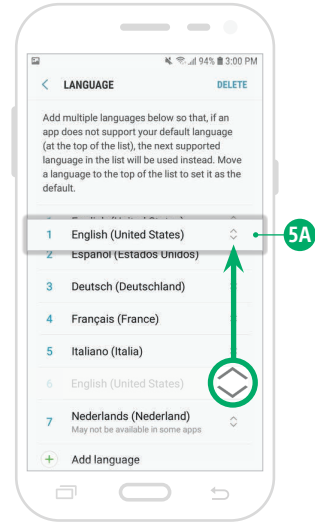

Step 6

Complete Setup


- 6** Tap **START SETUP** and follow the on-screen instructions.

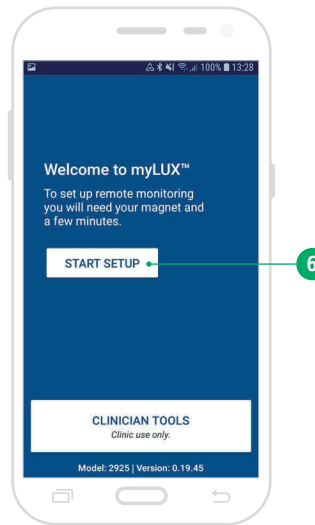

Step 7


All done!

Setup is complete! Please review the section: "Using myLUX" in this Quick Start Guide.

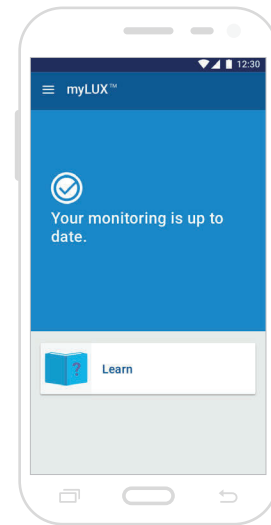

Using myLUX

- Keep your mobile device powered on at all times
- You do not need to bring your mobile device with you during the day, unless instructed by your clinic
- Charge your mobile device each night using the charging cord, and place within 6 feet (2 meters) from where you sleep

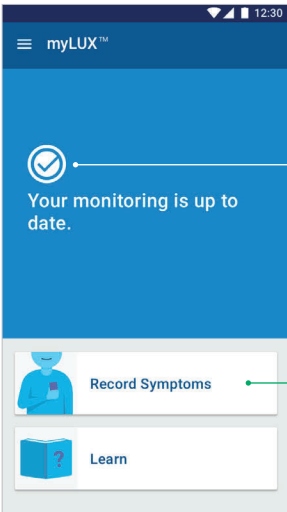

When you see this icon, you know that everything is working fine — no action is needed.

You may have a “Record Symptoms” button. **Use the button only when feeling symptoms that your clinic has asked you to record.** This button is not available to every patient.

How it works



Your insertable cardiac monitor (ICM) automatically records and stores irregular heart rhythms — whether you’re near your mobile device or not.



Each night, your mobile device collects irregular heart rhythms stored in your ICM and automatically sends that data for your clinic to review.



Your clinic will contact you if there is a need for further review.

CAUTION: The ICM system is not meant to assist with health emergencies. If you need immediate medical attention, call your health care provider or seek emergency medical services.

Your mobile device



A. Power Key

B. Home Button

C. Back Button

D. Charging Port

Boston Scientific Corporation
4100 Hamline Avenue North
St. Paul, MN 55112-5798 USA

Tel: 651.582.4000

Medical Professionals: 1.800.CARDIAC (227.3422)

Patients and Families: 1.866.484.3268

www.bostonscientific.com
www.bostonscientific.com/patients

2019-10

50942114-001 en USA



Quick Start Guide

myLUX™ Patient App

REF 2925

for LUX-Dx™ Insertable Cardiac
Monitor (ICM)

REF M301

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.

© 2019 Boston Scientific Corporation or its affiliates.
myLUX and LUX-Dx are trademarks of Boston Scientific Corporation or its affiliates.