



**LUX-Dx™**  
ICM System

# Review any LUX-Dx™ ICM Patient on LATITUDE Clarity™

**Boston Scientific**  
Advancing science for life™




## See how LUX-Dx helps streamline your workflow

The new LATITUDE Clarity Temporary Healthcare Provider Access allows healthcare providers who are not part of the patient's primary following clinic to obtain read-only observer access.

With this addition, the LUX-Dx ICM System provides convenient and efficient tools to enable patient data review for a variety of care settings.

- Two workflow needs – primary/following and acute care/observing – in one single LATITUDE Clarity login
- No more waiting for a following device clinic to send over the data or a Boston Scientific rep to come on-site

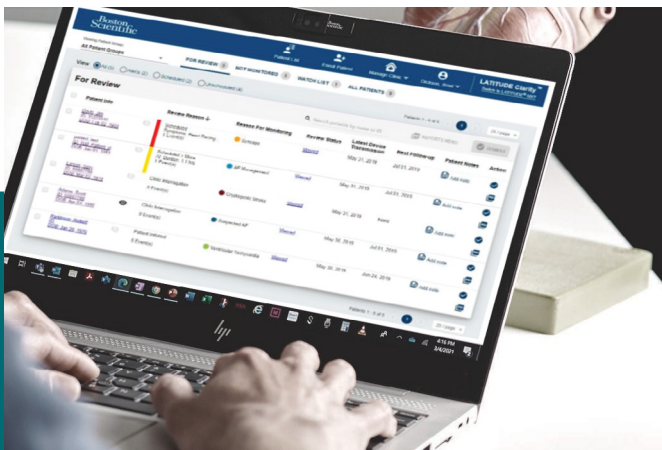


## What you need to get started with observer access

- Any smart device (smartphone, tablet, laptop, or desktop) to visit the LATITUDE Clarity website (<https://www.latitude.bostonscientific.com>)
- Access to secure Wi-Fi or cellular connection
- LATITUDE Clarity login and password
- LUX-Dx Clinic Assistant with magnet or myLUX Patient App with magnet
- Patient's ICM model number, serial number and date of birth (find them on myLUX, Clinic Assistant or patient's medical ID card)



### 1. Device interrogation

Clinicians can either use the LUX-Dx Clinic Assistant and magnet to interrogate the ICM device, or have the patient do a manual transmission on the myLUX™ Patient App. This step is recommended to get the latest transmission data.

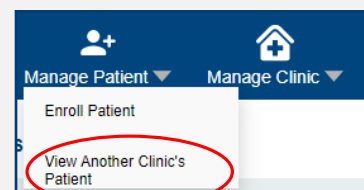

### 2. Data access

To view the data transmission, log in to the Clarity website. Select "Manage Patient" > "View Another Clinic's Patient" and input the patient's ICM model number, serial number, and date of birth to find the patient.



### 3. Review information

The patient data is now ready for the read-only review. This includes latest transmission, event history, trends and counters.

### 4. End access

The temporary observer data access to this patient will automatically expire after 24 hours; observing clinicians will no longer see this patient shown on their patient list.



**LATITUDE Clarity™ provides device checks and remote patient management for any LUX-Dx™ patient in the following settings:**


- Following clinics
- Non-following care settings:
  - Critical care
  - Inpatient
  - Cath lab
  - Surgery centre
  - Patient clinic
  - Emergency room/emergency department

- **Timely patient data access for any patient right at the point of care**

- **Patient data remains centralised and consistent on LATITUDE Clarity for following and observing clinicians**

The Temporary Healthcare Provider Access is for clinicians with a LATITUDE Clarity account set at the right access levels (Limited or Complete). To interrogate the patient, LUX-Dx Clinic Assistant and magnet are required.

- For an initial setup of a LATITUDE account and required accessories, visit <https://latitude.bostonscientific.eu/latitude-customer-support.html> and contact your local customer support team

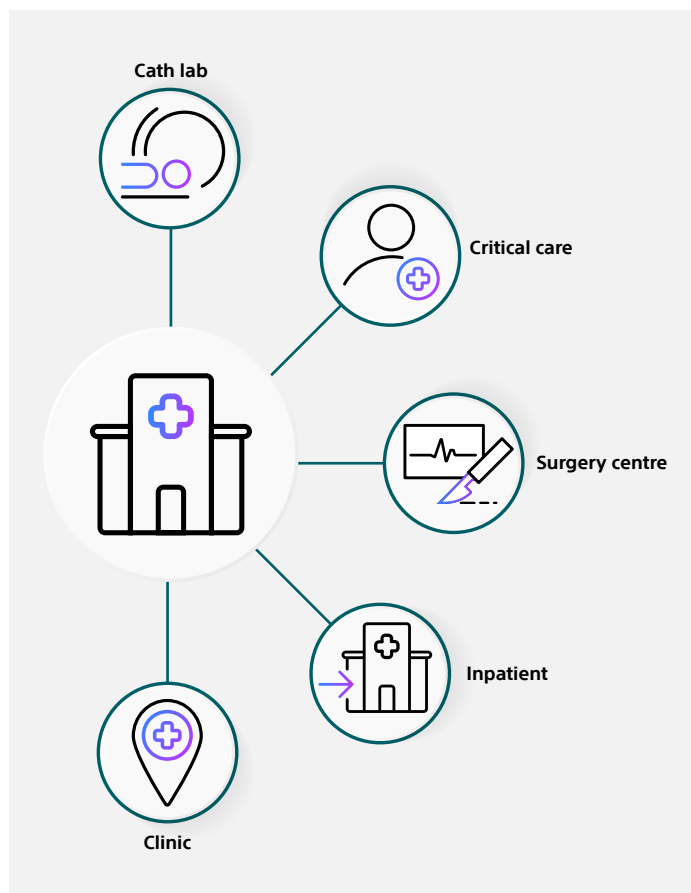

**Boston  
Scientific**

Advancing science for life™

[www.bostonscientific.eu](http://www.bostonscientific.eu)

CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings, and instructions for use can be found in the product labelling supplied with each device or at [www.IFU-BSCI.com](http://www.IFU-BSCI.com). Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries. This material not intended for use in France.

© 2024 Boston Scientific Corporation or its affiliates. All rights reserved.

CRM-1907303-AA

CE 2797