

Refer to the device directions for use for complete instructions on device use.

**Caution:** Federal Law (US) restricts this device to sale by or on the order of a physician.

Cautions can be found in the product labeling supplied with each device.

### Intended Use

This device is intended for use in endoscopic retrieval of foreign objects, food bolus, tissue fragments and excised tissue such as polyps.

### Warnings

- **This device is designed for single use only.** Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilizing may compromise the structural integrity of the device and/or lead to device failure which may in turn result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.
- Do not use this device for any purpose other than the stated intended use.
- Visually inspect the package prior to use for any tears, irregularities or damage. If an abnormality is detected, the sterile condition of the device may have been compromised. Do not use the device.
- Before use, confirm that the net, catheter and handle are not damaged. If any abnormalities such as significant deformations or excessive bends are found, do not use the device.
- Confirm that there are no significant deformations, sharp parts, burrs or edges.
- Gently run your fingertips over the entire length of the catheter to check for any crushes, excessive bends, broken areas or other damage.
- Make sure that there are no cracks on the handle
- Do not store the package containing the device in places where it may become damaged, wet or improperly sealed. Otherwise, the sterility of the device may be compromised and pose an infection control risk or cause tissue irritation.
- After use, dispose of product and packaging in accordance with hospital, administrative and/or local government policy.

### Contraindications

Contraindications include those specific to the primary endoscopic procedure to be performed in gaining access to the desired site.

## Precautions

- Refer to the package label for minimum channel size required for this device.
- The endoscopic retrieval of foreign objects, food bolus, tissue samples or excised polyps should only be performed by persons having adequate training in clinical endoscopic technique.
- Avoid blindly passing the device past any foreign object, particularly if the entire lumen is blocked.
- Use care to avoid inadvertently grasping tissue or organs not intended for retrieval.
- Keep gentle traction on the device during retrieval so that the retrieved object does not become loosened, dislodged or aspirated into the trachea.
- This device is not recommended for the retrieval of sharp foreign objects.
- After extraction, if the retrieved material does not exit the net upon deployment, the material should be removed from the net by rinsing the net in a basin of water; do not “pick” retrieved material from the net with a finger or tool as this may damage the net.