


# Agile™

## Esophageal Fully & Partially Covered Stent System

Featuring a Through-the-Scope  
Delivery System

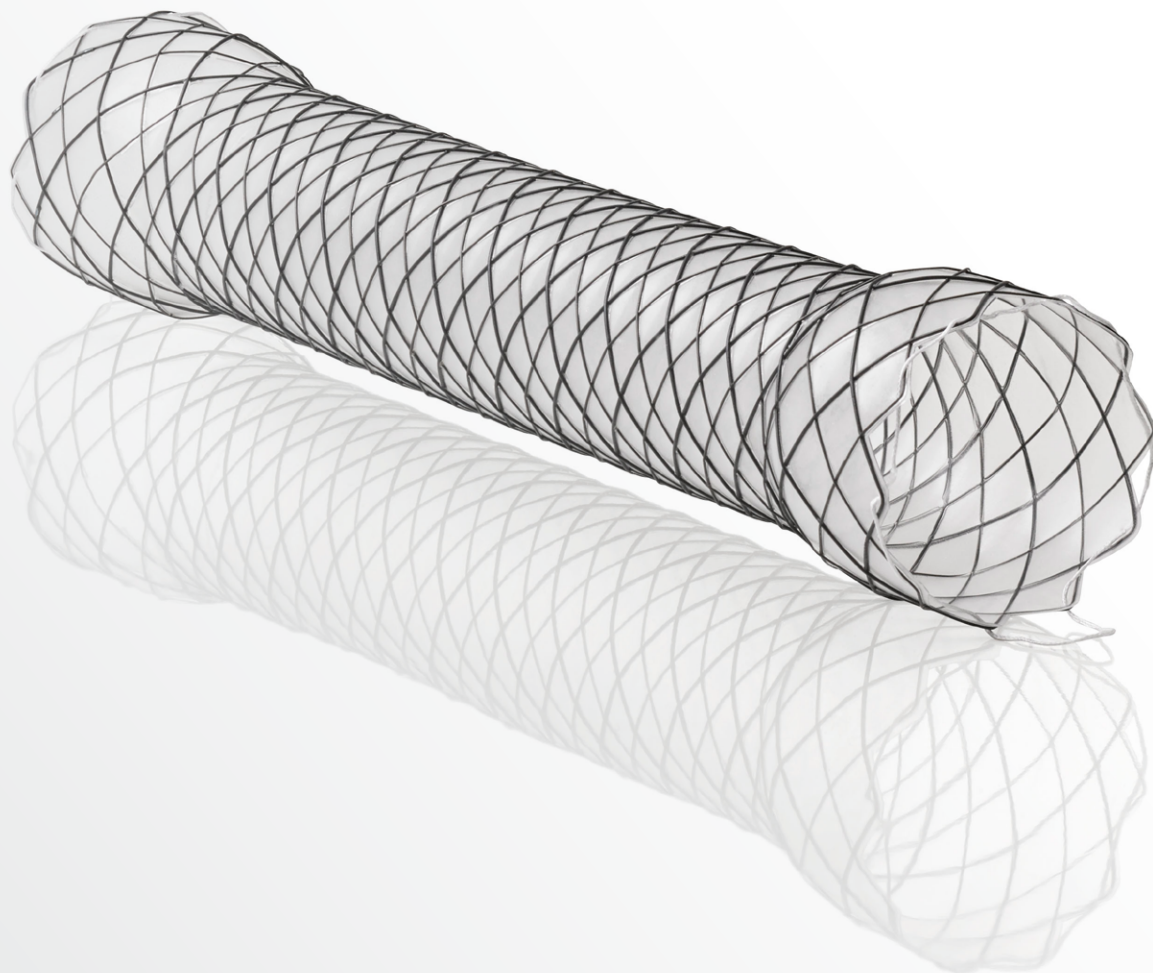


# About Agile™

The Agile Esophageal Stent System offers a flexible and conformable self-expanding metal stent on a through the scope delivery system which enables direct visualization during deployment and may help minimize reliance on fluoroscopy. The Agile portfolio is offered in fully covered and partially covered designs.

- Offering new sizing options including a 60mm length and 14mm diameter designed to meet a variety of clinical needs, all size configurations are offered in fully covered and partially covered options.
- Braided dual gauge nitinol wire provides an implant that is more flexible and conformable.\*
- Atraumatic looped ends to help minimize risk of trauma
- Retrieval suture on both ends of stent
- Tissue In-Growth Prevention—The silicone covering extends the entire length of the Fully Covered Agile Esophageal Stent and is designed to address tumor ingrowth.

\*Compared to WallFlex™ Esophageal 18mm stent diameter. WallFlex Esophageal is not available in a 14mm stent diameter for direct comparison.

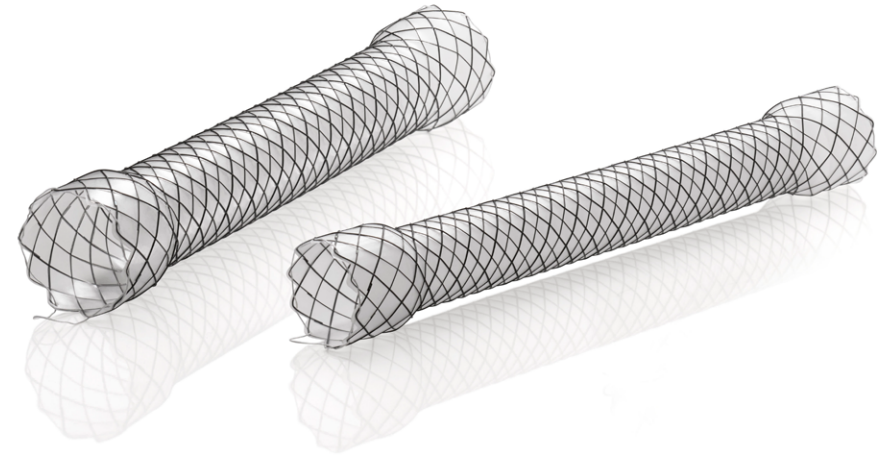

# Delivery System

## Through-the-Scope design

- Provides direct visualization to aid physicians in stent placement and help minimize fluoroscopic dependence.
- Designed to offer flexibility and trackability to enhance physician control.

## Low profile delivery system


Compatible with a 3.7mm working channel, designed to navigate through tight strictures.

## 5 radiopaque markers

Markers signal the ends of the stent, the midpoint of the stent, and the point at which the stent can no longer be recaptured, aiding in deployment. The fifth marker at the leading end of the exterior tube indicates how far the stent has been deployed.

## Reconstrainability

In the event that the stent is not exactly where a physician intends it to be, the stent can be reconstrained up to 2x and redeployed.

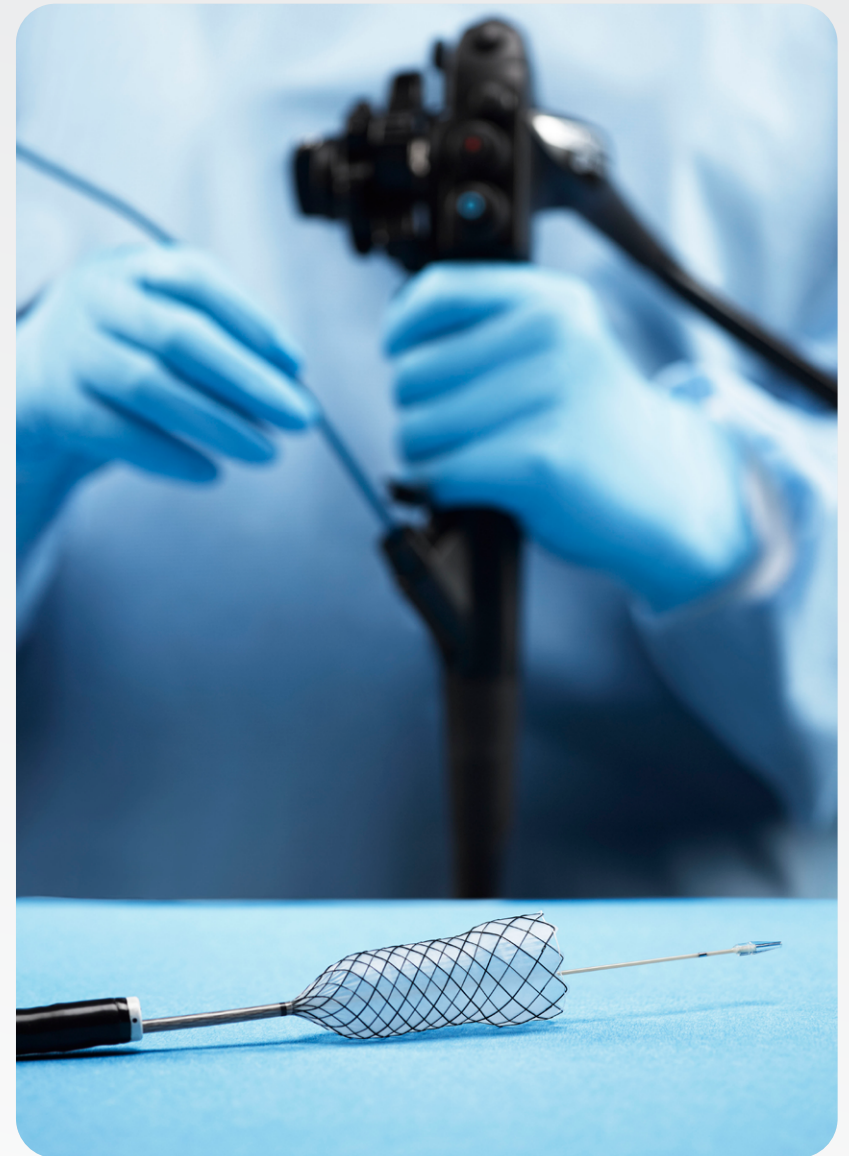

# Ordering Information

The Agile™ Esophageal Stent System is intended for maintaining esophageal luminal patency in esophageal strictures caused by intrinsic and/or extrinsic malignant tumors and occlusion of concurrent esophageal fistulas.

Agile Esophageal Stent System—Partially Covered					
Order Number	GTIN	Stent O.D. (mm)	Proximal/Distal Flares O.D.(mm)	Stent Length (mm)	Delivery System Diameter (Fr)
M00517200	08714729972907	14	19	62	10.5
M00517210	08714729972914	14	19	102	10.5
M00517220	08714729972921	14	19	119	10.5
M00517230	08714729972938	14	19	148	10.5
M00517240	08714729972945	18	23	59	10.5
M00517250	08714729972952	18	23	97	10.5
M00517260	08714729972969	18	23	119	10.5
M00517270	08714729972976	18	23	149	10.5

[View fully covered](#)

# Ordering Information

Agile Esophageal Stent System—Fully Covered					
Order Number	GTIN	Stent O.D. (mm)	Proximal/Distal Flares O.D.(mm)	Stent Length (mm)	Delivery System Diameter (Fr)
M00517400	08714729973027	14	19	62	10.5
M00517410	08714729973034	14	19	102	10.5
M00517420	08714729973041	14	19	119	10.5
M00517430	08714729973058	14	19	148	10.5
M00517440	08714729973065	18	23	59	10.5
M00517450	08714729973072	18	23	97	10.5
M00517460	08714729973089	18	23	119	10.5
M00517470	08714729973096	18	23	149	10.5

[View partially covered](#)

All photographs owned by Boston Scientific.

All trademarks are the property of their respective owners.

Caution: U.S. Federal law restricts this device to sale by or on the order of a physician.



Boston Scientific Corporation  
300 Boston Scientific Way  
Marlborough, MA 01750  
[www.BostonScientific.com](http://www.BostonScientific.com)

© 2020 by Boston Scientific  
Corporation or its affiliates.  
All rights reserved.

ENDO-815304-AA