

Resolution™ Clip

Boston
Scientific

Advancing science for life™

Procedural Steps*

1 Carefully insert the **Resolution Clip** through the biopsy channel, as shown in Figure 1. When the Resolution Clip can be observed exiting the scope, **remove the RED stopper**.




2 Push the **WHITE handle** distally up against the **BLUE over-sheath grip**, to fully expose the **jaws**, as shown in Figure 2.

3 To **open the clip**, move the slider distally away from the thumb ring. To **close the clip**, move the slider proximally until tactile resistance is felt in the handle, as shown in Figure 2.

NOTE: Do not continue to close the clip beyond the tactile resistance until you are ready to deploy the clip, otherwise you may not be able to re-open the clip. If you hear or feel a click, the clip cannot be re-opened.

4 To **deploy the Resolution Clip**, continue moving the slider proximally beyond the tactile resistance point until it reaches the thumb ring and the clip has been deployed, as shown in Figure 3.

Once the **Resolution Clip** has been deployed, gently move the slider distally to separate the clip from the delivery device. Once the clip separates, release the slider and advance the over-sheath distally. Then, replace the stopper and withdraw the catheter from the scope.

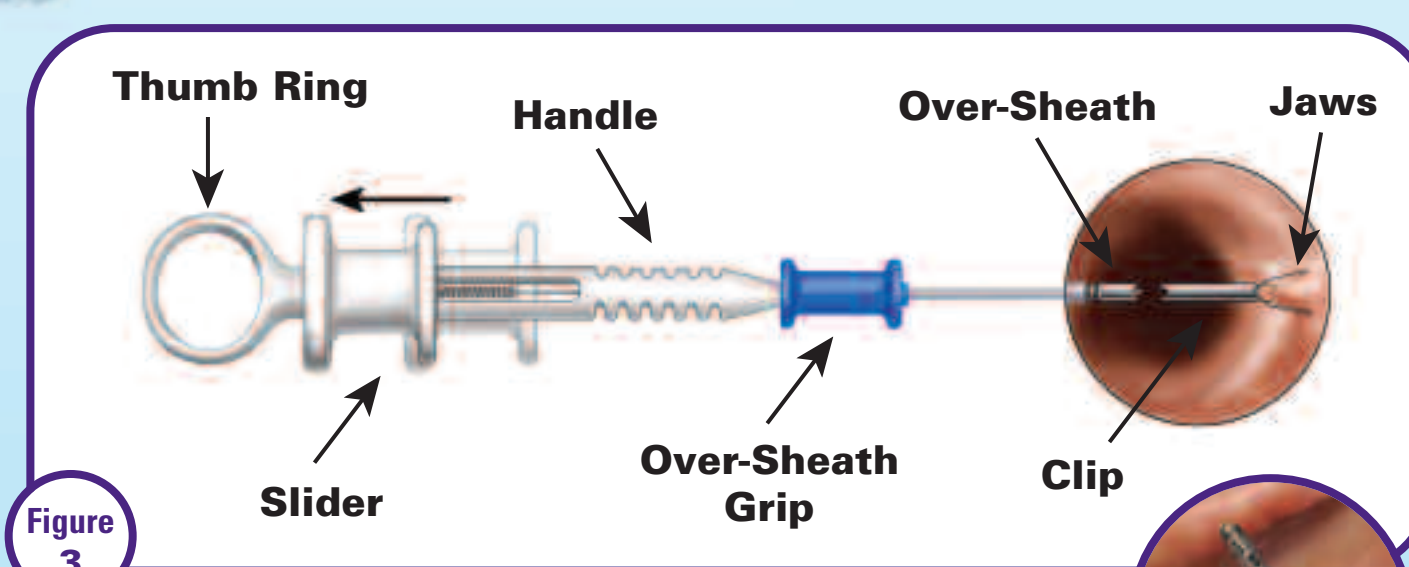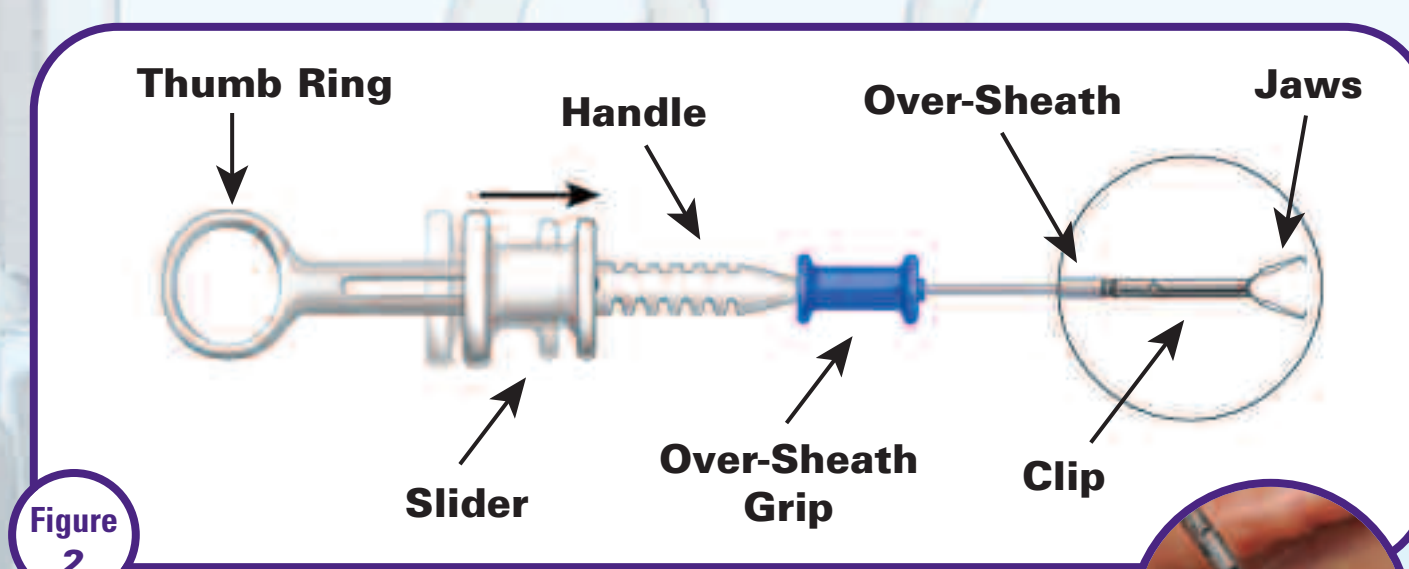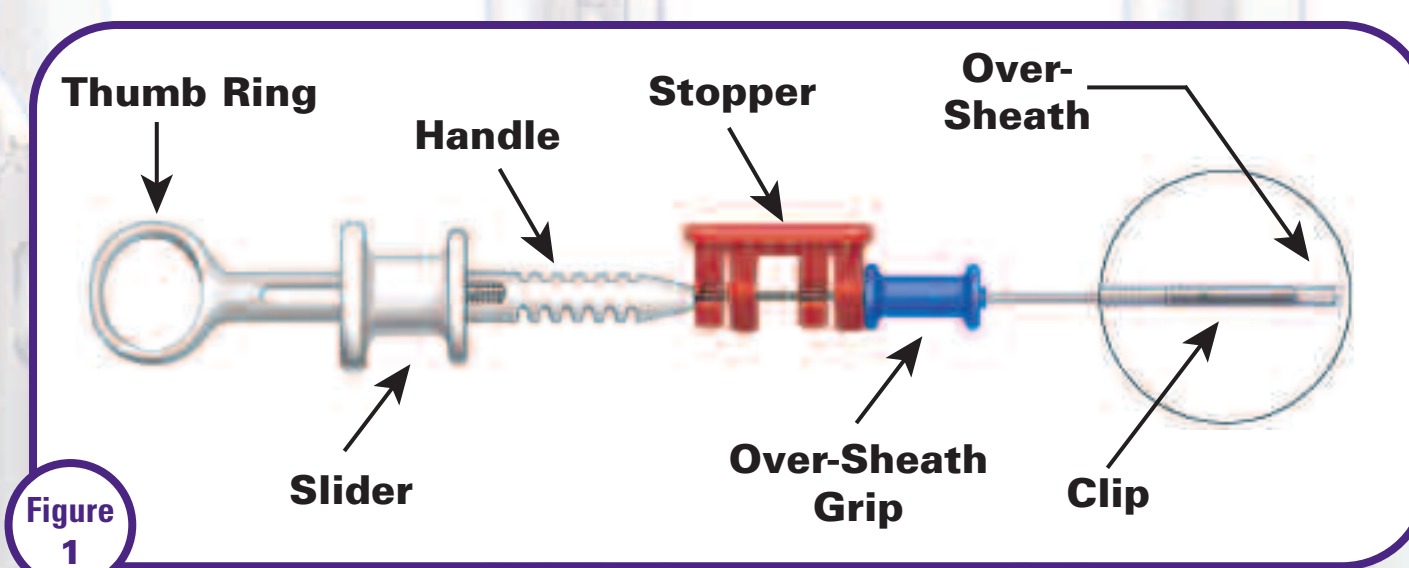

The Resolution Clip is engineered to open and close up to *five times* prior to deployment to aid in repositioning the clip.†



†Reopening and closing capability may be limited by clinical circumstances and patient anatomy, among other factors.

*This overview is provided for illustrative purposes only and is intended only as a brief summary of how the procedural steps using the Resolution Clip are generally performed. Please refer to the Directions For Use for complete instructions.

Resolution is a registered trademark of Boston Scientific Corporation or its affiliates.

Indications, contraindications, warnings and instructions for use can be found in the product labeling supplied with each device.

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.