Scientific

Advancing science for life

The Acquire FNB needle has allowed us to procure larger tissue specimens in a more efficient manner, even in cases where FNA had not been successful before. - Dr. Nirav Thosani

#### **Acquire**<sup>™</sup> Endoscopic Ultrasound Fine Needle Biopsy Device

The Franseen needle tip design of the Acquire Endoscopic Ultrasound Fine Needle Biopsy device is an optimized, proven solution to procure larger tissue specimens for histological analysis based on 50 years of clinical use in interventional radiology.


oic \_\_\_\_

The three points provide stability at puncture while the high quality, fully formed heels are designed to maximize tissue capture and minimize fragmentation, which may result in improved diagnostic yield and specimen adequacy to support oncology research.




### Procedure using Acquire 22ga FNB Needle.

Image courtesy: Dr. Krishnavel Chathadi Director of Endoscopic Services Henry Ford System, Michigan USA

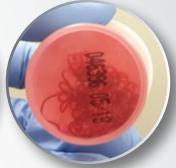

## Acquire sample in formalin using wet suction.

Image courtesy: Dr. Mankanwal Sachdev Medical Director Endoscopy St. Joseph's Hospital, Arizona USA

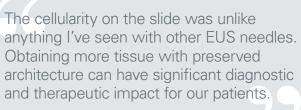


- Dr. Shantel Hebert-Magee




FNB of pancreatic mass showing tumor with neuroendocrine features (H&E stained slide, 10x) with inset showing positivity with immunohistochemical stain (chromogranin stain, 10x).

Image courtesy: Dr. Barbara Chadwick Pathology, University of Utah

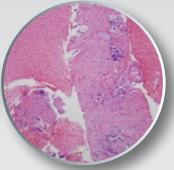


This image shows an EUS-guided FNB also obtained with a 22 gauge Acquire needle, showing an invasive adenocarcinoma with desmoplastic stroma (H&E stained slide, 10x).

Image courtesy: Dr. Barbara Chadwick Pathology, University of Utah

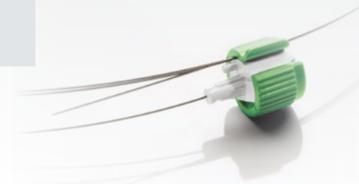

# The Acquire<sup>™</sup> EUS-FNB needle is designed to capture bigger tissue specimens

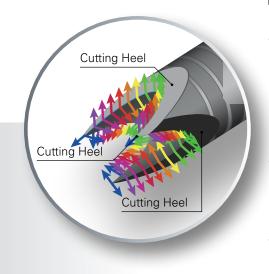

Design Feature	Intended Benefit		
Three Symmetrical Cutting Surfaces with Fully Formed Heels	For precise cutting capabilities, stability, and device control		
Cobalt Chromium (Acquire 22ga and 25ga)	To provide benefits over some stainless steel alloys including greater needle hardness and excellent tensile properties to deliver:**  Improved needle penetration Improved pushability and kink resistance Increased resistance to deformation after multiple passes		
Nitinol (Acquire 19ga Flex)	Nitinol is more flexible than Stainless Steel.* Acquire 19ga Flex needle is designed to pass through the tortuous anatomy. It provides flexibility and passability comparable to the 22ga Aquire Needle.*		
Echogenic Pattern Extends to the Needle Tip	To provide precise guidance within the target site		
Custom Sheath Sizes	Designed to improve passability		
Control Zone and Lubricomp Polymer	Two ergonomically defined areas designed to optimize control during actuation		

<sup>\*\*</sup>Keehan E, Gergely L. 2009. Catheter and Specialty Needle Alloys. Poster session presented at:

Materials & Processes for Medical Devices Conference & Exposition: Minneapolis

#### Potential clinical and economic benefits

Designed to improve physician confidence in sample quality

There are no published, standard specimen requirements or evaluation criteria for diagnoses of certain cancers among pathologists. However, a double-blinded survey with GI doctors (Boston Scientific Market Research 2016) found the following:

- More than half of the GIs surveyed agree that a larger tissue sample will give them more confidence for an accurate diagnosis in the absence of Rapid Onsite Evaluation (ROSE).
- More than half of the GIs who agreed that larger tissue will give them more confidence for accurate diagnosis, also agree that a larger tissue sample will lead to more accurate diagnosis.
- 52% report needing more tissue as the most important reason to use an FNB needle. Among those who reported more tissue as the most important reason, some considered more tissue to lead to better diagnosis and reduced chance of a repeat procedure.
- On average the surveyed GI physicians say that in 20% of their cases they are told (by the pathology or cytopathology departments) that they have an inadequate sample due to size.


Acquire is designed to obtain more tissue, providing your physician greater confidence that the samples they obtain may improve diagnostic yield and may be sufficient to support further oncology research.



– Dr. Krishnavel Chathadi


## **Assessment:** EUS-guided biopsy using a Franseen needle design

Note: Retrospective studies used the Acquire Needle 22ga only

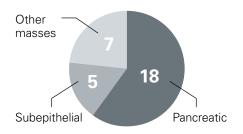

#### **Total Patients = 30**

Over a 3-month period (May to July 2016) Total = 30 (male = 21; median age, 71.5 years)

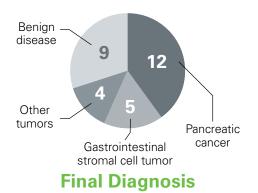



96.6% Achieved
Diagnostic Adequacy
at ROSE



Established Histological Diagnosis



**Conclusions:** Preliminary data suggest that the Franseen-tip needle yields diagnostic material for ROSE and histology in greater than 95% of patients.<sup>1</sup>

Prospective studies involving larger cohorts of patients are required to confirm these findings.

#### **Acquire**<sup>™</sup>

Endoscopic Ultrasound Fine Needle Biopsy (FNB) Device

Order Number	Description	Needle Size	Minimum Working Channel	Sheath Diameter	Packaging (color coded)
M005 <b>5558</b> 0	19ga Acquire FNB Needle, Flexible	19ga (1.14mm)	2.8mm	1.73mm	Box 1
M005 <b>5554</b> 0	22ga Acquire FNB Needle	22ga (0.72mm)	2.4mm	1.65mm	Box 1
M005 <b>5554</b> 1	22ga Acquire FNB Needle	22ga (0.72mm)	2.4mm	1.65mm	Box 5
M005 <b>5556</b> 0	25ga Acquire FNB Needle	25ga (0.52mm)	2.4mm	1.52mm	Box 1
M005 <b>5556</b> 1	25ga Acquire FNB Needle	25ga (0.52mm)	2.4mm	1.52mm	Box 5

- Packaging includes a 20cc syringe and one-way stopcock
- Working length: 137.5cm to 141.5cm, adjustable
- Needle length: 0cm to 8cm, adjustable

All cited trademarks are the property of their respective owners.

CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labelling supplied with each device. Information for use only in countries with applicable health authority registrations. Material not intended for use in France.

Product available in the European Economic Area (EEA) only. Please check availability with your local sales representative or customer service. ENDO-474101-AA. Printed in The Netherlands by De Budelse/Gosling.



www.bostonscientific.eu

<sup>&</sup>lt;sup>1</sup> Bang, J. Y., Hebert-Magee, S., Hasan, M. K., Navaneethan, U., Hawes, R. and Varadarajulu, S. (2016), Endoscopic ultrasonography-guided biopsy using a Franseen needle design: Initial assessment. Digestive Endoscopy. doi:10.1111/den.12769.