

\* The therapy referred to here may be one of several left heart procedures that require transseptal access. Additional procedural steps and/or devices may be required to deliver the therapy. Before use, consult Instructions for Use for any devices accordingly.

† WATCHMAN™ Access Sheaths as shown in this brochure is sold separately.

‡ The VersaCross Connect™ Transseptal Dilator is for use with a 12F (4.09 mm) ID WATCHMAN™ Access Sheath that is 75 cm in length, specifically: WATCHMAN FXD Curve™ Access System [Models: M635TS80010, M635TS80020].

CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings, and instructions for use can be found in the product labelling supplied with each device or at [www.IFU-BSCI.com](http://www.IFU-BSCI.com).

Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries.

This material not intended for use in France.

**Boston  
Scientific**  
Advancing science for life™

[www.bostonscientific.eu](http://www.bostonscientific.eu)

© 2025 Boston Scientific Corporation  
or its affiliates. All rights reserved.

EP-1536805-AC C€ 1639



**VersaCross Connect™**  
LAAC Access Solution


*ZERO EXCHANGE*

**WATCHMAN™** Access Systems

# ZERO EXCHANGE

**WATCHMAN™** Access Sheath Delivery<sup>†</sup>

## EXCHANGELESS 3-in-1 RF Wire

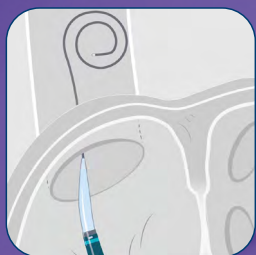

**1** Start at the SVC



**2** RF transseptal puncture



**3** Deliver **WATCHMAN™** Sheath

INSTANTLY DEPLOY


**0.035" WIRE**

to deliver **WATCHMAN™** Access Systems<sup>‡</sup> with confidence

## DELIVER THERAPY ON TARGET




**TRUform™**  
Shapeable Technology

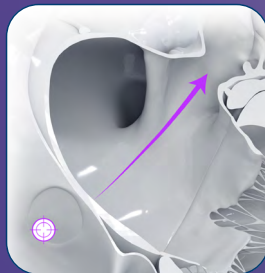

**Precision RF**  
Puncture Technology




the right **CURVE** at the right **TIME**

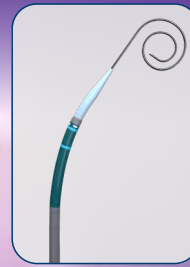


## TRANSEPTAL CURVE

**VersaCross Connect™**  
Transseptal Dilator with **TRUform™** shapeable technology optimizes transseptal approach

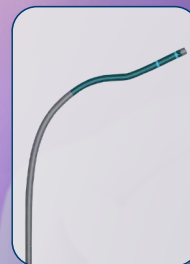

## THERAPY DELIVERY CURVE

**WATCHMAN™** Access  
Sheaths retain their curve for therapy delivery

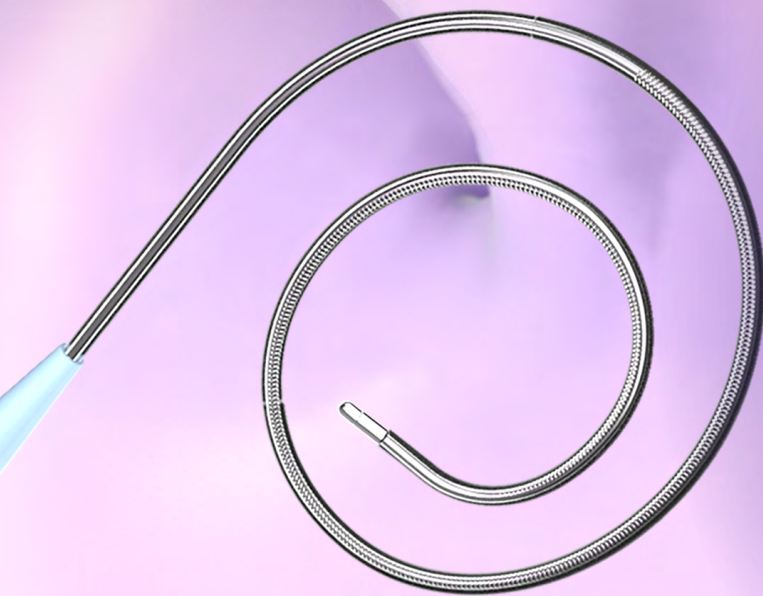

## Personalize your solution

1. Choose your **VersaCross™**  
RF Wire



Length: 180 cm, 230 cm  
Diameter: 0.035"

2. Choose your **VersaCross Connect™**  
Transseptal Dilator



Inner Diameter: 0.035"  
Outer Diameter: 12F (4.09 mm)  
Useable Length: 85 cm

## Solution with Pigtail Wire

Part No.	Dilator curve	Wire length (cm)
VXAK3001	D0	180
VXAK3003	D1	180
VXAK3005	D0	230
VXAK3007	D1	230

## Solution with J-Tip Wire


Part No.	Dilator curve	Wire length (cm)
VXAK3009	D0	180

## OMNIVIZ™ Technology


Reliably locate the VersaCross Connect solution on fluoroscopy and ultrasound. Positional markers indicate position of RF tip within dilator.

# FAST TRACK

to your therapy delivery  
in a **SINGLE SOLUTION**

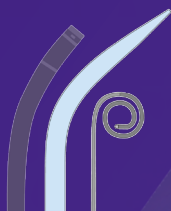

**VersaCross Connect™**  
LAAC Access Solution†

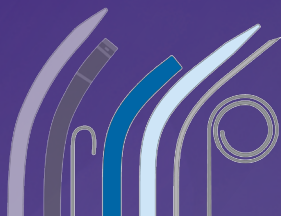

**Needle based  
workflow**

VersaCross™  
RF Wire,  
VersaCross  
Connect™  
Dilator,  
WATCHMAN™  
Sheath



**INSERT**

**RETRACT**  
guidewire

**INSERT**  
needle

**POSITION**  
on fossa

**CONFIRM**  
position

**TRANSEPTAL**  
access

**RETRACT**  
needle

**INSERT**  
exchange wire

**RETRACT**  
dilator/sheath

**INSERT**  
therapy sheath/dilator

**RETRACT**  
dilator

**INSERT**  
WATCHMAN  
Delivery Catheter\*



Guidewire  
& sheath

## VersaCross Connect™ LAAC Access Solution

DESIGNED for use with  
**WATCHMAN**  
Access Systems†





**3-IN-1  
RF WIRE**

for ZERO EXCHANGE delivery  
of WATCHMAN™ Access Sheath†

**Smooth transition** between  
VersaCross Connect™ Dilator and  
WATCHMAN™ Access Sheath†



**Snap-lock** designed for  
WATCHMAN™ Access Sheaths†

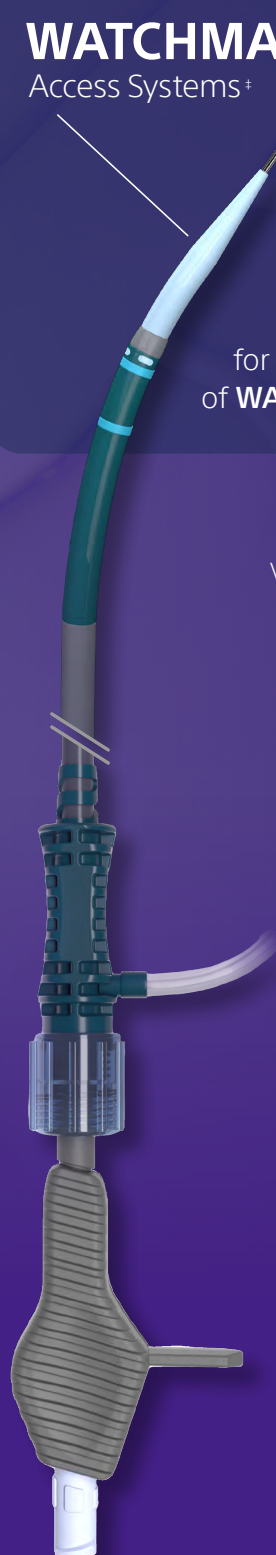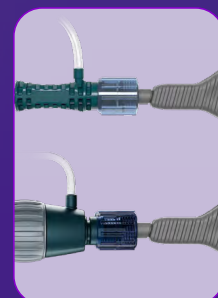

† WATCHMAN™ Access Sheaths as shown in this brochure is sold separately.