


## Communicating with the Implant

For best communication, your Remote Control should face your Implant.



Signal strength may change depending on Remote Control position. Check your signal strength indicator for optimal communication.

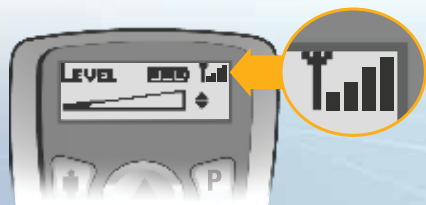

If communication is poor, you may see the following screens:



Try repositioning your Remote Control

OR



Press **P** to try again or press **i** to cancel.

Not intended as a substitute for the provided *Patient System Handbook* (9055425-001) or the *Patient Trial Handbook* (9055426-001). The contents of the handbook must be fully understood before using your Precision Plus System.

All cited trademarks are the property of their respective owners. **CAUTION:** The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labelling supplied with each device. Information for the use only in countries with applicable health authority product registrations. Precision is a registered trademark of Boston Scientific Corporation or its affiliates.

© 2014 Boston Scientific Corporation or its affiliates. All rights reserved.

9055581-003 Rev B

3-300263\_MAR2014  
Printed in the U.S.A.

**CE** Authorized to affix  
0123 the CE mark in 2013

**Boston Scientific**

**Neuromodulation**  
25155 Rye Canyon Loop  
Valencia, CA 91355

866.360.4747 Toll-free  
877.464.2940 Fax  
[ControlYourPain.com](http://ControlYourPain.com)

**PRECISION™ PLUS**  
Spinal Cord Stimulator System

**Boston Scientific**

Remote Control  
Quick Start Guide



Model SC-5212

## 1 Insert Batteries

Requires three AAA batteries.



## 2 Unlock Your Remote Control

Press and hold **P** until the screen changes to "Release P to Unlock."



## 3 Stimulation

Press this button at any time to turn stimulation on/off.



Press the Up/Down Arrow buttons to increase and decrease stimulation.



## 4 Program Settings

From the Level Screen:



Press **P** for programs. Press **P** again to cycle through the programs.

Press the Up Arrow button to activate the highlighted program.

OVER ►

My Physician \_\_\_\_\_

Name \_\_\_\_\_

Phone \_\_\_\_\_

Hospital \_\_\_\_\_