


4 Recharging the Remote Control



Your Remote will display a message reminding you to recharge whenever the battery power becomes low.


Recharge the Remote battery by connecting it to the USB Power Supply provided.

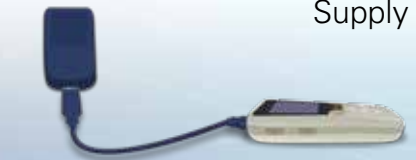



5 Accessing Programs

If your healthcare provider has configured Programs that can be changed, there are two ways to view the Programs menu:



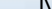

1 Press the Programs button  on the left side of the Remote


2 From the Main Menu , select Programs **P** by pressing 

6 Changing Programs





1 From the Home Screen, press  to display the Main Menu 





2 Press  again to enter the Programs menu **P**




3 Press  again to enter the Change Program menu 

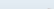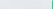

4 Press  or  to select the desired Program



5 Press  to activate the selected Program

Not intended as a substitute for the provided *Precision Spectra™ Remote Control Handbook* (90668562/90668563). The contents of the handbook must be fully understood before using your remote control.

All cited trademarks are the property of their respective owners. **CAUTION:** The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labeling supplied with each device. Information for the use only in countries with applicable health authority product registrations. Precision Spectra is a registered trademark of Boston Scientific Corporation or its affiliates.

© 2014 Boston Scientific Corporation or its affiliates. All rights reserved.

90819321-01 Rev B

3-300185_MAR2014
Printed in the U.S.A.

 Authorized to affix the CE mark in 2012



Neuromodulation
25155 Rye Canyon Loop
Valencia, CA 91355

866.360.4747 Toll-free
877.464.2940 Fax
ControlYourPain.com

**PRECISION
SPECTRA™**
Spinal Cord Stimulation System

**Boston
Scientific**

Remote Control
Quick Start Guide



Model SC-5232

1 Remote Control Buttons



Screen Display



2 Start/Stop Stimulation

Stimulation can be turned on and off by pressing the Stimulation **ON/OFF** button. ☀

To turn stimulation **ON**, you must unlock the Remote first; then, press the Stimulation **ON/OFF** button.

To turn stimulation **OFF**, press the Stimulation **ON/OFF** button again. Stimulation can be turned off at any time, even when the Remote is locked.



To **START/STOP** stimulation, press ☀

3 Quick Start Guide



OVER ▶

My Physician

Name

Phone

Hospital

Boston Scientific

Defining tomorrow, today.™

www.bostonscientific-international.com