

MAJESTIC Clinical Trial

3-Year Follow-Up¹

MAJESTIC Stenting of the Superficial Femoral and/or Proximal Popliteal Artery Project with Boston Scientific's (ELUVIA™) Drug-Eluting Stent

OBJECTIVE:

To evaluate the performance of the Eluvia Drug-Eluting Vascular Stent System in the treatment of superficial femoral (SFA) and/or proximal popliteal artery (PPA) lesions.

TRIAL DESIGN:



- Prospective, single-arm, non-randomized, multi-center, core lab adjudicated
- Primary patency, ankle-brachial index (ABI), Rutherford classification and stent fracture evaluated at 12 and 24 months
- Freedom from TLR evaluated at 12, 24 and 36 months
- Eligible patients with chronic, symptomatic (Rutherford categories 2, 3 or 4) lower limb ischemia and stenotic, restenotic or occlusive lesions in the native superficial femoral artery or proximal popliteal artery (n=57)
- Degree of stenosis $\geq 70\%$ (visual angiographic assessment)
- Vessel diameter ≥ 4 mm and ≤ 6 mm
- Total lesion length ≥ 30 mm and ≤ 110 mm

BASELINE CHARACTERISTICS:

Patient Demographics	n = 57 subjects	Lesion Characteristics (Core Lab)	n = 57 lesions
Age (Years)	69.3 \pm 9.3	Reference Vessel Diameter	5.2 \pm 0.8
Male Gender	82.5%	Target Lesion Length	70.8 \pm 28.1
Diabetes Mellitus	35.1%	Severely Calcified	64.9%
History of Smoking	87.7%	Percent Diameter Stenosis	86.3% \pm 16.2%
Hypertension	73.7%	Total Occlusions	46.2%
Hyperlipidemia	63.2%	% Extending into Distal SFA	77.2%
Coronary Artery Disease	38.6%	% Extending into PPA	8.8%

3-YEAR RESULTS:

The Eluvia Stent continues to demonstrate unprecedented clinical outcomes with an 85.3% freedom from TLR at 3 years, one of the highest reported in comparable SFA clinical trials.

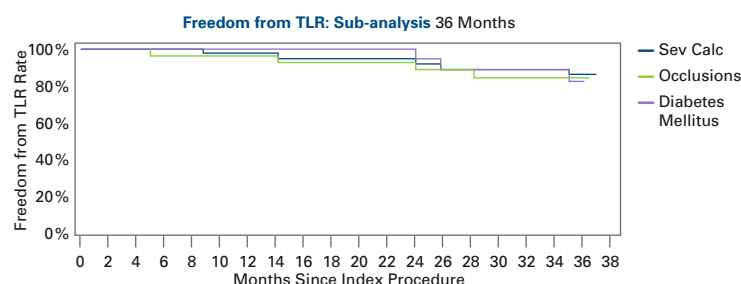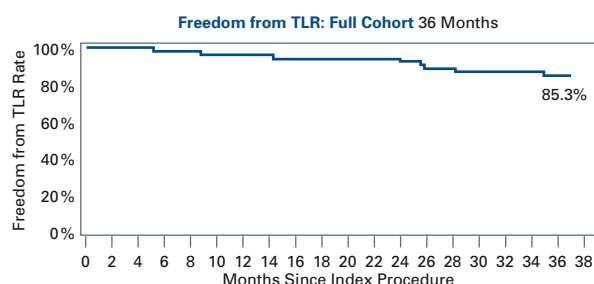

	12 Months	24 Months
Primary Patency ^a	96.4%	83.5%
Assisted Primary Patency ^b	98.2%	88.9%

Note: Kaplan-Meier Estimates. Per study protocol, primary patency was not evaluated at 36 months.

^a Duplex ultrasound peak systolic velocity ratio ≤ 2.5 and absence of TLR or bypass.

^b No TLR and those with TLR not for complete occlusion or bypass who were free of restenosis at 24 months.

¹ Müller-Hülsbeck S, et al. Long-Term Results from the MAJESTIC Trial of the Eluvia Paclitaxel-Eluting Stent for Femoropopliteal Treatment: 3 Year Follow up. Cardiovasc Interv Radiol. 2017, in press.

MAJESTIC Clinical Trial

3-Year Follow-Up¹

SAFETY RESULTS:

- 15.1% composite major adverse event rate at 36 months
- No deaths or amputations; 8 TLRs (8/53)


Overall	
36-Month MAE	15.1 %
All-Cause Death at 1 Month	0.0 %
Target Limb Major Amputation	0.0 %
Target Lesion Revascularization (TLR)	15.1 %

STENT INTEGRITY:

- No stent fractures at 24 months upon angiographic core lab analysis²

PATIENT OUTCOMES:

- 91% of patients presented with no or minimal claudication (Rutherford 0-1) at 24 months²
- 91% of patients sustained clinical improvement through 24 months
- Mean ABI improved from 0.73 ± 0.22 at baseline to 0.93 ± 0.26 at 24 months

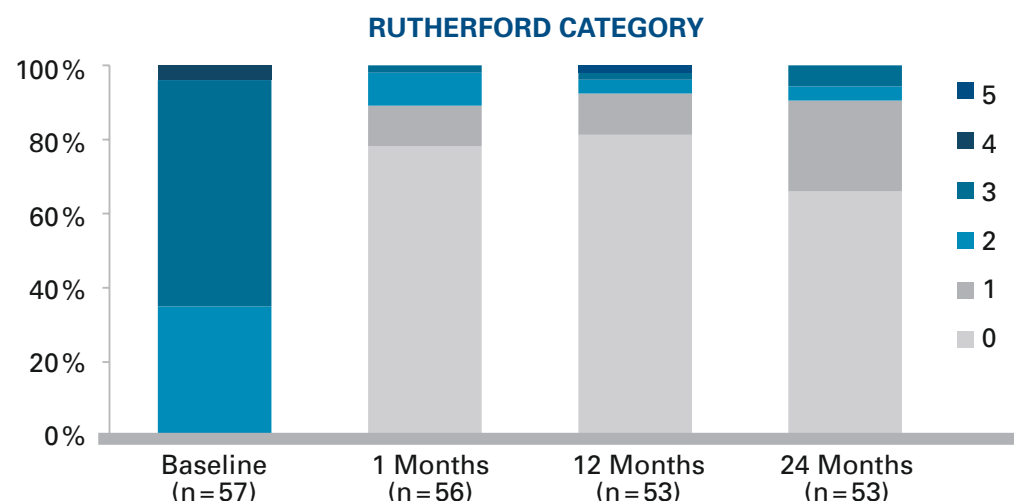

CONCLUSIONS:

- Long-term results from the MAJESTIC study of the Eluvia stent continue to demonstrate good clinical outcomes (assessed through 2 years) and a low reintervention rate (through 3 years).
- Subgroup analysis suggests that the low TLR rate was representative of outcomes for patients with risk factors such as diabetes, severe calcification, and occlusion

1 Müller-Hülsbeck S, et al. Long-Term Results from the MAJESTIC Trial of the Eluvia Paclitaxel-Eluting Stent for Femoropopliteal Treatment: 3 Year Follow up. Cardiovasc Interv Radiol. 2017, in press.

2 Per study protocol, duplex ultrasound (DUS), systematic x-ray fracture evaluation, ankle-brachial index (ABI), and Rutherford classification assessments were not evaluated at 3 years.

CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labeling supplied with each device. Information for use only in countries with applicable health authority product registrations. This material is not for use or distribution in France. Eluvia is a registered or unregistered trademark of Boston Scientific Corporation or its affiliates. All other trademarks are property of their respective owners.

PI-494010-AA AUG2017 Printed in Germany by medicalvision.

**Boston
Scientific**
Advancing science for life™

www.bostonscientific.eu

© 2017 Boston Scientific Corporation
or its affiliates. All rights reserved.
DINPER4753EA