


COMPARE CLINICAL TRIAL¹

COMPARE is the world's first head-to-head prospective, RCT (1:1) comparing low dose Ranger™ DCB (2 µg/mm²) to higher dose IN.PACT™ DCB (3.5 µg/mm²)

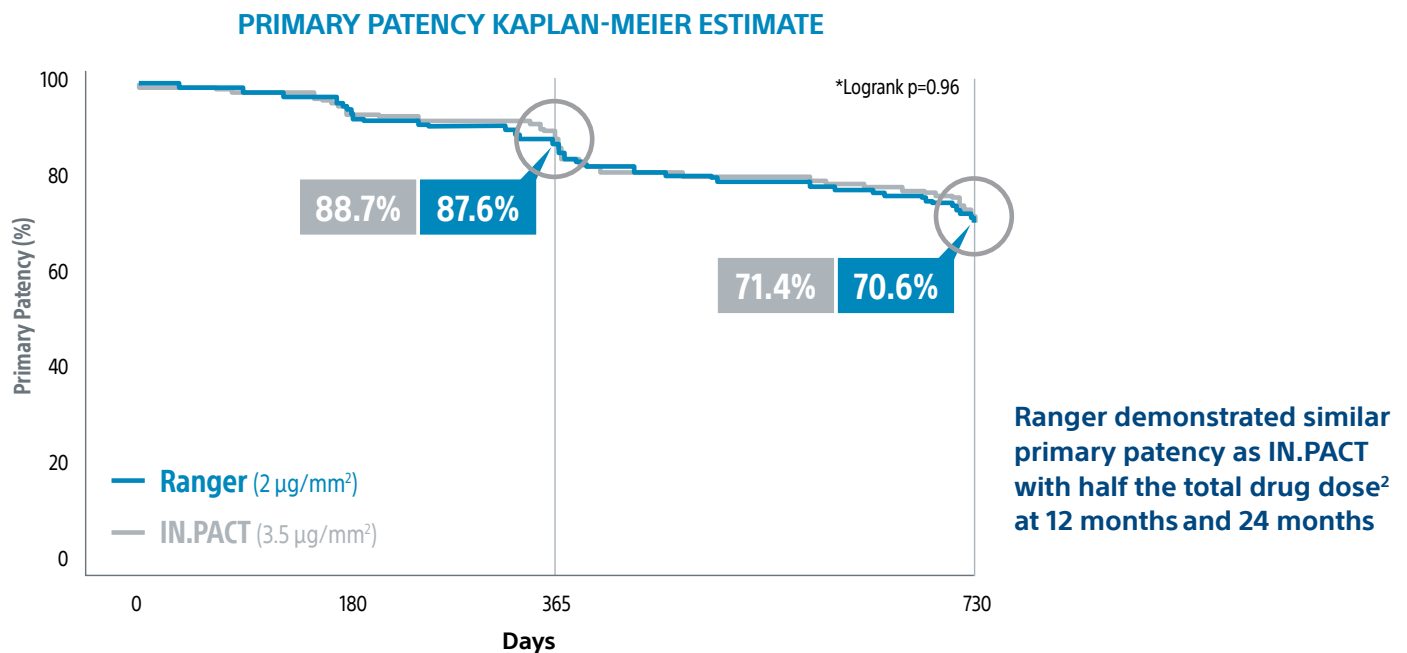

At time point zero: **Ranger** n=207 **IN.PACT** n=207

*Log-rank p-value compares the entire K-M curves from time zero to full two-year follow-up window.

1. COMPARE Clinical Trial 24-Month Results presented by Sabine Steiner, MD. LINC 2021.

2. Based on total drug dose for 4mm x 60mm or averages for full size matrix per the IN.PACT™ Admiral™ Drug-Coated Balloon Instructions for Use, www.medtronic.com and the Ranger™ Paclitaxel-Coated PTA Balloon Catheter Instructions for Use.

BASELINE CHARACTERISTICS

	RANGER (n=207)	IN.PACT (n=207)	p-value
Age	68.2	68.4	0.79
Female	38.2%	36.2%	0.68
Current/Former Smoker	77.3%	75.3%	0.63*
Total Occlusions	41%	43%	0.62
Total Occlusion Length	131 mm	113 mm	0.23
Target Lesion Length	124 mm	128 mm	0.65
Moderate to Severe Calcification**	51%	57%	***
Diabetics	31%	37%	0.18

* p-value based on entire distribution Never, Former or Current Smokers

** PACSS Grade 3/4 may be considered similar to moderate/severe calcification.

*** p-value for entire distribution of PACSS Calcium Grades 0, 1, 2, 3, 4 calcium for RANGER vs. IN.PACT. p-value was 0.20.

COMPARE TRIAL DETAILS	RANGER (n=207)	IN.PACT (n=207)	p-value
Excipient	TransPax™ citrate ester	Urea	
Paclitaxel dose density	2.0 µg/mm ²	3.5 µg/mm ²	
Average total paclitaxel dose per patient in trial	6,971 µg	13,035 µg	<0.0001

24-MONTH KEY RESULTS	RANGER (n=207)	IN.PACT (n=207)	p-value
Mortality: All Cause	3.6% (7/196)	2.2% (4/181)	0.6
Mortality: Device or Procedure Related	0%	0%	1.0
CD-TLR	17.3%	13.0%	0.3

12-MONTH KEY RESULTS ³	RANGER (n=207)	IN.PACT (n=207)	p-value
Binary Primary Patency*	83.0% (156/188)	81.5% (141/173)	P _{non-inferiority} <0.01
Freedom from Major Adverse Events*	91.0% (182/200)	92.6% (175/189)	P _{non-inferiority} <0.01
Mortality: All Cause	2.5%	1.6%	0.73
Mortality: Device or Procedure Related	0%	0%	N/A
CD-TLR	9.0%	7.4%	0.59

* Primary Endpoint Met

12-Month Results Published in the European Heart Journal

COMPARE: prospective, randomized, non-inferiority trial of high vs. low dose paclitaxel drug-coated balloons for femoropopliteal interventions. doi.org/10.1093/eurheartj/ehaa049

3. Sabine Steiner, et al. COMPARE: prospective, randomized, non-inferiority trial of high- vs. low-dose paclitaxel drug-coated balloons for femoropopliteal interventions, European Heart Journal, Volume 41, Issue 27, 14 July 2020, Pages 2541-2552, <https://doi.org/10.1093/eurheartj/ehaa049>.

Definitions:

Primary safety endpoint: composite of freedom from device and procedure-related death through 30 days and freedom from major target limb amputation and CD-TLR through 12 months post index-procedure.

Primary efficacy endpoint: primary patency at 12 months defined as absence of clinically driven target lesion revascularization (CD-TLR) or binary restenosis determined as a peak systolic velocity ratio > 2.4 evaluated by duplex ultrasound core laboratory analysis.

CD-TLR: a reintervention performed for ≥ 50% diameter stenosis (confirmed by angiography) within ± 5 mm proximal and/or distal to the target lesion after documentation of recurrent clinical symptoms of PAD (increase of 1 Rutherford class or more) and/or drop of ABI (≥20% or >0.15 when compared to maximum early post-procedural level).

**Boston
Scientific**
Advancing science for life™

Peripheral Interventions
www.bostonscientific.eu

CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labelling supplied with each device. Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries. This material is not intended for use in France.

Ranger is a registered or unregistered trademark of Boston Scientific Corporation or its affiliates. All other trademarks are property of their respective owners.

© 2021 Boston Scientific Corporation or its affiliates. All rights reserved.

PI-961805-AA