

ELUVIA™ Drug-Eluting Vascular Stent System

12-month results from IMPERIAL, the world's first head-to-head DES SFA Trial¹

Presented at LINC 2020

OBJECTIVE:

Evaluate the safety and effectiveness of the Boston Scientific Corporation ELUVIA™ Drug-Eluting Vascular Stent System for treating Superficial Femoral Artery (SFA) and/or Proximal Popliteal Artery (PPA) lesions up to 140 mm in length.

IMPERIAL TRIAL DESIGN:


Global multi-center, 2:1 randomization against Cook Medical's Zilver™ PTX™ Stent, controlled, single-blind, non-inferiority trial; core lab adjudicated

- 465 (RCT) patients across 64 sites
- 5-year follow-up
- Degree of stenosis ≥ 70% (visual angiographic assessment)
- Vessel diameter ≥ 4 mm and ≤ 6 mm
- Total lesion length ≥ 30 mm and ≤ 140 mm

BASELINE CHARACTERISTICS:

| Patient Demographics | Eluvia (n=309) | Zilver PTX (n=156) | Lesion Characteristics | Eluvia (n=309) | Zilver PTX (n=156) |
|----------------------|----------------|--------------------|---------------------------|----------------|--------------------|
| Age (Years) | 68.5±9.5 | 67.8±9.4 | Target Lesion Length (mm) | 86.5±36.9 | 81.8±37.3 |
| Male Gender | 66.0% | 66.7% | Severely Calcified | 40.1% | 32.3% |
| Diabetes Mellitus | 41.7% | 43.6% | Total Occlusions | 31.2% | 30.3% |
| History of Smoking | 86.1% | 84.0% | Extending into Distal SFA | 66.3% | 65.4% |

12-MONTH RESULTS: Eluvia demonstrated a statistically significant difference in primary patency and half the TLR rate vs. Zilver PTX at 12 months

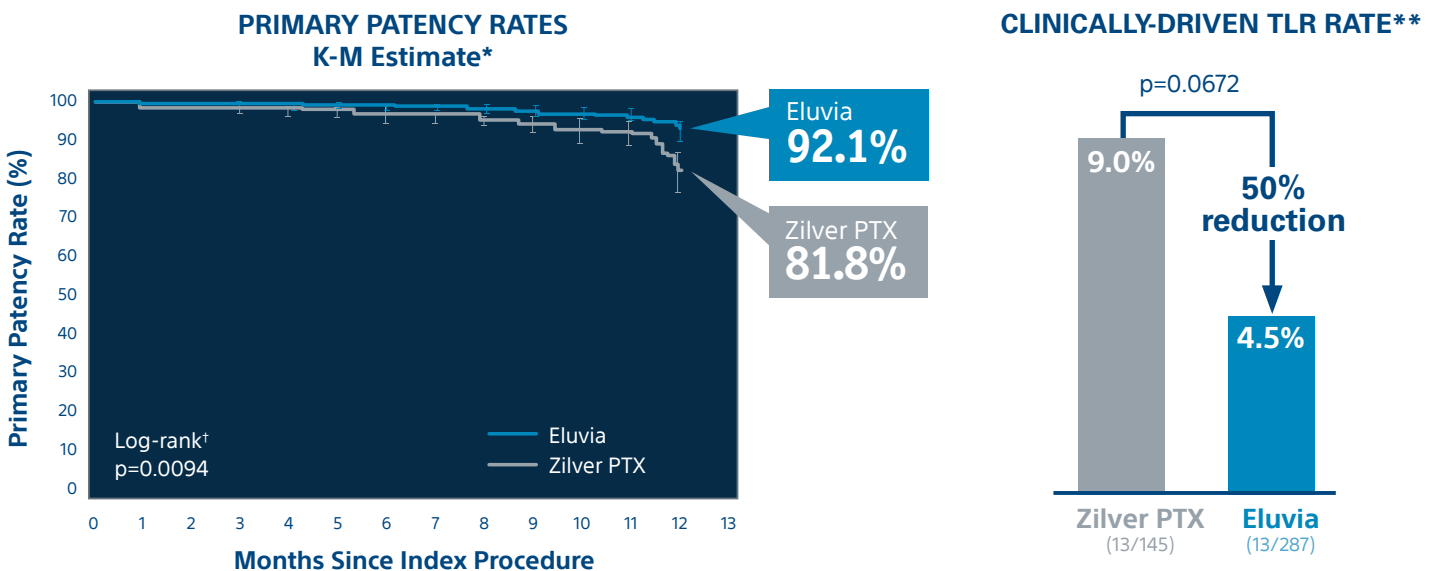

In IMPERIAL RCT and as reported in The Lancet, 1-year ITT, CEC adjudicated mortality rates were 2.1% (6/292) for Eluvia and 4.0% (6/150) for Zilver PTX (p=0.23).


*Kaplan Meier Estimate; Primary patency as determined by duplex ultrasound (DUS) Peak Systolic Velocity Ratio (PSVR) is ≤ 2.4 at the 12-month follow-up visit, in the absence of clinically-driven TLR or bypass of the target lesion.

** Gray, WA TCT 2018.


† Log-rank p-value compares the entire K-M curves from time zero to full one year follow-up window.

1. IMPERIAL Trial: A global randomized controlled multi-center trial with 2:1 randomization of the Eluvia™ Drug-Eluting Stent against Cook Medical's Zilver™ PTX™ Stent, single-blind, non-inferiority design; independent core lab adjudication. Superiority determined in a post hoc analysis that was specified prior to unblinding. 12-Month Primary Patency rate of 86.8% in the Eluvia arm vs. 77.5% in the Zilver PTX arm (p-value = 0.0144).

IMPERIAL Diabetic[^] Subgroup 12-Month Results: Eluvia demonstrated a higher primary patency and a statistically significant TLR reduction of greater than 70% in diabetic patients vs. Zilver PTX

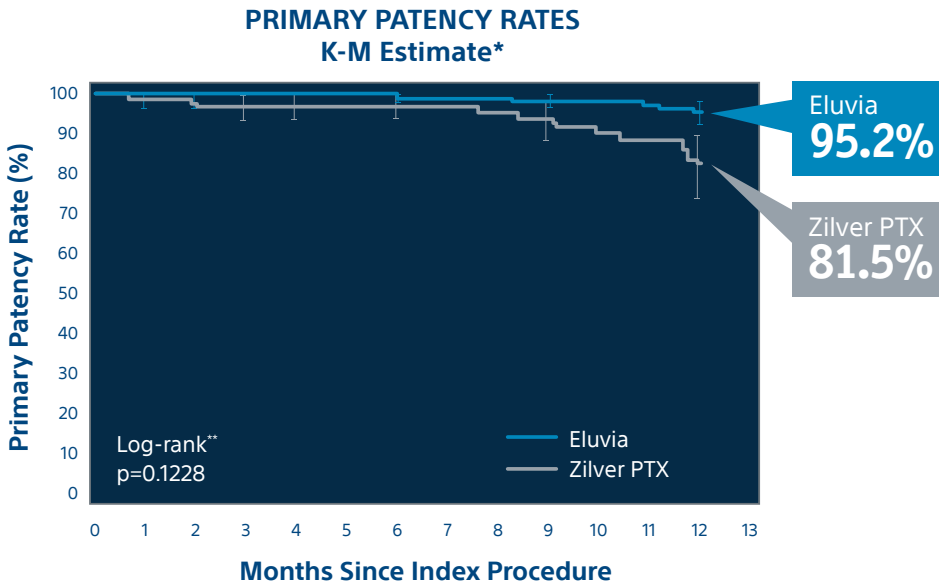

CLINICALLY-DRIVEN TLR RATE¹

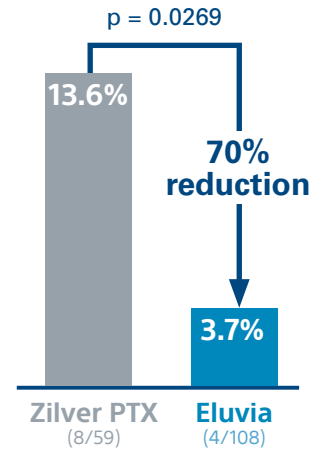

12-Month Remarkable and Consistent Results in Complex Lesions

| | IMPERIAL RCT ² (n = 309) | IMPERIAL Long Lesions (n = 50) | IMPERIAL Diabetic Subgroup Analysis ³ (n = 116) | IMPERIAL Severe / Moderate Calcium Subgroup Analysis (n = 193) | IMPERIAL CTO Subgroup Analysis (n = 96) | Münster Registry (n = 62) |
|---------------------------------------|--|--|---|---|---|--------------------------------|
| Study Design | RCT, multicenter, global | Single arm multicenter, global | RCT, multicenter, global | RCT, multicenter, global | RCT, multicenter, global | Single-center registry |
| 12-month primary patency rate* | 92.1% | 91.0% | 95.2% | 92.5% | 86.4% | 87.0% ⁴ |
| Lesion length (mm) | 86.5 | 162.8 | 87.0 | 89.9 | 94.4 | 200 |
| Severe calcification | 40% | 28% | 46% | n/a | n/a | 42% ⁵ |
| Total occlusions | 31% | 32% | 25% | n/a | 100% | 79% |
| | Statistically significantly higher primary patency | Remarkable primary patency in long lesions | Statistically significantly lower TLR (3.7%) and stent thrombosis rate (0.9%) | Remarkable primary patency and 2.8% TLR in heavy calcium | Low TLR (7.9%) and stent thrombosis rate (2.2%) | CLI in nearly half of patients |

[^]Diabetes = medically-treated diabetic patients.

*Kaplan Meier Estimate; Primary patency as determined by duplex ultrasound (DUS) Peak Systolic Velocity Ratio (PSVR) is ≤ 2.4 at the 12-month follow-up visit, in the absence of clinically-driven TLR or bypass of the target lesion.

** Log-rank p-value compares the entire K-M curves from time zero to full one year follow-up window.

1. Intention to treat. Müller-Hülsbeck S, LINC 2019.

2. In IMPERIAL RCT, Eluvia K-M Primary Patency was 92.1% vs. 81.8% for Zilver PTX at 12 months.

3. In IMPERIAL Diabetic Subgroup, Eluvia K-M Primary Patency was 95.2% vs. 81.5% for Zilver PTX at 12 months.

4. PSVR < 2.0 .

5. Moderate and severely calcified.

Eluvia is a registered or unregistered trademark of Boston Scientific Corporation or its affiliates. All other trademarks are property of their respective owners.

CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labeling supplied with each device. Information for use only in countries with applicable health authority registrations. Material not intended for use in France.

Boston Scientific
Advancing science for life™

www.bostonscientific.eu

© 2020 Boston Scientific Corporation or its affiliates. All rights reserved.

PI-567103-AE