




ICESEED™ 1.5 CX and ICESEED™ 1.5 CX Short Cryoablation Needles

WHEN PRECISION MATTERS

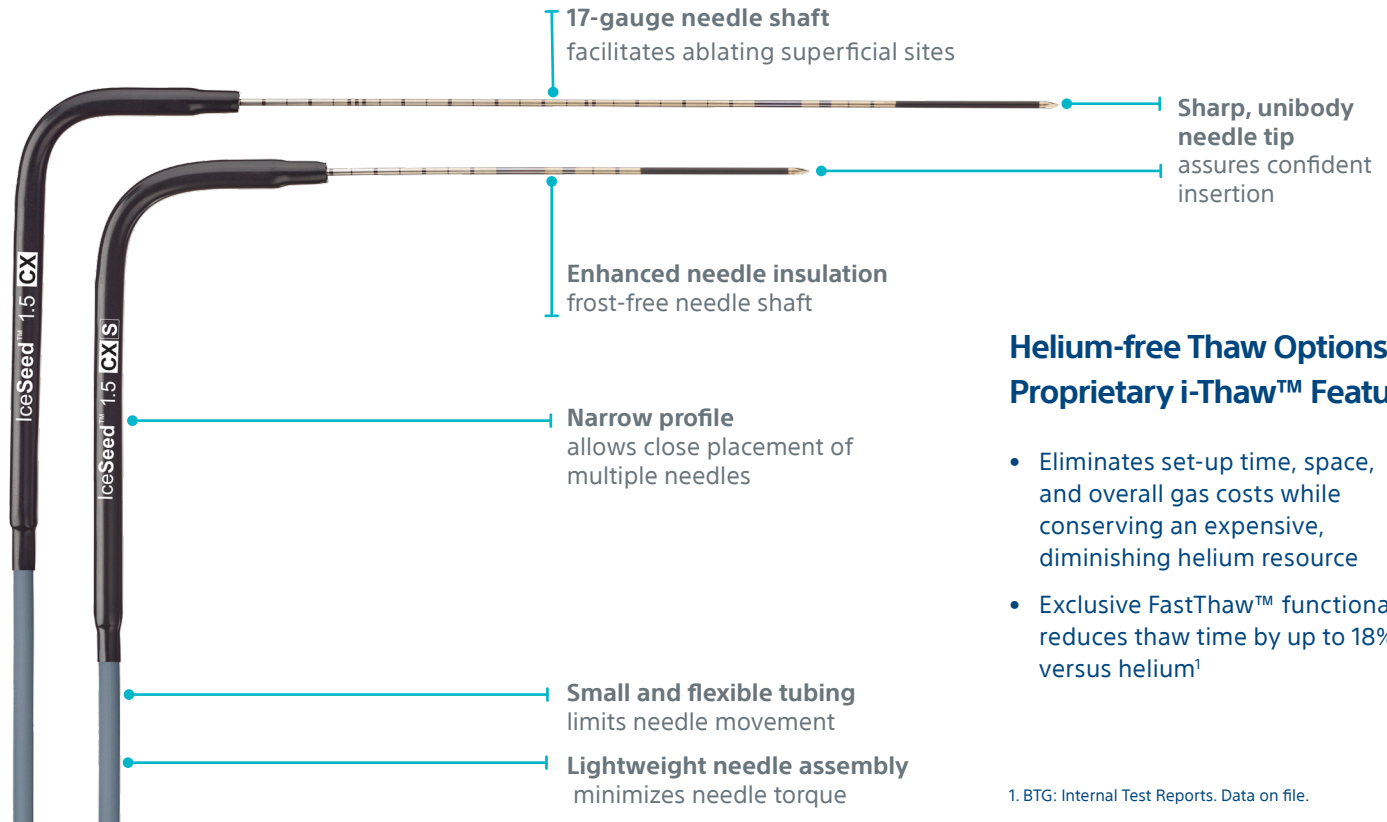


Helium-free Thaw Options via Proprietary i-Thaw™ Feature

- Eliminates set-up time, space, and overall gas costs while conserving an expensive, diminishing helium resource
- Exclusive FastThaw™ functionality reduces thaw time by up to 18% versus helium¹

1. BTG: Internal Test Reports. Data on file.

Precise Control...Refined Therapeutic Choice

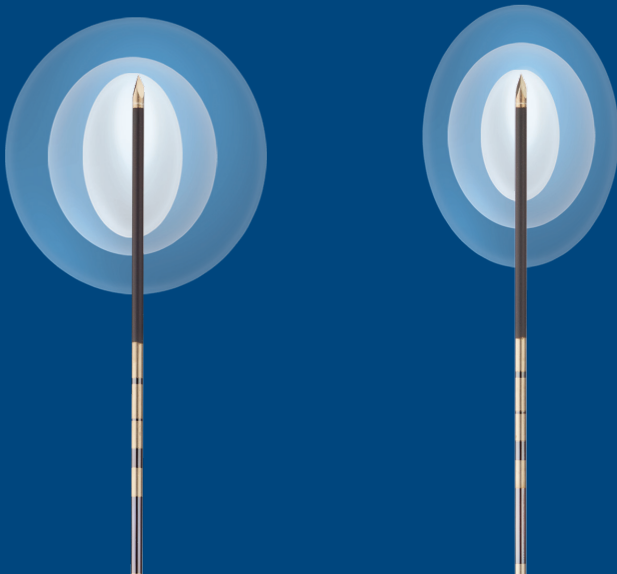

With the benefits of a shorter shaft length combined with the smaller, sculptable iceball size of IceSeed CX, you'll have flexibility of treatments for

- benign or malignant tumors
- cryoneurolysis
- small or superficial lesions


ICESEED™ 1.5 CX and ICESEED™ 1.5 CX Short Cryoablation Needles

IceSeed™ 1.5 CX

IceSeed 1.5 CX Needle:
H7493967433170

Shaft Length:
17.5 cm

Shaft Diameter:
1.5 mm / 17G

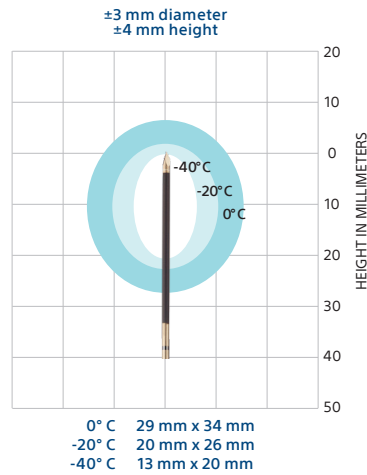


IceSeed™ 1.5 CX Short

IceSeed 1.5 CX Short Needle:
H7493967233100

Shaft Length:
10 cm

Shaft Diameter:
1.5 mm / 17G

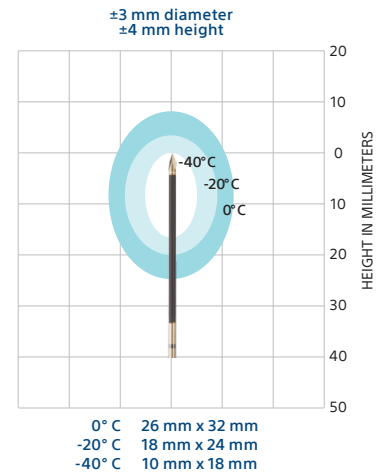

Multi-Needle Isotherm Values (diameter x height in mm)

CRYOABLATION NEEDLES	SINGLE-NEEDLE ISOTHERM			TWO-NEEDLE ISOTHERM			THREE-NEEDLE ISOTHERM			FOUR-NEEDLE ISOTHERM		
	0° C	-20° C	-40° C	0° C	-20° C	-40° C	0° C	-20° C	-40° C	0° C	-20° C	-40° C
IceSeed™ 1.5 CX	29 x 34	20 x 26	13 x 20	41 x 39	31 x 29	23 x 21	48 x 49	38 x 39	29 x 31	52 x 52	42 x 42	33 x 34
IceSeed™ 1.5 CX S	26 x 32	18 x 24	10 x 18	39 x 36	29 x 26	21 x 17	43 x 45	34 x 36	25 x 27	49 x 51	40 x 41	31 x 33

SKIN PROTECTION: When performing cryoneurolysis on shallow nerves make sure to place a sterile glove filled with warm saline or warmed ultrasound gel at the surface of the skin. Also, avoid direct contact between skin and needle tubing with the use of towels or draping.

Ordering Information Cryoablation Portfolio of Needles

CRYOABLATION NEEDLES	PART NUMBER	CONFIGURATION	SHAFT DIAMETER (MM / GAUGE)	SHAFT LENGTH (CM)	HANDLE COLOR	TRACK ABLATION RADIAL WIDTH / LENGTH
IceSeed™ 1.5 CX	H7493967433170	Angled 90°	1.5 mm / 17 G	17.5 cm	Black	1.6 / 14 mm
IceSeed™ 1.5 CX S	H7493967233100	Angled 90°	1.5 mm / 17 G	10 cm	Black	1.6 / 14 mm

Boston Scientific

Advancing science for life™

www.bostonscientific.eu

© 2024 Boston Scientific Corporation or its affiliates. All rights reserved.

PI-1240404-AB

All cited trademarks are the property of their respective owners.

CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labeling supplied with each device. Information for the use only in countries with applicable health authority product registrations. Material not intended for use in France.