


LithoVue Empower™

Retrieval Deployment Device

Assembly Instructions


Please refer to directions for use for intended use statement, relevant warnings, precautions, side effects and contraindications of the device.

A) Connect to a Nitinol Basket

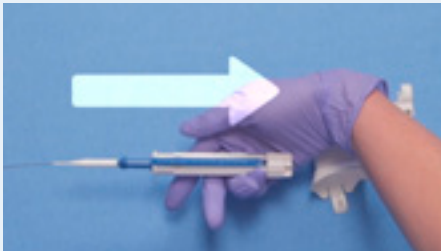

STEP 1

To secure the retrieval device onto LithoVue Empower Device, the basket must be in the full open position by sliding the handle back completely.

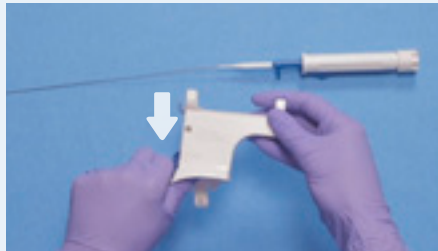

STEP 2

Pick up the LithoVue Empower Device and hold down the trigger.



STEP 3

Snap your Boston Scientific retrieval device into LithoVue Empower Device by matching the blue ridge on the handle to the blue indentation on LithoVue Empower Device. The handle will snap into place when secured correctly.

B) Connect to the LithoVue™ Ureteroscope



STEP 4

To connect LithoVue Empower Device to your LithoVue Ureteroscope, align the pin with the D/U markers on the ureteroscope.

This side must be attached first for the device to function properly. When aligned correctly, pressing down will snap the device into place.



STEP 5


Verify that your final assembly matches the above image.



STEP 6

Feed the basket through your scope adaptor. Ensure the adaptor is completely open before feeding your device through or use an introducer.

*Please note that if you are using an adaptor other than a tuohy borst style adaptor, you **MUST** use an introducer to insert the basket into the working channel of the scope.*

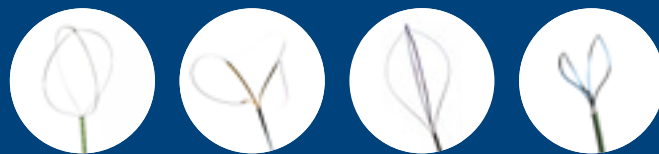

Maximize the value of the LithoVue™ Ureteroscope

The LithoVue Empower™ Retrieval Deployment Device works with our LithoVue™ Ureteroscope. Now you can turn your trusted LithoVue Ureteroscope into a physician-controlled retrieval device.

Integrate seamlessly

Compatible with Boston Scientific's Zero Tip™, Dakota™, Escape™ and Graspit™ Nitinol Retrieval Devices.

Ordering information

Product Number	Description	Quantity
M0067919900	LithoVue Empower Retrieval Deployment Device	1 unit



Want to learn more about the LithoVue Empower Device and how you can take control of stone removal?

Visit www.bostonscientific.eu

**Boston
Scientific**
Advancing science for life™

All cited trademarks are the property of their respective owners. CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labeling supplied with each device. Information for the use only in countries with applicable health authority product registrations. Information not intended for use or distribution in France.

URO-565403-AA JUL 2018 Printed in Germany by medicalvision.

www.bostonscientific.eu

© 2018 Boston Scientific Corporation or its affiliates. All rights reserved.
DINURO2323EA