

# AdVance™ Male Sling System

## Post-procedure instructions

**Congratulations, you have taken this important step to improve your quality of life!**

The AdVance Male Sling is a proven, reliable treatment for male stress urinary incontinence. To ensure continued success with this treatment, carefully review and follow these instructions.

**Follow these instructions **post-procedure weeks 1-6** or based on your urologist's recommended timeline.**

1. No lifting anything greater than 10-15 lbs.
2. No bending, squatting, climbing, (e.g., stepping up into high vehicles), extreme leg spreads, biking, weightlifting or jogging.
3. No intercourse.

**Follow these instructions **post-procedure weeks 1-2** or based on your urologist's recommended timeline.**

1. No bath or hot tubs; however, you may shower 24 hours after your procedure.

**Additional expectations following your procedure:**

1. Your urologist may prescribe a stool softener, pain medication and antibiotic for you following your procedure.
2. It is not uncommon to experience some pain at the procedural site. This is typically controlled with pain medication prescribed by your urologist.
3. Initially, you may have to strain a bit to void, and it is not uncommon for your urine stream to slightly decrease following your procedure. This will usually resolve. If you are unable to urinate following your procedure, you may need a catheter for a short period of time.

**CAUTION:** Federal (U.S.) law restricts this device to sale by or on the order of a physician.

Your doctor is your best source for information on the risks and benefits of the AdVance™ Male Sling System. Talk to your doctor for a complete listing of risks, warnings and important safety information.

The AMS AdVance™ Male Sling System is intended for the treatment of male stress urinary incontinence (SUI). These devices are not for patients with urinary tract infections, a tendency to bleed easily; a blood clotting disorder, the inability to fight infection or any other condition that would interfere with healing, decreased kidney function; or relative blockage of the kidneys.

Additional information is provided in the product Patient Manuals, available through your doctor. MH-547816-AA



Boston Scientific Corporation  
300 Boston Scientific Way, Marlborough, MA 01752  
[www.amsmenshealth.com](http://www.amsmenshealth.com)

All trademarks are the property of their respective owners.  
© 2016 by Boston Scientific Corporation or its affiliates.  
All rights reserved. MH-390215-AA JUN 2016