To request a new card:

Please contact the AMS Patient Liaison at:
U.S. Toll Free: 1 800 328 3881 ext. 6261
or Tel: +1 952 930 6261

American Medical Systems, Inc.

10700 Bren Road West
Minnetonka, MN 55343 U.S.A.
U.S. Toll Free: 1 800 328 3881
Tel: +1 952 930 6000
Tel: +31 20 593 8800

American Medical Systems Europe B.V.

Haarlerbergweg 23 G
1101 CH Amsterdam Zuid-Oost
The Netherlands

©2016 Boston Scientific Corporation or its affiliates. en
All rights reserved. All trademarks are property of the respective owners.
21900005AW (A/W Rev P) (2016-11)

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American Medical Systems Europe B.V.

Haarlerbergweg 23 G
1101 CH Amsterdam Zuid-Oost
The Netherlands

©2016 Boston Scientific Corporation or its affiliates. fr
All rights reserved. All trademarks are property of the respective owners.
21900005AW (A/W Rev P) (2016-11)

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American Medical Systems Europe B.V.

Haarlerbergweg 23 G
1101 CH Amsterdam Zuid-Oost
The Netherlands

©2016 Boston Scientific Corporation or its affiliates. de
All rights reserved. All trademarks are property of the respective owners.
21900005AW (A/W Rev P) (2016-11)

AMS™

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Contattare AMS Patient Liaison:
Numero verde USA:
1 800 328 3881 int. 6261 o
Telefono: +1 952 930 6261

American Medical Systems, Inc.

10700 Bren Road West
Minnetonka, MN 55343 U.S.A.
U.S. Toll Free: 1 800 328 3881
Tel: +1 952 930 6000
Tel: +31 20 593 8800

American Medical Systems Europe B.V.

Haarlerbergweg 23 G
1101 CH Amsterdam Zuid-Oost
The Netherlands

©2016 Boston Scientific Corporation or its affiliates. it
All rights reserved. All trademarks are property of the respective owners.
21900005AW (A/W Rev P) (2016-11)

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American Medical Systems Europe B.V.

Haarlerbergweg 23 G
1101 CH Amsterdam Zuid-Oost
The Netherlands

©2016 Boston Scientific Corporation or its affiliates. es
All rights reserved. All trademarks are property of the respective owners.
21900005AW (A/W Rev P) (2016-11)

AMS™

Patient Identification Card

AMS Ambicor™ Penile Prosthesis

Patient Name
Patient Telephone
Implant Date
Implanting Surgeon
Surgeon Telephone
Clinic or Hospital Name
Clinic or Hospital City
Clinic or Hospital State
Clinic or Hospital Country

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
Ambicor™ Two-Piece Penile Implant

Important Physician Information

This patient has an AMS Ambicor Inflatable Penile Prosthesis implanted to treat erectile dysfunction.

Design: The prosthesis is a closed, saline-filled system consisting of two components: a pair of cylinders implanted in the corpora cavernosa and a pump implanted in the scrotum (Figure 1). Kink-resistant tubing connects the components, which are made from silicone elastomer.

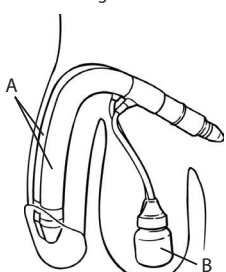Figure 1

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
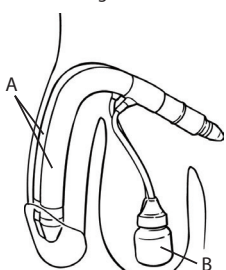
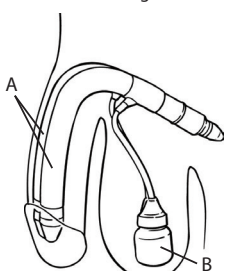
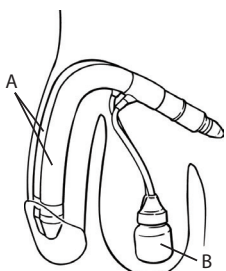
Figura 1

A. Cilindri B. Bulbo della pompa


To inflate: Squeeze and release the pump bulb several times. This transfers fluid from the reservoirs located at the proximal end of the cylinders to the main portion of the cylinders, creating an erection (Figure 2).

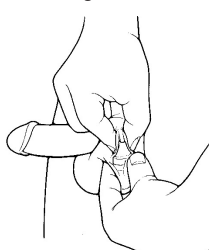Figure 2

To deflate: Bending the penis to an angle of 55°–65°, holding it there for 6–12 seconds, and then returning to a horizontal position will return the fluid to the reservoirs and the penis will relax (Figure 3).

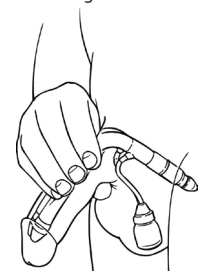Figure 3


Caution: In the event of a medical emergency, please be aware of the presence of this prosthesis.

MR safety information:
This prosthesis is
MR Conditional to 3.0 Tesla.

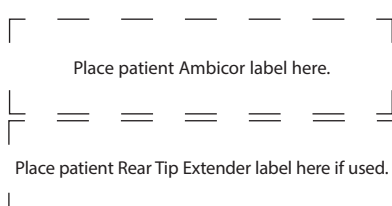Additional product information is available at www.amslabeling.com.

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