To request a new card:

Please contact the AMS Patient Liaison at:
U.S. Toll Free: 800 328 3881 ext. 6261 or
Tel: +1 952 930 6261

American Medical Systems, Inc.

10700 Bren Road West
Minnetonka, MN 55343 U.S.A.
U.S. Toll Free: 1 800 328 3881
Tel: +1 952 930 6000
Tel: +31 20 593 8800

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American Medical Systems Europe B.V.

Haarlerbergweg 23 G
1101 CH Amsterdam Zuid-Oost
The Netherlands

©2016 Boston Scientific Corporation or its affiliates. All rights reserved. All trademarks are property of the respective owners.
21900008AW (A/W Rev M) (2016-08)

es

AMS™

Patient Identification Card

AMS 700™ MS Series
Penile Prosthesis

Patient Name
Patient Telephone
Implant Date
Implanting Surgeon
Surgeon Telephone
Clinic or Hospital Name
Clinic or Hospital City
Clinic or Hospital State
Clinic or Hospital Country

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
AMS 700™
Three-Piece Penile Implant

Important Physician Information

This patient has an AMS 700 MS Series Three-Piece Inflatable Penile Prosthesis implanted to treat erectile dysfunction.

Design: The prosthesis is a closed, saline-filled system consisting of 3 components: a reservoir, a pair of cylinders implanted in the corpora cavernosa, and a pump implanted in the scrotum (Figure 1). Kink-resistant tubing connects the components, which are made from silicone elastomer.

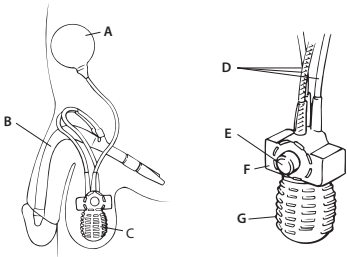Figure 1


A. Reservoir B. Cylinders C. Pump
D. Tubing E. Deflation button
F. Deflation block G. Pump bulb

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
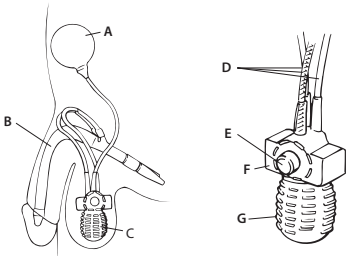
To inflate: Grasp the tubing above the deflation block with one hand. Using the other hand, squeeze and release the pump bulb quickly and firmly. Then squeeze and release the bulb several times until the cylinders fill with fluid and become erect. Do not squeeze the deflation button while squeezing the bulb (Figure 2).

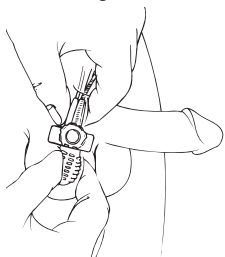Figure 2

To deflate: Grasp the tubing above the deflation block with one hand. Using the other hand, place the thumb and forefinger on opposite sides of the deflation block. Squeeze the deflation button for about 4 seconds, and then release it (Figure 3).

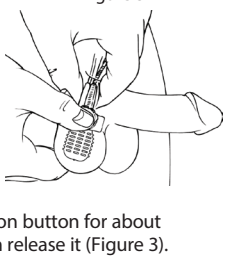Figure 3

Caution: In the event of a medical emergency, please be aware of the presence of this prosthesis.

MRI safety information:
This prosthesis is
MR Conditional to 3.0 Tesla.

Additional product information is available at www.amslabeling.com.

Place patient AMS 700 Cylinder label here, or if using a preconnected system, place label here.

Place AMS 700 MS Pump label here if a preconnected system was not used.

Place patient AMS 700 Reservoir label here.

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