### To request a new card:

Please contact the AMS Patient Liaison at:  
U.S. Toll Free: 800 328 3881 ext. 6261 or  
Tel: +1 952 930 6261

**American Medical Systems, Inc.**  
10700 Bren Road West  
Minnetonka, MN 55343 U.S.A.  
U.S. Toll Free: 800 328 3881  
Tel: +1 952 930 6000  
www.AmericanMedicalSystems.com

**American Medical Systems Europe B.V.**  
Haarlerbergweg 23 G  
1101 CH Amsterdam Zuid-Oost  
The Netherlands  
Tel: +31 20 593 8800

©2015 American Medical Systems, Inc.  
All rights reserved.  
21900008 (A/W Rev L) 2015-04

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**American Medical Systems, Inc.**  
10700 Bren Road West  
Minnetonka, MN 55343 U.S.A.  
U.S. Toll Free: 800 328 3881  
Tel: +1 952 930 6000  
www.AmericanMedicalSystems.com

**American Medical Systems Europe B.V.**  
Haarlerbergweg 23 G  
1101 CH Amsterdam Zuid-Oost  
The Netherlands  
Tel: +31 20 593 8800

©2015 American Medical Systems, Inc.  
Todos los derechos reservados.  
21900008 (A/W Rev L) 2015-04

es

### Patient Identification Card

AMS 700™ MS Series  
Penile Prosthesis

Patient Name \_\_\_\_\_  
Patient Telephone \_\_\_\_\_  
Implant Date \_\_\_\_\_  
Implanting Surgeon \_\_\_\_\_  
Surgeon Telephone \_\_\_\_\_  
Clinic or Hospital Name \_\_\_\_\_  
Clinic or Hospital City \_\_\_\_\_  
Clinic or Hospital State \_\_\_\_\_  
Clinic or Hospital Country \_\_\_\_\_

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
Three-Piece Penile Implant

## Important Physician Information

This patient has an AMS 700 MS Series Three-Piece Inflatable Penile Prosthesis implanted to treat erectile dysfunction.

**Design:** The prosthesis is a closed, saline-filled system consisting of 3 components: a reservoir, a pair of cylinders implanted in the corpora cavernosa, and a pump implanted in the scrotum (Figure 1). Kink-resistant tubing connects the components, which are made from silicone elastomer.

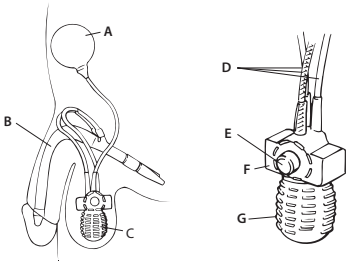Figure 1


A. Reservoir B. Cylinders C. Pump  
D. Tubing E. Deflation button  
F. Deflation block G. Pump bulb

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
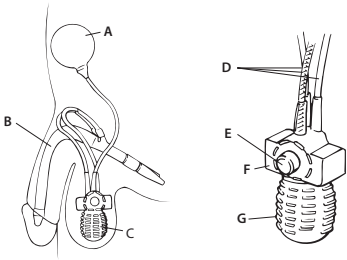
**To inflate:** Grasp the tubing above the deflation block with one hand. Using the other hand, squeeze and release the pump bulb quickly and firmly. Then squeeze and release the bulb several times until the cylinders fill with fluid and become erect. Do not squeeze the deflation button while squeezing the bulb (Figure 2).

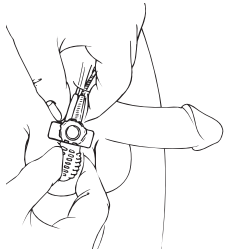Figure 2

**To deflate:** Grasp the tubing above the deflation block with one hand. Using the other hand, place the thumb and forefinger on opposite sides of the deflation block. Squeeze the deflation button for about 4 seconds, and then release it (Figure 3).

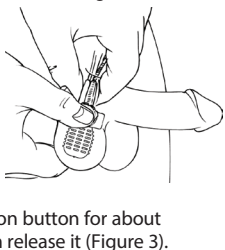Figure 3

**Caution:** In the event of a medical emergency, please be aware of the presence of this prosthesis.

**MRI safety information:**  
This prosthesis is  
MR Conditional to 3.0 Tesla.

Additional product information is available at [www.amslabeling.com](http://www.amslabeling.com).

Place patient AMS 700 Cylinder label here, or if using a preconnected system, place label here.

Place AMS 700 MS Pump label here if a preconnected system was not used.

Place patient AMS 700 Reservoir label here.

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