




LithoVue Empower™

Retrieval Deployment Device

Assembly instructions*

Connecting a Nitinol Basket

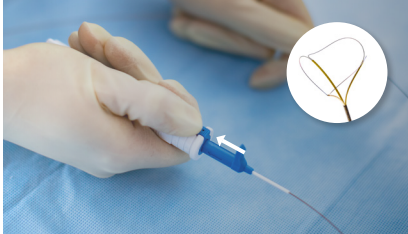

Step 1:
Open basket



Step 2:
Hold down trigger of LithoVue
Empower Device



Step 3:
Attach basket

Seat the basket onto the LithoVue Empower Device brackets

- **BLUE** basket slider facing **BLUE** LithoVue Empower Device slider
- Basket wire facing in the same direction as the trigger


Press down to snap into bracket



Step 4:
Inspect


- Release trigger
- Cap at the end of the basket should sit outside of the LithoVue Empower Device bracket

Connecting to the LithoVue™ Ureteroscope

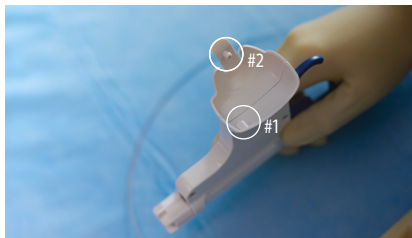

Step 5: Find pin holes

on LithoVue Empower™ Device

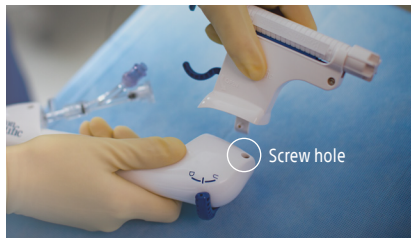

Step 6: Insert pin #1

of LithoVue Empower Device into
LithoVue scope screw hole



Step 7: Rotate

the LithoVue Empower Device
until pin #2 snaps into place



Step 8: Insert basket

into scope adaptor and confirm device
is operating correctly by pressing
down and releasing trigger

Caution: U.S. Federal law restricts this device to sale by or on the order of a physician.

CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings, and instructions for use can be found in the product labelling supplied with each device or at www.IFU-BSCI.com. Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries. This material not intended for use in France. Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries. Please check availability with your local sales representative or customer service.

All images are the property of Boston Scientific. All trademarks are the property of their respective owners.

Boston Scientific Corporation
300 Boston Scientific Way
Marlborough, MA 01752-1234
www.bostonscientific.com

© 2019 Boston Scientific Corporation
or its affiliates. All rights reserved.

URO-570713-AA JAN 2019

CE 0344 LithoVue Empower™
Retrieval Deployment Device