Title: SpaceOAR® System – 10mL IFU

Document Number: LCN 80-1010-001

Revision: L

Change History

Revision	Change Request #	Date	Description	Originator
A	CR0108	4/28/10	Original Issue	J. Mayol
B	CR0137	8/10/10	Add precaution statement “Do not use if the PEG powder is not free flowing” and correction to Figure 1	J. Mayol
C	CR0187	4/19/11	Add Swedish, German, Italian, French, Spanish, and Dutch to English IFU	J. Mayol
D	CR0207	8/22/11	Revise all references in regard to using the prepared precursor solution within “30 minutes” to using the prepared precursor solution within “1 hour”	J. Mayol
E	CR0211	11/8/11	Change “absorbed in 4-5 months” to absorbed in about 6 months”. Change “Note” box to a “Warning” box for consistency. Add “Note” box with recommended equipment for procedure, add “Warning” related to needle penetration of rectal lumen, add new Risks, updated pictorials, and further clarification of user instructions.	J. Mayol
F	CR0338	4/10/13	Add Turkish instructions to IFU, update to indicate that SpaceOAR is a registered trademark, add missing “no latex” symbol to Spanish section.	J. Mayol
G	CR0466	1/7/15	Warnings, Precautions and Risks sections updated. Provide further details to application technique instructions. Figure 1 has improved labeling and Figure 2 was added. Various sections reworded. Authorized Representative address updated. Required Equipment section added. Removed Note and Warning boxes.	C. Reinhold
H	CR0658	8/1/16	Added information on MRI Compatibility per ASTM F2503-13, add Danish instructions to IFU.	C. Reinhold
J	CR0807	9/6/17	Address Change	S. Tulipano
K	CR1100	7/31/18	Removed last sentence of overview Added first bullet point under precautions in the following Languages: Danish, German, Spanish, French, Italian, Dutch, Swedish and Turkish.	S. Tulipano
L	CR1179	12/17/18	Change Medpass Address	I.Fedorov

SpaceOAR® System

Product Code SO-1010

INSTRUCTIONS FOR USE

Indication:

SpaceOAR System is intended to temporarily position the anterior rectal wall away from the prostate during radiotherapy for prostate cancer and in creating this space it is the intent of SpaceOAR System to reduce the radiation dose delivered to the anterior rectum.

Contraindications:

None

Overview:

The SpaceOAR System consists of components for preparation of a synthetic, absorbable hydrogel spacer and a delivery system packaged in a single use kit. The *in situ* formed hydrogel spacer creates a temporary space between the prostate and rectum during radiation therapy. The spacer is formed by mixing two solutions, the Precursor and the Accelerator. The Precursor solution is formed through the mixing of the Diluent solution (Trilysine buffer solution) with the PEG powder. The Accelerator solution is a salt buffer solution. When mixed together, the solutions cross-link to form a soft hydrogel. The mixing of the solutions is accomplished as the materials pass through a static mixer in the Y-Connector prior to passing through the injection needle. SpaceOAR hydrogel implant is MR Safe.

The hydrogel spacer maintains space for approximately 3 months and is absorbed in about 6 months, sufficient time to support the intended use.

The SpaceOAR System is provided sterile and consists of the following components:

Powder Vial, with blue label	
Diluent Syringe, with blue label	
Accelerator Syringe	
Y-Connector	
Syringe Holder	
Plunger Cap	
18G x 15cm Needle	

* No components are comprised of any latex material.

Required Equipment:

For this procedure a side-fire transrectal ultrasound (TRUS) probe and a stepper are required; a stand-off balloon is recommended.

Warnings:

- SpaceOAR System must only be administered via an aseptic transperineal route. Do not administer transrectally.
- If the needle enters the rectal lumen at any time during the procedure, abandon the procedure to avoid infection. The perirectal space may not open during hydrodissection, e.g., scar tissue. If the perirectal space does not open with saline do not inject SpaceOAR.

Precautions:

- SpaceOAR Hydrogel System procedures should only be performed by physicians who have received appropriate training for proper hydrogel spacer deployment technique.
- Users of SpaceOAR System should be familiar with ultrasound needle placement during transperineal procedures.
- The SpaceOAR System is provided sterile. Do not use if packaging or seal has been damaged or opened. Do not re-sterilize.
- Do not use if the PEG powder is not free flowing.
- All kit components are intended for single-use only. SpaceOAR components cannot be re-used.
- Discard opened and unused product.
- Use only with delivery system provided. Appropriate mixing of the Precursor and Accelerator solutions will not occur if the supplied Y-Connector is not used.
- Use within 1 hour of preparing the Precursor solution. Discard entire kit if not used within 1 hour.
- If placing fiducials, do so with a transperineal approach prior to SpaceOAR hydrogel injection.
- SpaceOAR System injection should proceed uninterrupted, without stopping. Stopping during injection may result in device plugging, requiring the preparation of a replacement kit.

Risks:

Potential complications that may be associated with the use of SpaceOAR System include, but are not limited to: pain associated with SpaceOAR hydrogel injection; pain or discomfort associated with SpaceOAR hydrogel; needle penetration of the bladder, prostate, rectal wall, rectum, or urethra; injection of SpaceOAR hydrogel into bladder, prostate, rectal wall, rectum, or urethra; local inflammatory reactions; infection; injection of air, fluid, or SpaceOAR hydrogel intravascularly; urinary retention; rectal mucosal damage, ulcers, necrosis; bleeding; constipation; and rectal urgency.

Detailed Kit Preparation


The use of SpaceOAR System consists of three steps:

- A. Preparing the Precursor Syringe
- B. Assembling the delivery components for injection
- C. Positioning the needle and injection of SpaceOAR hydrogel

A. Preparing the Precursor Syringe


1. Using sterile technique, transfer the contents of the SpaceOAR System onto the sterile field.
2. Remove the blue cap from the Diluent Syringe and discard.
3. Attach the Diluent Syringe (blue label) to the Powder Vial (blue label)

4. Without depressing the syringe plunger, pierce the vial seal by pushing the syringe into the vial cap until it is fully depressed (twisting not required). The entire reference line should disappear below the vial rim.
5. Inject syringe contents into the vial.
6. Shake the vial/syringe assembly until the powder is completely dissolved and set aside for approximately one minute. The solution may appear to be milky with bubbles.
7. Invert the vial/syringe assembly and draw 5 mL of Precursor back into the syringe.
8. Unscrew the Syringe from the Powder Vial and discard the vial. This is the Precursor Syringe.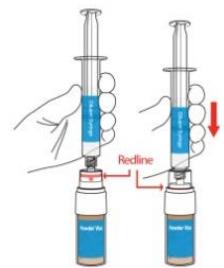

B. Assembling the Delivery Components for Injection


1. Remove the cap from the Accelerator Syringe and expel liquid as needed so that 5 mL remains in the syringe. Check that the Diluent and the Accelerator Syringe fluid levels are equal.

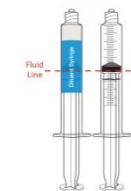
2. Pull back 1 mL of air into each syringe.


3. With the syringes held upright attach Precursor and Accelerator Syringes to the Y-Connector.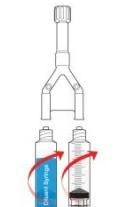

- Attach Syringe Holder to syringe barrels.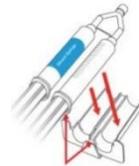


- Carefully attach the Plunger Cap to the plungers of both syringes while holding the plungers to avoid dispensing solutions into the Y-Connector.

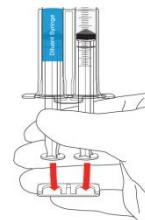- Advance the syringe plungers to expel most of the air, but not all, introduced in Step B2. Do not allow Precursor fluids to enter the Y-Connector at this time.

SpaceOAR Delivery System prior to needle attachment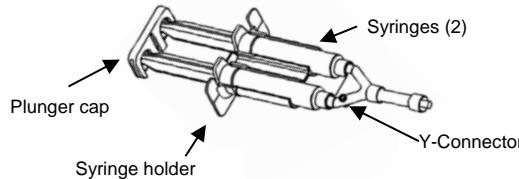


Detailed Procedure Instruction

C. Positioning the Needle and Injection of SpaceOAR hydrogel

- Position the TRUS probe to enable visual guidance of the needle into the space between the prostate and rectum. If needed, lower probe to reduce pressure on anterior rectal wall and prostate, allowing for creation of perirectal space.
- Attach the 18G x 15cm needle (provided) to a syringe containing saline (not provided).
- Insert the needle approximately 1-2 cm anterior to the anal opening. Angle the needle as needed to reach the perirectal fat between the anterior rectal wall and the prostate.


- Advance needle through the rectourethralis muscle to the perirectal fat approximately midgland (Figure 1). Confirm midline needle position in sagittal and axial views and verify that the needle tip is in the perirectal fat.

Figure 1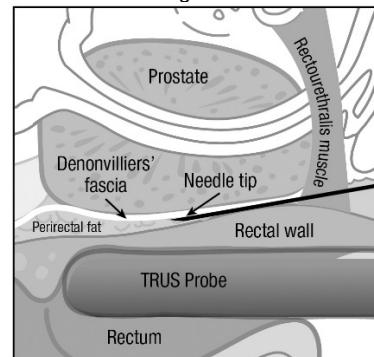

- Inject small amounts of saline to hydrodissect the space between Denonvilliers' fascia and anterior rectal wall.
- Access axial view to confirm needle is in the correct location (mid-gland and centered) and aspirate to ensure needle is not intravascular.
- While maintaining the desired needle position, carefully disconnect the syringe from the 18G needle.
- Being careful to maintain needle position, attach the SpaceOAR delivery system to the 18G needle.
- Under ultrasound guidance in the sagittal view, use a smooth, continuous injection technique to dispense the SpaceOAR hydrogel into the space between the prostate and rectal wall (Figure 2). Inject entire syringe contents without stopping.

Figure 2

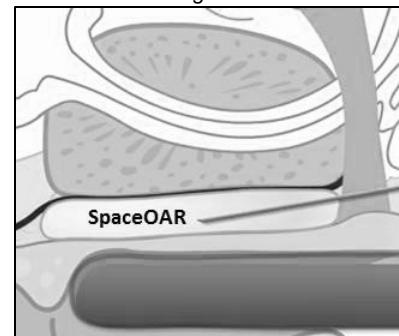- Withdraw the needle/syringe assembly and discard the SpaceOAR System using appropriate precautions for sharp objects.

Not made with
natural rubber
latex

SpaceOAR System must be stored at or below 25°C (77°F) in order to maintain product performance.

Manufacturer:

Augmenix, Inc.
201 Burlington Rd
Bedford, MA 01730 USA

Authorized Representative:
MedPass SAS
95 bis Boulevard Pereire
75017 Paris – France

SpaceOAR is a registered trademark of Augmenix, Inc.

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SpaceOAR è un marchio registrato appartenente a Augmenix, Inc.