



Consultation with Hepatic Artery Infusion (HAI) provider to learn about HAI therapy requirements and eligibility

Scans to rule out cancer spread and confirm eligibility for HAI procedure

Systemic chemotherapy is discontinued 3-4 weeks before surgery



Surgery to implant HAI pump and pump is filled



After-surgery hospital stay



Return every 2 weeks for lab testing and pump refills alternating between fluorouracil and heparinized saline

First HAI pump refill 2 weeks after surgery



Discharge from hospital

Post-surgery pump flow scan



If you cannot keep a required refill appointment, call your HAI provider to ensure your pump doesn't run dry

Systemic chemotherapy is restarted 4-6 weeks after surgery

Continued follow-up care as determined by your HAI provider to monitor response to treatment



During HAI therapy, inform your HAI provider about travel and activities you plan to do



HAI provider will determine when to discontinue HAI therapy and whether to maintain the pump with periodic refills



Need support along the way?
Scan to connect with a Patient Educator



INTERA 3000™ Hepatic Artery Infusion Pump | PATIENT JOURNEY

Intera 3000 Hepatic Artery Infusion Pump Important Safety Information

The Intera 3000 Hepatic Artery Infusion Pump is used to deliver medications such as chemotherapy to the blood vessels in the liver.

With all medical procedures, there are risks associated with the procedure to place the pump in your abdomen and with the use of the pump to deliver medications. The risks associated with placement and use of the pump include but are not limited to bleeding, blood clots, blockage within the device, damage to the blood vessels, fluid or blood build-up around the device under the skin, infection, movement or displacement of the pump or catheter, pump damage, and leakage or incorrect delivery of the medications.

Special care may be required after placement of the Intera 3000 Pump and includes but is not limited to avoiding rough physical activity, avoiding deep sea or scuba diving, avoiding saunas, avoiding long periods in hot tubs and avoiding direct sun exposure, heating pads or hot water bottles over the area of your abdomen where the pump is placed. Additional care may be needed for air travel, moving to or visiting a high-altitude place, or if you have a fever.

The Intera 3000 Pump is MRI Conditional, which means that certain conditions must be met to safely have an MRI after placement of the pump.

Be sure to talk with your doctor so that you thoroughly understand all of the risks and benefits associated with the use of the device.