




ICESEED™ 1.5 CX and ICESEED™ 1.5 CX Short Cryoablation Needles

WHEN PRECISION MATTERS

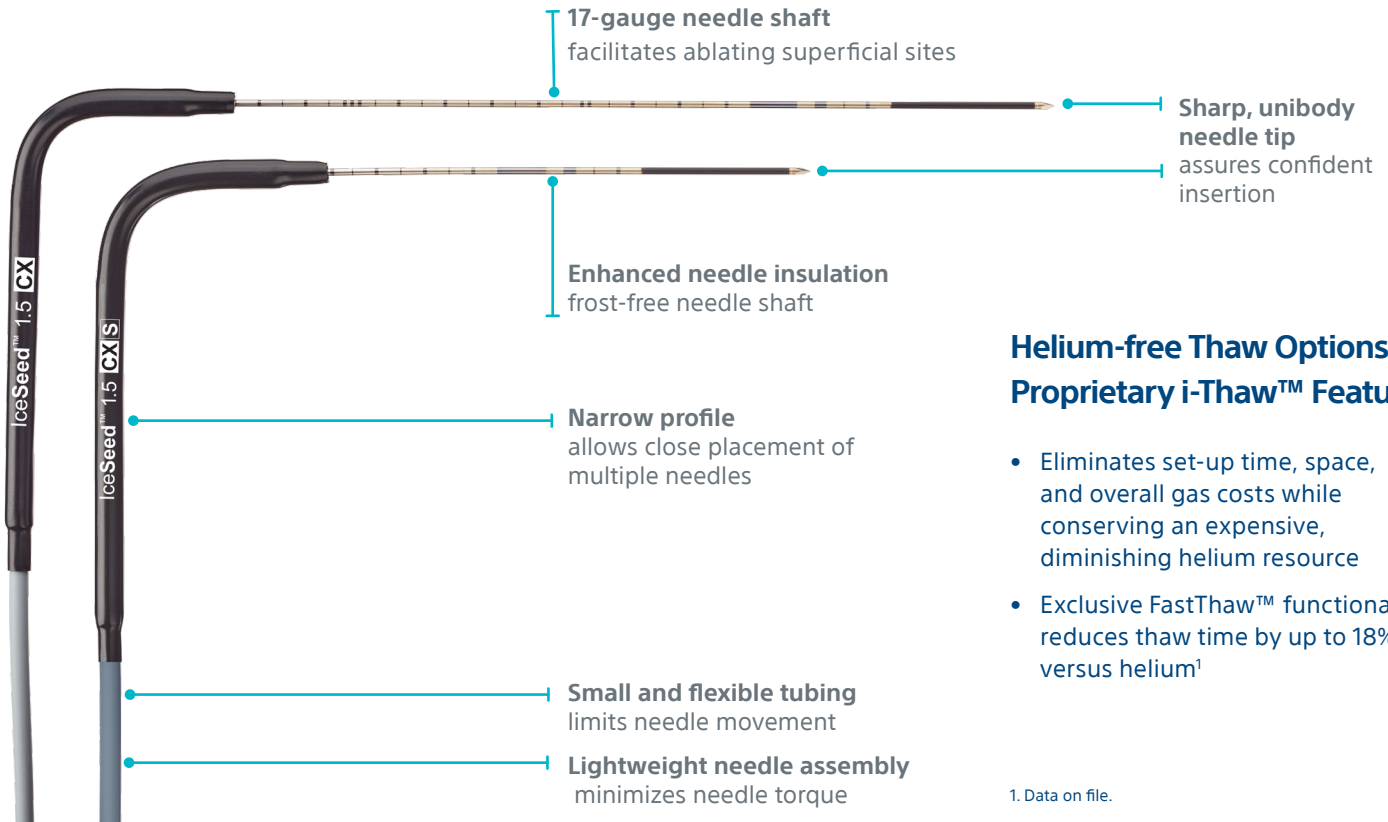


Helium-free Thaw Options via Proprietary i-Thaw™ Feature

- Eliminates set-up time, space, and overall gas costs while conserving an expensive, diminishing helium resource
- Exclusive FastThaw™ functionality reduces thaw time by up to 18% versus helium¹

1. Data on file.

Precise Control...Refined Therapeutic Choice

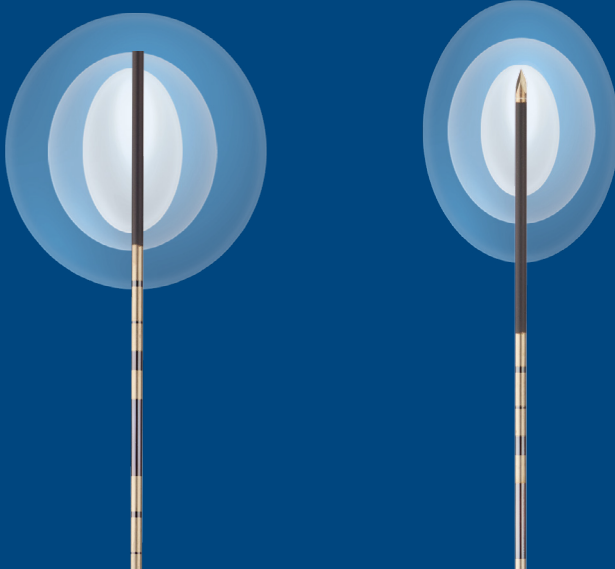

With the benefits of a shorter shaft length combined with the smaller, sculptable iceball size of IceSeed CX, you'll have flexibility of treatments for

- benign or malignant tumors
- cryoneurolysis
- small or superficial lesions


ICESEED™ 1.5 CX and ICESEED™ 1.5 CX Short Cryoablation Needles

IceSeed™ 1.5 CX

IceSeed 1.5 CX Needle:
H7493967433170

Shaft Length:
17.5 cm

Shaft Diameter:
1.5 mm / 17G

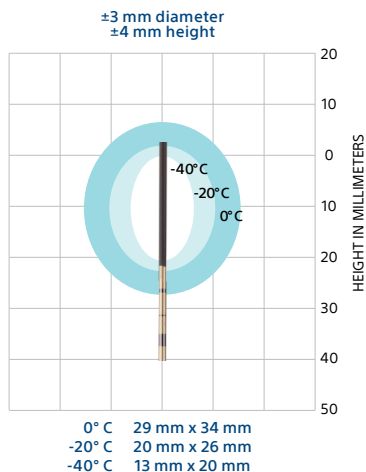


IceSeed™ 1.5 CX Short

IceSeed 1.5 CX Short Needle:
H7493967233100

Shaft Length:
10 cm

Shaft Diameter:
1.5 mm / 17G

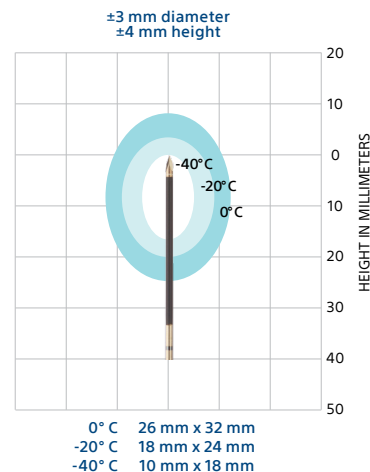

Multi-Needle Isotherm Values (diameter x height in mm)

| CRYOABLATION NEEDLES | SINGLE-NEEDLE ISOTHERM | | | TWO-NEEDLE ISOTHERM | | | THREE-NEEDLE ISOTHERM | | | FOUR-NEEDLE ISOTHERM | | |
|----------------------|------------------------|---------|---------|---------------------|---------|---------|-----------------------|---------|---------|----------------------|---------|---------|
| | 0° C | -20° C | -40° C | 0° C | -20° C | -40° C | 0° C | -20° C | -40° C | 0° C | -20° C | -40° C |
| IceSeed™ 1.5 CX | 29 x 34 | 20 x 26 | 13 x 20 | 41 x 39 | 31 x 29 | 23 x 21 | 48 x 49 | 38 x 39 | 29 x 31 | 52 x 52 | 42 x 42 | 33 x 34 |
| IceSeed™ 1.5 CX S | 26 x 32 | 18 x 24 | 10 x 18 | 39 x 36 | 29 x 26 | 21 x 17 | 43 x 45 | 34 x 36 | 25 x 27 | 49 x 51 | 40 x 41 | 31 x 33 |

SKIN PROTECTION: When performing cryoneurolysis on shallow nerves make sure to place a sterile glove filled with warm saline or warmed ultrasound gel at the surface of the skin. Also, avoid direct contact between skin and needle tubing with the use of towels or draping.

Ordering Information Cryoablation Portfolio of Needles

| CRYOABLATION NEEDLES | PART NUMBER | CONFIGURATION | SHAFT DIAMETER (MM / GAUGE) | SHAFT LENGTH (CM) | HANDLE COLOR | TRACK ABLATION RADIAL WIDTH / LENGTH |
|----------------------|----------------|---------------|-----------------------------|-------------------|--------------|--------------------------------------|
| IceSeed™ 1.5 CX | H7493967433170 | Angled 90° | 1.5 mm / 17 G | 17.5 cm | Black | 1.6 / 14 mm |
| IceSeed™ 1.5 CX S | H7493967233100 | Angled 90° | 1.5 mm / 17 G | 10 cm | Black | 1.6 / 14 mm |



Cryoablation Needles Indications, Safety, and Warnings

<https://www.bostonscientific.com/cryoneedles-indications>



ICEfx™ Cryoablation System Indications, Safety, and Warnings

<https://www.bostonscientific.com/icefx-indications>



Visual ICE™ Cryoablation System Indications, Safety, and Warnings

<https://www.bostonscientific.com/visualicecryo-indications>

Boston Scientific
Advancing science for life™

Interventional Oncology and Embolization

300 Boston Scientific Way
Marlborough, MA 01752-1234
www.bostonscientific.com

To order product or for more information
contact customer service at 1.888.272.1001.

© 2026 Boston Scientific Corporation
or its affiliates. All rights reserved.

PI-1240404-AC