

**Implantable  
Cardioverter  
Defibrillator  
Therapy  
Patient Handbook**





## Your ICD system information

Have your doctor or nurse complete these forms before you go home from the hospital.

ICD Model Number: \_\_\_\_\_

ICD Serial Number: \_\_\_\_\_

ICD Model Type: ☐ ICD ☐ ICD w/ AVT

ICD Features: ☐ RF telemetry ☐ LATITUDE™

Implant Date: \_\_\_\_\_

Lead Model/Serial Numbers: \_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_

\_\_\_\_\_

## Your medical contact information

Electrophysiologist Name/Phone Number:

---

Cardiologist Name/Phone Number:

---

Hospital Name/Address/Phone Number:

---

---

Medications (list):

---

---

**By Mail:**

Boston Scientific Corporation  
4100 Hamline Avenue North  
St. Paul, MN 55112-5798 USA

**By Telephone:**

Worldwide: 651.582.4000  
Medical Professionals: 1.800.CARDIAC (227.3422)  
Patients and Families: 1.866.484.3268

**On the Internet:**

[www.bostonscientific.com](http://www.bostonscientific.com)  
[www.bostonscientific.com/patientlabeling](http://www.bostonscientific.com/patientlabeling)



# Table of contents

<b>Introduction</b> .....	<b>1</b>
When is this device used?, 2	
When is this device not used?, 3	
How reliable is this device?, 3	
<b>Glossary</b> .....	<b>5</b>
<b>Your heart's natural pacemaker</b> .....	<b>20</b>
Arrhythmias and your device, 22	
Ventricular tachycardia, 24	
Ventricular fibrillation, 25	
Atrial fibrillation, 29	
Bradycardia, 33	
<b>Sudden cardiac arrest</b> .....	<b>36</b>
Risk factors, 36	

Identifying your SCA risk, 37

**Your ICD system ..... 40**

The device, 40

The leads, 41

**Implanting your ICD system .....42**

Implant risks, 44

**After your Implant ..... 49**

Medications, 51

Activities and exercise, 51

Your ICD system information, 52

Your identification card, 52

**Living with your ICD..... 54**

Preparing for ICD shock therapy, 56

How therapy feels, 59  
Special considerations, 61  
Replacing your system, 70

**Important safety information.....73**

Operating household appliances and tools, 74  
Theft detection and security systems, 83  
Airport security, 85  
Cellular phones, 86  
Dental and medical procedures, 88

**Visit us on the Internet ..... 94**

**Summary..... 95**

**Visit us on the Internet .....97**

Notes and questions .....	98
Symbols on Packaging .....	100
Index .....	101

The following are trademarks of Boston Scientific or its affiliates: ZIP, LATITUDE

# Introduction

Your doctor has recommended an implantable cardioverter defibrillator (ICD) system to treat your irregular heart rhythm. As a person with an irregular heart rhythm, you may experience symptoms that can affect your quality of life. An ICD is designed to monitor and treat heart rhythm problems, greatly reducing the risks associated with them.

This handbook will tell you how an ICD system treats heart rhythms that are too fast and/or too slow. It will discuss activities you can begin and those you should avoid after your surgery. It will talk about some of the changes that may occur in your life. It will also answer many questions patients typically have. If you have questions about what you read in this handbook, ask your doctor or nurse. They are your best resource for information.

The glossary is located at the front of the handbook. It defines many of the words you will see in the upcoming pages, as well as those you may hear from your doctors and nurses.

### **When is this device used?**

Your doctor has decided that you should receive a defibrillator because you have an increased risk of sudden cardiac death due to ventricular rhythm disturbances. Sudden cardiac death is a result of sudden cardiac arrest, which occurs when electrical problems in the heart cause a dangerously fast and irregular heart rhythm. Other conditions, such as a prior heart attack, symptomatic heart failure that limits everyday activities, and an ejection fraction of 35 percent or less, could also add to your risk of sudden cardiac death. You may also have, or might develop,

atrial rhythm disturbances. If you have any questions about when this device is used, ask your doctor.

### **When is this device not used?**

Patients whose ventricular rhythm disturbances have a reversible or temporary cause should not receive this device. In addition, there may be types of atrial rhythm disturbances for which this device should not be used. If you have any questions about when this device is not used, ask your doctor.

### **How reliable is this device?**

It is Boston Scientific's intent to provide implantable devices of high quality and reliability. However, these devices may exhibit malfunctions that may result in lost or compromised ability

to deliver therapy. Refer to Boston Scientific's *CRM Product Performance Report* on [www.bostonscientific.com](http://www.bostonscientific.com) for more information about device performance, including the types and rates of malfunctions that these devices have experienced historically. While historical data may not be predictive of future device performance, such data can provide important context for understanding the overall reliability of these types of products. Talk with your doctor about this product performance data, and the risks and benefits associated with the implantation of this system.

# Glossary

## **Adaptive rate**

The ability of a device to increase or decrease its pacing rate in response to bodily needs, activity, or exercise.

## **Antitachycardia pacing (ATP)**

A series of small, rapid, low-energy pacing pulses delivered to the heart to slow a rapid heartbeat to its normal rhythm.

## **Arrhythmia**

An abnormal heartbeat that is too fast, too slow, or irregular.

## **Asynchrony**

A condition in which the heart fails to maintain a normal timing sequence between atrial and ventricular contractions.

### **Atrial fibrillation (AF)**

An irregular heart rhythm caused by abnormal electrical signals starting from several areas of the atria. The atria of a heart in AF may beat between 200 and 600 beats per minute. While not usually immediately life-threatening, untreated AF can greatly increase the risk of stroke or heart muscle damage.

### **Atrioventricular (AV) node**

A cluster of cells located in the wall between the right and left atrium, just above the ventricles. This part of the heart's electrical pathway helps carry signals from the atria to the ventricles.

### **Atrioventricular (AV) synchrony**

The normal timing sequence for an atrial contraction followed, after a fraction of a second, by a ventricular contraction.

### Atrium (plural: atria)

One of the two upper chambers of the heart—specifically, the right atrium and left atrium. The atria collect blood as it comes into the heart and pump blood into the lower chambers (ventricles).

### Bradycardia

An abnormally slow heartbeat, typically fewer than 60 beats per minute.

### Cardiac arrest

See *sudden cardiac arrest (SCA)*.

### Cardioversion

Procedure in which a fast heart rate (i.e., ventricular tachycardia or atrial fibrillation) is restored to a normal rhythm with a low-to

moderate-energy electrical shock that is carefully timed with your heartbeat.

### Catheter

A thin, flexible tube or wire inserted into the body for a variety of purposes. Catheters are inserted into the heart during an electrophysiology (EP) test to monitor your heart's electrical activity. Hollow catheters are also used to carry a lead through a blood vessel. See also *electrophysiology (EP) test or study*.

### Communicator

See *LATITUDE Communicator*.

### Defibrillation

Procedure in which a fast heart rate (i.e., ventricular fibrillation, ventricular tachycardia) is restored to a normal rhythm by delivering an electrical shock.

## Defibrillator

A device that delivers an electrical shock to the heart to restore an extremely rapid and sometimes irregular heart rate to a normal rhythm. A defibrillator may be an implanted medical device or external medical equipment.

## Device

See *pulse generator*.

## ECG/EKG (electrocardiogram)

A graphic representation of your heart's electrical signals. The graph shows how electrical signals travel through your heart. Your doctor can tell what kind of rhythm you have by looking at the pattern of your heartbeat.

### Ejection fraction

The percentage of blood ejected from the left ventricle with each heartbeat. A healthy ejection fraction is usually higher than 55%, although this can vary depending on the individual. Patients with a low ejection fraction may have an increased risk of sudden cardiac arrest. Talk with your doctor about your ejection fraction and what impact it has on your health.

### Electromagnetic field

Invisible lines of force that result from electrical fields (produced by voltage) and magnetic fields (produced by current flow). Electromagnetic fields decrease in strength the farther they are from their source.

### Electromagnetic interference (EMI)

Interference that occurs when an electromagnetic field interacts with an implanted device. See also *electromagnetic field*.

### Electrophysiology (EP) test or study

A test in which catheters (thin, flexible tubes or wires) are inserted into your heart to identify and measure the type of electrical signals in your heart. The test results can help your doctor identify the origins of your abnormal heart rhythms, determine how well medications work, and decide what treatment is best for your condition. The test can also be used to see how well your device operates during your abnormal heart rhythm.

### Fibrillation

See *atrial fibrillation* and *ventricular fibrillation*.

### Heart attack

See *myocardial infarction (MI)*.

### **Heart block**

A condition in which the electrical signals of your heart's natural pacemaker (SA node) are delayed or do not reach the ventricles.

### **Heart rhythm**

A series of heartbeats. You may hear your doctor refer to your rhythm as being normal or irregular. A normal heart rate typically ranges from 60 to 100 beats per minute at rest.

### **Implantable Cardioverter Defibrillator (ICD) system**

A device (also called a pulse generator) and leads. An ICD system is implanted to monitor your heart rhythm and help treat dangerously fast or slow arrhythmias.

### **Interrogation**

The process whereby a computerized device (programmer or LATITUDE Communicator) uses telemetry communication signals

to gather identification and status information from your device. Your doctor uses this information to evaluate how your device is performing and check for any arrhythmia episodes you may have had. See also *telemetry communication*.

### **LATITUDE Communicator**

An in-home monitoring system that communicates with your device. The Communicator can gather and send device data to the LATITUDE Patient Management System, which your physician can then view via the Internet. Your device may or may not be configured to use the LATITUDE Patient Management System. See also LATITUDE Patient Management System.

### **LATITUDE Patient Management System**

A remote monitoring system that collects important data from your device. This patient information can be viewed via the Internet, only by members of your health care support team. Your

device may or may not be configured to use the LATITUDE Patient Management System. See also *LATITUDE Communicator*.

### **Lead (pronounced "leed")**

An insulated wire that is implanted in the heart and connected to the device. The lead senses your heartbeat and delivers pacing pulses and/or shocks from the device to the heart. The leads are usually passed into your heart through a vein.

### **Myocardial infarction (MI)**

Also called a heart attack. A myocardial infarction occurs when an artery that supplies blood to the heart becomes blocked. As a result, blood does not reach some parts of the heart, and some of the heart tissue dies. Symptoms of a myocardial infarction may include pain in the chest, arm, or neck; nausea; fatigue; and/or shortness of breath.

## **Pectoral**

The area above the breast and below the collarbone. This is a common area for a device implant.

## **Programmer**

Microcomputer-based equipment that is used to communicate with the device. The programmer is used during testing and follow-up exams to gather and display information from the device. The doctor or technician also uses the programmer to adjust the device so that it senses and treats your arrhythmias.

## **Pulse generator**

Also called a device. The pulse generator is the part of the ICD system that contains the electronics and the battery; it is

implanted under the skin in the pectoral (or, in some cases, abdominal) area. See also *pectoral*.

### **Radio frequency (RF) telemetry communication**

Technology that allows the device to exchange information with a programmer or LATITUDE Communicator by communicating over radio signals. RF telemetry is sometimes referred to as ZIP™ Wandless Telemetry. Your device may or may not be configured for RF telemetry communication. See also *telemetry communication*.

### **Sinoatrial (SA) node**

The heart's natural pacemaker. The SA node is a small group of specialized cells in the upper right chamber of the heart (right atrium) that normally generates an electrical signal. This signal runs through the heart and causes the heart to beat.

### Sudden cardiac arrest (SCA)

The sudden, abrupt loss of heart function (i.e., cardiac arrest) usually due to electrical problems in the heart that cause a dangerously fast and irregular heart rhythm. If untreated, SCA can lead to death (also called sudden cardiac death).

### Sudden cardiac death (SCD)

Death occurring from sudden cardiac arrest. See also *sudden cardiac arrest (SCA)*.

### Supraventricular tachycardia (SVT)

A fast heart rhythm caused by signals coming from a specific area above the ventricles, usually in the atria. A heart with SVT may beat over 150 beats per minute, which may produce palpitations and fluttering in the chest.

## Telemetry communication

Technology that allows a device to exchange information with a programmer or LATITUDE Communicator by using radio frequency (RF) telemetry communication.

## Ventricle

One of two lower chambers of the heart. The right ventricle pumps blood to the lungs, and the left ventricle pumps oxygen-carrying blood from the lungs to the rest of the body.

## Ventricular fibrillation (VF)

A very fast, irregular heart rhythm caused by abnormal electrical signals starting from several areas of the ventricle. The ventricle beats so fast that it pumps very little blood to the body. A heart in VF may beat more than 300 beats per minute. Without immediate medical attention, VF can be fatal. Defibrillation is the only way to treat VF once it occurs.

### **Ventricular tachycardia (VT)**

A fast rhythm caused by abnormal electrical signals coming from the ventricle. The rapid rate of 120 to 250 beats per minute may produce dizziness, weakness, blind spots, and eventual unconsciousness. VT may progress to ventricular fibrillation.

### **Wanded telemetry communication**

Technology that allows a device to exchange information with a programmer or LATITUDE Communicator through a wand that is placed over the skin near the device. See also *telemetry communication*.


### **ZIP™ Wandless Telemetry**

See *radio frequency (RF) telemetry communication*.

## Your heart's natural pacemaker

Your heart works as both a mechanical pump and an electrical organ. It is able to beat because it produces electrical signals. These signals travel through the electrical pathways of your heart (Figure 1), causing the muscle contraction that pumps blood throughout your body.

Normally these signals come from a small area in your heart called the sinoatrial (SA) node. This area is located in the upper right chamber, or right atrium. When the SA node signals the two upper chambers of the heart (the atria), they contract at the same time. The atrial contraction fills the two lower chambers (the ventricles) with blood (Figure 2). As the electrical signal travels through the ventricles, it causes them to contract, which pumps blood out to your body. The contraction of the heart muscle

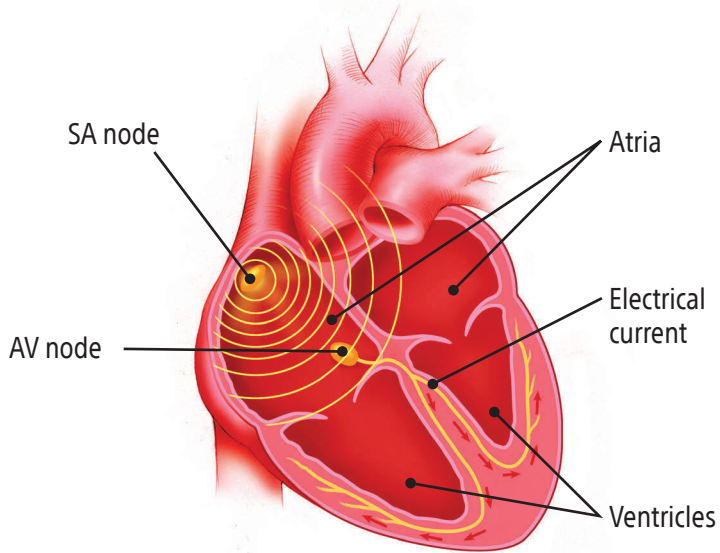

**Figure 1. The heart and its electrical pathways.**

(ventricles) is what you feel as a heartbeat. After a brief rest, the cycle begins again.


## **Arrhythmias and your device**

An arrhythmia occurs when something goes wrong in the heart's electrical system. An arrhythmia is a series of abnormal, often irregular, heartbeats. If the arrhythmia continues, it may prevent the heart from pumping enough blood throughout your body.

### **What your device does**

Your device is designed to monitor and treat certain rhythm problems and greatly reduce the risks associated with them.

Blood flow to  
atria

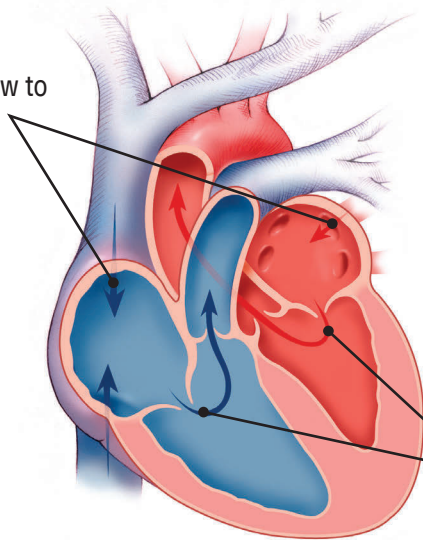

Blood flow  
through  
ventricles

Figure 2. The heart and its blood flow.

Several types of arrhythmias are described in the following paragraphs. Ask your doctor which of these arrhythmias you may experience, and consider recording this information in the “Notes and questions” space on page 98.

## **Ventricular tachycardia**


One type of arrhythmia you may experience is ventricular tachycardia (VT). With this type of arrhythmia, your heart’s electrical signals may come from one of the ventricles instead of the SA node (Figure 3). The electrical signal does not pass through the heart normally and causes a fast, sometimes irregular heartbeat. As your heart beats faster, it pumps less blood to your body. If this rapid heartbeat continues, you may

feel skipped beats or dizziness. You could eventually become unconscious, and your heart might stop beating (cardiac arrest).

VT can sometimes be treated with medication. In other cases, an external defibrillator—such as those used by paramedics—or an ICD may be used to stop the abnormal signals and return your heart to a more normal rhythm.

## **Ventricular fibrillation**


Another type of arrhythmia is ventricular fibrillation (VF). With this arrhythmia, irregular electrical signals come from several spots in the ventricles (Figure 4). This causes a rapid heart rate. In some cases, the heart will beat more than 300 beats per minute.

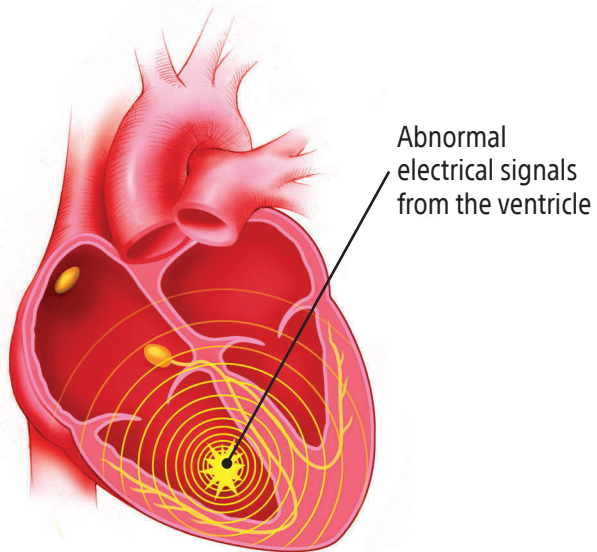

**Figure 3. An example of ventricular tachycardia.**

When you experience VF, very little blood is pumped from your heart to the rest of your body. When your heart is in VF, you will become unconscious very quickly. Like ventricular tachycardia, VF can be treated with a defibrillator. The defibrillator produces an electrical shock that passes through the heart. The shock stops the abnormal signals and allows the SA node to return the heart to a more normal rhythm.

If an episode of VT or VF continues without medical treatment, your heart cannot supply enough oxygen-carrying blood to your brain and body tissues. Without oxygen, your brain and body tissues cannot function normally, which could be fatal.

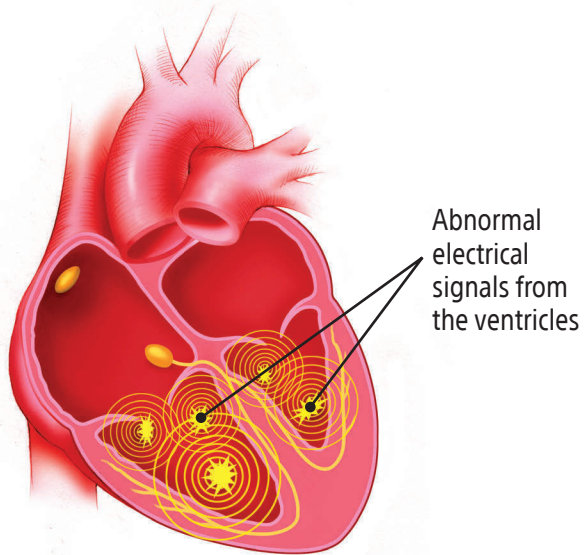

**Figure 4. An example of ventricular fibrillation.**

## Atrial fibrillation


Atrial fibrillation (AF) is a common arrhythmia. When you have AF, your heart has lost atrioventricular (AV) synchrony. Instead of normal electrical conduction, signals start irregularly from several sites in the atria. This causes the atria to quiver rapidly. While in this irregular rhythm, the atria cannot work together with the ventricles to effectively pump blood throughout your body. During AF, your atrial rate increases to between 200 and 600 beats per minute. Because not all of the electrical signals get through to the ventricles, your resulting heart rate is irregular (Figure 5).

AF is not usually an immediately life-threatening arrhythmia. However, it can affect your health in many ways. You may experience palpitations (sudden fluttering, racing, or skipped

heartbeats), chest pain, dizziness, fatigue, or shortness of breath. You could also faint. In addition to these symptoms, people who have AF may also have an increased risk of stroke. It is important that you talk with your doctor about the symptoms associated with this arrhythmia.

## **Types of atrial fibrillation**

There are three types of AF. If you are diagnosed with AF, your doctor will explain which type you have and how your device can treat your atrial arrhythmias. Use the “Notes and questions” space on page 98 to write down important information about your AF.

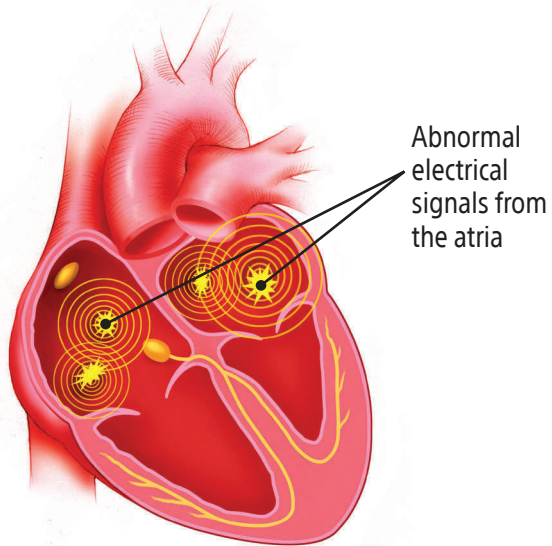

**Figure 5. An example of atrial fibrillation.**

## **Paroxysmal AF**

With paroxysmal AF, your heart rhythm is normal most of the time. When episodes of AF develop, they end on their own without treatment, although they can reoccur.

## **Persistent AF**

With this type of arrhythmia, episodes of AF are more frequent. They also tend to last longer than episodes of paroxysmal AF, and will not end on their own. This arrhythmia can sometimes be treated with medication. For many patients, external electrical cardioversion (restoring a fast heart rate to normal with a low-to moderate-energy electrical shock) may be used to stop the abnormal signals and return the heart to a more normal rhythm.

An ICD with atrial therapy can also provide therapy to maintain a normal heart rhythm.

### **Permanent AF**


With this type of arrhythmia, your heart is always in AF. Unlike paroxysmal or persistent AF, permanent AF will not end on its own nor will it respond to cardioversion.

### **Bradycardia**

Sometimes the heart beats too slowly. This can be caused by the SA node not working properly or by a condition called heart block (Figure 6). Heart block exists when there is a problem with the electrical pathway between the atria and the ventricles. The

natural pacemaker signals sent out by the SA node could be delayed or may not reach the ventricles.

During bradycardia, the chambers of the heart do not contract often enough to supply the proper amount of blood to your body. If you have bradycardia, you may feel tired or dizzy, or you may faint.

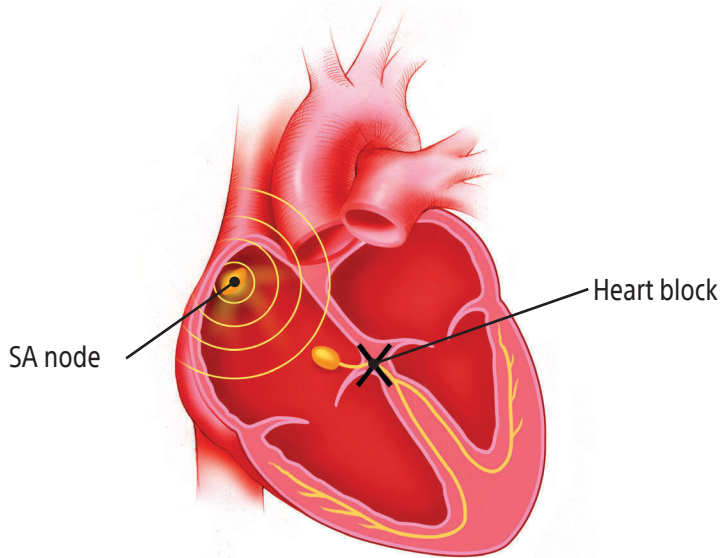

**Figure 6. An example of heart block.**

## Sudden cardiac arrest

If you have had a heart attack, you may also be at risk for sudden cardiac arrest (SCA). Sudden cardiac arrest occurs when the heart beats very fast and irregularly as a result of abnormal electrical signals (VF), causing it to pump very little blood to the body. Because the heart does not pump enough blood throughout the body, most people tend to lose consciousness suddenly. If SCA is not treated, it can lead to sudden cardiac death (SCD). The only way to stop this type of arrhythmia is to deliver an electrical shock with a defibrillator.

### Risk factors

Most people do not have obvious symptoms of SCA, so it is important to be aware of possible risk factors:

- Previous heart attack

- Impaired pumping function of the heart muscle
- Rapid, abnormal heart rhythms coming from the ventricles
- A family history of SCA or SCD

Early identification of your SCA risk is the key to prevention. If you are at risk, it is important to talk to your doctor.

## Identifying your SCA risk

Your doctor may perform one or more of the following tests to assess your risk for SCA.

**Echocardiogram:** An echocardiogram is a test that measures your heart's ejection fraction. The ejection fraction determines your heart's pumping function. During this test, ultrasound waves are used to provide a moving image of your heart. Based upon the

results of this test, your doctor will determine if further testing is needed.

**Holter monitoring:** A Holter monitor is an external monitor that is worn for an extended period. The monitor records your heart's electrical activity, including any arrhythmias you experience. Your doctor analyzes the recording to determine if you experience any abnormal rhythms.

**Electrophysiology (EP) testing:** An EP test identifies and measures the type of electrical signals in your heart. During this test, your doctor will insert catheters (thin, flexible tubes or wires) into your heart. The catheters record electrical signals within your heart. Your doctor can also use the catheters to stimulate your heart to see if you could develop an arrhythmia. This test can help your doctor recognize if you have an abnormal heart rhythm

and identify its origins. It will also determine how well certain medications or an implanted device would work to treat your heart rhythm. Your doctor can then decide what treatment is best for your condition.

## Your ICD system

Your ICD system is designed to monitor and treat your heart arrhythmias. The system consists of a pulse generator (also called a device), which is typically implanted in your chest, and one or more leads, which are implanted in your heart and connected to the device.

### The device

The device is a small computer. It runs on a battery that is safely sealed within its case. The device continuously monitors your heart rhythm and delivers electrical energy (as programmed by your physician) to your heart when it senses an arrhythmia. The device can act as a pacemaker, cardioverter, or defibrillator. For more information about these types of therapies, see "How therapy feels" on page 59.

As the device monitors your heart rhythm, it can also store information about your heart. Your doctor can review this information using a special computer called a programmer. The programmer communicates with the device from outside your body (see "Patient follow-up options" on page 65). With the programmer, your doctor can better evaluate the programmed therapy for your heart rhythm and adjust the settings if necessary.

## The leads


A lead is an insulated wire implanted in your heart and connected to the device. The lead carries the heart signal to the device. It then carries energy from the device back to the heart to coordinate your heart rhythm.

## Implanting your ICD system

An ICD system is implanted during a surgical procedure. To keep you as comfortable as possible, you will be sedated for this surgery. During the procedure, your doctor will insert the lead into a vein, usually through a small incision near your collarbone. The doctor will then pass the lead through the vein into the right ventricle of your heart, where the tip of the lead will rest directly against your heart's inner wall (Figure 7).

If your doctor decides your heart condition requires a dual-chamber system, an additional lead will be implanted. One lead is placed in the right atrium, and the other lead is placed in the right ventricle.

After the leads are positioned, they will be tested to make sure they clearly sense your heart signal and can adequately pace your heart. After this testing, the device will be connected to the

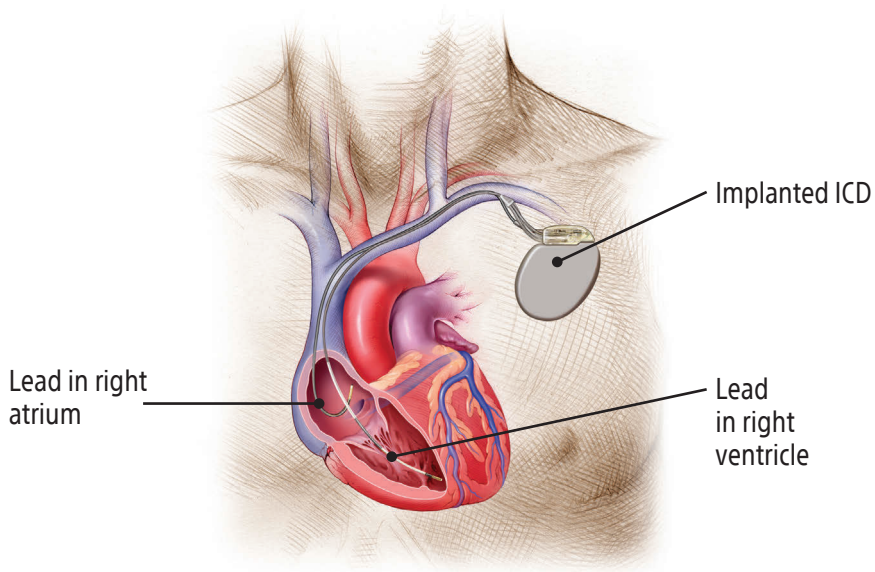

**Figure 7. An implanted ICD system.**

leads and placed in position (usually below the collarbone, just beneath the skin).

Your doctor will then test your ICD system. During this test, your doctor will start an arrhythmia in your heart. The device will recognize the rhythm and give the programmed treatment.

After your doctor has finished testing your system, the incision will be closed. You may experience some discomfort from the incision as you recover from the surgery. You should be able to return to normal activities soon after the procedure.

## **Implant risks**

As with any surgical procedure, it is important to understand that, while complications do not happen very often, there are risks

associated with the implantation of a device or lead. You should talk with your doctor about these risks, including those listed below.

Some of the risks encountered during the implant procedure include, but are not limited to, the following:

- Bleeding
- Formation of a blood clot
- Damage to adjacent structures (tendons, muscles, nerves)
- Puncture of a lung or vein
- Damage to the heart (perforation or tissue damage)
- Dangerous arrhythmias
- Heart attack

- Stroke
- Death

Some of the risks encountered after the system is implanted may include, but are not limited to, the following:

- You may develop an infection.
- You may experience erosion of the skin near your device.
- The lead(s) may move out of place in the heart.
- The electrodes on the lead or the pacing pulses may cause an irritation or damaging effect on the surrounding tissues, including heart tissue and nerves.
- The device may move from the original implant site.

- You may have difficulty coping with having an implanted device.
- The device might be prevented from shocking or pacing due to electromagnetic interference (see "Important safety information" on page 73).
- You may receive a shock or pacing therapy when it is not needed (unnecessary therapy).
- The device might not be able to detect or appropriately treat your heart rhythms.
- The device may exhibit malfunctions that may result in lost or compromised ability to deliver therapy. See "How reliable is this device?" on page 3.

Be sure to talk with your doctor so that you thoroughly understand all of the risks and benefits associated with the implantation of this system.

## After your Implant

As you recover from your implant surgery, you will find that your device may allow you to return to an active lifestyle. It is important that you become actively involved in your recovery by following your doctor's instructions, including:

- Report any redness, swelling, or drainage from your incisions.
- Avoid lifting heavy objects as instructed by your doctor.
- Walk, exercise, and bathe according to your doctor's instructions.
- Do not wear tight clothing that could irritate the skin over your device.

- Contact your doctor if you develop a fever that does not go away in two or three days.
- Ask your doctor any questions you may have about your device, heart rhythm, or medication.
- Avoid rubbing your device or the surrounding chest area.
- If directed by your doctor, limit arm movements that could affect your lead system.
- Avoid rough contact that could result in blows to your implant site.
- Tell your other doctors, dentists, and emergency personnel that you have an implanted device and show them your Medical Device Identification card.

- Contact your doctor if you notice anything unusual or unexpected, such as new symptoms or symptoms like the ones you experienced before you received your device.

## Medications

Your device is designed to help treat your heart condition. However, you may need to continue taking certain medications as well. It is important that you follow your doctor's instructions regarding any medications.

## Activities and exercise

Your doctor will help you decide what level of activity is best for you. He or she can help answer your questions about lifestyle changes, travel, exercise, work, hobbies, and sexual intimacy.

## **Your ICD system information**

Have your doctor or nurse complete the “Your ICD system information” form at the front of this handbook before you go home from the hospital.

## **Your identification card**

Whether you are going away for the weekend or running a quick errand, carry your Medical Device Identification card with you. In an emergency, the card will alert medical and security personnel that you have an implanted device.

You will be given a temporary Medical Device Identification card when you receive your device. Boston Scientific will mail you a

permanent Medical Device Identification card about six to eight weeks after your implant.

Your Medical Device Identification card contains your name, your doctor's name and phone number, and the model numbers of your device and leads.

## **Updating your identification card**

If you move or select a new doctor, please call Boston Scientific Medical Records at 1.800.728.3282 to update your records. When you notify us of a change, we will send you a new identification card reflecting the change.

## Living with your ICD

It is important to follow your doctor's instructions as well as these recommendations:

- Your doctor will arrange a follow-up plan with you to check your device and overall health on a regular basis. It is important that you attend your scheduled in-office follow-up visits, even if you are enrolled in the LATITUDE Patient Management System. LATITUDE does not eliminate the need for in-office visits, although it can minimize the number of them. For more information about the LATITUDE Patient Management System, see "Patient follow-up options" on page 65.

- If you are enrolled in the LATITUDE Patient Management System, use your LATITUDE Communicator as directed by your doctor or clinician.
- Ask your doctor if you have any questions about or notice anything unusual with your device.
- Take the medications prescribed for you as instructed by your doctor.
- Carry your identification card and medication list with you at all times.
- Tell your family doctor, dentist, and emergency personnel that you have an implanted device.

## Preparing for ICD shock therapy

While the device's monitoring of your heart won't cause any noticeable sensations, shock therapy for an arrhythmia may be very noticeable. It is important that you know what to expect.

Before you experience symptoms or receive a shock, discuss with your doctor or nurse a plan for contacting your doctor and, if necessary, emergency personnel. Use the forms in this handbook to write down important telephone numbers and information about your current medications. It might be helpful to keep this information near your phone.

If you have symptoms of a fast heart rate, it is likely that your device will deliver therapy within a few seconds. Try to remain

calm, and find a place to sit or lie down. The sensation from receiving therapy should only last a moment.

It is possible, however, that you may require additional medical attention. Be sure to talk with your doctor about what you should do, and consider the following suggestions:

1. If possible, have someone who is prepared to perform cardiopulmonary resuscitation (CPR)—should you need it—stay with you through the event.
2. Make sure a friend or family member knows to phone your local emergency response system if you remain unconscious.
3. If you are conscious but do not feel well after a shock, have someone call your doctor.

4. If you feel fine after a shock and no more symptoms appear, it may not be necessary to seek medical help immediately. However, follow your doctor's instructions for when to call his or her office. For example, if a shock occurs at night, your doctor may tell you to call him or her the next morning. Someone at the doctor's office will ask you questions such as:
- What were you doing right before the shock?
  - What symptoms did you notice before the shock?
  - At what time did the shock occur?
  - How did you feel right after the shock?
5. It is possible that you could feel symptoms of an arrhythmia but not receive therapy. This depends on the programmed settings of your device. For example, an arrhythmia may cause

symptoms, but it may not be fast enough for your device to deliver therapy. In any case, if your symptoms are severe or continue for more than a minute or so, you should seek immediate medical attention.

## How therapy feels

Your device is designed to always monitor your heart rhythm. If it senses an arrhythmia, it will deliver therapy to your heart. Remember that your doctor has programmed your device to meet your individual needs. The type of therapy you receive and when you receive it is based upon those programmed settings.

**Antitachycardia pacing (ATP):** If your arrhythmia is fast but regular, your device can deliver a series of small, rapid pacing pulses to interrupt the arrhythmia and return your heart to its

normal rhythm. You may not feel the pacing therapy, or you may have a feeling of fluttering in your chest. Most patients who receive this pacing therapy say it is painless.

**Cardioversion:** If your arrhythmia is very fast but regular, your device can deliver a low- to moderate-energy shock to stop the arrhythmia and return your heart to its normal rhythm. Many patients say cardioversion is mildly uncomfortable, like a thump on the chest. This sensation will only last for a moment.

**Defibrillation:** If your arrhythmia is very irregular and fast, your device can deliver a high-energy shock to stop the arrhythmia and return your heart to its normal rhythm. Many patients faint or become unconscious shortly after a very fast VT or VF rhythm begins. As a result, many patients do not feel these high-energy shocks. Some describe the sudden but brief shock as a “kick in

the chest.” This sensation will only last for a moment. While many find the shock reassuring, other patients may be upset for a short time after the shock is delivered.

**Bradycardia pacing:** If your heart signals are too slow, your device can pace your heart. It sends signals to your upper and/or lower chambers, telling them to contract more frequently to meet your body’s needs. This can help maintain your heart rate until your body’s natural pacemaker is able to take control. Patients do not typically feel the electrical pulses used to pace the heart.

## Special considerations

Your doctor might ask you to avoid activities where the risk of unconsciousness could endanger you or others. These activities

might include driving, swimming or boating alone, or climbing a ladder.

## **Driving**

Driving laws and symptoms caused by your arrhythmia are often the deciding factors in whether you will be allowed to drive. Your doctor will advise you about what is best for your safety and the safety of others.

## **Sexual intimacy**

For most patients, sexual intimacy is not a medical risk. The natural heart rate increase that occurs during sex is the same as the heart rate increase when you exercise. Exercise testing at the hospital will help your doctor program your device settings

so you should not get a shock during sex. If you receive a shock during sex, your partner may feel a tingling sensation. The shock is not harmful to your partner. Be sure to let your doctor know if you receive a shock during sex so he or she can consider reprogramming your device.

## **When to call your doctor**

Your doctor will provide guidelines for when you should contact him or her. In general, phone your doctor if you:

- Receive any arrhythmia therapy from your device and have been instructed to call.
- Have symptoms of an abnormal heart rhythm and have been instructed to call.

- Notice any swelling, redness, or drainage from your incisions.
- Develop a fever that does not go away in two or three days.
- Have questions about your device, heart rhythm, or medications.
- Plan to travel or move away. Work with your doctor to develop a follow-up plan while you are away.
- Hear any beeping sounds from your device. This indicates that your device needs to be checked immediately. See "What should you do if your device starts to beep?" on page 70.
- Notice anything unusual or unexpected, such as new symptoms or symptoms like the ones you had before you received your device.

Remember that your device is designed to monitor and treat your life-threatening arrhythmias. It can be a great source of reassurance for you and your friends and family.

## **Patient follow-up options**

Your doctor will schedule regular follow-up sessions to check your device and monitor your condition. There are two follow-up options: in-office visits and remote follow-up sessions.

### **In-office follow-up visits**

Your doctor will schedule regular in-office follow-up visits. It is important that you attend these visits, even if you are feeling well.

During your visit, the doctor or nurse will use a programmer to check your device. The programmer is a special external computer that can communicate with your device in two ways:

1. By using radio frequency (RF) telemetry communication, if you have an RF-enabled device.
2. By using wand telemetry communication. In this case, the doctor or nurse will place a wand over your skin near your device.

A typical follow-up visit takes about 20 minutes. During your visit, your doctor or nurse will use the programmer to interrogate, or check, your device. They will review your device's memory to evaluate its performance since your last visit and check for any arrhythmia episodes you may have had. If necessary, they will

adjust your device's programmed settings. They will also check the battery to see how much energy is left.

### **Remote follow-up sessions**

Your doctor may want you to use the LATITUDE Patient Management System. When using the LATITUDE Patient Management System, you will receive a home monitoring unit called a Communicator. The Communicator is used to interrogate your device on a regular schedule that is set by your physician. The Communicator then sends the data gathered from your device to the LATITUDE Patient Management secure database. Your doctor can then access this database using an Internet-enabled personal computer.

While use of the Communicator does not eliminate the need for in-office visits that may be scheduled by your physician, it can minimize the number of them. The Communicator cannot reprogram or change any functions of your device. Your physician can only do this using a programmer during an office visit.

### **What you should know about your device's battery**

A battery, safely sealed inside your device, provides the energy needed to monitor your heart rhythm, pace your heart, or deliver electrical therapy. Just like any other type of battery, the battery in your device will be used up over time. Since the battery is permanently sealed within your device, it cannot be replaced when its energy is depleted. Instead, your entire device will need to be replaced (see "Replacing your system" on page 70). How

long your device's battery lasts depends upon the settings your doctor programs and how much therapy you receive.

How will you know if your device's battery is running down?

Device batteries have very predictable behavior over time. Your device will regularly check its own battery. At every follow-up visit, the doctor or nurse will also check to see how much energy is remaining in the battery. When the battery's energy level decreases to a certain point, your device will need to be replaced.

Your doctor can turn on a feature that will cause your device to beep when replacement time is near. See "What should you do if your device starts to beep?" on page 70.

## **What should you do if your device starts to beep?**

Under certain conditions, your device will beep 16 times every 6 hours. Call your doctor immediately any time you hear beeping tones from your device. Your doctor or nurse can demonstrate these beeping tones so you'll recognize them.

## **Replacing your system**

Eventually, the energy in your device's battery will decrease to a point where your device will need to be replaced (see "What you should know about your device's battery" on page 68). Your doctor will monitor your device's battery levels and determine when to replace your device.

To replace your device, your doctor will surgically open the pocket of skin where your device is located. He or she will disconnect your old device from your leads and then check to make sure your leads work properly with your new device.

In rare instances, your leads may not work properly with your new device, and your doctor may need to replace the leads. Your doctor will determine if your leads should be replaced.

Should a lead need to be replaced, your doctor will insert a new lead into a vein, similar to how the original lead was implanted. See "Implanting your ICD system" on page 42.

Your doctor will then connect the leads to your new device. Finally, he or she will test your new system to make sure it is working properly.

After the testing is complete, the pocket of skin will be closed. You may experience some discomfort from the incision as you recover from the surgery. You should be able to return to normal activities soon after the procedure.

## **Risks**

Risks encountered during a device and/or lead replacement procedure are similar to the risks of the initial implant, such as infection, tissue damage, and bleeding. See "Implant risks" on page 44.

Be sure to talk with your doctor about the potential risks when making decisions about replacing your system.

## Important safety information

Your device has built-in features that protect it from interference produced by most electrical equipment. Most of the things you handle or work around on a daily basis are not going to affect your device. However, your device is sensitive to strong electromagnetic interference (EMI) and can be affected by certain sources of electric or magnetic fields.

If your employment requires you to be close to large industrial generators or sources of radar, you may need special consideration before returning to work. If your work takes place in such an environment, please talk with your physician.

## Operating household appliances and tools

Use the following guidelines for safe interaction with many common tools, appliances, and activities.

### Items that are safe under normal use:

- Air purifiers
- Blenders
- CD/DVD players
- Clothes washing machines and dryers
- Electric blankets
- Electric can openers

- Electric invisible fences
- Electric toothbrushes
- Fax/copy machines
- Hair dryers
- Heating pads
- Hot tubs/whirlpool baths

*NOTE: Consult with your doctor before using a hot tub. Your medical condition may not permit this activity; however, it will not harm your device.*

- Laser tag games
- Microwave ovens
- Ovens (electric, convection, and gas)

- Pagers
- Patient alert devices
- Personal computers
- Personal digital assistants (PDAs)

*NOTE: PDAs that also function as cell phones should be kept at least 6 inches (15 cm) away from your device. See "Cellular phones" on page 86.*

- Portable space heaters
- Radios (AM and FM)
- Remote controls (TV, garage door, stereo, camera/video equipment)
- Stoves (electric or gas)

- Televisions
- TV or radio towers (safe outside of restricted areas)
- Tanning beds
- Vacuum cleaners
- VCRs
- Video games

## **Warnings and precautions**

If you use any of the following items, it is important that you keep them the recommended distance away from your device to avoid interaction.

Items that should not be placed directly over your device, but are otherwise safe to use:

- Cordless (household) telephones
- Electric razors
- Hand-held massagers
- Portable MP3 and multimedia players (such as an iPod®) that do not also function as a cellular phone (see "Cellular phones" on page 86).

*NOTE: While portable MP3 players themselves should not interfere with your device, the headphones or earbuds should be stored at least 6 inches (15 cm) away from your device, and you should avoid draping the headphones around your neck.*

---

iPod is a trademark of Apple Inc.

**Items that should remain at least 6 inches (15 cm) away from your device:**

- Cellular phones, including PDAs and portable MP3 players with integrated cellular phones

*NOTE: For more information about cellular phones, see "Cellular phones" on page 86.*

- Devices transmitting Bluetooth® or Wi-Fi signals (cellular phones, wireless Internet routers, etc.)

---

The Bluetooth word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by Boston Scientific Corporation is under license. Other trademarks and trade names are those of their respective owners.

- Headphones and earbuds

*NOTE: It is safe to use headphones and earbuds, but you should refrain from draping them around your neck and from storing them in a breast or other shirt pocket that places them within 6 inches (15 cm) of your device.*

- Magnetic wands used in the game of Bingo
- Purses, attaché bags, backpacks, bracelets, and electronic device cases/holders with magnetic closures/snaps; respiratory masks (e.g., CPAP masks) with magnetic straps; and clothing with built-in magnets

**Items that should remain at least 12 inches (30 cm) away from your device:**

- Battery-powered cordless power tools

- Chain saws
- Corded drills and power tool
- Home power generators
- Lawn mowers
- Leaf blowers
- Remote controls with antennas
- Shop tools (drills, table saws, etc.)
- Slot machines
- Snow blowers
- Stereo speakers

**Items that should remain at least 24 inches (60 cm) away from your device:**

- Arc welders
- CB and police radio antennas
- Running motors and alternators, especially those found in vehicles

*NOTE: Avoid leaning over running motors and alternators of a running vehicle. Alternators create large magnetic fields that can affect your device. However, the distance required to drive or ride in a vehicle is safe.*

**Items that should not be used:**

- Body-fat measuring scales
- Jackhammers

- Magnetic mattresses and chairs
- Stun guns

If you have questions about the EMI safety of a particular appliance, tool, or activity, please call Boston Scientific Patient Services at 1.866.484.3268.

## **Theft detection and security systems**

Electronic antitheft systems (including tag deactivation) and security gates or tag readers that include radio frequency identification (RFID) equipment (often found in store and library doorways, at checkout counters, and in point-of-entry access control systems) should not cause you any worry if you follow these guidelines:

- Walk through theft detection and security systems at a normal pace.
- Do not lean against or linger near these systems.
- Do not lean against checkout counter-mounted or handheld tag deactivation systems.
- Avoid lingering near entrance and exit doorways, as some theft detection systems may be hidden in the walls or the floor in these areas.
- If you are near an electronic antitheft, security, or entry control system and suspect interaction (experience symptoms) with your device from one of these systems, promptly move away from equipment nearby and inform your doctor.

- Most home security systems are unlikely to affect the proper function of your implanted system.

Your Boston Scientific implantable device is unlikely to set off the alarm from an electronic antitheft or security system.

## Airport security

Your device contains metal parts that may set off airport security metal detector alarms. The security archway will not harm your device. Tell security personnel that you have an implanted device and show them your Medical Device Identification card.

Airport security wands could temporarily affect your device or turn it off if the wand is held over it for a period of time (about 30 seconds). If possible, ask to be hand-searched instead of being

searched with a handheld wand. If a wand must be used, inform the security personnel that you have an implanted device. Tell the security personnel not to hold the wand over your device and to perform the search quickly.

If you have questions about airport security, call your doctor or Boston Scientific Patient Services at 1.866.484.3268.

## Cellular phones

Keep your cellular phone at least 6 inches (15 cm) away from your device. Your cellular phone is a source of EMI and could affect your device's operation. This interaction is temporary, and moving the phone away from your device will return it to proper function. To reduce the chance of interaction, follow these precautions:

- Maintain a distance of at least 6 inches (15 cm) between the cellular phone and your device. If the phone transmits more than 3 watts, increase the distance to 12 inches (30 cm).
- Hold the cellular phone to your ear on the opposite side of your body from your device.
- Do not carry a cellular phone in a breast pocket or on a belt if that places the phone within 6 inches (15 cm) of your device.

These precautions apply only to cellular phones, not to household cordless phones. However, you should avoid placing your household cordless phone receiver directly over your device.

## Dental and medical procedures

Some medical procedures could damage or otherwise affect your device. Be sure to always tell your dentist and physicians that you have an implanted device so that they can take the necessary precautions. Be especially careful with the following procedures:

- **Magnetic Resonance Imaging (MRI):** This is a diagnostic test that uses a strong electromagnetic field. Some defibrillation systems have been evaluated to allow the patient to undergo MRI scans under specific conditions. Talk to your physician about the capabilities of your defibrillation system. If your system is not one of those eligible to be scanned, or if the required conditions are not met, MRI scans can severely damage your device and should not be performed. Hospitals keep MRI equipment in rooms marked with signs that

indicate magnets are inside. Do not go inside these rooms unless your physician has confirmed that your defibrillation system is eligible and you meet the requirements for an MRI scan.

- **Diathermy:** This uses an electrical field to apply heat to tissues in the body and could damage your device or injure you. Diathermy should not be performed.
- **Electrocautery:** This is used during surgical procedures to stop vessels from bleeding. It should be used only when your device is turned off. Talk with your heart doctor and the doctor performing the medical procedure to determine who turns off your device.

- **Electrolysis and Thermolysis:** These are dermatology or hair removal procedures that pass electrical current into the skin. Talk with your heart doctor before having any electrolysis or thermolysis treatment.
- **External defibrillation:** This is a procedure, typically used in medical emergencies, that uses external equipment to deliver an electrical shock to your heart to restore a rapid and irregular heart rate to a normal rhythm. External defibrillation can affect your device, but can still be performed if necessary. If you receive external defibrillation, be sure to contact your physician as soon as possible following the emergency to verify that your device is functioning properly.
- **Lithotripsy:** This is a medical procedure that is used to break up stones in the urinary tract (e.g., kidney stones). Lithotripsy

can damage your device if certain precautions are not taken. Talk with your heart doctor as well as the doctor performing the procedure about what can be done to protect your device.

- **Therapeutic radiation treatment for cancer:** This procedure can affect your device and will require special precautions. If you should need radiation treatment, talk with your heart doctor as well as the doctor performing the medical procedure.
- **Transcutaneous Electrical Nerve Stimulation (TENS) unit:** This is a device prescribed by physicians or chiropractors for control of chronic pain. A TENS unit can affect your device and will require special precautions. If you must use a TENS unit, talk with your heart doctor.

Most medical and dental procedures will not affect your device.  
Some examples include:

- Dental drills and cleaning equipment
- Diagnostic X-rays
- Diagnostic ultrasound procedures
- Mammograms

*NOTE: Mammograms will not interfere with your device. However, your device could be damaged if it gets compressed in the mammogram machine. Make sure the doctor or technician knows that you have an implanted device.*

- EKG machines
- CT scans

If you need to undergo any surgical procedures, tell your dentist and/or doctor that you have an implanted device. They can contact the physician who monitors your device to find the best way to provide treatment.

If you have questions about a specific appliance, tool, medical procedure, or piece of equipment, please talk with your doctor or call Boston Scientific Patient Services at 1.866.484.3268.

## Visit us on the Internet

Boston Scientific's website, [www.bostonscientific.com](http://www.bostonscientific.com), offers a variety of information of interest to people with cardiac devices, including frequently asked questions about living with your implanted device, health tips, patient stories, and links to additional resources.

**[www.bostonscientific.com](http://www.bostonscientific.com)**

## Summary

It is natural for you to feel anxious or nervous about receiving a device. You have been identified by your physician as having a significant risk of sudden cardiac death due to your medical conditions. Remember that your device can be a great source of reassurance for you and your friends and family.

Talking with other ICD patients is often helpful while adjusting to your new device. Ask your doctor, nurse, or Boston Scientific representative if there is a local ICD patient support group in your area.

The information presented in this handbook is intended to help you understand more about your heart condition and your device. If you have questions about what you have read, be

sure to ask your doctor or nurse. They are your best resource for information about your particular needs or situation.

## Visit us on the Internet

Boston Scientific's website, [www.bostonscientific.com](http://www.bostonscientific.com), offers a variety of information of interest to people with cardiac devices, including frequently asked questions about living with your implanted device, health tips, patient stories, and links to additional resources.

**[www.bostonscientific.com](http://www.bostonscientific.com)**

# Notes and questions

Use this space to write down questions or additional information about your device:

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---

---


---

---

---

---

# Symbols on Packaging

Symbol	Definition
	Manufacturer

# Index

## A

- Activities, 51,61
- Airport security, 85
- Antitachycardia pacing, 59
- Arrhythmia, 22
  - atrial fibrillation, 29
  - ventricular fibrillation, 25
  - ventricular tachycardia, 24
- Atria, 20
- Atrial fibrillation, 29
  - paroxysmal, 32
  - permanent, 33
  - persistent, 32
  - types of AF, 30

## B

- Battery, 68
  - beeping tones, 64,70
  - end of life, 69,70
- Beeping tones, see Battery
- Boating, 62
- Bradycardia, 33
- Bradycardia pacing, 61

## C

- Calling your doctor, 63
- Cardioversion therapy, 60
- Cellular phones, 79,86

Communicator, *see* LATITUDE  
Patient Management System

Cordless phones, 78,87

CT scans, 92

## D

Defibrillation therapy, 60

Dental equipment, 92

Dental procedures, 88

Device, 40

- implanting, 42

- reliability, 3

- replacing, 70

Diathermy, 89

Driving, 62

## E

Echocardiogram, 37

EKG machines, 92

Electrocautery, 89,90

Electrolysis and Thermolysis, 90

Electromagnetic interference  
(EMI), 73

Electronics

- precautions, 74

Electrophysiology (EP) test, 38

Exercise, 51

External defibrillation, 90

## F

Follow-up visits, 65  
    in-office, 65  
    remote, 67

## G

Glossary, 5

## H

Heart block, 33  
Heart function, 20  
Holter monitoring, 38  
Household appliances  
    precautions, 74

## I

ICD system, 40  
    device, 40  
    implanting, 42  
    leads, 41  
    reliability, 3  
    replacing, 70  
    risks, 44  
Identification card, 52  
Implanting the system, 42  
    recovery, 49  
    risks, 44

## L

Ladders, 62

LATITUDE Patient Management System, 54,67

Communicator, 55,67

Leads, 41

implanting, 42

replacing, 71

Lithotripsy, 90

Living with your ICD, 54

ID card, 52

preparing for therapy, 56

## M

Mammograms, 92

Medical procedures, 88

Medications, 51

MRI, 88

## P

Paroxysmal AF, 32

Permanent AF, 33

Persistent AF, 32

Precautions, 73

airport security, 85

cellular phones, 79,86

dental procedures, 88

diathermy, 89

electrocautery, 89,90

external defibrillation, 90

household appliances, 74

lithotripsy, 90

- medical procedures, 88
- MRI, 88
- radiation treatment, 91
- TENS units, 91
- theft detection systems, 83
- tools, 74

Programmer, 41, 66

Pulse generator, *see* Device

## R

Radiation treatment, 91

Radio frequency (RF)  
telemetry, 66

Recovery, 49

Reliability, 3

Replacing the system, 70  
risks, 72

Risks, *see* Precautions

- electromagnetic  
interference, 73
- implant procedure, 45
- post-implant, 46
- replacement procedure, 72
- sudden cardiac death, 36

## S

Safety, *see* Precautions

Security systems, 83

Sexual intimacy, 62

Sinoatrial (SA) node, 16

Sudden cardiac arrest, 36

    diagnosis, 37

    risk factors, 36

Sudden cardiac death, *see* Sudden cardiac arrest

Swimming, 62

## T

Telemetry communication

    radio frequency (RF), 66

    wanded, 66

TENS units, 91

Theft detection systems, 83

Therapy

    antitachycardia pacing, 59

    bradycardia pacing, 61

    cardioversion, 60

    defibrillation, 60

    how it feels when delivered, 59

    preparing for, 56

Tools

    precautions, 74

Traveling, 51,64

    airport security, 85

## U

Ultrasound, 92

## V

Ventricles, 20

Ventricular fibrillation (VF), 25

Ventricular tachycardia (VT), 24

## W

Wanted telemetry, 66

Warnings, see Precautions

## X

X-rays, 92





# Boston Scientific

Advancing science for life™



## Manufacturer:

**Boston Scientific Corporation**

4100 Hamline Avenue North

St. Paul, MN 55112-5798 USA

Tel: 651.582.4000

Medical Professionals:

1.800.CARDIAC (227.3422)

Patients and Families: 1.866.484.3268

[www.bostonscientific.com/patientlabeling](http://www.bostonscientific.com/patientlabeling)

© 2020 Boston Scientific Corporation or its  
affiliates. All rights reserved.

## ICD

51114096-001 en USA 2020-12

