


Step 1

Attach Magnet to Mobile Device Case


- 1A** Remove the blue magnet from the metal box.
- 1B** Use the tab to peel off the adhesive backing from the back of the mobile device case.
- 1C** Place the logo side of the magnet on the adhesive and press firmly.

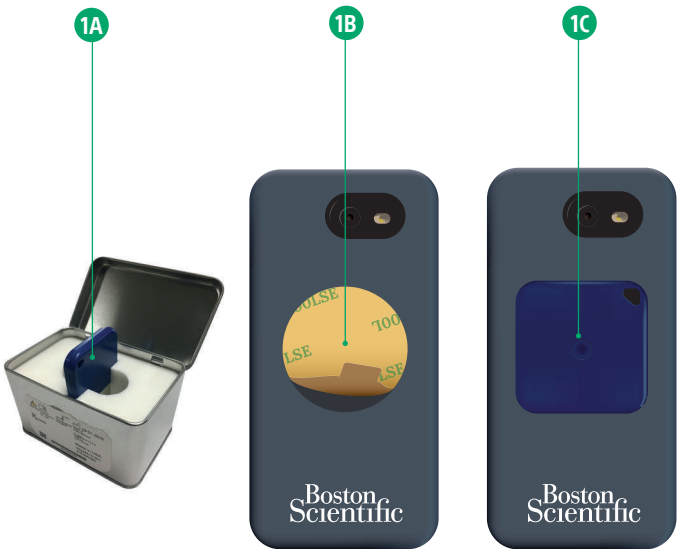

Step 2

Power On your Device

- 2** Press and hold the Power Key.

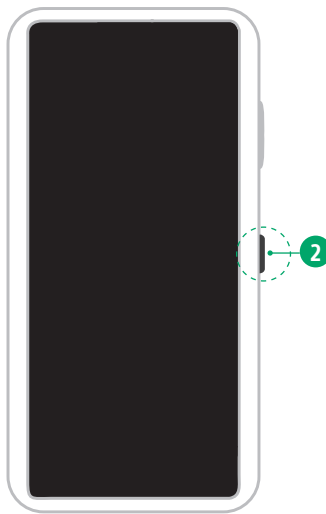


Step 3

Open your App

- 3** Tap the app icon to open your app.


Touchscreen Tip: Gently and quickly tap the screen with your fingertip. Do not use your fingernail.

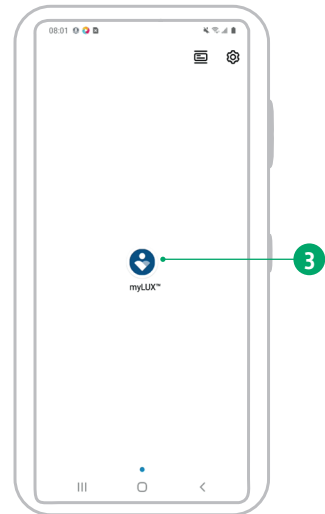

Step 4

If you see this screen...


You're all set up! Please review the section: "Using myLUX" in this Quick Start Guide.

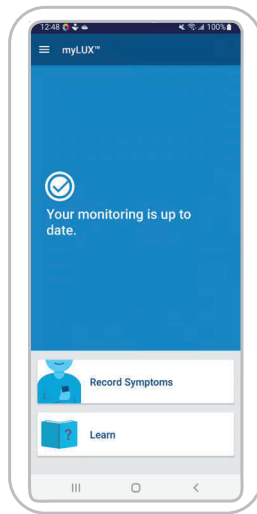

Or, if you see this screen...

- 4A** ... and the app is set to your preferred language, tap **NEXT** and go to step 6.

- 4B** Or, if you prefer, tap **CHANGE LANGUAGE**.


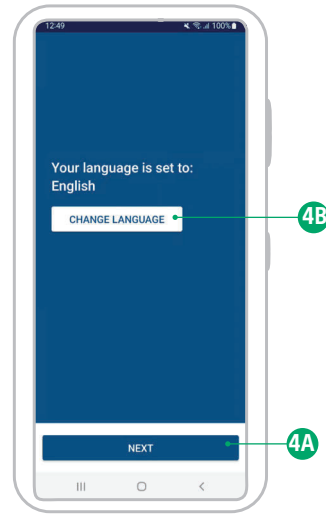


Step 5

Change Language

- 5A** Select your preferred language by tapping the language you want and then tap **APPLY**.


- 5B** Tap the **BACK** button. The screen in step 4A will display in the language you selected. Tap **NEXT**.

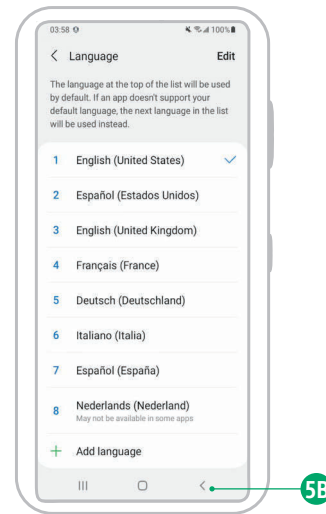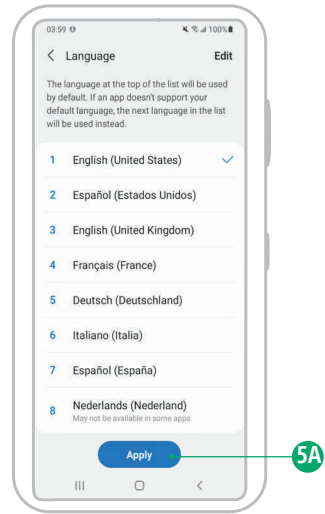

Step 6

Complete Setup


- 6** Tap **START SETUP** and follow the on-screen instructions.

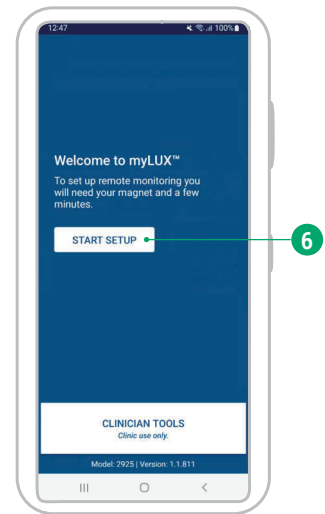

Step 7


All done!

Setup is complete! Please review the section: "Using myLUX" in this Quick Start Guide.

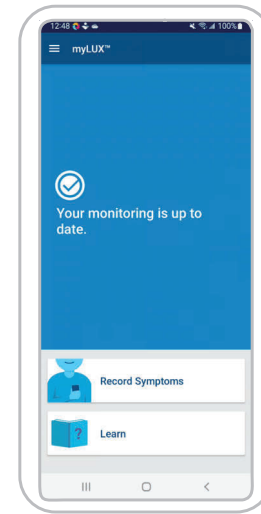

Using myLUX

- Keep your mobile device powered on at all times
- You do not need to bring your mobile device with you during the day, unless instructed by your clinic
- Charge your mobile device each night using the charging cord, and place within 6 feet (2 meters) from where you sleep

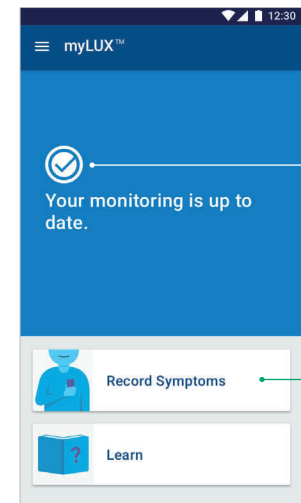

When you see this icon, you know that everything is working fine — no action is needed.

You may have a “Record Symptoms” button. **Use the button only when feeling symptoms that your clinic has asked you to record.** This button is not available to every patient.

How it works



Your insertable cardiac monitor (ICM) automatically records and stores irregular heart rhythms — whether you’re near your mobile device or not.




Each night, your mobile device collects irregular heart rhythms stored in your ICM and automatically sends that data for your clinic to review.



Your clinic will contact you if there is a need for further review.

CAUTION: The ICM system is not meant to assist with health emergencies. If you need immediate medical attention, call your health care provider or seek emergency medical services.

Your mobile device

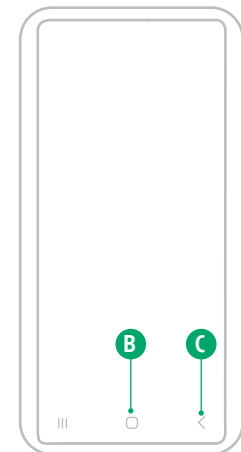

A. Power Key

B. Home

C. Back

D. Charging Port

Figures and instructions that describe the mobile device are representative. Your actual mobile device may vary slightly in appearance.



Manufacturer

Boston Scientific Corporation
4100 Hamline Avenue North
St. Paul, MN 55112 USA

Tel: 651.582.4000

Medical Professionals: 1.800.CARDIAC (227.3422)

Patients and Families: 1.866.484.3268

www.bostonscientific.com
www.bostonscientific.com/patientlabeling

2023-03

51222801-001 en USA



Related professional use document:
51583079-001

Quick Start Guide



For a Boston Scientific Provided myLUX™ Mobile Device

myLUX™ Patient App

REF 2925

for LUX-Dx™, LUX-Dx II™, and
LUX-Dx II+™ Insertable Cardiac
Monitor (ICM)

REF M301, M302, M312

Reference number

CAUTION: Federal law (USA) restricts this device to sale by or on the order of a licensed healthcare practitioner.

© 2023 Boston Scientific Corporation or its affiliates.
myLUX, LUX-Dx, LUX-Dx II, and LUX-Dx II+ are trademarks of Boston Scientific Corporation or its affiliates.